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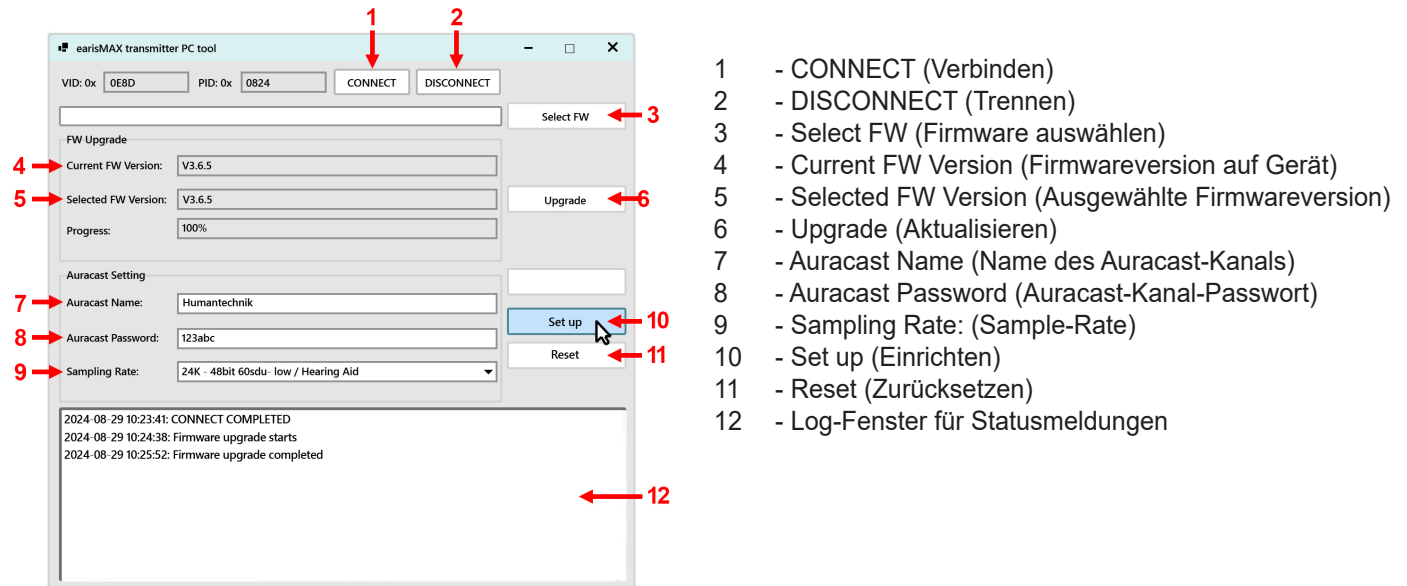
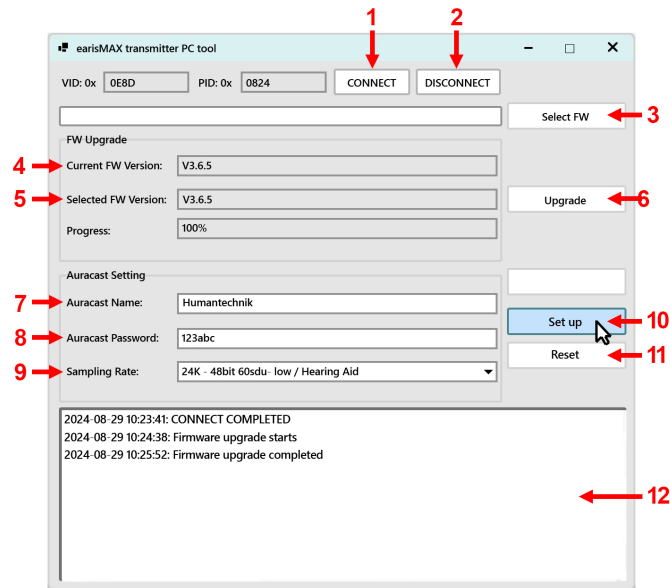
# Setting up and using the earisMAX Auracast transmitter PC tool

Download the zip-packed PC tool (Windows only) from the Humanteknik website:  
<https://www.humanteknik.com/downloads/earisMAX-PC-tool.zip>

Unzip the folder into a directory of your choice. This directory now contains the programme „earisMAX transmitter PC tool.exe“.

Before running the tool, the earisMAX Auracast transmitter must be connected to your computer via USB cable. Double-click to start the programme. The tool is now open:

- 1- CONNECT
- 2- DISCONNECT
- 3- Select FW (Select firmware)
- 4- Current FW Version (firmwareversion on the device)
- 5- Selected FW Version
- 6- Upgrade
- 7- Auracast Name
- 8- Auracast Password
- 9- Sampling Rate
- 10- Set up
- 11- Reset
- 12- Log window for status messages

Click on „CONNECT“ (1) to connect the device to the app. The currently installed firmware version is displayed under (4).

## Firmware update

If a more recent firmware version is available (can be viewed on the product website at <https://www.humanteknik.com/produkte/tv-hoersysteme/earismax-das-system/earismax-bluetooth-sender>), the zip-packed firmware package can be downloaded from there.

The selected firmware version should now be displayed under „Selected FW Version“ (5). This is transferred to the earisMAX Auracast transmitter by clicking on „Upgrade“ (6). Do not switch off the computer during this process and do not disconnect the transmitter’s USB cable - otherwise the device could become unusable.

The process can take up to 10 minutes. Once completed, a corresponding confirmation is displayed in the log window (12).

## Encryption of the Auracast stream and setting the sample rate

The reception of the Auracast stream generated by the transmitter can be restricted by setting a password. The name of the Auracast stream can be set under (7) and the password under (8). Please only use letters and numbers for this. The password should consist of 4 to 8 characters. If the password field is left blank, the stream is freely accessible to all listeners within range.

The sample rate can be selected under (9). If hearing systems are among the Auracast receivers, a setting of 24K may be necessary. The default setting of 48K is used for transmission in high sound quality.

These settings are transferred to the transmitter by clicking on „Set up“ (10). The changes can be recognised via the earisMAX app (or other Auracast switching apps) when the stream is received.

Click on „Reset“ (11) to reset the name of the stream, the password and the sample rate to the default values (no stream name, no password, 48K).

**Important:** Please disconnect the transmitter from the programme by clicking on „Disconnect“ (2) after completing all processes. The programme can then be closed and the transmitter removed from the computer.

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