HUMANTECHNIK

lisa

D

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Seite 2

Feueralarmsender akustisch

GB

Operating Instructions

Page 8

Fire alarm transmitter acoustic

F

Mode d'emploi

Page 12

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
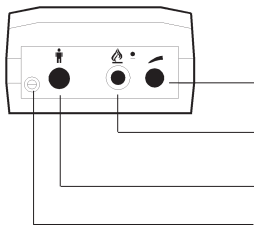
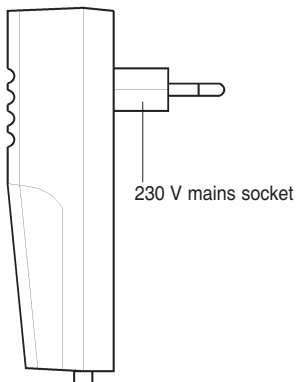
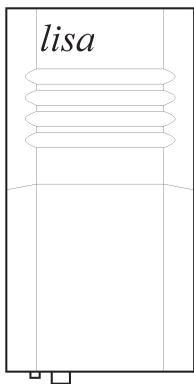
Dial switch for setting the microphone sensitivity

Microphone socket

Person call button

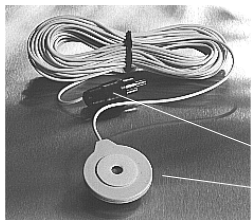Channel selector switch

Microphone cable 5 m/10 m

Order Number:

A-2955-0 (length 5 m)

A-2956-0 (length 10 m)

Plug to connection socket
fire alarm transmitter acoustic

Microphone to stick next to the acoustic output of the smoke detector.

Congratulations on purchasing the *fire alarm transmitter acoustic*. We hope it will give you many years of reassuring operation. Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

Standard components

Please check if all following components are included:

- *lisa fire alarm transmitter acoustic*
- Operating instructions
- Warranty card

lisa system introduction

An alerting system requires at least one transmitter and a receiver. The transmitter captures sound signals and transforms them into digital signals that are passed to a receiver by the existing mains electricity supply. The receiver then transforms these digital signals into light signals.

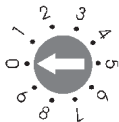
Channel selection

A digital transmission system ensures the correct recognition of each of the digital signals from each transmitter. The system has 10 different selectable channels, 0-9. The factory setting for all *lisa* devices is channel 0. Only where more than one installation is required in the same building will it be necessary to select a different channel for each system.

To select a channel, turn the rotary switch on the underside of the transmitter to the required channel number using a small screwdriver.

Example:

Set channel number 0

Personal call button

Pressing this button causes a certain flash sequence on each receiver. It is possible to call people in other rooms this way. This button also allows the *lisa* system to be tested without any connecting cables.

Duration of flashing signals

The factory setting for the duration of the flash is 28 seconds for all *lisa* receivers. However, you can reduce this to 8 seconds if you wish.

To change the flash duration for a particular transmitter, remove the transmitter from the mains socket and, while pressing in the personal call button, plug the transmitter back into the socket. To switch back to 28 seconds remove the transmitter from the socket and plug back in again this time without pressing the personal call button.

Smoke detector connection

Use the adhesive ring to glue the microphone next to the sound outlet of your smoke detector (see illustration) and plug the microphone cable into the connection jack on the bottom of the transmitter. To do this, pull off the backing foil from the adhesive ring on the microphone.

Installation example

For optimum function of the transmitter, its sensitivity dial switch must be set so the transmitter reliably responds to the smoke detector.

Setting the sensitivity

1. Set the sensitivity dial switch to its lowest level (turn to the anticlockwise to stop).
2. Activate the test alarm of the smoke detector and gradually increase the sensitivity of the transmitter (by turning the dial clockwise) until the transmitter reacts to the smoke detector.

Important note

All *lisa* devices connected to the mains supply have a low power consumption which may lead to the device feeling warm. This is normal and does not interfere with the normal function of the devices.

These signalling systems have a high operational safety. Should any faults occur in spite of proper installation and use, please contact your dealer. High and low voltage spikes (e.g. during a thunderstorm) can occur on the mains supply. The *lisa* devices have an integrated electronic controller which may stop working after these conditions. If this happens simply unplug each of the devices and plug them in again.

As a precaution remove all devices from their sockets during a thunderstorm.

The use of halogen lamps or other equipment (e.g. Microwave Ovens) may, in rare cases, interfere with the *lisa* signals. In these cases the use of mains filters or suppressors is advisable. Please ask your dealer.

Maintenance and care

All *lisa* devices are maintenance free. To clean, remove from the mains socket and wipe with a moistened cloth. Never use cleaning agents.

lisa devices should not be exposed to excessive heat, dust, humidity or mechanical shock.

Note: This product is **not** protected against splash water. against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The *lisa fire alarm transmitter acoustic* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the pak-

kaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical data




Power supply:	230 V, 50 Hz
Power consumption:	approx. 1 W
Height:	121 mm
Width:	65 mm
Depth:	35 mm
Weight:	200 g

This product complies with European Union norms.

You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

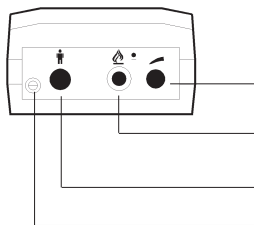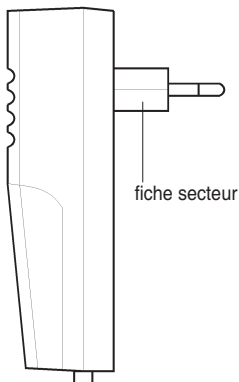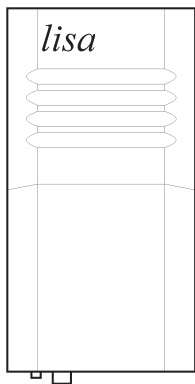Technical specifications subject to change without notice.

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Sarabec Ltd