D

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Seite 2

Türklingelsender galvanisch

GB

Operating Instructions

Page 8

Doorbell transmitter direct

F

Mode d'emploi

Page 12

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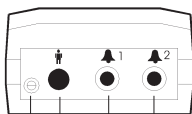
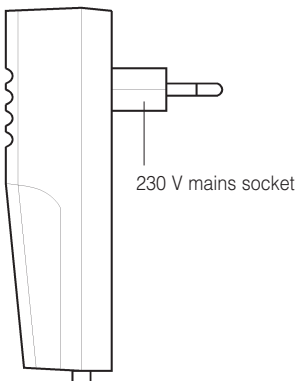
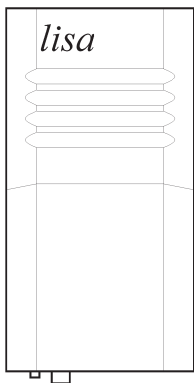
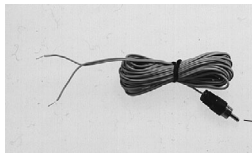
- Connection socket for door II
- Connection socket for door I
- Person call button
- Channel selector switch

Direct connection doorbell cable
Order number:
A-2950-0 (length 5 m)
A-2951-0 (length 10 m)

— Plug to connection socket

Congratulations on purchasing the *doorbell transmitter direct*.

We hope it will give you many years of reassuring operation. Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

Standard components

Please check if all following components are included:

- *lisa doorbell transmitter direct*
- Operating instructions
- Warranty card

lisa system introduction

An alerting system requires at least one transmitter and a receiver. The transmitter captures signals and transforms them into digital signals that are passed to a receiver by the existing mains electricity supply. The receiver then transforms these digital signals into light signals.

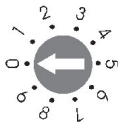
Channel selection

A digital transmission system ensures the correct recognition of each of the digital signals from each transmitter. The system has 10 different selectable channels, 0-9. The factory setting for all *lisa* devices is channel 0. Only where more than one installation is required in the same building will it be necessary to select a different channel for each system.

To select a channel, turn the rotary switch on the underside of the transmitter to the required channel number using a small screwdriver.

Example:

Set channel number 0

Person call button

Pressing this button causes a certain flash sequence on each receiver. It is possible to call people in other rooms this way. This button also allows the *lisa* system to be tested without any connecting cables.

Duration of flashing signals

The factory setting for the duration of the flash is 28 seconds for all *lisa* receivers. However, you can reduce this to 8 seconds if you wish.

To change the flash duration for a particular transmitter, remove the transmitter from the mains socket and, while pressing in the person call button, plug the transmitter back into the socket. To switch back to 28 seconds remove the transmitter from the socket and plug back in again this time without pressing the person call button.

Doorbell connection

Plug the transmitter into a convenient mains socket. Plug the 2.5 mm cinch plug of the doorbell cable into the socket at the underside of the transmitter. Lay the cable to the bell push or door chime. Connect both wires to the screw contacts of the bell push or door chime.

Warning: The transmitter is only designed to work with low voltage doorbells. Do not connect the thin doorbell cable to the mains supply. This could be extremely dangerous for the user and may destroy the transmitter. The doorbell voltage must be at least 3 Volts and must not exceed 15 Volts. Both AC and DC can be used. Pressing your doorbell will now cause each receiver to flash with the corresponding number of flashes. A second doorbell may be connected into the remaining doorbell socket. The receiver will flash differently for doorbell 1 and doorbell 2.

Failure causes

- You did not press the doorbell for long enough. Ring the bell for longer.
- If your doorbell system uses direct current voltage, you have not connected the wires with the correct polarity. Swap over the wires.
- The doorbell voltage of your system is too low. Either contact an electrician to have the doorbell system converted or use the doorbell transmitter acoustic.

Important note

All *lisa* devices connected to the mains supply have a low power consumption which may lead to the device feeling warm. This is normal and does not interfere with the normal function of the devices.

These signalling systems have a high operational safety. Should any faults occur in spite of proper installation and use, please contact your dealer. High and low voltage spikes (e.g. during a thunderstorm) can occur on the mains supply. The *lisa* devices have an integrated electronic controller which may stop working after these conditions. If this happens simply unplug each of the devices and plug them in again.

As a precaution remove all devices from their sockets during a thunderstorm.

The use of halogen lamps or other equipment (e.g. Microwave Ovens) may, in rare cases, interfere with the *lisa* signals. In these cases the use of mains filters or suppressors is advisable. Please ask your dealer.

Maintenance and care

All *lisa* devices are maintenance free. To clean, remove from their mains sockets and wipe over with a moistened cloth. Never use alcohol, thinner or other organic materials.

lisa devices should not be exposed to excessive direct sunlight, heat, thick dust, humidity or mechanical shock.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The *lisa doorbell transmitter direct* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product

is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical data




Power supply:	230 V, 50 Hz
Power consumption:	approx. 1 W
Input voltage (door I and door II):	3–15 V AC/DC
Height:	121 mm
Width:	65 mm
Depth:	35 mm
Weight:	200 g

This product complies with European Union norms.

You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

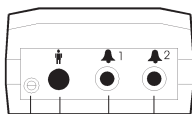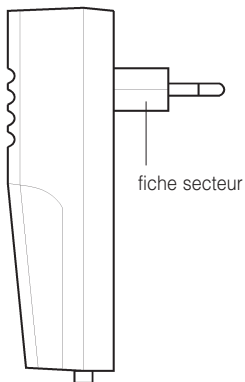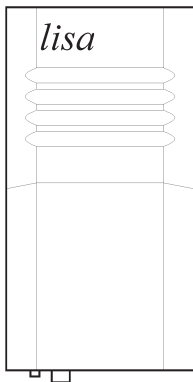Technical specifications subject to change without notice

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Humantechnik GHL AG