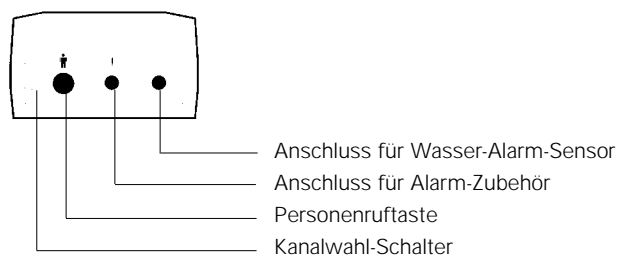
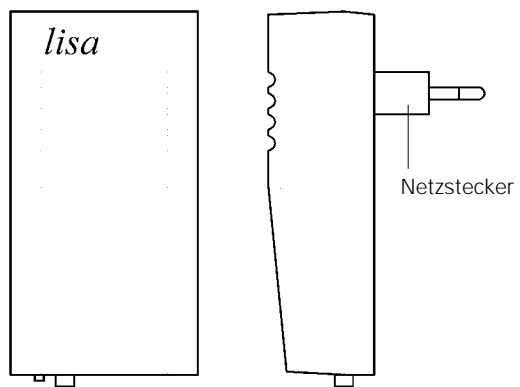
lisa

D	Bedienungsanleitung Alarmsender	Seite 2
GB	Operating Instructions Alarm transmitter	Page 8
F	Mode d'emploi Emetteur d'alarme	Page 14
NL	Gebruiksaanwijzing Alarmzender	Pagina 20
I	Istruzioni per l'uso Trasmettitore di allarmi	Pagina 26
E	Instrucciones de servicio Emisor de alarma	Página 32

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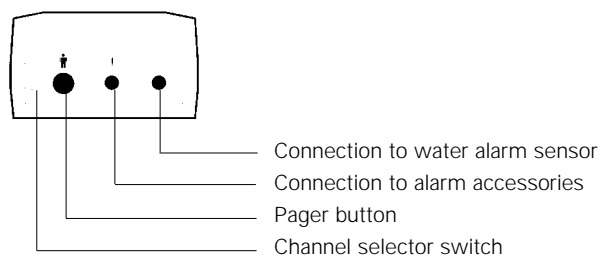
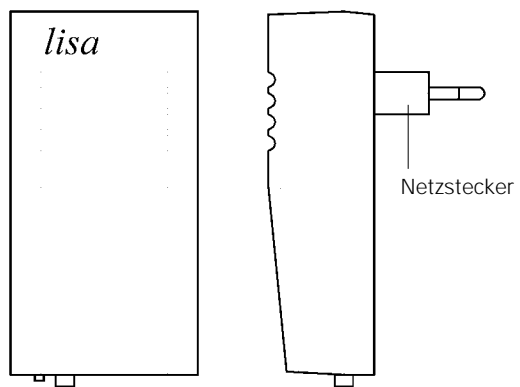
Congratulations on purchasing the *Alarm Transmitter*. We hope it will give you many years of reassuring operation. Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

***lisa* system introduction**

An alerting system requires at least one transmitter and a receiver. The transmitter captures sound signals and transforms them into digital signals that are passed to a receiver by the existing mains electricity supply. The receiver then transforms these digital signals into light signals.

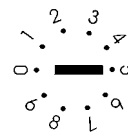
Channel selection

A digital transmission system ensures the correct recognition of each of the digital signals from each transmitter. The system has 10 different selectable channels, 0-9. The factory setting for all *lisa* devices is channel 0. Only where more than one installation is required in the same building will it be necessary to select a different channel for each system.

To select a channel, turn the rotary switch on the underside of the transmitter to the requi-

red channel number using a small screwdriver.

Example:
channel
setting
no. 0

Personal Call button

Pressing this button causes a certain flash sequence on each receiver. It is possible to call people in other rooms this way. This button also allows the *lisa* system to be tested without any connecting cables.

Duration of flashing signals

The factory setting for the duration of the flash is 28 seconds for all *lisa* receivers. However, you can reduce this to 8 seconds if you wish.

To change the flash duration for a particular transmitter, remove the transmitter from the mains socket and, while pressing in the Personal Call button, plug the transmitter back into the socket. To switch back to 28 seconds remove the transmitter from the socket and plug back in again this time without pressing the Personal Call button.

Alarm transmitter operation

Plug the transmitter into a convenient mains socket and plug the respective alarm accessories into the socket on the underside of the transmitter.

Note:

A separate socket must be used for the water alarm sensor. This socket is identified with a double wave symbol.

Available accessories (check availability with your dealer):**Alarm mat (A-2921-0)**

Plug the alarm mat into the socket of the transmitter marked by a triangle framing an exclamation mark. Place the alarm mat under the doormat.

**Alarm call button (A-2935-0)**

Plug the alarm call button into the socket of the transmitter marked by a triangle framing an exclamation mark. The alarm signal is triggered by pressing the button (e.g. from the patient's bed)

Water alarm sensor

(A-2924-0)

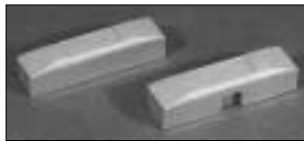
Plug the water alarm sensor into the socket of the transmit-


ter marked with a double wave. The water alarm sensor is suitable for nearly all fluids. Stick the sensor with the supplied double-sided adhesive tape to the wall so that the contacts touch the floor. The signal is triggered as soon as both contacts are wetted by the fluid.

Mount-on contact (A-2922-0)

For connection to doors and windows. The mount-on contact consists of a permanent magnet and a reed contact. Plug the alarm call button into the socket of the transmitter marked by a triangle framing an

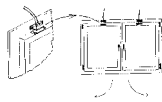exclamation mark. Contact is open when magnet is applied. An alarm is signalled if the magnet is separated from the contact.

Smoke detector (A-2926-0)

Attach the smoke detector with fastening screws to the ceiling of the room to be monitored and connect it with the signal cable to the alarm transmitter. Use the socket marked by a triangle framing an exclamation mark. The smoke detector is triggered by smoke from smouldering fires and signals an alarm.

Important note

All *lisa* devices connected to the mains supply have a low power consumption which may lead to the device feeling warm. This is normal and does not interfere with the normal function of the devices.

These signalling systems have a high operational safety. Should any faults occur in spite of proper installation and use, please contact your dealer.

High and low voltage spikes (e.g. during a thunderstorm) can occur on the mains supply. The *lisa* devices have an integrated electronic controller which may stop working after these conditions. If this happens simply unplug each of the devices and plug them in again. As a precaution remove all devices from their sockets during a thunderstorm.

The use of halogen lamps or other equipment (e.g. Microwave Ovens) may, in rare cases, interfere with the *lisa* signals. In these cases the use of mains filters or suppressors is advisable. Please ask your dealer.

Maintenance and care

All *lisa* devices are maintenance free. To clean, remove from their mains sockets and wipe over with a moistened cloth. Never use alcohol, thinner or other organic materials.

lisa devices should not be exposed to excessive direct sunlight, heat, thick dust, humidity or mechanical shock.

Warranty

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people who are not authorised to do so (destruction of the seal on the unit).

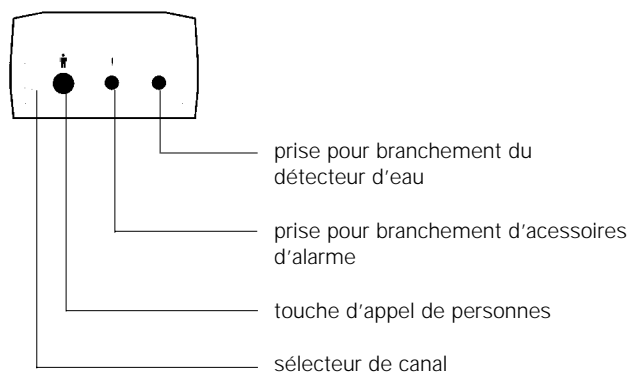
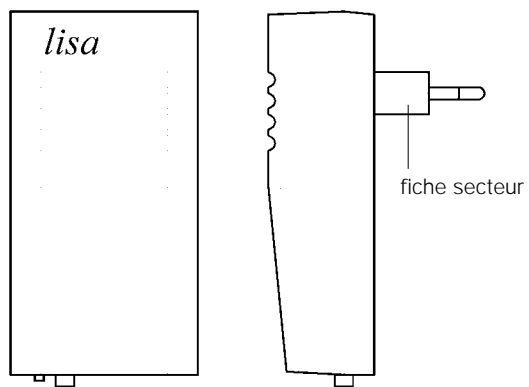
Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer.

Technical data

Power supply: 230 V, 50 Hz
Power consumption: approx. 1 W

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Humantechnik GHL AG