HUMANTECHNIK

lisa

D

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Seite 2

combi II Sender

GB

Operating Instructions

Page 8

combi II transmitter

F

Mode d'emploi

Page 12

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NL

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Pagina 18

combi II Zender

I

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Pagina 24

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
E

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
Página 30

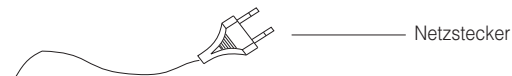
combi II Emisor

Netzstecker

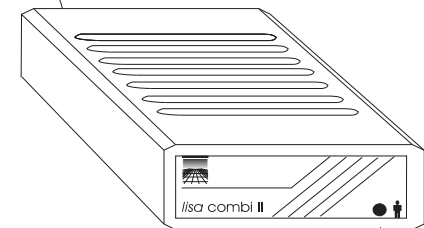Personenruf-
taste

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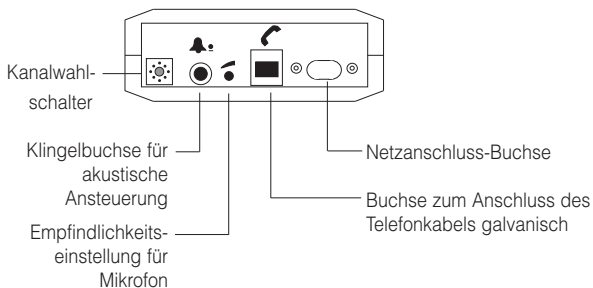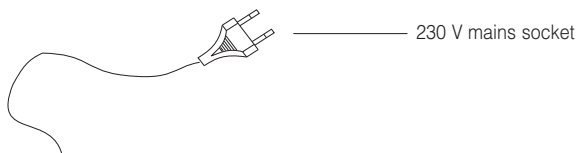
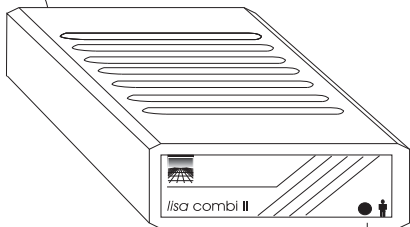
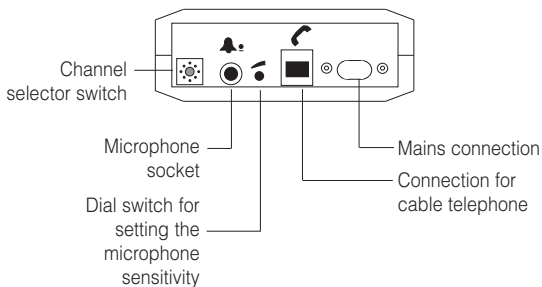
Channel selector switch

Microphone socket

Dial switch for setting the microphone sensitivity

Mains connection

Connection for cable telephone

Microphone cable order number:

A-2950-0 (length 5 m)

A-2951-0 (length 10 m)

Plug to connection to *combi II*

Microphone to stick over the acoustic output of the doorbell.

Congratulations on purchasing the *lisa combi II*. We hope it will give you many years of reassuring operation. Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

Standard components

Please check if all following components are included:

- *combi II*
- Operating instructions
- Warranty card

lisa system introduction

An alerting system requires at least one transmitter and a receiver. The transmitter captures sound signals and transforms them into digital signals that are passed to a receiver by the existing mains electricity supply. The receiver then transforms these digital signals into light signals.

Channel selection

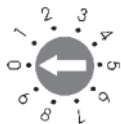
A digital transmission system ensures the correct recognition of each of the digital signals from each transmitter. The system has 10 different selectable channels, 0-9.

The factory setting for all *lisa* devices is channel 0. Only where more than one installation is required in the same building will it be necessary to select a different channel for each system.

To select a channel, turn the rotary switch on the underside of the transmitter to the required channel number using a small screwdriver.

Example:

Set channel number 0

Personal Call button

Pressing this button causes a certain flash sequence on each receiver. It is possible to call people in other rooms this way. This button also allows the *lisa* system to be tested without any connecting cables.

Duration of flashing signals

The factory setting for the duration of the flash is 28 seconds for all *lisa* receivers. However, you can reduce this to 8 seconds if you wish. To change the flash duration for a particular transmitter, remove the transmitter from the mains socket and, while pressing in the personal call button, plug the transmitter back into the socket. To switch back to 28 seconds remove the transmitter from the socket and plug back in again this time without pressing the personal call button.

Doorbell connection

Plug the transmitter into a convenient mains socket. Plug the doorbell microphone lead into the respective

socket at the back of the transmitter. Lay the cable to the door chime and attach the microphone to the door chime by peeling off the paper circle to reveal the sticky ring beneath. For optimal operation of the transmitter the sensitivity may be adjusted so that only the doorbell activates the transmitter and not background noise.

Setting the sensitivity

1. Set the sensitivity controller to its lowest level (turn to the anticlockwise to stop).
2. Activate the doorbell and gradually increase the sensitivity of the transmitter (by turning the dial clockwise) until the transmitter reacts to the doorbell.
3. If your intercom system operates with different signal tones, make sure that both signal tones are reliably detected.

Telephone connection

Plug the transmitter into a convenient mains socket. Plug the end of the cable with the transparent clip-in connector (6-pin plug) into the socket at the back of the transmitter. Plug the other end into the telephone socket. If the required telephone socket is in use plug a 3-way telephone adaptor into the socket and plug your telephone into one of the sockets on the adaptor.

Warning

Only use the supplied telephone cable otherwise correct operation can not be guaranteed. The *lisa combi II* is only designed for connection to analogue systems. If a connection is required to a digital system (e.g. ISDN or part of a PBX exchange system) please use a ISDN telephone transmitter, a telephone transmitter acoustic, the Universal-*lisa* or the *lisa combi III*.

Important note

All *lisa* devices connected to the mains supply have a low power consumption which may lead to the device feeling warm. This is normal and does not interfere with the normal function of the devices.

High and low voltage spikes (e.g. during a thunderstorm) can occur on the mains supply. The *lisa* devices have an integrated electronic controller which may stop working after these conditions. If this happens simply unplug each of the devices and plug them in again.

As a precaution remove all devices from their sockets during a thunderstorm.

The use of halogen lamps or other equipment (e.g. Microwave ovens) may, in rare cases, interfere with the *lisa* signals. In these cases the use of mains filters or suppressors is advisable. Please ask your dealer.

Maintenance and care

All *lisa* devices are maintenance free. To clean, remove from their mains sockets and wipe over with a moistened cloth. Never use alcohol, thinner or other organic materials. *lisa* devices should not be exposed to excessive direct sunlight, heat, thick dust, humidity or mechanical shock.

Note: This product is not protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The *combi II* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer. The warranty covers the repair of the product and returning it to you free of charge during the warranty period. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt. **Always specify the product number in any event.**



Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical data


| | |
|--------------------|--------------|
| Power supply: | 230 V, 50 Hz |
| Power consumption: | approx. 1 W |
| Height: | 41 mm |
| Width: | 117 mm |
| Depth: | 133 mm |
| Weight: | 300 g |

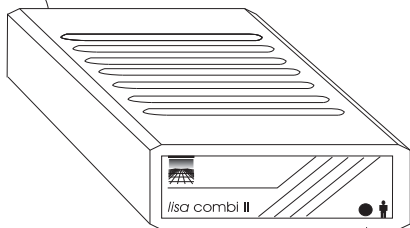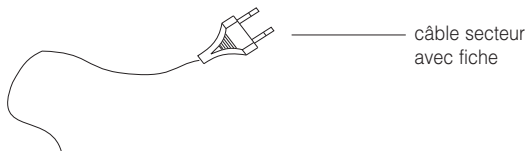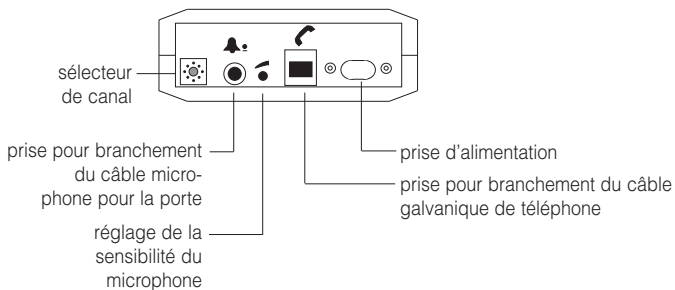This product complies with European Union norms. You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

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Humantechnik GHL AG