*lisa*

**D**

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Seite 2

Blitzlampe

**GB**

**Operating Instructions**

Page 8

Flash lamp

**F**

**Mode d'emploi**

Page 14

Lampe-flash

**NL**

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Pagina 20

Flitslamp

**I**

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Pagina 26


Lampada flash

**E**

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Página 32

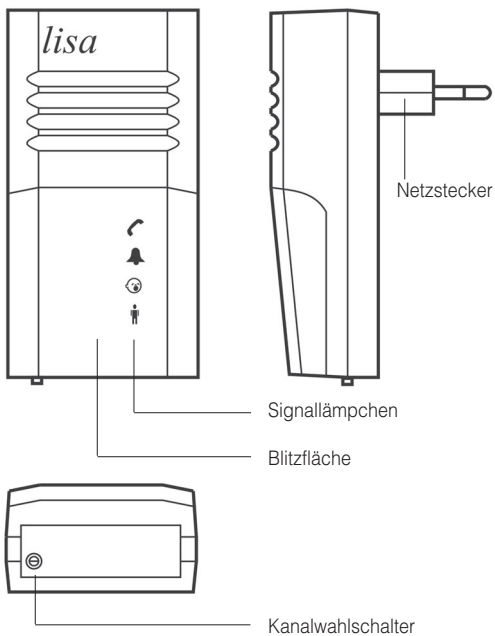Lámpara de destellos

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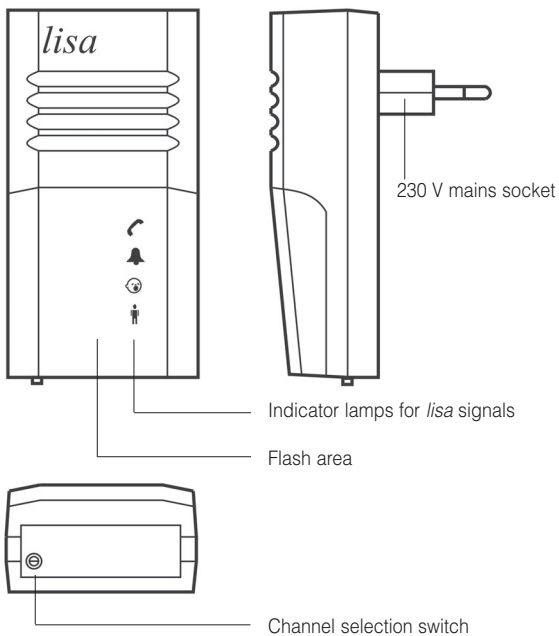
Congratulations on purchasing the *lisa flash lamp*. We hope it will provide you with many years of service. You have chosen a modern and reliable system.

Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

### Standard components

Please check if all following components are included:

- *lisa flash lamp*
- Operating instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

### Functional principle

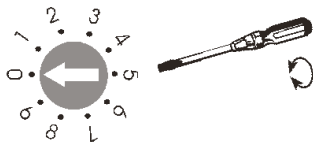
An alerting system requires at least one transmitter and a receiver. The transmitter captures signals and transforms them into digital signals that are passed to a receiver by the existing mains electricity supply. The receiver then transforms these digital signals into light signals.

### Channel selection

A digital transmission system ensures the correct recognition of each of the digital signals from each transmitter. The system has 10 different selectable channels, 0-9. The factory setting for all *lisa* devices is channel 0. Only where more than one installation is required in the same building, it will be necessary to select a different channel for each system.

To select a channel, turn the rotary switch on the underside of the transmitter to the required channel number using a small screwdriver.

All transmitters and receivers must be set to the same channel.

**Example:** Set channel number 0

## Transmission signal detection

The *lisa flash lamp* converts the various transmission signals into different sequences of flashing and lamp signals.

**Telephone:** Telephone LED lights

flashing sequence 

B	P14	B	P30	W
---	-----	---	-----	---

**Doorbell I:** Doorbell LED lights

flashing sequence 

B	P14	B	P14	B	P14	B	P40	W
---	-----	---	-----	---	-----	---	-----	---

**Doorbell II:** Doorbell LED flashes

flashing sequence 

B	P20	B	P20	B	P20	B	P40	W
---	-----	---	-----	---	-----	---	-----	---

**Babymonitor:** Babymonitor LED lights

flashing sequence 

B	P35	B	P40	W
---	-----	---	-----	---

**Person call:** Person call LED lights

flashing sequence 

B	P12	B	P12	W
---	-----	---	-----	---

**Alarm:** All LEDs flash

flashing sequence 

B	P20	B	P20	B	P14	B	P14	B	P14	W
---	-----	---	-----	---	-----	---	-----	---	-----	---

**Water alarm:** Water alarm LED lights

flashing sequence 

B	P12	B	P20	B	P12	B	P12	B	P12	W
---	-----	---	-----	---	-----	---	-----	---	-----	---

**Fire alarm:** Fire alarm LED lights

flashing sequence 

B	P12	B	P20	B	P12	B	P20	W
---	-----	---	-----	---	-----	---	-----	---

**P12** Pause 1,2 seconds

**P14** Pause 1,4 seconds

**P20** Pause 2,0 seconds

**B** flash

**P30** Pause 3,0 seconds

**P35** Pause 3,5 seconds

**P40** Pause 4,0 seconds

**W** repeat

The indicator LEDs remain on for 40 seconds.

### **Important note**

All *lisa* devices connected to the mains supply have a low power consumption which may lead to the device feeling warm. This is normal and does not interfere with the normal function of the devices.

These signalling systems have a high operational safety. Should any faults occur in spite of proper installation and use, please contact your dealer. High and low voltage spikes (e.g. during a thunderstorm) can occur on the mains supply. The *lisa* devices have an integrated electronic controller which may stop working after these conditions. If this happens simply unplug each of the devices and plug them in again.

**As a precaution remove all devices from their sockets during a thunderstorm.**

The use of halogen lamps or other equipment (e.g. Microwave Ovens) may, in rare cases, interfere with the *lisa* signals. In these cases the use of mains filters or suppressors is advisable. Please ask your dealer.

### **Maintenance and care**

All *lisa* devices are maintenance free.

**Note:** To clean, remove from their mains sockets and wipe over with a moistened cloth. Never use alcohol, thinner or other organic materials.

Do not set up the *lisa* devices where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

## Warranty

The *lisa flashing lamp* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

**Disposal** of used electrical and electronic equipment (applicable in the countries of the European Union and in other European countries which operate a separate collection system for these devices). The symbol on the product or its packaging indicates that this product should not be treated as normal household waste. It should be handed in to a collection center for recycling electrical and electronic equipment. By playing your part in ensuring the correct disposal of this product, you will be safeguarding the environment and the health of other people. The environment and health are jeopardized by incorrect disposal. Material recycling helps to reduce the consumption of raw materials. You can obtain further information about recycling this product from your local council, from local waste disposal operators and from the shop where you bought the product.

**Technical data**

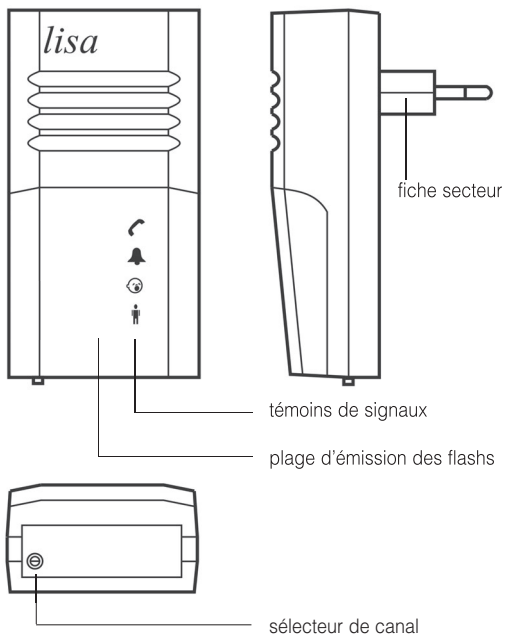
Power supply:	230 V, 50 Hz
Power consumption:	approx. 1 W
Height:	121 mm
Width:	65 mm
Depth:	35 mm
Weight:	215 g

This product complies with European Union norms.

You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

Technical specifications subject to change without notice.

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**Humantechnik GHL AG**