HUMANTECHNIK

lisa

D

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Seite 2

Tischblitzlampe

GB

Operating Instructions

Page 8

Table flash lamp

F

Mode d'emploi

Page 14

Lampe-flash de table

NL

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Pagina 20

Tafelflitslamp

I

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Pagina 26

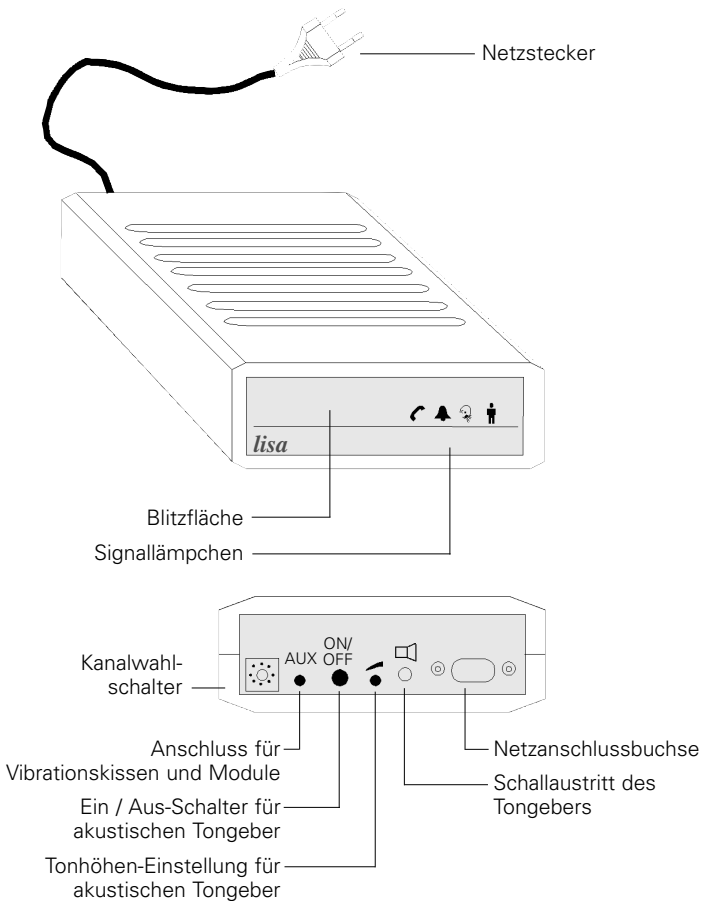
Lampada flash da tavolo

E

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Página 32

Lámpara de destellos de mesa

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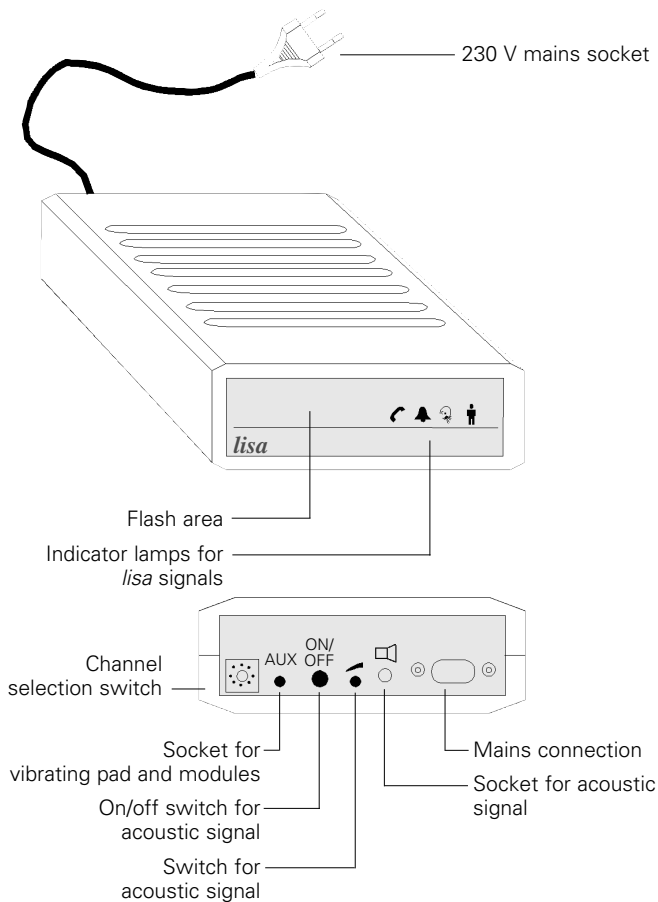
Congratulations on purchasing the *lisa Table Flash Lamp*. We hope it will provide you with many years of service. You have chosen a modern and reliable system.

Please read the operating instructions through carefully to be able to start the unit correctly and to familiarise yourself with all of the systems features.

Standard components

Please check if all following components are included:

- *lisa table flash lamp*
- 230 V power supply unit
- Operating instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

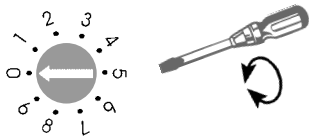
Functional principle

An alerting system requires at least one transmitter and a receiver. The transmitter captures sound signals and transforms them into digital signals that are passed to a receiver by the existing mains electricity supply. The receiver then transforms these digital signals into light signals.

Channel selection

A digital transmission system ensures the correct recognition of each of the digital signals from each transmitter. The system has 10 different selectable channels, 0-9. The factory setting for all *lisa* devices is channel 0. Only where more than one installation is required in the same building will it be necessary to select a different channel for each system.

To select a channel, turn the rotary switch on the backside of the transmitter to the required channel number using a small screwdriver. All transmitters and receivers must be set to the same channel.

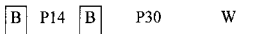
Example:

Set channel number 0

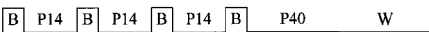
Transmission signal detection

The *table flash lamp* converts the various transmission signals into different sequences of flashing and lamp signals.

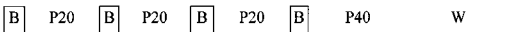
Telephone: Telephone LED lights

flashing sequence 

Doorbell I: Doorbell LED lights

Blitzrhythmus 

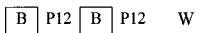
Doorbell II: Doorbell LED lights

flashing sequence 

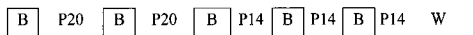
Babymonitor: Babymonitor LED lights

flashing sequence 

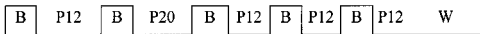
Pager: Pager LED lights

flashing sequence 

Alarm: All LEDs flash

flashing sequence 

Water alarm: Water alarm LED lights

flashing sequence 

Fire alarm: Fire alarm LED lights

flashing sequence 

P12 Pause 1,2 seconds

P30 Pause 3,0 seconds

P14 Pause 1,4 seconds

P35 Pause 3,5 seconds

P20 Pause 2,0 seconds

P40 Pause 4,0 seconds

B flash

W repeat

The indicator LEDs remain on for 40 seconds.

Important note

All *lisa* devices connected to the mains supply have a low power consumption which may lead to the device feeling warm. This is normal and does not interfere with the normal function of the devices.

These signalling systems have a high operational safety. Should any faults occur in spite of proper installation and use, please contact your dealer.

High and low voltage spikes (e.g. during a thunderstorm) can occur on the mains supply. The *lisa* devices have an integrated electronic controller which may stop working after these conditions. If this happens simply unplug each of the devices and plug them in again.

As a precaution remove all devices from their sockets during a thunderstorm.

The use of halogen lamps or other equipment (e.g. Microwave Ovens) may, in rare cases, interfere with the *lisa* signals. In these cases the use of mains filters or suppressors is advisable. Please ask your dealer.

Maintenance and care

All *lisa* devices are maintenance free. **Note:** To clean, remove from their mains sockets and wipe over with a moistened cloth. Never use alcohol, thinner or other organic materials.

lisa devices should not be exposed to excessive direct sunlight, heat, thick dust, humidity or mechanical shock.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The *lisa table flash lamp* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt. **Always specify the product number in any event.**

Environmental note/ Waste disposal

Disposal of used electrical and electronic equipment (applicable in the countries of the European Union and in other European countries which operate a separate collection system for these devices).

The symbol on the product or its packaging indicates that this product should not be treated as normal household waste. It should be handed in to a collection center for recycling electrical and electronic equipment.

By playing your part in ensuring the correct disposal of this product, you will be safeguarding the environment and the health of other people. The environment and health are jeopardized by incorrect disposal. Material recycling helps to reduce the consumption of raw materials. You can obtain further information about recycling this product from your local council, from local waste disposal operators and from the shop where you bought the product.

Technical data

Power supply: 230 V, 50 Hz

Power consumption: approx. 1 W

Height: 42 mm

Width: 117 mm

Depth: 133 mm

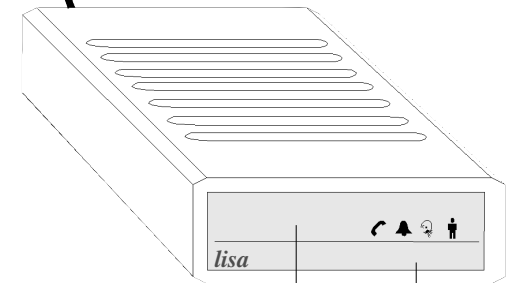
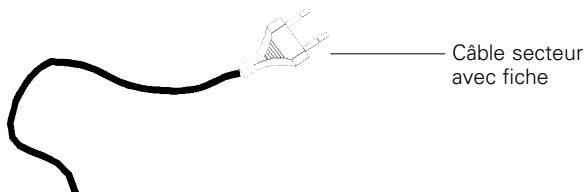
Weight: 325 g

This product complies with European Union norms.

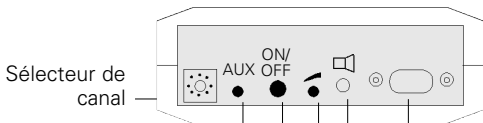
You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

Technical specifications subject to change without notice.

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Humantechnik GHL AG