HUMANTECHNIK

*lisa*

D

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Seite 2

Universal-*lisa*

GB

**Operating Instructions**

Page 10

Universal-*lisa*

F

**Mode d'emploi**

Page 18

Universal-*lisa*

NL

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Pagina 26

Universal-*lisa*

I

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Pagina 34

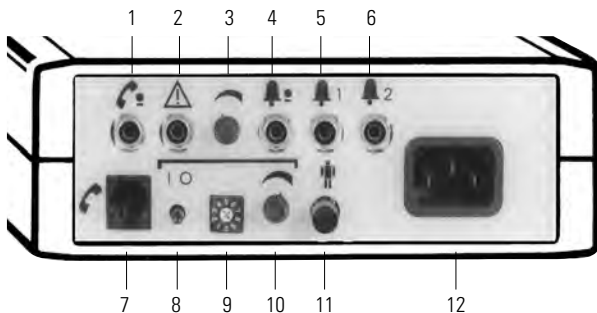
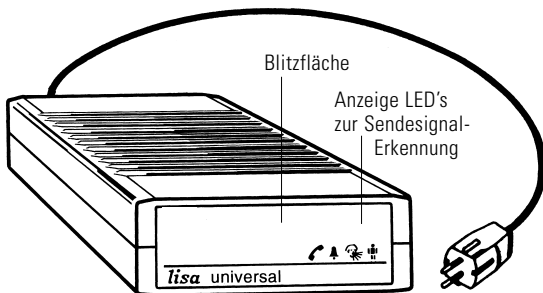
Universal-*lisa*

E

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Página 42

Universal-*lisa*

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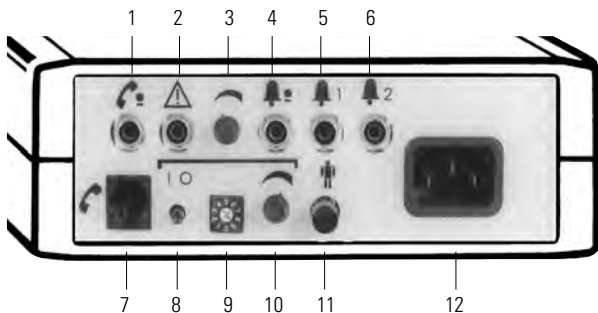
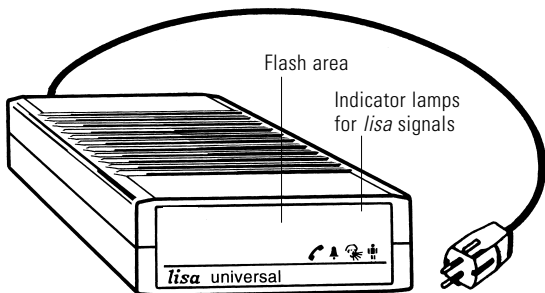
- 1 Socket for telephone (acoustic)
- 2 Socket for alarm accessories
- 3 Switch for acoustic signal (doorbell microphone)
- 4 Socket for doorbell (acoustic)
- 5 Socket for doorbell I (galvanic)
- 6 Socket for doorbell II (galvanic)
- 7 Socket for telephone (galvanic)
- 8 on/off switch for acoustic signal
- 9 Channel selection switch
- 10 Switch for acoustic signal (tone control treble)
- 11 Pager button
- 12 Mains connection

Congratulations on purchasing the *universal-lisa*. We hope it will give you many years of reassuring operation. You have chosen a modern and reliable system. Please read the operating instruction through carefully to be able to start the unit correctly and to familiarize yourself with all of the system features.

### Standard components

Please check if all following components are included:

- *Universal-lisa*
- Connecting cable
- Operating instruction
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

The ***universal-lisa*** combines **transmitter** (door bell, telephone, alarm, person call) **and receiver** (flash lamp) in one housing.

### System introduction

An alerting system requires at least one transmitter and a receiver. The transmitter captures sound signals and transforms them into digital signals that are passed to the build-in receiver. The receiver then transforms these digital signals into light signals.

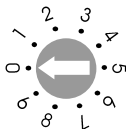
### Channel selection

A digital transmission system ensu-

res the correct recognition of each of the digital signals from each transmitter. The system has 10 different selectable channels, 0-9. The factory setting for all *lisa* devices is channel 0. Only where more than one installation is required in the same building will it be necessary to select a different channel for each system.

To select a channel, turn the rotary switch on the underside of the transmitter to the required channel number using a small screwdriver. All transmitters and receivers must be set to the same channel.

**Example:**  
channel  
setting 0

### Personal Call button

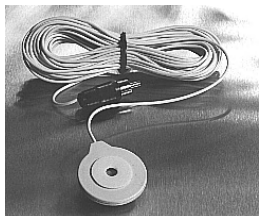
Pressing this button causes a certain flash sequence. It is possible to call people in other rooms this way. This button also allows the *lisa* system to be tested without any connecting cables.

### Duration of flashing signals

The factory setting for the duration of the flash is 28 seconds for all *lisa* receivers. However, you can reduce this to 8 seconds if you wish.

### Function of the transmitter

Plug the transmitter into a convenient mains socket and connect the cables according to the drawing.

Picture:  
microphone cable 5 m or 10 m for  
the acoustic telephone and door bell  
connection.

A-2955-0 (length 5m)

A-2956-0 (length 10m)

### Telephone connection - acoustic

Plug the microphone cable into the respective socket at the *universal-lisa*. Lay the cable to the telephone. Peel off the paper circle on the microphone to reveal the sticky ring beneath. Attach the microphone to the area of the telephone where the ringer is situated. Adjust the volume of the ringer so that the transmitter reacts to the ringing.

**Note:** If the telephone handset is replaced too harshly it may activate the telephone transmitter.

### Telephone connection - galvanic

Plug the end of the cable with the transparent clip-in connector (6-pin plug) into the respective socket at the *universal-lisa*. Plug the other end into the telephone socket.

**Note:** Only use the supplied telephone cable otherwise correct operation can not be guaranteed.

### Connection of alarm contacts

Plug the chosen alarm accessory into the respective socket at the *universal-lisa*.

**Note** The connection socket for alarm contacts is designed for normally open contacts (contact closed when alarm).

### Available accessories (check availability with your dealer):

Alarm mat (A-2921-0)

Alarm call button (A-2935-0)

Water alarm sensor (A-2924-0)

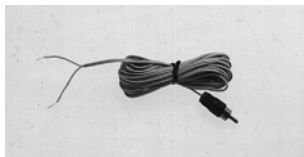
Mount-on contact (A-2922-0)

Smoke detector (A-2926-0)

### Doorbell connection - galvanic

Plug the 2.5 mm jack plug of the doorbell cable into the respective socket at the *universal-lisa*. Lay the cable to the bell push or door chime. Connect both wires to the screw contacts of the bell push or door chime.

**Warning: The Transmitter is only designed to work with low voltage doorbells. Do not connect the thin doorbell cable to the mains supply. This could be extremely dangerous for the user and may destroy the transmitter.**

Cable for doorbell connection  
A-2950-0 ( 5 m)  
A-2951-0 (10 m)

The doorbell voltage must be at least 3 Volts and must not exceed 15 Volts. Both AC and DC can be used.

Pressing your respective doorbell will now cause each receiver to flash with the corresponding number of flashes. The receiver will flash differently for doorbell 1 and doorbell 2.

**Note:** For the connection of 2 doorbells, two cables are necessary.

### **Failure causes**

- If your doorbell system uses direct current voltage, you have not connected the wires with the correct polarity. Swap over the wires.

- The doorbell voltage of your system is too low. Either contact an electrician to have the doorbell system converted or use the doorbell transmitter - acoustic.

### **Doorbell connection - acoustic**

Plug the doorbell microphone lead into the respective socket at the *universal-lisa*. Lay the cable to the door chime and attach the microphone to the door chime by peeling off the paper circle to reveal the sticky ring beneath. For optimal operation of the transmitter the sensitivity may be adjusted so that only the doorbell activates the transmitter and not background noise.

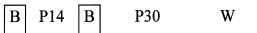
### **Setting the sensitivity**

1. Set the sensitivity controller to its lowest level (turn to the anti-clockwise to stop).
2. Activate the doorbell and gradually increase the sensitivity of the transmitter (by turning the dial clockwise) until the transmitter reacts to the doorbell.
3. If your intercom system operates with different signal tones, make sure that both signal tones are reliably detected.

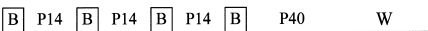
## Transmission signal detection

The *universal-lisa* converts the various transmission signals into different sequences of vibration and lamp signals.

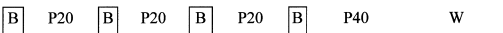
**Telephone:** Telephone LED lights

Flashing rhythm  A horizontal timeline showing a sequence of events: a box labeled 'B', followed by a box labeled 'P14', then another box labeled 'B', followed by a box labeled 'P30', and finally a box labeled 'W'.

**Doorbell I:** Doorbell LED lights

Flashing rhythm  A horizontal timeline showing a sequence of events: a box labeled 'B', followed by a box labeled 'P14', then another box labeled 'B', followed by a box labeled 'P14', then another box labeled 'B', followed by a box labeled 'P14', then another box labeled 'B', followed by a box labeled 'P40', and finally a box labeled 'W'.

**Doorbell II:** Doorbell LED lights

Flashing rhythm  A horizontal timeline showing a sequence of events: a box labeled 'B', followed by a box labeled 'P20', then another box labeled 'B', followed by a box labeled 'P20', then another box labeled 'B', followed by a box labeled 'P20', then another box labeled 'B', followed by a box labeled 'P40', and finally a box labeled 'W'.

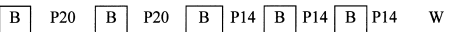
**Babymonitor:** Babymonitor LED lights

Flashing rhythm  A horizontal timeline showing a sequence of events: a box labeled 'B', followed by a box labeled 'P35', then another box labeled 'B', followed by a box labeled 'P40', and finally a box labeled 'W'.

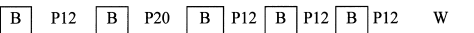
**Pager:** Pager LED lights

Flashing rhythm  A horizontal timeline showing a sequence of events: a box labeled 'B', followed by a box labeled 'P12', then another box labeled 'B', followed by a box labeled 'P12', and finally a box labeled 'W'.

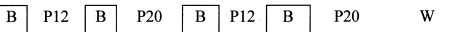
**Alarm:** All LEDs flash

Flashing rhythm  A horizontal timeline showing a sequence of events: a box labeled 'B', followed by a box labeled 'P20', then another box labeled 'B', followed by a box labeled 'P20', then another box labeled 'B', followed by a box labeled 'P14', then another box labeled 'B', followed by a box labeled 'P14', then another box labeled 'B', followed by a box labeled 'P14', and finally a box labeled 'W'.

**Water alarm:** Water alarm LED lights

Flashing rhythm  A horizontal timeline showing a sequence of events: a box labeled 'B', followed by a box labeled 'P12', then another box labeled 'B', followed by a box labeled 'P20', then another box labeled 'B', followed by a box labeled 'P12', then another box labeled 'B', followed by a box labeled 'P12', then another box labeled 'B', followed by a box labeled 'P12', and finally a box labeled 'W'.

**Fire alarm:** Fire alarm LED lights

Flashing rhythm  A horizontal timeline showing a sequence of events: a box labeled 'B', followed by a box labeled 'P12', then another box labeled 'B', followed by a box labeled 'P20', then another box labeled 'B', followed by a box labeled 'P12', then another box labeled 'B', followed by a box labeled 'P20', and finally a box labeled 'W'.

**P12** pause 1,2 seconds  
**P14** pause 1,4 seconds  
**P20** pause 2,0 seconds  
**B** flash

**P30** pause 3,0 seconds  
**P35** pause 3,5 seconds  
**P40** pause 4,0 seconds  
**W** repeat

The indicator LEDs remain on for 60 seconds.

During a quick call, the individual signals on the *Universal-lisa* will flash continuously for 8 seconds.

---

### **Duration of flashing signals**

The factory setting for the duration of the flash is 28 seconds for all *lisa* receivers. However, you can reduce this to 8 seconds if you wish.

To change the flash duration for a particular transmitter, remove the transmitter from the mains socket and, while pressing in the Personal Call button, plug the transmitter back into the socket. To switch back to 28 seconds remove the transmitter from the socket and plug back in again this time without pressing the Personal Call button.

### **Important note**

All *lisa* devices connected to the mains supply have a low power consumption which may lead to the device feeling warm. This is normal and does not interfere with the normal function of the devices.

These signalling systems have a high operational safety. Should any

faults occur in spite of proper installation and use, please contact your dealer.

High and low voltage spikes (e.g. during a thunderstorm) can occur on the mains supply. The *lisa* devices have an integrated electronic controller which may stop working after these conditions. If this happens simply unplug each of the devices and plug them in again.

**As a precaution remove all devices from their sockets during a thunderstorm.**

The use of halogen lamps or other equipment (e.g. Microwave Ovens) may, in rare cases, interfere with the *lisa* signals. In these cases the use of mains filters or suppressors is advisable. Please ask your dealer.

## Maintenance and care

All *lisa* devices are maintenance free. To clean, remove from their mains sockets and wipe over with a moistened cloth. Never use alcohol, thinner or other organic materials. *lisa* devices should not be exposed to excessive direct sunlight, heat, thick dust, humidity or mechanical shock.

**Note:** This product is not protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

## Warranty

The *RF flashing lamp* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out

under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt. **Always specify the product number in any event.**

## Environmental note/ Waste disposal

Disposal of used electrical and electronic equipment (applicable in the countries of the European Union and in other European countries which operate a separate collection system for these devices).

The symbol on the product or its packaging indicates that this product should not be treated as normal household waste. It should be handed in to a collection center for recycling electrical and electronic equipment.

By playing your part in ensuring the correct disposal of this product, you will be safeguarding the environment and the health of other people. The environment and health are jeopardized by incorrect disposal.

Material recycling helps to reduce the consumption of raw materials. You can obtain further information about recycling this product from your local council, from local waste disposal operators and from the shop where you bought the product.

## Technical data

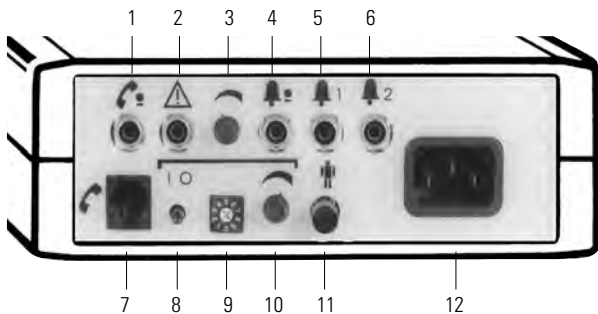
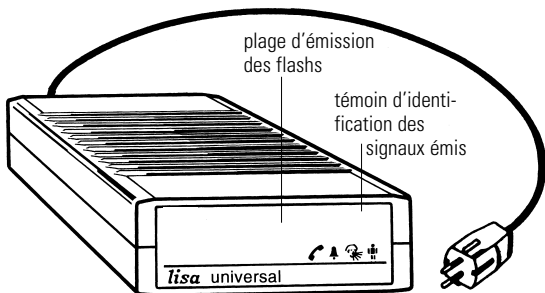
Power supply: 230 V, 50 Hz  
Power consumption: approx. 4 W

Height: 57 mm  
Width: 170 mm  
Depth: 190 mm  
Weight: 720 g

This product complies with European Union norms.  
You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

Technical specifications subject to change without notice.

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## **Humantechnik GHL AG**