lisa

D

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Seite 2

Tischblitzlampe

GB

Operating Instructions

Page 8

Table flash lamp

F

Mode d'emploi

Page 14

Lampe-flash de table

NL

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Pagina 20

Tafelflitslamp

I

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Pagina 26

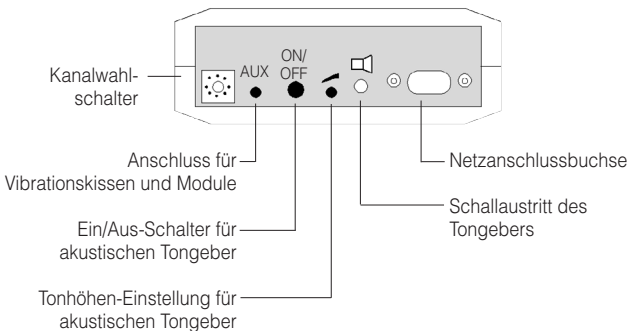
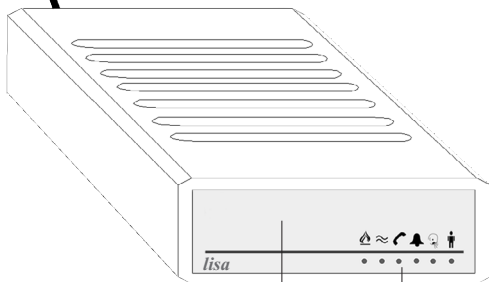
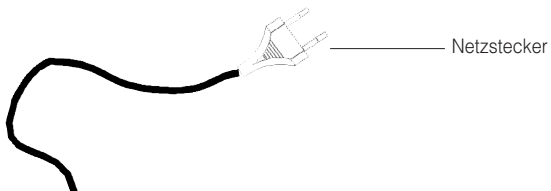
Lampada flash da tavolo

E

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Página 32

Lámpara de destellos de mesa

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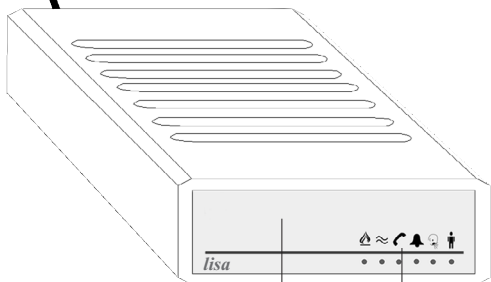
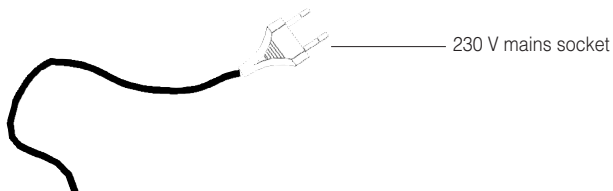
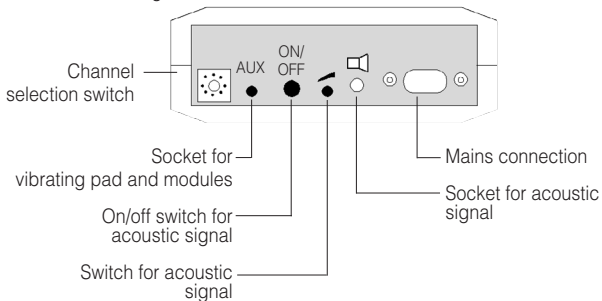
Flash area

Indicator lamps for *lisa* signals

Congratulations on purchasing your *RF table flash lamp*. We hope it will give you many years of reassuring operation. You have purchased a modern and reliable system.

Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

Standard components

Please check if all following components are included:

- *lisa RF table flash lamp*
- 230 V power supply unit
- Operating instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

Functional principle

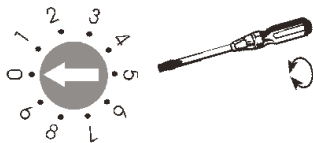
The *RF table flash lamp* receives the radio signals from various radio transmitters and converts them into light signals. It possesses a powerful flash tube which alerts you to a signal as well as LEDs which indicate the type of signals which are being received. In addition, it is possible to connect a vibrating pad and modules (accessories).

Setting up the system – Setting the correct radio channel

A special digital transmission process ensures that the radio signals are picked up without errors. This process also makes it possible to set 10 different radio channels.

When supplied from the factory, all *RF table flash lamps* are set to channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building.

Set the channel you want by adjusting the channel selector switch on the backside of the unit to the required channel number. Use a small screwdriver to adjust this switch.

All transmitters and receivers must be set to the same channel.

Startup -

Installing the RF table flash lamp

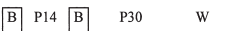
Simply connect the *lisa RF table flash lamp* to a power point. The unit is immediately ready to use.

Make sure the flashing light can illuminate the room with as few obstacles as possible.

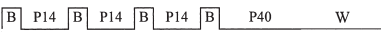
Transmission signal recognition

The *RF table flash lamp* converts the various transmission signals into varying sequences of flashes and signal displays.

Telephone: Telephone LED lights

flashing sequence 

Doorbell I: Doorbell LED lights

flashing sequence 

Doorbell II: Doorbell LED flashes

flashing sequence 

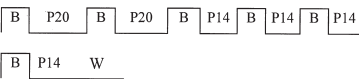
Babymonitor: Babymonitor LED lights

flashing sequence 

Person call: Person call LED lights

flashing sequence 

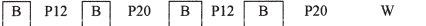
Alarm: All LEDs flash

flashing sequence 

Water alarm: The water alarm LED lights

flashing sequence 

Fire alarm: Fire alarm LED lights

flashing sequence 

P12 Pause 1,2 seconds

P30 Pause 3,0 seconds

P14 Pause 1,4 seconds

P35 Pause 3,5 seconds

P20 Pause 2,0 seconds

P40 Pause 4,0 seconds

B flash

W repeat

The indicator LEDs remain on for 40 seconds.

Power failure backup

The built-in battery can provide backup power for a mains power outage of up to 7 hours. The following functions are available during this period:

1. Acoustic signal
2. Triggering of indicator LEDs
3. Triggering of a vibration cushion or modules (if connected)

Important: The flash light is not triggered during a power outage.

Radio range

The radio signals are designed to have a range of 80 meters under optimum conditions. The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.

- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

Maintenance and care

All *lisa* devices are maintenance free.

Note: To clean, remove from their mains sockets and wipe over with a moistened cloth. Never use alcohol, thinner or other organic materials.

lisa devices should not be exposed to excessive direct sunlight, heat, thick dust, humidity or mechanical shock.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The *lisa RF table flash lamp* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electrical and electronic equipment (applicable in the countries of the European Union and in other European countries which operate a separate collection system for these devices). The symbol on the product or its packaging indicates that this product should not be treated as normal household waste. It should be handed in to a collection center for recycling electrical and electronic equipment. By playing your part in ensuring the

correct disposal of this product, you will be safeguarding the environment and the health of other people.

The environment and health are jeopardized by incorrect disposal. Material recycling helps to reduce the consumption of raw materials. You can obtain further information about recycling this product from your local council, from local waste disposal operators and from the shop where you bought the product.

Technical Data

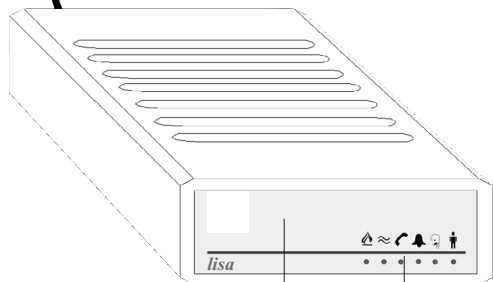
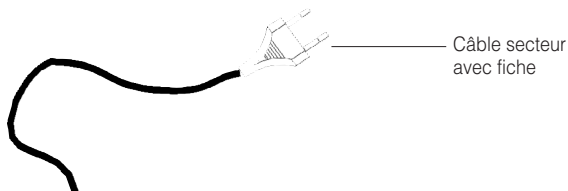
Power supply:	230 V, 50 Hz,
Power consumption:	approx. 1 W
Frequency:	868,35 MHz
Height:	42 mm
Width:	117 mm
Depth:	133 mm
Weight:	approx. 350 g

This product complies with European Union norms.

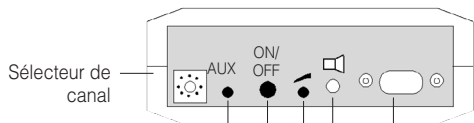
You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

Technical specifications subject to change without notice.

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Humantechnik GHL AG