HUMANTECHNIK

lisa

DE

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Seite 2

Converter lisa -> Funk

EN

Operating Instructions

Page 8

Converter lisa -> radio

FR

Mode d'emploi

Page 14

Convertisseur lisa -> radio

NL

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Pagina 20

Convertor lisa -> radio

IT

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Pagina 26

Convertitore lisa -> radio


ES

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
Página 32

Convertidor lisa -> radio

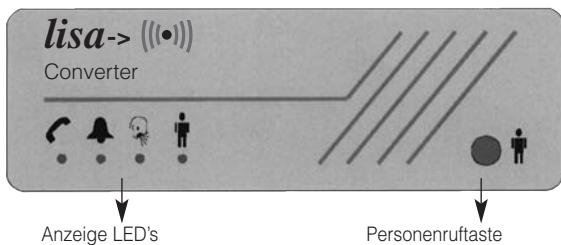Vorderseite

Rückseite

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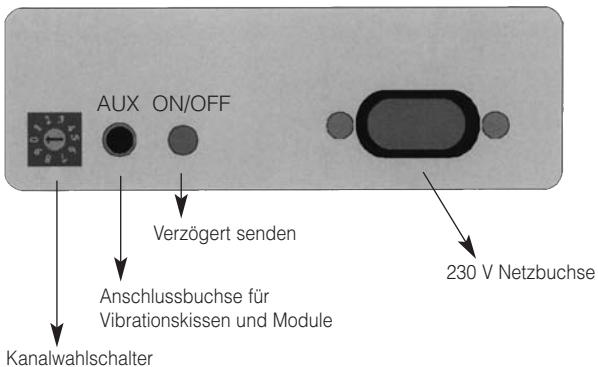
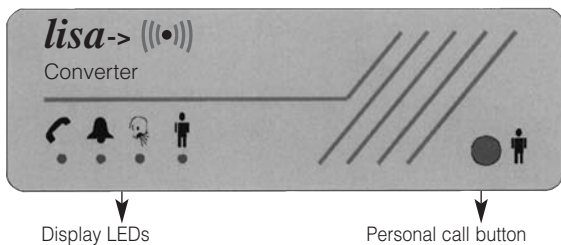
Front

Back

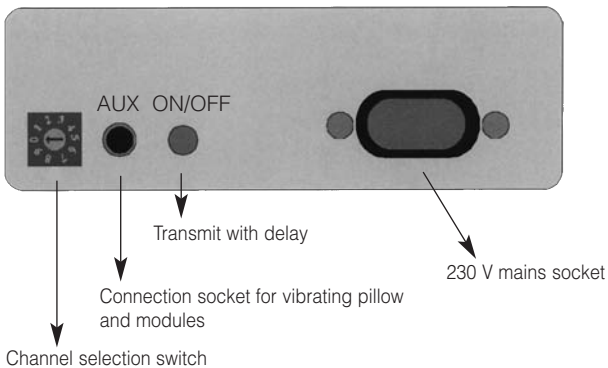Congratulations on purchasing your *converter lisa -> radio*. We hope that it will serve you well for many years to come. You have opted for a modern and reliable system.

Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

Standard components

Please check if all following components are included:

- *converter lisa -> radio*
- connecting cable
- operating Instructions
- warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

Functional principle

With the *converter lisa -> radio*, *lisa* systems operating through the 230 V power supply can be combined with the radio receivers. The *converter lisa -> radio* receives the signals from various *lisa* transmitters and sends them to the cable-free radio receivers. All *lisa* transmitters (acoustic/ galvanic telephone transmitters, acoustic/galvanic doorbell transmitters, baby alarm transmitter, alarm transmitters) can be used as transmitters.

Operating and display elements

- person call button
- display LEDs
- channel selection switch
- connection socket for vibrating pillow
- transmit with delay (this operating element is only important if two *converters lisa -> radio* are used).


Note 1

The person call button located on the converter does not have any function for 60 seconds if an alarm signal/person call signal has been sent.

Note 2

If the "Transmit with delay" switch is on position 1 (pressed in) the radio signals are sent with a delay of 1.5 seconds.

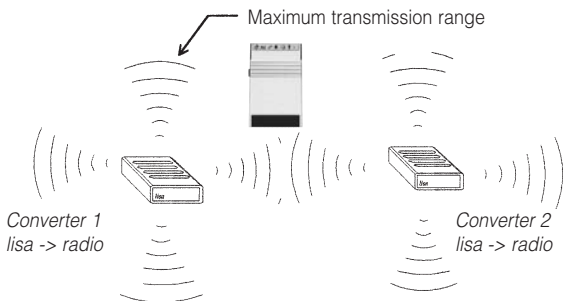Relay function – operation with two *converters lisa -> radio* Converters If the radio transmission range of a *converter lisa -> radio* is not sufficient (e.g. in large houses), you can incorporate a second *converter lisa -> radio* into the system.

Important: *Converter 1 lisa -> radio* :
"Transmit with delay" is pressed

Converter 1 lisa -> radio :
"Transmit with delay" is not pressed

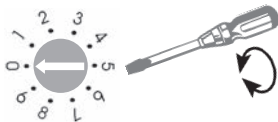
Channel selection

A special digital transmission process ensures that the radio signals are picked up without errors. This process also makes it possible to set 10 different radio channels. When supplied from the factory, all *lisa* devices are set to channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building.

Set the required channel by using a small screwdriver to turn the channel selector switch, which is located on

the backside of the *converter lisa -> radio*, to the required channel number.

All transmitters and receivers must be set to the same channel.

Example: Set channel number 0

Startup

The *converter lisa* -> *radio* is immediately ready for use after being connected to a 230 V socket.

Transmission range

The transmission range of the radio signals of the *converter lisa* -> *radio* is designed for 80 metres under optimal conditions.

The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones.
Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavorable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.

- Having units operating at similar frequencies located close to one another may also result in mutual interference.

Positioning

In order to attain as wide a range as possible, you should follow the installation information below :

- The *converter lisa* -> *radio* should not be positioned near devices which emit strong interference signals such as television sets, monitors or mobile telephones.
- The *converter lisa* -> *radio* should not be positioned on metallic objects as otherwise the transmission range is reduced by the metallic screening.
- Position the *converter lisa* -> *radio* as centrally as possible in the house and at a medium room height.

Maintenance and care

The *converter lisa -> radio* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth.

Note: remove the *converter lisa -> radio* from mains socket. Never use spirits, thinners or other organic solvents.

Do not set up the *converter lisa -> radio* where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The *converter lisa -> radio* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical data

Power supply:	230 V, 50 Hz
Frequency:	868.35 MHz
Power consumpt.	approx. 1 W
Maximum transmitting power:	<10 mW
Height:	41 mm
Width:	117 mm
Depth:	133 mm
Weight:	approx. 290 g

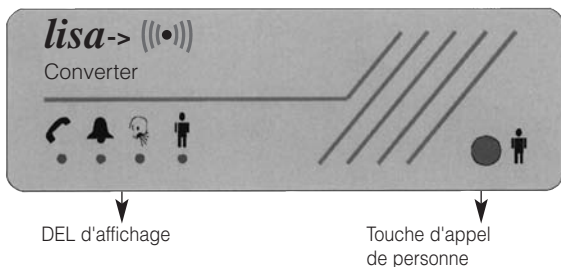
This device complies to the following EU directives:

- 2011 / 65 / EU RoHS directive
- 2012 / 19 / EU WEEE directive
- 2014 / 53 / EU RED directive


Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at www.humanttechnik.com/service.

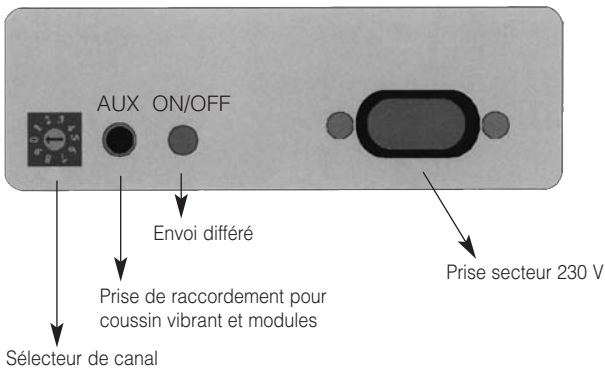Technical specifications subject to change without notice.

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