lisa

DE

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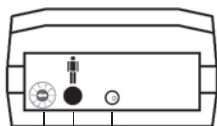
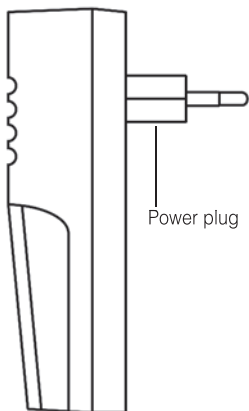
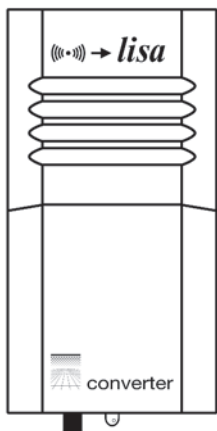
call button

channel selector

Congratulations on purchasing your *Radio-lisa converter*. We hope it will give you many years of reassuring operation. You have purchased a modern and reliable system. Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

Standard components

Please check if all following components are included:

- *Radio-lisa converter*
- operating Instructions
- warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

Functional principle

With the *Radio-lisa converter* it is possible to combine the *lisa* systems, which is working on the 230 V-powerline, with the Radio Transmitters. The *Radio-lisa converter* receives the signals from different Radio Transmitters and feed this signals into the 230 V-powerline. Following radio transmitters can be used: Radio Telephone Transmitter acoustic/ direct connection, Radio Doorbell Transmitter acoustic/ direct connection, Radio Baby Monitor, Radio Alarm Transmitter.

Controls

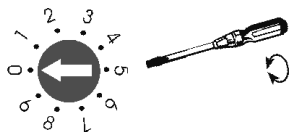
- »Transmitter« indicator
- Call button
- Channel selector

Note: The pager button on the *Radio-lisa converter* ceases to function for 40 seconds once a manual or alarm call has been detected.

Channel selection

A digital transmission system ensures the correct recognition of each of the digital signals from each transmitter. The system has 10 different selectable channels, 0-9. The factory setting for all *lisa* devices is channel 0. Only where more than one installation is required in the same building will it be necessary to select a different channel for each system.

To select a channel, turn the rotary switch on the underside of the transmitter to the required channel number using a small screwdriver.

All transmitters and receivers must be set to the same channel.

Installing the *Radio-lisa converter*

Simply connect the *Radio-lisa converter* to a power point. The unit is immediately ready to use.

Radio range

The transmitter for the *Radio-lisa converter* are designed to have a radio signal range of 80 meters under optimum conditions.

The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones.

Under certain circumstances, this may prevent the receiver from functioning at all.

- Positioning of the transmitter at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

Positioning the radio transmitter

Bear in mind the following points when positioning the radio transmitter, in order to achieve the greatest possible range from the radio transmitter:

- The radio transmitter should not be set up close to devices which emit powerful interference signals, such as TVs, computer monitors or mobile phones.
- The radio transmitter should not be placed on metallic objects, since otherwise the radio range will be restricted by metallic shielding.
- Position the radio transmitter as close to the centre of the building as possible, at a medium height in the room.

Maintenance and care

All *lisa* devices are maintenance free. To clean, **remove from their mains sockets** and wipe over with a moistened cloth. Never use alcohol, thinner or other organic materials.

lisa devices should not be exposed to excessive direct sunlight, heat, thick dust, humidity or mechanical shock.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The *Radio-lisa converter* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal

of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.




Technical data

Power supply:	230V/ 50Hz
Consumption:	approx. 2 W
Frequency:	868.35 MHz
Operating range:	0 – 40 °C
Height:	21 mm
Width:	65 mm
Depth:	35 mm
Weight:	approx. 200 g

This product complies with the guidelines of the European Union. You can obtain CE compliance declarations via www.humanteknik.com/service

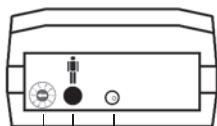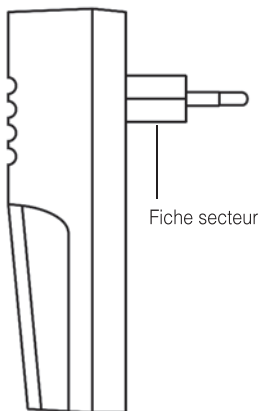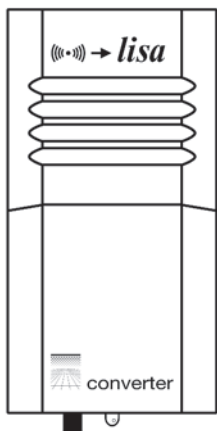Technical specifications subject to change without notice.

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Sarabec Ltd