HUMANTECHNIK

lisa

DE

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Seite 2

Funk-Alarm-Monitor »alarmo«

EN

Operation Instructions

Page 8

RF transmitter »alarmo«

FR

Mode d'emploi

Page 14

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

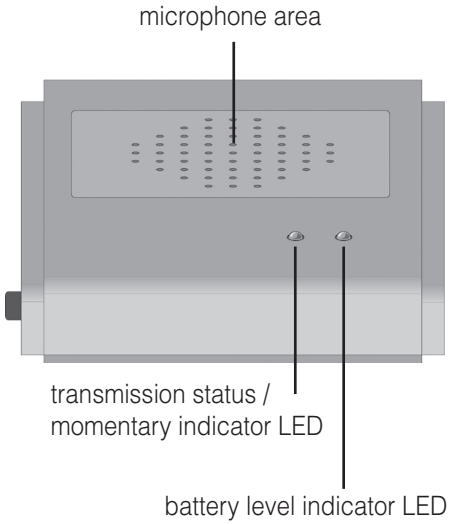
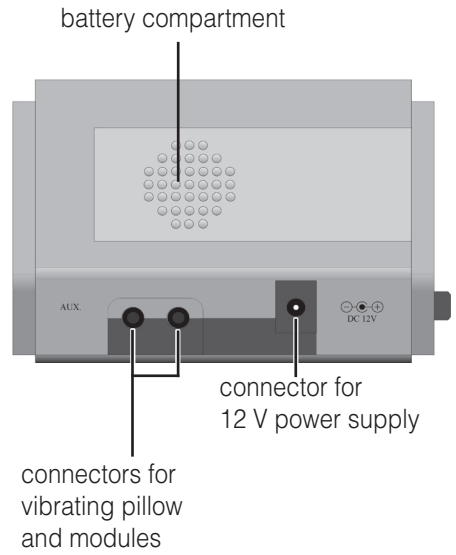
view of right side

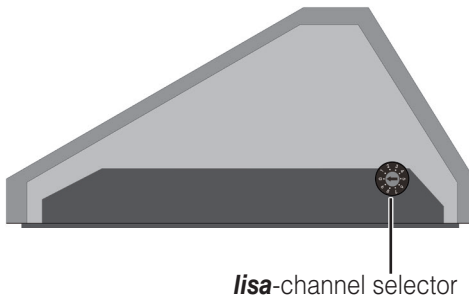view of left side

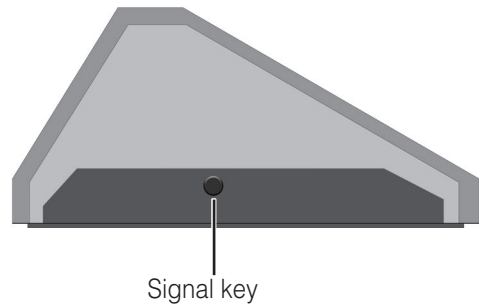Congratulations on the purchase of your **RF lisa** alarm monitor »alarmo«.

You have chosen a highly modern and reliable transmitter for the wireless **RF lisa** home signalling system.

Please read this manual thoroughly to be able to set up the unit correctly and to familiarise yourself with all of the features the system offers.

Package contents

Please check if all of the following components are included:

- »alarmo«
- rechargeable battery (in the compartment)
- 12 V power supply unit
- manual
- warranty card

Should any parts be missing, please immediately contact your dealer or the manufacturer directly.

Function

»alarmo« has been conceived to enable the easy incorporation of already existing smoke detectors into the **RF lisa**-signalling system, without need for modification or wiring.

»alarmo« scans environmental noises in proximity for sound patterns typical of smoke detectors and fire alarm systems of any type.

False alarms, possibly caused by alarm sounds not actually within in the room (e. g. passing emergency vehicles, TV and movie sound effects) are prevented by means of intelligent analysis and filtering of the registered sound.

If an alarm sound is identified as such, »alarmo« transmits a radio signal to all **RF lisa** receivers within range, which will indicate the incident with light, vibration or sound - depending on the type of receiver.

Controls

Battery status indicator

A red illuminated battery status indicator means that the inserted rechargeable battery is almost depleted. The battery needs to be recharged.

A permanent green light indicates a charged battery and an operational »alarmo«.

Transmission status / momentary indicator

The transmission status indicator lights up when a radio signal is sent to the **lisa**-receivers.

Signal key

This button is intended to check if the system is working correctly, even when no acoustic alarm is picked up. Function and signal range can be verified by pressing this button.

lisa-Channel selector

You can select the radio channel using the channel selector. Make sure all devices in your system are set to the same channel.

Note (special accessories)

»alarmo« is able to trigger various indicator modules, e. g. you can have a light flash directly released by the »alarmo«.

The following modules (not included) can be plugged into to the AUX output connectors:

- Vibrating pillow (A-3303-0)
- Flash module MF-1 (A-3320-0)
- Switch module MS-1 (A-3330-0)
- Acoustic module MA-1 (A-3340-0)

Setting up the system

1. Placement

»alarmo« scans and analyses the noises within its environment for sounds typical of smoke detectors and similar alarm systems. In order to assist this function, »alarmo« should be placed as close as possible to the sound source of the alarm, which is to be monitored (e. g. on a shelf underneath a smoke detector).

In order to gain the best possible transmission range, please note the following recommendations:

- The transmitter should not be placed close to devices emitting intense interference, like TV screens, computer monitors or mobile phones.
- The transmitter should not be placed on objects made of metal, as the transmission range is diminished by the metallic shielding effect.

Note:

The device is not able of identifying, recording or even transmitting voices and spoken conversation.

Therefore there is no possible invasion or endangerment of the user's privacy.

2. Mains operation

Insert the plug at the end of the 12 V power supply into the corresponding connector on the rear side of the »alarmo«.

Then plug the power supply unit into a mains power socket in your room.

Power failure backup

Please note that the device is designed for mains operation.

The rechargeable battery is used as backup only in the event of a power failure.

The stand-by time of the battery is approx. 2 hours.

3. Connecting the rechargeable battery (included)

Open the battery compartment by sliding the cover to the right of the device using your thumb.

To connect the rechargeable battery, insert the battery plug into the outlet in the battery compartment.

The battery is charged when »alarmo« is connected with the mains power supply.

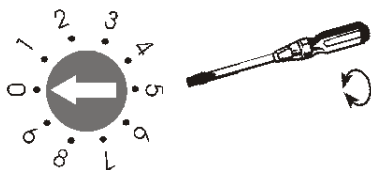
The battery has a charging time of 12 hours. Use and charge only the special battery A-2995-0.

2. Setting the correct radio channel

A special digital transmission process ensures that the radio signals are picked up without errors. This process also makes it possible to set 10 different radio channels. When supplied from the factory, all **RF lisa** devices are set to channel 0. It is only necessary to set units to other channels if you have several different users operating **RF lisa** systems in the same building. Set the required channel by using a small screwdriver to turn the channel selector switch, which is located on the right side of the unit, to the required channel number.

All transmitters and receivers must be set to the same channel.

Example:
Set channel number 0

Important notice:

»alarmo«'s radio alarm signal is triggered by the computer-assisted positive analysis of environment sounds.

Despite of digital filtering and programmed threshold values, the possibility of a false alarm cannot be fully excluded, due to the almost limitless variety of possible sounds (e. g. siren-like custom ringtones on mobile phones).

Due to these facts, the Humantechnik GmbH will take no responsibility for false alarms and their effects.

For this reason we advise to test the function of the »alarmo« by releasing a test alert on the alarm device (smoke detector, fire alarm system) which is to be monitored.

Sensitivity

»alarmo« has been developed and tested by considering and utilising many different types of alarm tones.

It is possible to alter the thresholds of your »alarmo«'s acoustic analysis, should it fail to respond to the monitored alarm device, even after repeated tests and alternating positions.

This involves opening the chassis and a certain process.

In this case, please contact the Humantechnik GmbH for advice.

Radio range

Note the following points when using the »alarmo«:

The radio signals are designed to have a range of 70 meters under perfect conditions. The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation.
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

Maintenance and care

»alarmo« does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth.

Note: Disconnect »alarmo« from the power supply!

Never use spirits, thinners or other organic solvents. Do not set »alarmo« up where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, above or near the product.

Please make sure that the batteries are not exposed to excessive heat, such as sunlight, fire or anything similar.

Warranty

The »alarmo« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by unauthorised persons (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical data

Power supply:	12 V-power supply unit at 110 - 230 V, 50-60 Hz or via rechargeable battery A-2995-0
Rechargeable battery standby time:	Approx. 2 hours
Frequency:	868.35 MHz
Operating range:	0° C to 40° C
Transmission indicator:	1 red LED
Battery level indicator:	1 red/green LED
Height:	65 mm
Width:	100 mm
Depth:	135 mm
Weight:	220 g

This device satisfies the following EU directives:

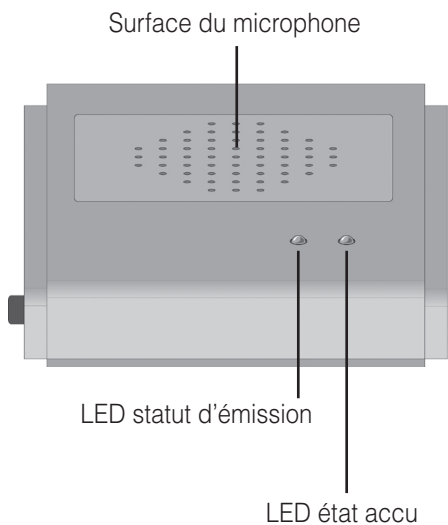
- 2011/65/EG RoHS directive
- 2002/96/EG WEEE directive
- 1999/5/EG R&TTE directive


Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at www.humanteknik.com.

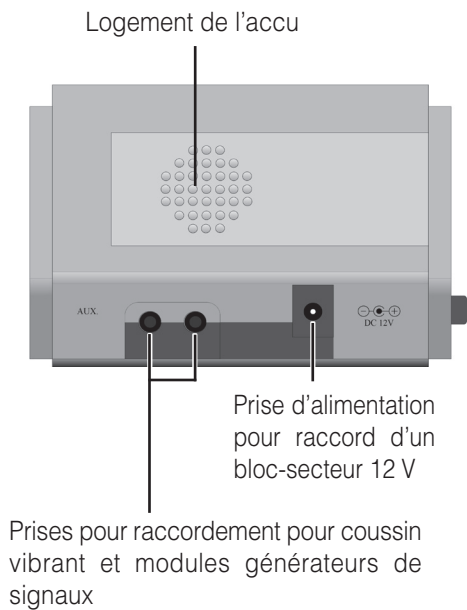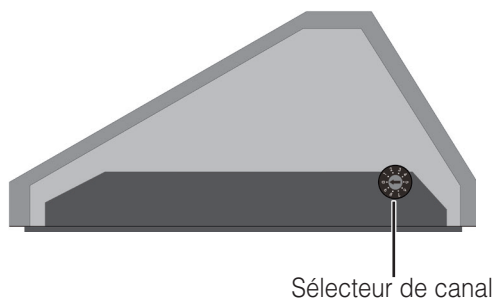
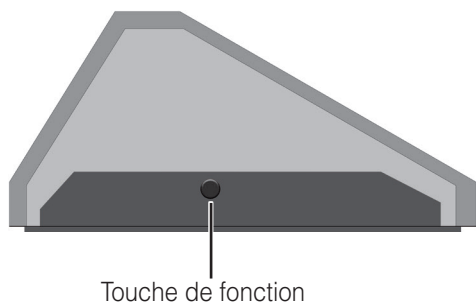
Technical specifications subject to change without prior notice.

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Sarabec Ltd