HUMANTECHNIK

*lisa*

**DE**

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Seite 2

Funk-Alarmsender galvanisch

**EN**

**Operation Instructions**

Page 8

RF alarm transmitter direct

**FR**

**Mode d'emploi**

Page 14

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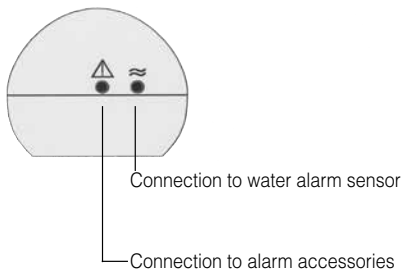
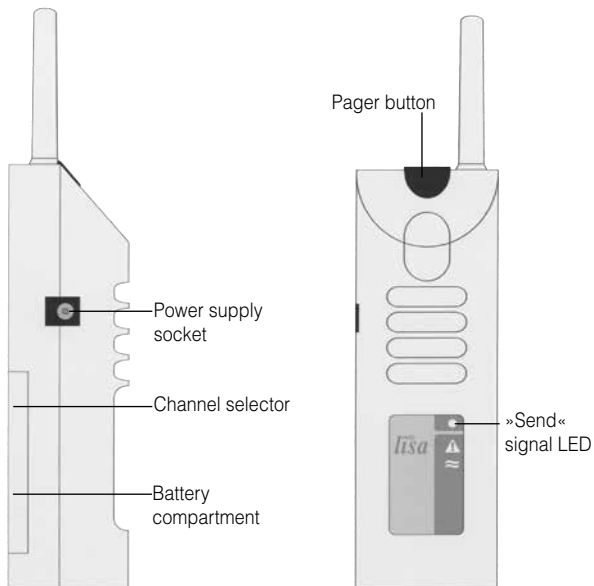
Congratulations on purchasing your *RF alarm transmitter direct*.

This is a product for people with hearing problems and helps to perceive acoustic events in the home through visible signals. Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

### Standard componets

Please check if all following components are included:

- lisa RF alarm transmitter
- 1 connection cable
- 9 V battery
- Operating instructions
- Warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

### Operating principle

An alerting system consists of at least one transmitter and one receiver. The *RF alarm transmitter* picks up the signals from various alarm sensors. It converts these signals into radio pulses (868.35 MHz) and transmits them to the receiver without the need for wires.

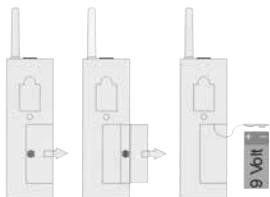
### Special features

When a signal is detected, a transmission is sent out for a total of 22 seconds. This increases both the reliability and the range of the system.

## Setup

### 1. Inserting the battery

There is a battery compartment on the back of the *RF alarm transmitter*. Open it by sliding the cover of the battery compartment to the right of the receiver using your thumb. Connect a 9 V block battery to the connector plug, insert the battery into the battery compartment and close the device. When the battery is connected, the receivers may be triggered once.

### 2. Setting the correct radio channel

A special digital transmission process ensures that the radio signals are picked up without errors. This process also makes it possible to set 10 different radio channels.

All *lisa RF* devices are set to channel 0 by factory default. It is only necessary to set units to other channels, if you have several independent users operating units in the same building. Set the required channel by using a small screwdriver to turn the channel selector switch, which is located

towards the top of the battery compartment, to the required channel number.

All transmitters and receivers must be set to the same channel.

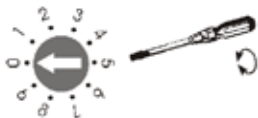
Fig. setting the correct radio channel.

### 3. Installing the alarm accessories

Plug the alarm accessories into the socket on the underside of the RF alarm transmitter.

This socket is identified by a triangle framing an exclamation mark:

#### Please note:

A separate socket must be used for the water alarm sensor.

This socket is marked with a double wave.

### Available accessories:

#### Alarm mat (A-2921-0)

Connect the plug from the alarm mat directly into the socket in the RF alarm transmitter (alarm accessories socket). Place the alarm mat under the doormat.

#### Alarm call button (A-2935-0)

Connect the alarm call button's plug directly into the socket of the RF alarm transmitter (alarm accessories socket). The alert is triggered by pressing the button (e.g. from the patient's bed).

#### Water alarm sensor (A-2924-0)

Plug the water alarm sensor into the socket marked with a double wave. The water alarm sensor is suitable for nearly all liquids.

Stick the sensor with the supplied double-sided adhesive tape to the wall so that the contacts touch the floor. The signal is triggered as soon as both contacts are in contact with the liquid.

#### Mount-on contact (A-2922-0)

For indicating opened doors and windows. The mount-on contact consists of a permanent magnet and a reed contact. The contact is open when the magnet is in contact. An alarm signal is triggered if the magnet is separated from the contact. Use the cable to connect the mount-on contact to the

RF alarm transmitter (alarm accessories socket).

### **Mains operation**

You can also operate your *RF alarm transmitter direct* via the plug-in power supply unit (A-2992-0).

To connect the PSU, insert the connector on the end of the PSU cable into the corresponding socket on the left-hand side of the device. Then plug the power supply unit into the normal 230 V socket in your house.

### **Radio range**

The radio signals are designed to have a range of 80 meters under optimum conditions. The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones.

Under certain circumstances, this may prevent the receiver from functioning at all.

- Positioning of the transmitter at an unfavourable distance from surfaces, which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.

- Particularly in towns and cities, there may be many other sources of radio waves, which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another, may also result in mutual interference.

### **Positioning the radio transmitter**

Bear in mind the following points when positioning the radio transmitter, in order to achieve the greatest possible range from the radio transmitter:

- The transmitter should not be set up close to devices which emit powerful interference signals, such as TVs, computer monitors or mobile phones.
- The radio transmitter should not be placed near metallic objects, since otherwise the radio range will be restricted by metallic shielding.

## Maintenance and care

The *RF alarm transmitter direct* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the *RF alarm transmitter direct* where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

### ATTENTION:

Only use commercially available 9 V block batteries!

There is a risk of explosion if the wrong type of battery is used. The batteries must not be exposed to high or low extreme temperatures and low atmospheric pressure during use, storage or transport at high altitude. Disposing of batteries in fires, ovens or mechanically crushing and cutting them can cause fires or explosions.

**Dispose of used batteries according to your local disposal regulations.**

**Use only the intended mains adapter A-2992-0 with the unit.**

## Warranty

The *RF alarm transmitter direct* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/ till receipt.

**Always specify the product number in any event.**

### Battery life

Under normal conditions (7 transmissions a day), the transmitter battery should last for about 1 year. Press the pager button on the transmitter to test the battery charge status. If the „Send“ signal LED on the transmitter does not come on, the battery needs to be replaced!

**Disposal** of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of these products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

### Technical data

Power supply:	9 V block battery or via a plug-in PSU (A-2992-0, accessory)
Battery life:	approx. 1 year
Frequency:	868.35 MHz
Operating range:	0° C – 40° C
Height (including the antenna):	147 mm (201 mm)
Width:	50 mm
Depth:	41 mm
Weight (including battery):	135 g

This device complies with the following EC directives:

- 2011 / 65 / + 2015 / 863 / EC RoHS directive
- 2012 / 19 / EC WEEE directive
- 2014 / 53 / EC RED directive

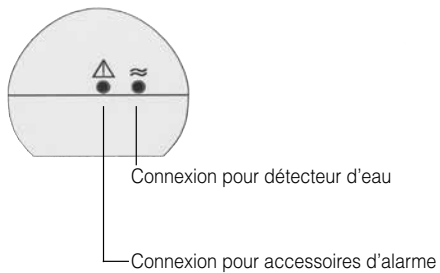
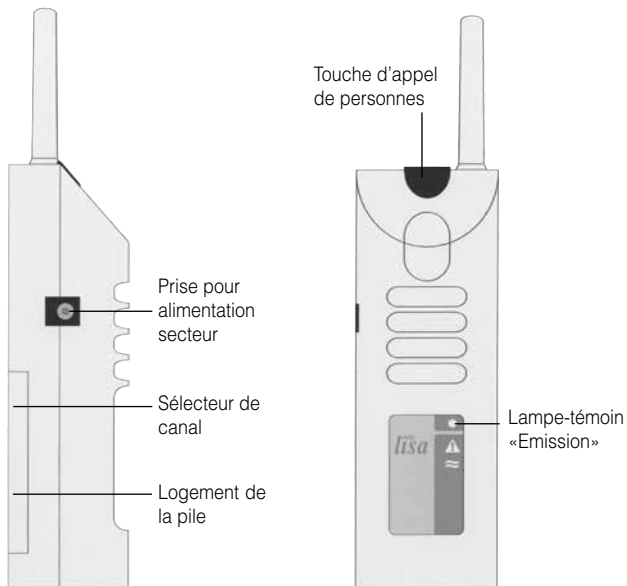
Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at [www.humantechnik.com/service](http://www.humantechnik.com/service).

Humantechnik's UK authorised representative:  
Sarabec Ltd., 15 High Force Road, MIDDLESBROUGH TS2 1RH, UK.

Sarabec Ltd., hereby declares that this device complies with all UK statutory instruments. UK declaration of conformity available from: Sarabec Ltd.

Technical specifications subject to change without prior notice.

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**Humantechnik GmbH**Im Wörth 25
**Humantechnik GHL AG**Rastatterstrasse 9
**Sarabec Ltd**15 High Force Road