HUMANTECHNIK

lisa

DE

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Seite 2

Funk-Kombisender
lisa combi II

EN

Operating Instructions

Page 8

Radio transmitter
RF *lisa* combi II

FR

Mode d'emploi

Page 14

Emetteur radio *lisa*
combi II RF

NL

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Pagina 20

Draadloze Kombizender
combi II RF

IT

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Pagina 26



Radiotrasmettitore
combi II RF


ES

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Página 32

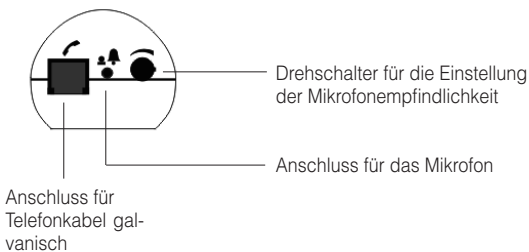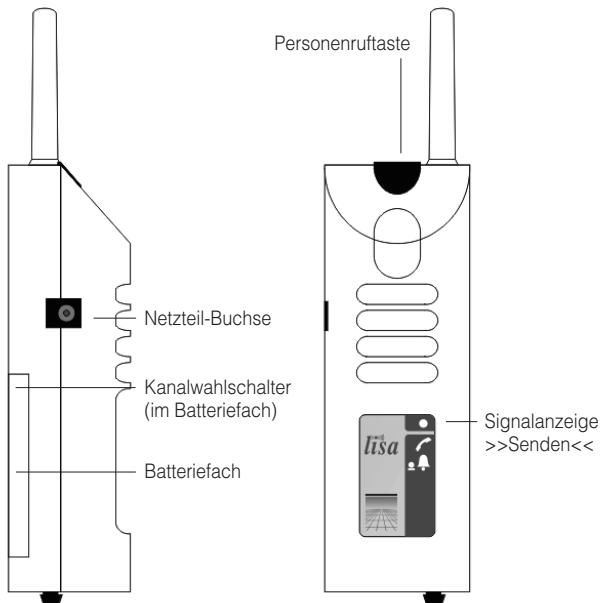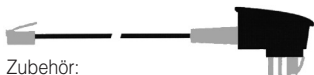Radioemisor
combi II RF

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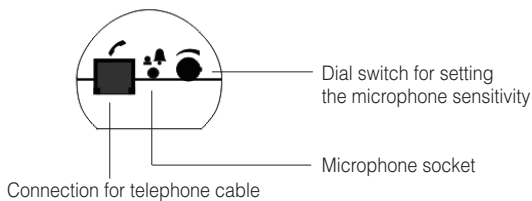
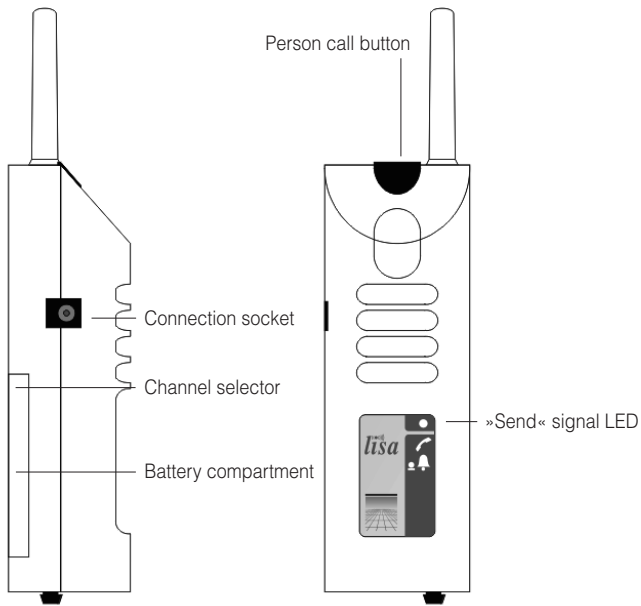
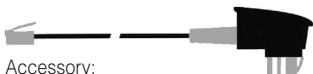
Accessory:
telephone cable

Accessory: microphone cable
with 2.5 mm jack plug

Congratulations on purchasing your *RF lisa combi II*. We hope it will give you many years of reassuring operation. Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

Included in the standard package

Please check if all following components are included:

- *RF lisa combi II*
- Telephone cable, length 6 m
- Microphone cable, length 1 m
- 9 V block battery
- Operating Instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

Operating principle

A signalling system consists of at least one transmitter and one receiver. The *RF lisa combi II* picks up the ringing signals from telephone and

doorbell through an electrical connection. It converts these signals into radio pulses (868.35 MHz) and transmits them to the receiver without the need for wires.

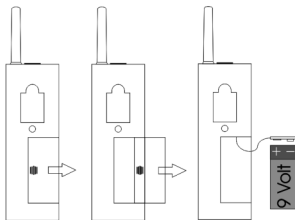
Special features

When a signal is detected, it is transmitted for a total of 22 seconds. This increases both the reliability and the range of the system.

Setting up the system -

1. Inserting the battery

There is a battery compartment on the back of the *RF lisa combi II*. Open it by sliding the cover of the battery compartment to the right of the receiver using your thumb. Connect a 9 V block battery to the connector plug, place the battery into the recess and close the receiver again. When the battery is connected, the receiving signals may be triggered once.

2. Setting the correct radio channel

A special digital transmission process ensures that the radio signals are picked up without errors. This process also makes it possible to set 10 different radio channels.

When supplied from the factory, all *RF lisa combi II* are set to channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building.

Set the required channel by using a small screwdriver to turn the channel selector switch, which is located towards the top of the battery compartment, to the required channel number. All transmitters and receivers must be set to the same channel.

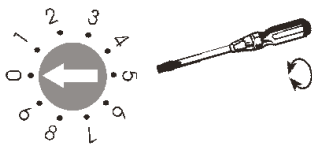
Fig. Setting the correct radio channel.

3a. Installing the telephone cable

Plug the end of the cable with the transparent clip-in connector (6-pin plug) into the socket on the underside of the *RF lisa combi II* transmitter. Plug the other end into a space telephone socket.

Important

Please only use the telephone cable supplied with the unit, otherwise proper function cannot be guaranteed.

The *RF lisa combi II* is designed to be connected to an analog telephone system.

Use the RF telephone transmitter acoustic if your phone system is ISDN or part of a PBX (exchange) system.

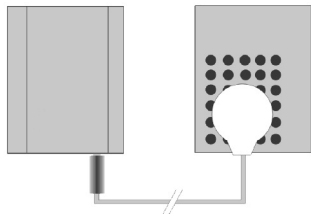
3b. Installing the microphone cable

Plug the microphone cable into the socket on the underside of the *RF lisa combi II*. Use the adhesive ring to stick the microphone housing onto the acoustic output cover of your doorbell/intercom system. To do this, pull off the backing foil from the adhesive ring on the microphone.

For optimum function of the transmitter, its sensitivity dial switch must be set so the transmitter reliably responds to the doorbell.

Pulling off the backing foil.

Plugging in the microphone cable and sticking the microphone over the acoustic output cover of your doorbell.

Person call button

Pressing the person call button on the *RF lisa combi II* causes a signal to be triggered on every receiver which is set to the same radio channel as the transmitter. The person call button lamp lights up to indicate this. This makes it possible to call people who are in different rooms. Also, this button can be used for checking the system is functioning even when the microphone cable is not connected.

Mains operation

You can also operate your *RF lisa combi II* via the plug-in power supply unit (A-2992xx). To connect the PSU, insert the connector on the end of the PSU cable into the corresponding socket on the left-hand side of the device. Then plug the power supply unit into the normal 230 V socket in your house.

Radio range

The radio signals are designed to have a range of 80 meters under optimum conditions.

The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.

- Positioning of the transmitter at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

Positioning the radio transmitter

Bear in mind the following points when positioning the radio transmitter, in order to achieve the greatest possible range from the radio transmitter:

- The radio transmitter should not be set up close to devices which emit powerful interference signals, such as TVs, computer monitors or mobile phones.
- The radio transmitter should not be placed on metallic objects, since otherwise the radio range will be restricted by metallic shielding.

Maintenance and care

The *RF lisa combi II* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the *RF lisa combi II* where it will be exposed to full sunlight for long periods.

In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Battery life

Under normal conditions (7 transmissions a day), the transmitter battery should last for about 1 year. Press the pager button on the transmitter to test the battery charge status. If the "Send" signal LED on the transmitter does not come on, the battery needs to be replaced!

Warranty

The *RF lisa combi II* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge.

It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units.

You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical data

Power supply:	9 V block battery or via a plug-in PSU
Battery life:	approx. 1 year
Frequency:	868.35 MHz
Maximum transmitting power:	<10 mW
Operating range:	0° C – 40° C
Height (including the antenna):	147 mm (201 mm)
Width:	50 mm
Depth:	41 mm
Weight (including battery):	145 g

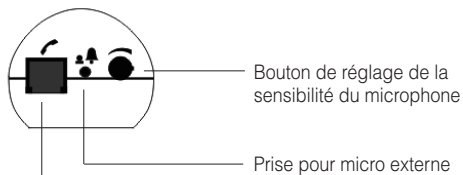
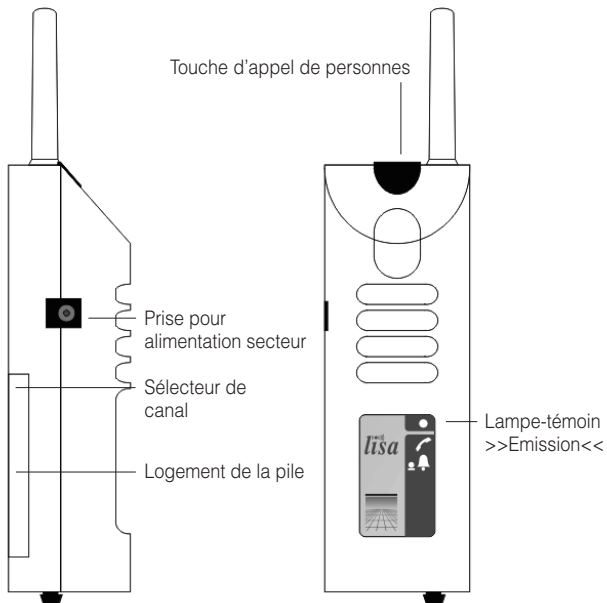
This device satisfies the following EC directives:

- 2011 / 65 / EC + 2015 / 863 / EC RoHS directive
- 2012 / 19 / EC WEEE directive
- 2014 / 53 / EC RED directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at www.humantechnik.com.

Technical specifications subject to change without notice.

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Sarabec Ltd