HUMANTECHNIK

visuTone.

DE

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Seite 2

EN

Operation Instructions

Page 6

FR

Mode d'emploi

Page 10

NL

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Pagina 14

IT

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Pagina 18

ES

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Página 22

SE

Bruksanvisning

Sida 26

NO

Betjeningsveiledning

Side 30

FI

Käyttöohje

Sivu 34

PL

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Strona 38


HU

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
Oldal 42

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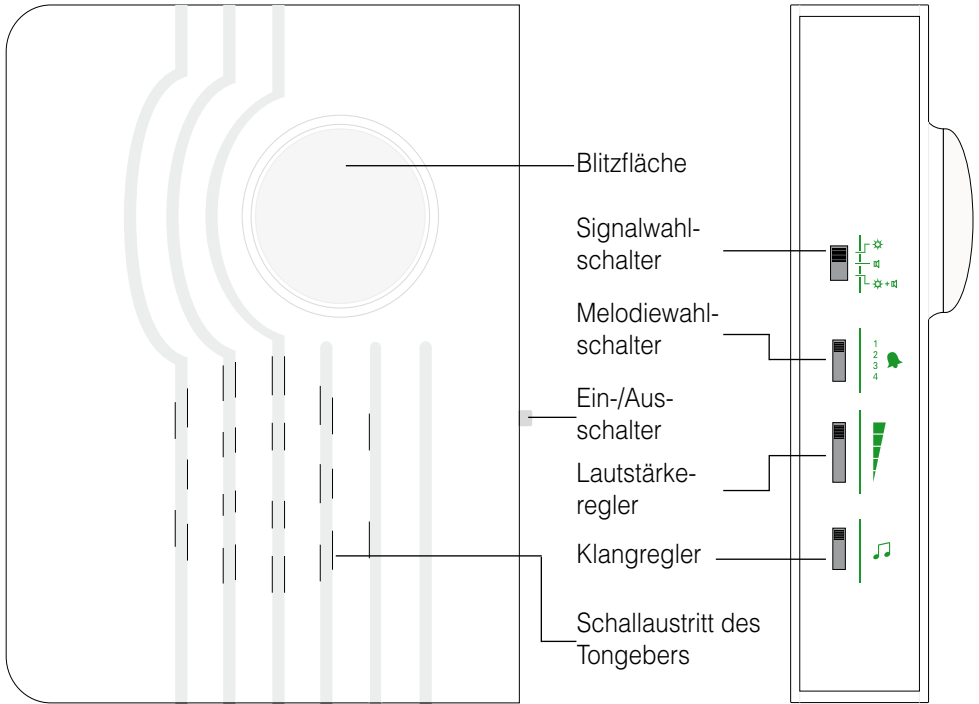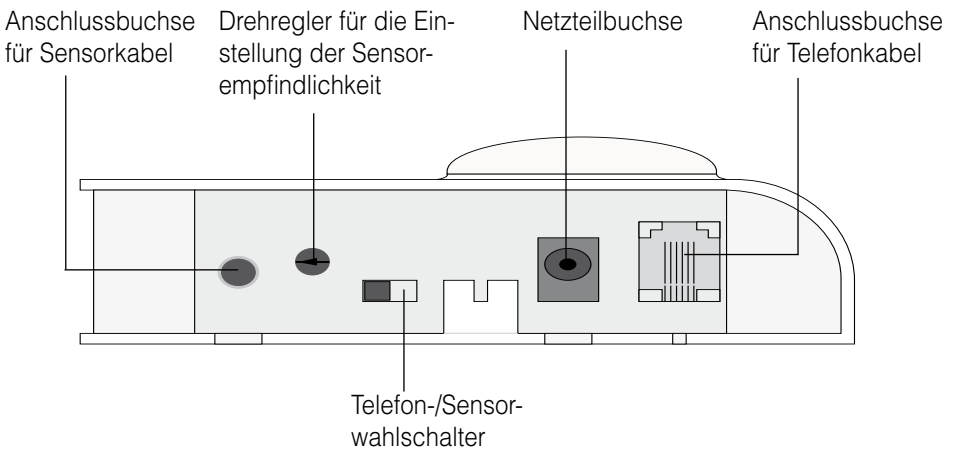
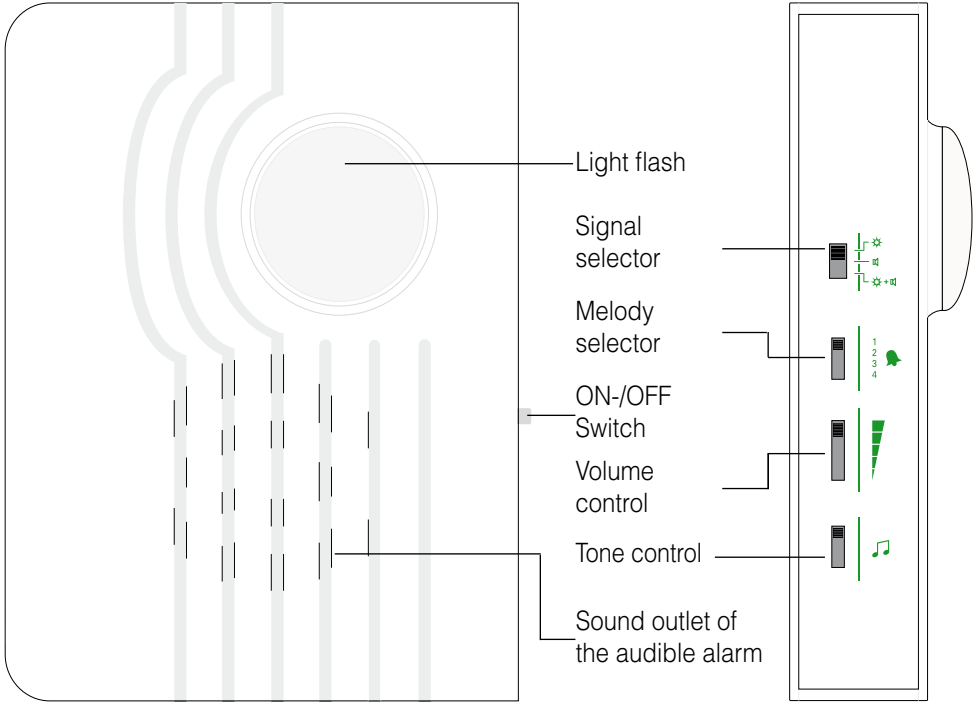
Device left side

Device underside

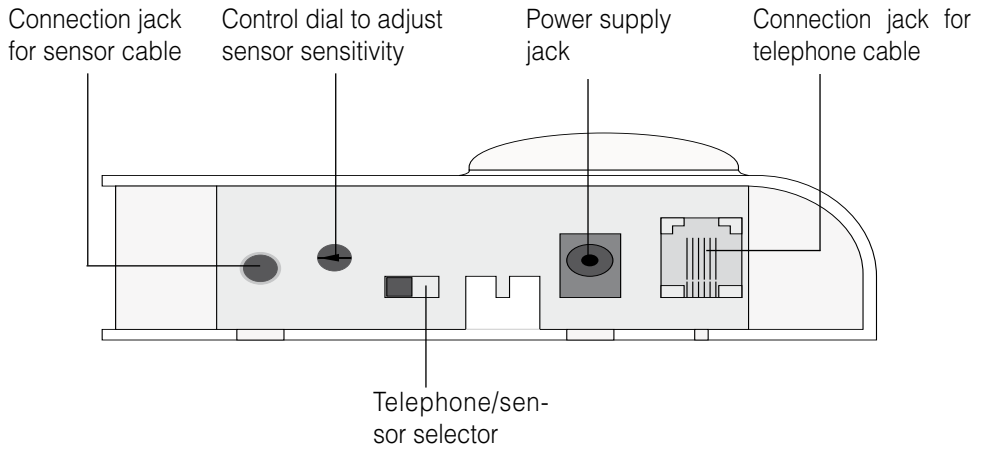Congratulations on purchasing your *visuTone*. You have chosen a modern and reliable system. Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

Standard componets

Please check if all following components are included:

- *visuTone*
- 4 Batteries (Mignon, AA, LR06)
- Telephone cable, length 5 m
- Sensor cable, length 5 m
- Operating instructions
- Warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

Operating principle

The *visuTone* picks up the ringing signals from your telephone or doorbell through an electrical connection and converts these signals into acoustic and optical signals.

1. Inserting the batteries

You will find a battery compartment on the rear side of the *visuTone*. Open it by gently pressing the small clip against the battery compartment lid with your thumb and then lift it up. Now insert the four batteries that came with the device into the compartment, **making sure that plus is matched with plus and minus with minus!** Replace the lid on the compartment.

2a. Installing the telephone cable

Plug the end of the cable with the transparent clip-in connector (6-pin plug) into the socket on the underside of the *visuTone*. Plug the other end into a space telephone socket.

If you use the telephone cable, the telephone/

sensor selector on the underside of the *visuTone* must be set to  .

Important

Please only use the telephone cable supplied with the unit, otherwise proper function cannot be guaranteed. The *visuTone* is designed to be connected to an analog telephone system.

2b. Installation of the sensor cable

Use the adhesive ring to affix the sensor disk to the sound emission opening on your doorbell and insert the sensor cable into the connection jack on the underside of the *visuTone*. To do so, make sure to remove the protective film over the adhesive ring on the sensor disk first.

If you use the sensor cable, the telephone/sensor selector on the underside of the *visuTone* must be set to  . To ensure that the *visuTone* functions properly, the sensitivity control dial must be set so that the sensor reacts to the signal promptly.

Note

You can use the *visuTone* to either indicate a ring at the doorbell or a ring of the telephone. The *visuTone* is not able to process and indicate both signals at the same time.

3. ON-/OFF switch

Set the ON-/OFF switch to „ON“. The *visuTone* is now ready to operate.

4a. Selecting the type of signaling

Use the signal selector to choose whether a doorbell or telephone ring is to be signaled visually only, acoustically only or both visually and acoustically. To do so, slide the signal selector to one of the three following settings:

- Selector at the top: visually only
- Selector in the middle: acoustically only
- Selector at the bottom: visually and acoustically

4b. Selecting the melody

Use the melody selector to select one of the four melodies available.

4c. Adjusting the volume

Use the volume control to adjust the volume of the ring signal to one of five volume levels.

4d. Adjusting the tone

Use the tone control to adjust the tone of the ring signal to one of four levels between a high-pitched and a low-pitched tone.

Note (special accessory)

A power cord available as a special accessory can be plugged into the jack marked "DC 6V" on the underside of the *visuTone*.

Maintenance and care

The unit does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. **Note: remove the *visuTone* from mains socket!**

Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

Warranty

The unit is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units.

You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal.

Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical data

Power supply:	4x 1.5V AA batteries or using the 6 V power cord (special accessory) plugged into a 230 V, 50 Hz power outlet
Battery life:	approx. 3 months (at 5 events a day)
Operating range:	0° C – 40° C
Optical signals:	Light flash
Acoustic signaling:	Ring signal with adjustable volume, tone and melody
Height:	150 mm
Width:	115 mm
Depth:	40 mm
Weight:	350 g (including battery):

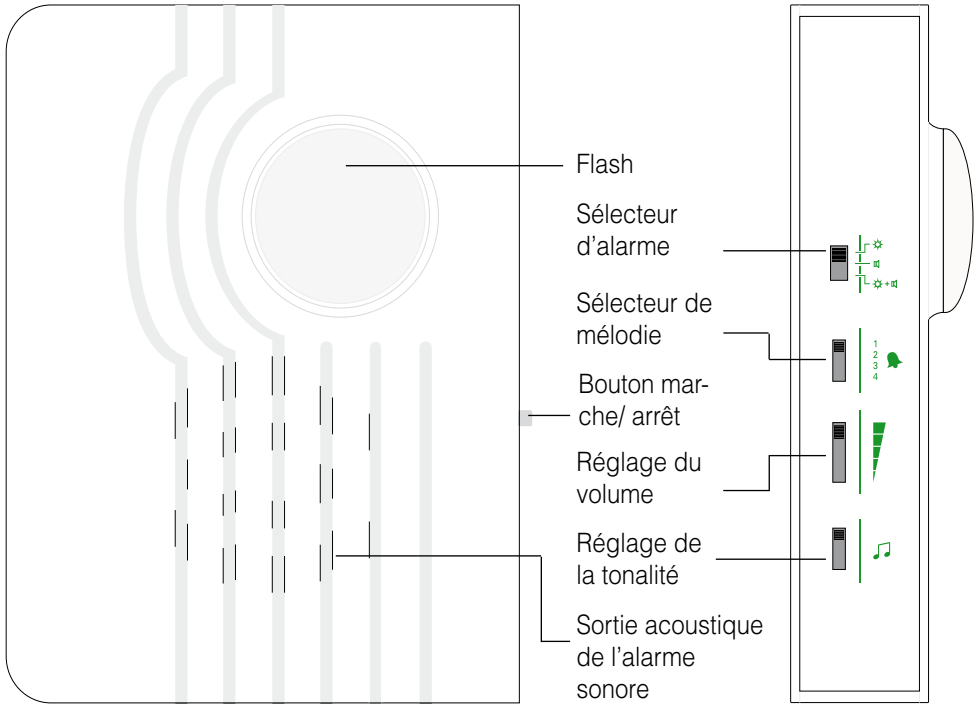
This product complies with European Union norms.

You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

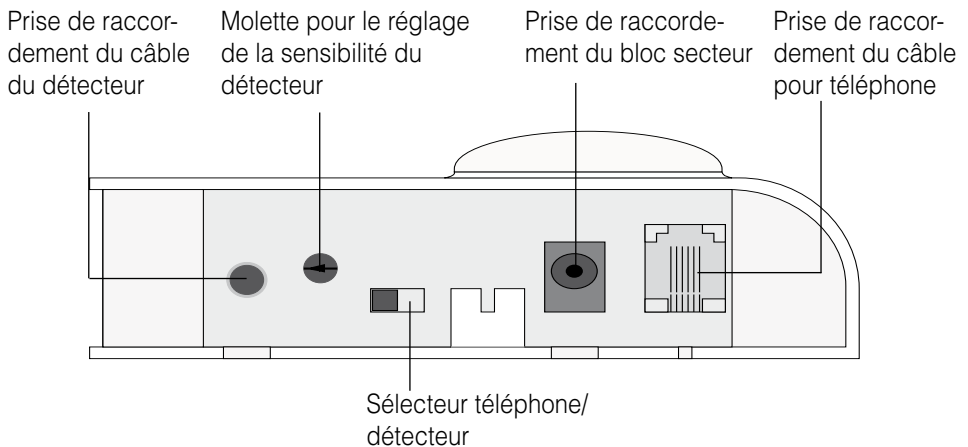
Technical specifications subject to change without notice.

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Sarabec Ltd