HUMANTECHNIK

**visu**Tone.air

**DE**

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Seite 2

**EN**

**User manual**

Page 12

**FR**

**Mode d'emploi**

Page 22

**NL**

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Pagina 32

**IT**

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
Pagina 42

**ES**

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
Página 52
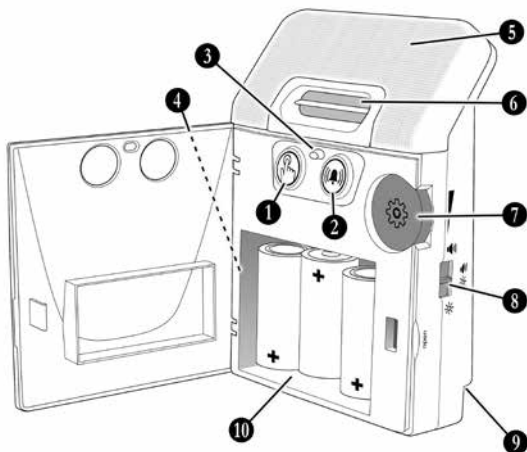
## Empfänger

## Türklingeltaste
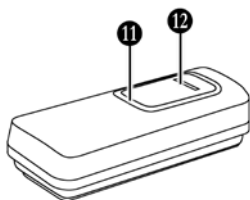
## Telefonsender

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

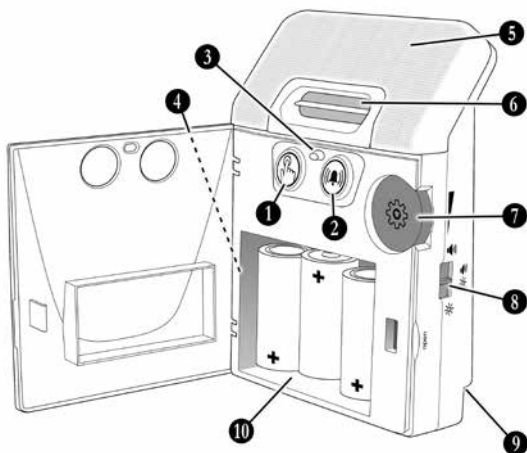
## Receiver

## Doorbell-pushbutton
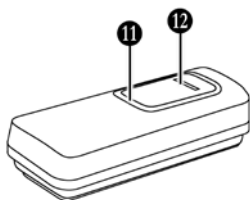
## Telephone transmitter (not included - available as accessory)

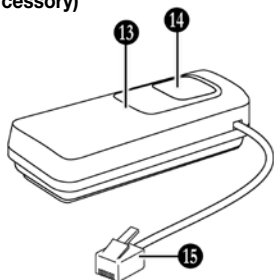Thank you for buying the *visuTone.air!* Please read this manual carefully. It will tell you how to set up the product correctly and explain all the features it has to offer.

## Package contents

Please check if the following items are included:

- »*visuTone.air*« receiver
- Doorbell-pushbutton
- 3 batteries type C (R14)
- 1 button cell CR2032
- 2 screws
- 2 wall plugs
- User manual
- Warranty card

Should any parts be missing, please contact your dealer or the manufacturer immediately.

## Basic function

The doorbell-pushbutton is installed next to or in place of the existing doorbell. When the doorbell button is pressed, a radio signal is transmitted to the receiver, which then indicates the pressing of the button either with sounds, light or both.

## Safety instructions

Prior to putting this product into operation, please read this manual thoroughly.

Store this user manual in a safe place so that it is also accessible for other users at any time.

When using power tools: Always follow the instructions of the manufacturer and wear protective equipment.

Always check before starting to drill if there are any electric cables or water pipes in the wall. In case of doubt we recommend using a cable detector.

## Controls and displays

Legend to illustrations page 12:

### Receiver:

- 1) Indicator for door (green)
- 2) Indicator for telephone (orange)
- 3) Battery indicator
- 4) Reset button (left side)
- 5) Flashing area
- 6) Speaker
- 7) Control wheel / button
- 8) Indication selector
- 9) DC-power socket
- 10) Battery compartment

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
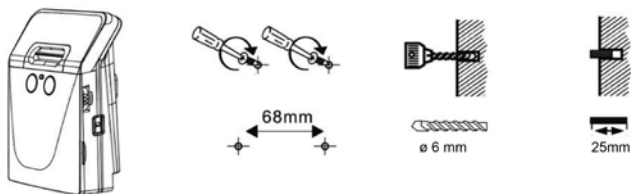
# 1. Installation

The receiver can either be stood or mounted on a wall anywhere within the radio range of the transmitters. To do this, fix the supplied screws into the wall (or wooden beam, cupboard, etc.) at a horizontal distance of 68 mm. When mounting on stone walls, it is necessary to use a power drill and masonry bit (6 mm). In this case, use the enclosed wall plugs for fastening. After attaching the screws, hook the »*visuTone.air*« receiver onto these via the recesses on the back of the unit.

## 1-2. Doorbell-pushbutton / Telephone transmitter

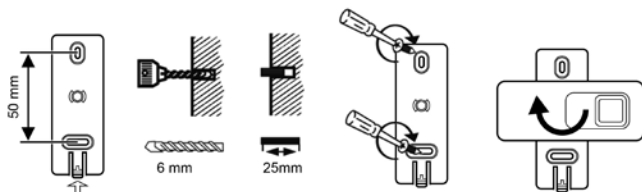
Lift the locking tab slightly and turn the wall bracket on the underside of the transmitter 90° to the housing. The mounting base can now be removed from the transmitter. Stick the wall bracket to the desired position on smooth and clean surfaces using the pre-mounted adhesive surface. For rough, uneven and dusty surfaces, it is recommended that the bracket be attached with screws. Insert two screws at a distance of 50 mm into the wall (or wooden beam, cupboard, etc.). When attaching to walls, it is necessary to use a power drill together with a masonry bit. In this case, use wall plugs to secure the bracket. Screw the bracket to the wall - be careful not to overtighten the screws. Then place and turn the transmitter housing back onto the bracket. The doorbell-pushbutton is protected against water and can be mounted outdoors (protection class IP56).

### Telephone transmitter:

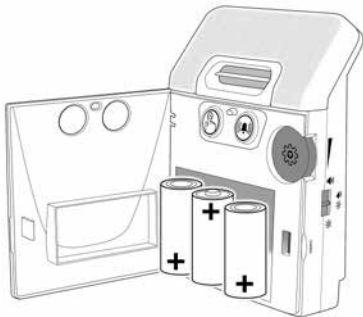Choose a location close to your telephone socket of your household or the telephone output of your router. If applicable, fix to a wall as described above. Connect the RJ-11 plug of the transmitter. If necessary, use adapters for this purpose (adapters are not included).

## **2. Inserting the batteries**

### **2-1. Receiver**


Open the door of the battery compartment and insert 3 batteries size C. Please ensure correct polarity of the batteries.

The power supply unit (item number A-2907-0, available as accessory) can alternatively be used. Plug the connector into the DC socket on the lower rear side of the receiver.

Battery indicator: When the batteries are at the end of their life, the red battery indicator flashes once every 15 seconds. The volume of the acoustic indicator is turned down and can only be turned up again after the batteries have been replaced.

### **2-2. Doorbell-pushbutton / Telephone transmitter**

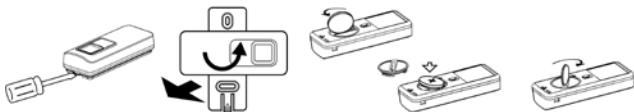Unlock the locking tab by using a small flat screwdriver. Rotate the mounting bracket and transmitter 90 degrees against each other. Remove the mounting bracket from the unit.

Rotate the battery cover using a coin. Remove the battery cover and insert the included CR2032 battery into battery compartment provided. Please ensure correct polarity of the battery. Insert the battery cover and lock it, using the coin again. Reattach the transmitter to the mounting bracket.

Battery indicator: If the transmitter battery reaches a low level, a warning signal is sent to the receiver. The respective symbol (green or orange) then flashes on the receiver once every 15 seconds. Replace the transmitter's button cell as soon as possible.


### 3. Operation

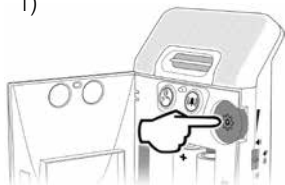The transmitter and receiver are already matched to each other at the factory and are immediately ready for operation after inserting the batteries. The coupling procedure described below can be carried out in the event of faults or the linking of further transmitters.

#### 3-1. Register a transmitter with the receiver:

The control wheel functions also as a button. Open the battery compartment door to operate it.

1)

2)

3.)

1) Press and hold the control wheel button for 2 to 3 seconds. The green doorbell event indicator lights up and starts flashing. If necessary, press the button again to change to the orange telephone ring event indicator. Otherwise, press the button again to cancel the pairing process.

2) When the desired event indicator flashes on the receiver, press the button of the transmitter you wish to be paired. If the pairing is successful, a beep is heard and the selected symbol flashes quickly. A double beep means a failed pairing. Make sure that the affected transmitter has not already been registered.

### 3-2. Setting the melody:

The standard melody is set by default. You can choose between 4 doorbell melodies and 2 telephone ringtones. Proceed as follows to change the melody:

- 1) Press and hold the control wheel button for 2 to 3 seconds. The green doorbell event indicator lights up and starts flashing. If necessary, press the button again to change to the orange telephone event indicator. Otherwise, press the button again to cancel the pairing process.
- 2) Select the desired melody for the acoustic alerting. To do this, turn the control wheel until the desired melody is heard. Press the control wheel button to select it.

### 3-3. Unregister a transmitter:

To unregister a transmitter, proceed as follows on the receiver unit:

- 1) Press and hold the control wheel button for 2 to 3 seconds. The green doorbell event indicator lights up and starts flashing. If necessary, press the button again to change to the orange telephone event indicator. Otherwise, press the button again to cancel the process.
- 2) When the according event indicator flashes on the receiver, press and hold the control wheel button for 2 to 3 seconds. A beep is heard if the unregistration was successful.

### 3-4. Indication selector:

The slider switch on the right-hand side of the unit can be used to select 3 different alerting modes:

1) Sound only

2) Sound and flash

3) Flash only

### **3-5. Adjusting the volume:**

8 volume levels can be set with the control wheel button. To do this, turn the wheel up or down. Each volume level is indicated by a beep, a double beep means that the maximum setting has been reached.

The volume can be changed at any time. Exceptions: The receiver is in pairing mode or the battery level indicator is lit.

Please note that the volume setting applies to both event indicators.

### **3-6. In case of device conflicts:**

If the »*visuTone.air*« behaves erratically or rings for no apparent reason, there may be an interference caused by a neighbouring RF device. This can be resolved by changing the transmitter code as follows:

- 1) Unregister the affected transmitter as described in **3-3**.
- 2) Remove the button cell battery from the affected transmitter.
- 3) Press and hold the transmitter button and reinsert the button cell. The button should then be held for another 5 seconds. The transmitter's LED flashes 5 times, indicating that a new coding has been generated.
- 4) Now register the transmitter again.

## **4. Troubleshooting**

### **4-1. A red indicator flashes and the ringtone volume has dropped:**

Replace the batteries of the receiver. Use „C“ (R14) size alkaline batteries.

### **4-2. An event indicator flashes every 15 seconds:**

Replace the battery (button cell type CR2032) of the corresponding transmitter.

### **4-3. I need to unregister a transmitter:**

Proceed as described in section **3-3**.

### **4-4. The receiver rings but no-one is at the door or the phone:**

Proceed as described under point **3-6**.

### **4-5. I cannot remove the transmitter from the bracket:**

There is a locking tab on one of the narrow sides of the bracket. This must be pressed slightly away from the transmitter - the transmitter can now be twisted and removed.

## 5. Accessories

### The following accessories are available:

visuTone.air telephone transmitter	Item no. A-2506-0
visuTone.air power supply	Item no. A-2907-0

## 6. Technical data

### Receiver:

Dimensions:	145 mm x 105 mm x 52 mm
Weight:	216 g (without batteries)
Transmission frequency:	868,4 MHz
Power supply:	3x batteries type C (R14)
Battery life at 3 events per day:	at 75% volume: approx. 12 months at 100% volume: approx. 9 months
Power supply (accessory):	AC 110V - 230 V / DC 12V 1A
Volume:	up to 100 dB at 1 m distance
Temperature range:	0 to 40°C

### Transmitter:

Dimensions:	80mm x 32mm x 17mm
Weight:	Doorbell pushbutton: 25 g Telephone transmitter: 30 g
Transmission frequency:	868,4 MHz
Transmission range:	up to 300 m (in open areas)
Max. transmission power:	<10 mW
Power supply:	1x button cell type CR2032
Battery life at 3 events per day, approximately:	Doorbell pushbutton: ~ 3 years Telephone transmitter: ~ 2 years
Protection class doorbellbutton:	IP56

Technical specifications subject to change without prior notice.

## Maintenance and care

The »visuTone.air« does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

**Note:** This product is **not** protected against splash water, with exception of the doorbell-pushbutton. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar. .

## Warranty

»visuTone.air« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

## Always specify the product number in any event.

### **Batteries**

The included batteries are recyclable. Please dispose the batteries through the battery container or the specialist shop. Please dispose only discharged batteries to ensure environmental protection.

**Disposal** of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to

a collecting point for the recycling of electric and electronic units.  
You protect the environment and health of your fellow men by the correct disposal of this products.

Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

This device complies with the following EC guidelines:

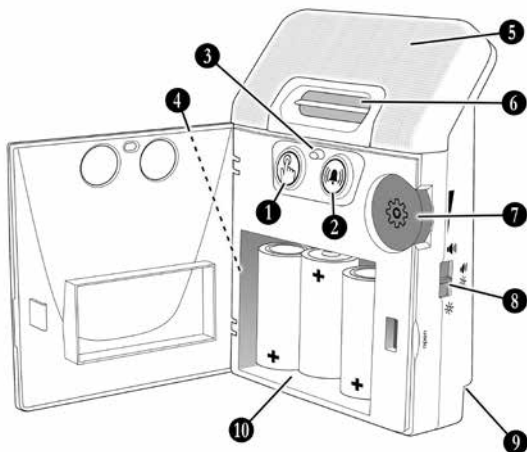
- 2011 / 65 / EC + 2015 / 863 / EC RoHS-directive
- 2012 / 19 / EC WEEE-directive
- 2014 / 53 / EC RED-directive


Compliance with the directives listed above is confirmed by the CE seal on the device.

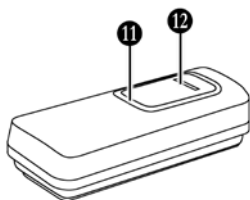Declarations of conformity are available on the Internet at [www.human-technik.com](http://www.human-technik.com).

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**Sarabec Ltd**