HUMANTECHNIK

signolux

**DE**

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Seite 2

**Empfänger S**

**EN**

**Operation Instructions**

Page 10

**Receiver S**

**FR**

**Mode d'emploi**

Page 18

**Récepteur S**

**NL**

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Pagina 26

**Ontvanger S**

**IT**

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Pagina 34

**Ricevitore S**

**ES**

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Página 42

**Receptor S**

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We would like to congratulate you on purchasing a »signolux«. You have chosen a modern and reliable system. Kindly read the user manual carefully in order to start using the device properly and to familiarise yourself with all the possibilities of the system.

### Standard delivery scope

Kindly check if all following parts are included:

- »signolux« receiver S
- 4 rechargeable batteries size AA / HR6
- Power supply unit
- Wall mount
- 2 screws
- 2 wall plugs
- Operating manual
- Warranty card

Should any parts be missing, kindly contact your specialist dealer or the manufacturer directly.

### Functioning principle

A »signolux« light signal system consists of at least one transmitter (e.g. doorbell pushbutton) and a »signolux« receiver. Up to 8 transmitters can be integrated.

If e.g. the doorbell pushbutton is pressed, a radio signal (radio impulses at 868,35 MHz) are transmitted to the »signolux« receiver. This one signals the received signals acoustically and visually with sounds and light signals.

### Safety instructions

- Prior to putting the device into operation and using it, kindly read this manual thoroughly and in full.
- Store this operating manual in a safe place so that it is also accessible for other users at any time.

- When using power tools always follow the instructions of the manufacturer and wear suitable protective equipment (e.g. protective goggles).
- Kindly check before starting to drill if there are any hidden electric cables and water pipes in the walls. In case of doubt we recommend using a cable detector.

## Setup

### 1. Inserting the rechargeable batteries into the »signolux« receiver

- a) At the rear side there is a battery compartment, its cover can be pushed downwards so that the compartment opens ( fig. 1).

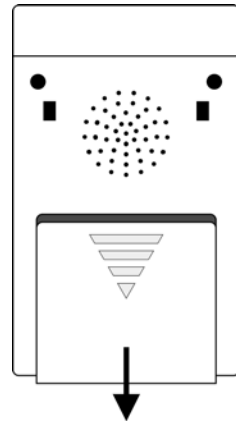
Figure 1: Battery compartment

- b) Now insert the 4 rechargeable batteries (included). Ensure correct polarity of the batteries.
- c) Close the battery cover again by inserting the cover and pushing it upwards.
- d) Connect the power supply to the receiver. The time needed for a full charge is approximately 6 hours.

## Setting and programming

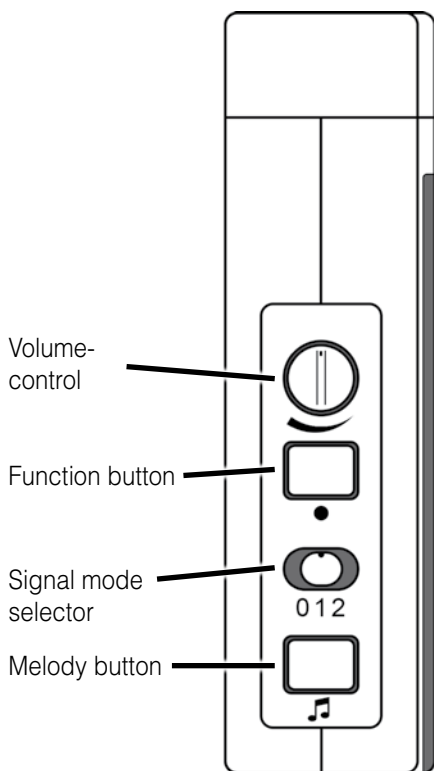
Figure 2: Controls, left side

### 2. Setting the signal mode

Using the signal mode slide switch (in fig. 2 “0 1 2”) you can select the signal mode. Kindly keep in mind that the event icons are always displayed by coloured LEDs.

- switch to position 0:  
alert tones, but no flash
- switch to position 1:  
alert tones and flash
- switch to position 2:  
no alert tones, flash only

Figure 3: Event icons and status LED

Each event icon can be assigned to a transmitter (e.g. doorbell pushbutton) and can be re-programmed at will.

### 3. Event icon signification

The symbols stand for:

Doorbell 1	Colour: Green
Doorbell 2	Colour: Green
Telephone	Colour: Orange
Person call	Colour: Yellow
Baby	Colour: White
Fire alarm	Colour: Red
Flood alarm	Colour: Blue
Alarm	Colour: Red

### 4. Setting the volume

On the left side of the »signolux« receiver you will find the rotating volume control with which you can set the volume of the alert tone.

#### **Warning:**

This device can generate sounds of up to 90 dB!

In order to protect your hearing, please set a low volume to begin with and only then adjust to the desired volume.

## 5. Programming signals and assignment to the event icons

Up to 8 transmitters can be assigned to each event icon. This assignment can be adapted by programming. Kindly keep in mind that a transmitter can only be programmed if it is not yet known in the memory of the »signolux« receiver. If required, you first might have to delete the transmitter from the memory (see step 7).

- a) Keep the function button »●« (fig. 2) pressed for some time. The event icons start to light up in sequence. Let go of the key as soon as you see the desired icon.
- b) Trigger your transmitter (e.g. operate the doorbell pushbutton or test button).
- c) Wait for about 20 seconds and test the function.

## 6. Setting the melody

You can adapt the melody for each icon:

- a) First trigger an alert (e.g. operate doorbell pushbutton).
- b) The icon assigned to the transmitter should light up and the previously chosen tone should be heard.
- c) Press and hold the melody setting button »♪« (fig. 2) for two seconds. Let go of the button and press it repeatedly until you have found the desired melody (you can choose from 15 different tones).
- d) Once the icon ceases to flash, the selected tone has been assigned.

## 7. Deleting transmitters from receiver memory

Do the following if you want to delete a transmitter from the memory:

- a) Keep the function button »●« (fig. 2) pressed for some time. The event icons light up in sequence. Let go of the key as soon as you see the desired icon.
- b) Now press the function button »●« and the melody button »♪« (fig. 2) at the same time, until all icons briefly light up.

Please note that this will remove all assigned transmitters from the selected event icon.

## 8. Battery level indication

The receiver informs about the state of the batteries of the receiver as well as the batteries of the transmitters:

- A sufficient battery charge level of the receiver is indicated by the red status LED flashing every 5 to 6 seconds.
- A low battery level of the receiver is indicated by the red status LED flashing in short sequence (once per second). Please recharge the receiver batteries as soon as possible (attach it to the power supply unit).
- A low battery level of a transmitter is indicated by the status LED blinking together with the icon of the according transmitter when its signal is triggered. Please replace the batteries of the according transmitter as soon as possible.

## Further functions

### 9. Recalling the last alert, cancelling the current alert

By pressing the »•« button (fig. 2) the last registered alert can be displayed again. You can also cancel an active displayed alert by pressing this button.

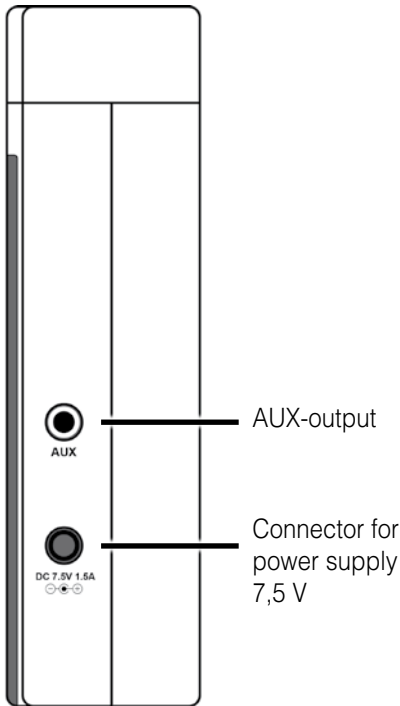
Figure 4: Lateral connectors

### 10. AUX-socket and power connector

The »AUX«-jack and the connector for the power adapter can be found on the right side of the receiver.

All Humanteknik modules can be connected to the AUX-output (vibrating pillow etc, available as accessories)

## Mounting

Important note: Kindly keep in mind that metal parts and other building materials may interfere with the transmission of the radio signal. As they can be either visible or hidden at the desired mounting location you should always carry out a functional test at the mounting location before starting the mounting process, and, if required, choose a different mounting or installation location if the reception is not guaranteed.

### 11. Mounting the »signalux« receiver

The »signalux« receiver is suited either for free-standing installation or it can also be mounted using the supplied wall mount. To use it, fix the wall mount onto a wall (or rafter, cupboard etc.) When mounting onto a brick- or stone wall, you may need a power drill and stone bit to do so. Use the supplied wall plugs in this case. Once fixed, hang the »signalux« receiver onto the wall mount bracket.

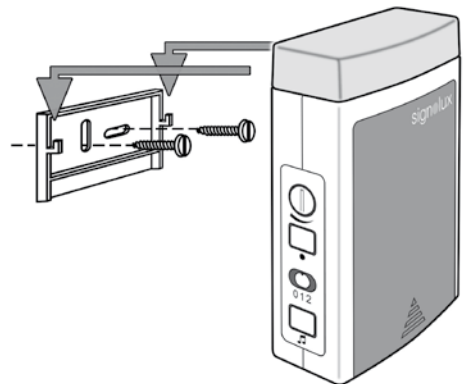
Figure 5: Wall mount

## Troubleshooting and solutions

Symptoms	Cause	Solutions
When trying to program a transmitter, a different icon lights up...	The transmitter you are trying to program is already known by the receiver.	See step 7. Delete the transmitter from the receiver's memory before it can be assigned to a new icon.
The red status LED light up with an alert...	This indicates a low battery in the according transmitter.	Insert a new battery in the transmitter.
The »signolux« receiver does not work...	Ensure that the rechargeable batteries have been inserted correctly.	Ensure correct polarity of the rechargeable batteries!
	Check if the batteries are charged	Connect the receiver to the power supply.
	Maybe the triggered transmitter has not yet been paired with the »signolux« receiver.	Assign it to an icon.
	Perhaps the »signolux« receiver is out of reach of the transmitter.	Try using the transmitter or receiver at different location.
There is no alert tone...	mode selector switch is on position 2 (fig. 2)	Check the position of the switch (fig. 2) on the »signolux« receiver.
		Ensure that the volume is not set to the lowest setting.
The flash light does not work...	mode selector switch is on position 0 (fig. 2)	Check the position of the switch (fig. 2) on the »signolux« receiver.

<b>Symptoms</b>	<b>Cause</b>	<b>Solutions</b>
The range is reduced...	Metal structures and other building materials can reduce the reach.	If possible do not mount the »signolux« transmitter or receiver near or on metal parts.
	Other devices may interfere with the radio signals, thus reducing the radio signals and the function of the »signolux« system.	Avoid setting up the »signolux« devices close to mobile phones and display screens.
	Walls and ceilings reduce the range.	Reduce the distance between transmitters and receivers or try a different position of the devices.
	Low battery charge reduces the range.	Charge the batteries by connecting the power supply unit.

## Maintenance and care

The »signalux« does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

## Warranty

The »signalux« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

**Disposal** of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

## Technical Data

Work range	0 to 40°C
Radio frequency	868,35 MHz*
Sound volume	90 dB at 1 m
Rechargeable battery type	4 x AA NiMH 1,2 V / 1700 mAh
Service time per charge (5 activations per day)	3 weeks
Power adapter	7,5 V / 1,5 A connection
Dimensions (HxWxD)	145 x 90 x 40 mm
Weight	298 g

\* Export versions for USA, CA, AU and IL have a frequency of 915 MHz.

This device complies to the following EC standards:

- 2011 / 65 / EC RoHS-Directive
- 2002 / 96 / EC WEEE-Directive
- 1999 / 5 / EC R&TTE-Directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at [www.humantechnik.com](http://www.humantechnik.com).

Technical specifications subject to change without prior notice.

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**Sarabec Ltd**