HUMANTECHNIK

signolux

DE

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Seite 2

Empfänger S

EN

Operation Instructions

Page 12

Receiver S

FR

Mode d'emploi

Page 22

Récepteur S

NL

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Pagina 32

Ontvanger S

IT

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Pagina 42

Ricevitore S

ES

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Página 52

Receptor S

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Thank you for purchasing the modern and reliable »signolux« alerting system. Please read the user manual carefully for the proper use of the device and so that you can familiarise yourself with all the possibilities this system offers.

Standard delivery scope

Kindly check if all following parts are included:

- »signolux« receiver S
- 4 rechargeable batteries size AA / HR6
- Power supply unit
- Wall mount
- 2 screws + 2 wall plugs
- User manual + Warranty card

Should any parts be missing, kindly contact your specialist dealer or the manufacturer directly.

Function

A »signolux« light signal system consists of at least one transmitter (e.g. doorbell pushbutton) and a »signolux« receiver. Up to 8 transmitters can be assigned to each event icon.

If e.g. the doorbell pushbutton is pressed, a radio signal (radio impulses at 868,425 MHz) are transmitted to the »signolux« receiver. This one signals the received signals acoustically and visually with sounds and light signals.

Safety instructions

- Prior to putting the device into operation and using it, kindly read this manual thoroughly and in full.
- Store this operating manual in a safe place so that it is also accessible for other users at any time.
- Keep Signolux devices out of the reach of children! This packaging contains small parts which may present a choking hazard!

- Flashes of light can trigger epileptic seizures in people with photosensitive epilepsy (PSE).
- When using power tools always follow the instructions of the manufacturer and wear suitable protective equipment (e.g. protective goggles).
- Kindly check before starting to drill if there are any hidden electric cables and water pipes in the walls. In case of doubt we recommend using a cable detector.

Setup

Inserting the rechargeable batteries into the receiver

- a) At the rear side there is a battery compartment, its cover can be pushed downwards so that the compartment opens (fig. 1).

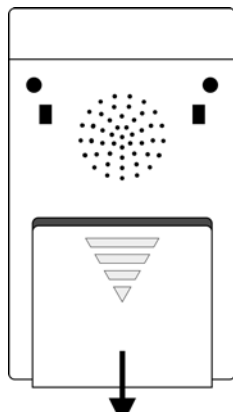
Figure 1: Battery compartment

- b) Now insert the 4 rechargeable batteries (included). Ensure correct polarity of the batteries.
- c) Close the battery cover again by inserting the cover and pushing it upwards.

d) Connect the power supply to the receiver. The time needed for a full charge is approximately 6 hours.

– switch to position 2:
no alert tones, flash only

Display and features

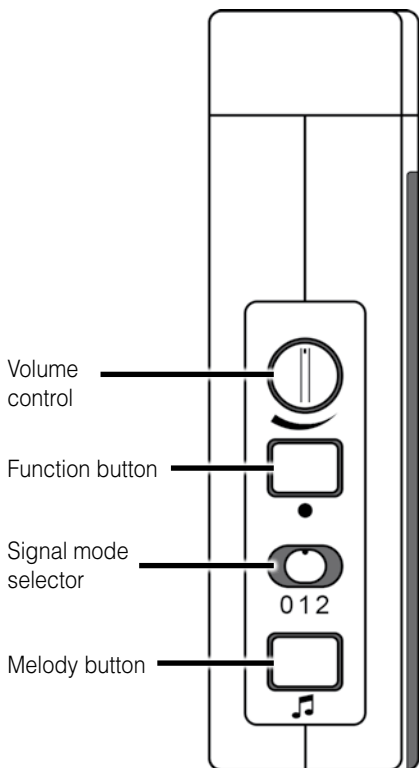
Figure 2: Controls, left side

Setting the signal mode

Using the signal mode slide switch (in fig. 2 "0 1 2") you can select the signal mode. Kindly keep in mind that the event icons are always displayed by coloured LEDs.

- switch to position 0:
alert tones, but no flash
- switch to position 1:
alert tones and flash

Figure 3: Event icons and status LED

Event icon signification

The icons stand for:

Doorbell 1	Colour: Green
Doorbell 2	Colour: Green
Telephone	Colour: Orange
Person call	Colour: Yellow
Baby	Colour: White
Fire alarm	Colour: Red
Flood alarm	Colour: Blue
Alarm	Colour: Red

Setting the volume

On the left side of the »signolux« receiver you will find the rotating volume control with which you can set the volume of the alert tone.

Warning:

This device can generate sounds of up to 90 dB!

In order to protect your hearing, please set a low volume to begin with and only then adjust to the desired volume.

Preset transmitter indication

Signalux devices can be used together immediately and without prior pairing when delivered.

Which event icon is displayed depends on the type of transmitter. This information can be found in the user manual of the according transmitter.

Optional manual pairing

Note 1:

If a transmitter is to be displayed with an event icon of your own choice, this can be done as described in the chapter *“Pairing transmitters and adapting them to the event icons”*.

However, this causes the receiver to lose the preset transmitter indication and all transmitters previously shown on the receiver must be individually paired.

Note 2:

Should there be repeated false alerts when using the preset transmitter indication, this may hint at a neighbouring Signalux installation causing interference. In this case, the problem can be solved by individually pairing all transmitter and receiver units within the setup (see *“Pairing transmitters and adapting them to the event icons”*).

Reset to factory defaults

Delete the pairing of each event icon (see *“Deleting transmitters”*).

When all previously assigned transmitters have been removed from the memory, the preset transmitter indication is restored and the transmitters are automatically displayed again.

Pairing transmitters and adapting them to the event icons

Up to 8 transmitters can be assigned to each icon. This assignment can be adapted by programming. Kindly keep in mind that a transmitter can only be programmed if it is not yet known in the memory of the »signalux« receiver. If required, you first might have to delete the transmitter from the memory (see *“Deleting transmitters”*).

- a) Keep the function button »●« (fig. 2) pressed for some time. The icons light up in sequence. Let go of the button as soon as you see the desired icon.
- b) Now trigger the alert using your transmitter (e.g. operate the doorbell push-button).
- c) Wait for about 20 seconds and test the function.

Setting the melody

You can adapt the melody for each icon. The signal mode switch needs to be in position 0 or 1.

- a) First trigger an alert (e.g. operate doorbell pushbutton).
- b) The icon assigned to the transmitter should light up and the previously chosen melody should be heard.
- c) Press and hold the melody setting button »♪« (fig. 2) for two seconds. Let go of the button and press it repeatedly until you have found the desired melody (you can choose from 16 melodies).
- d) Once the icon ceases to flash, the selected melody has been assigned.

Deleting transmitters

Do the following if you want to delete a transmitter from the memory:

- a) Keep the function button »●« (fig. 2) pressed for some time. The icons light up in sequence. Let go of the key as soon as you see the desired event icon.
- b) Now press the function button »●« and the melody setting button »♪« (fig. 2), at the same time, until all icons briefly light up.

Please note that this will remove all assigned transmitters from the selected event icon.

Battery level indication

The receiver informs about the state of the batteries of the receiver as well as the batteries of the transmitters:

- A sufficient battery level of the receiver is indicated by the red control lamp flashing every 5 to 6 seconds.
- A low battery level of the receiver is indicated by the red control lamp flashing in short sequence (once per second). Please recharge the receiver batteries as soon as possible.
- A low battery level of a transmitter is indicated by the red control lamp shining together with the event icon of the according transmitter when it's signal is triggered.

Please replace the batteries of the according transmitter as soon as possible.

Further functions

Recalling the last alert, cancelling the current alert

By pressing the »●« button (fig. 2) the last registered alert can be displayed again. You can also cancel an active displayed alert by pressing this button.

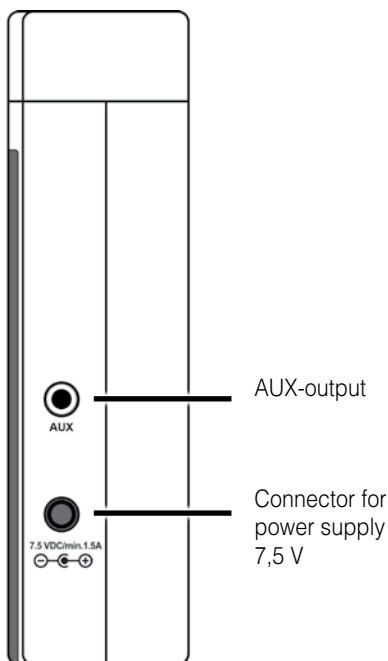
Figure 4: Lateral connectors

AUX-socket and power connector

The »AUX«-jack and the connector for the power adapter can be found on the right side of the receiver.

All Humantechnik modules can be connected to the AUX-output (vibrating pillow etc, available as accessories)

Mounting

Important note: Kindly keep in mind that metal parts and other building materials may interfere with the transmission of the radio signal. As they can be either visible or hidden at the desired mounting location you should always carry out a functional test at the mounting location before starting the mounting process, and, if required, choose a different mounting or installation location if the reception is not guaranteed.

Mounting the »signalux« receiver

The »signalux« receiver is suited either for free-standing installation or it can also be mounted using the supplied wall mount. To use it, fix the wall mount onto a wall (or rafter, cupboard etc.) When mounting onto a brick- or stone wall, you may need a power drill and stone bit to do so. Use the supplied wall plugs in this case. Once fixed, hang the »signalux« receiver onto the wall mount bracket.

Safety instructions for rechargeable batteries

Only use the specified battery type NiMH AA (HR6)!

There is a risk of explosion if the wrong type of battery is used. Batteries must not be exposed to high or low extreme temperatures and low air pressure during use, storage or transport at high altitude.

Disposing of batteries in fires, ovens or mechanically crushing and cutting them may cause fire or explosion. Exposing batteries to extremely high temperatures may cause an explosion or leakage of flammable liquid or gas.

Batteries exposed to extremely low air pressure may explode or emit flammable liquid or gas.

Use only the original power supply with the receiver.

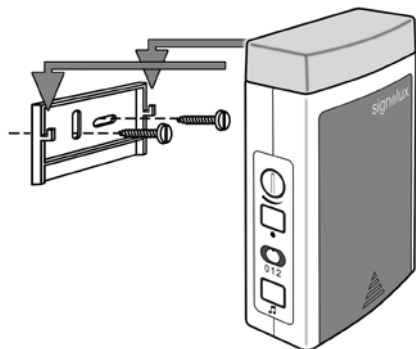
Figure 5: Wall mount

Maintenance and care

The »signolux« does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

Warranty

The »signolux« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of these products.

Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Battery disposal

Please dispose of used rechargeable batteries and batteries properly, according to your local regulations.

Troubleshooting and solutions


Symptoms	Cause	Solutions
When trying to program a transmitter, a different icon lights up...	The transmitter you are trying to program is already known by the receiver.	See step 7. Delete the transmitter from the receiver's memory before it can be assigned to a new icon.
The red status LED light up with an alert...	This indicates a low battery in the according transmitter.	Insert a new battery in the transmitter.
The »signolux« receiver does not work...	Ensure that the rechargeable batteries have been inserted correctly.	Ensure correct polarity of the rechargeable batteries!
	Check if the batteries are charged	Connect the receiver to the power supply.
	Maybe the triggered transmitter has not yet been paired with the »signolux« receiver.	Assign it to an icon.
	Perhaps the »signolux« receiver is out of reach of the transmitter.	Try using the transmitter or receiver at different location.
There is no alert tone...	mode selector switch is on position 2 (fig. 2)	Check the position of the switch (fig. 2) on the »signolux« receiver.
		Ensure that the volume is not set to the lowest setting.
The flash light does not work...	mode selector switch is on position 0 (fig. 2)	Check the position of the switch (fig. 2) on the »signolux« receiver.

Symptoms	Cause	Solutions
The range is reduced...	Metal structures and other building materials can reduce the reach.	If possible do not mount the »signolux« transmitter or receiver near or on metal parts.
	Other devices may interfere with the radio signals, thus reducing the radio signals and the function of the »signolux« system.	Avoid setting up the »signolux« devices close to mobile phones and display screens.
	Walls and ceilings reduce the range.	Reduce the distance between transmitters and receivers or try a different position of the devices.
	Low battery charge reduces the range.	Charge the batteries by connecting the power supply unit.

Specifications

Work range	0 to 40°C
Radio frequency	868,425 MHz
Sound volume	90 dB at 1 m
Rechargeable battery type	4 x AA NiMH 1,2 V (HR6)
Service time per charge (5 activations per day)	3 weeks
Power supply unit	7,5 V DC / 1,5 A
Dimensions (HxWxD)	145 x 90 x 40 mm
Weight	298 g

This device complies with the following EC directives:

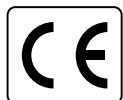- 2011 / 65 / EC + 2015 / 863 / EC RoHS-Directive
- 2012 / 19 / EC WEEE-Directive
- 2014 / 53 / EC RED-Directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at www.humantechnik.com.

Humantechnik's UK authorised representative:
Sarabec Ltd.
15 High Force Road
MIDDLESBROUGH TS2 1RH
United Kingdom

Sarabec Ltd., hereby declares that this device complies with all UK statutory instruments. UK declaration of conformity available from: Sarabec Ltd.

Technical specifications subject to change without prior notice.

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