HUMANTECHNIK

signolux

DE

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Seite 2

EN

**User manual
Receiver T**

Page 12

FR

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Page 22

NL

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Pagina 32

IT

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Pagina 42

ES

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We would like to congratulate you on purchasing the modern and reliable »signolux« alerting system. Please read the user manual carefully for the proper use of the device and so that you can familiarise yourself with all the possibilities this system offers.

Scope of Delivery

Kindly check if all following parts are included:

- »signolux«-Receiver T
- 4 rechargeable batteries size AAA
- Power supply unit
- User manual
- Warranty card

Should any parts be missing, kindly contact your specialist dealer or the manufacturer directly.

How it works

A »signolux« alerting system consists of at least one transmitter (e.g. doorbell pushbutton) and a receiver. Up to 8 transmitters can be assigned to each event icon.

If e.g. the doorbell pushbutton is pressed, a radio signal is transmitted to the receiver. This, in turn, displays the received signal audibly and visually with sounds and flashing lights.

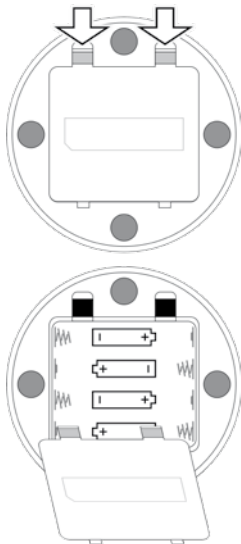
Safety instructions

- Prior to putting the device into operation and using it, please read this manual thoroughly.
- Keep this manual in a safe place so that it is also accessible for other users at any time.
- Keep Signolux devices out of the reach of children! This packaging contains small parts which may present a choking hazard!
- Flashes of light can trigger epileptic seizures in people with photosensitive epilepsy (PSE).

Setup

Inserting the rechargeable batteries

The battery compartment is located on the underside. Press the latches of the battery compartment cover slightly towards the center of the bottom of the device and lift the cover off.

Now insert the 4 supplied rechargeable batteries. Make sure they are inserted in the correct direction according to the markings.

Close the battery compartment by replacing the cover and pressing it shut.

Connect the power supply unit. The charging time is approx. 6 hours.

Display and features

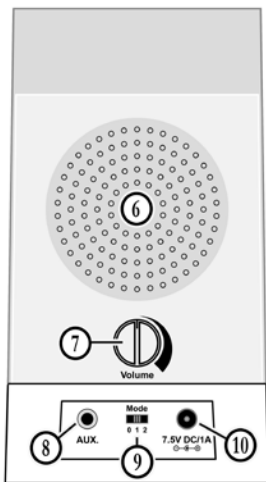
1. Flashing zone
2. Event icons
3. Function button
4. Status LED
5. Melody button

Function

A »signalux« alerting system consists of at least one transmitter (e.g. doorbell pushbutton) and a receiver. Up to 8 transmitters can be assigned to each event icon.

If e.g. the doorbell pushbutton is pressed, a radio signal is transmitted to the receiver. This, in turn, displays the received signal audibly and visually with sounds and flashing lights.

6. Speaker
7. Volume control
8. AUX output for modules
9. Mode selector switch
10. Connector for 7,5 V power supply unit

Setting the alerting mode

Using the mode selector switch (9) you can select the alerting mode.

Please keep in mind that the event icons are always indicated.

- switch to position 0:
alerting tones, but no flash
- switch to position 1:
alerting tones and flash
- switch to position 2:
no alerting tones, flash only

Setting the volume

Use the rotating volume control to set the volume of the alerting tone.

Warning:

This device can generate sounds of up to 90 dB! In order to protect your hearing, please set a low volume to begin with and only then adjust it to the desired volume.

Event icons

Each event icon can be linked to a transmitter (e.g. doorbell pushbutton) and can be re-assigned at will.

The icons symbolize:

	Doorbell 1	Colour: Green
	Doorbell 2	Colour: Green
	Telephone	Colour: Orange
	Person call	Colour: Yellow
	Baby	Colour: White
	Fire alarm	Colour: Red
	Flood alarm	Colour: Blue
	Alarm	Colour: Red

Preset transmitter indication

Signolux devices can be used together immediately and without prior pairing when delivered.

Which event icon is displayed depends on the type of transmitter. This information can be found in the user manual of the according transmitter.

Optional manual pairing

Note 1:

If a transmitter is to be displayed with an event icon of your own choice, this can be done as described in the chapter „*Linking transmitter alerts with event icons*“.

However, this causes the receiver to lose the preset transmitter indication and all transmitters previously shown on the receiver must be individually paired.

Note 2:

Should there be repeated false alerts when using the preset transmitter indication, this may hint at a neighbouring Signolux installation causing interference. In this case, the problem can be solved by individually pairing all transmitter and receiver units within the setup (see „*Linking transmitter alerts with event icons*“).

Reset to factory defaults

Delete the pairing of each event icon (see „*Deleting transmitters*“).

When all previously assigned transmitters have been removed from the memory of the receiver, the preset transmitter indication is restored and the transmitters are automatically displayed.

Linking transmitter alerts with event icons

Up to 8 transmitters can be assigned to each event icon. This is achieved by pairing a transmitter with an icon of your choice. Please keep in mind that a transmitter can only be paired if it is not already stored in the memory of the receiver. If required, you might first have to delete the transmitter from the memory (see „*Deleting transmitters*“).

- Keep the function button »●« (3) pressed for some time. The event icons start to light up in sequence. Let go of the button as soon as the desired icon is lit.

- b) Trigger your transmitter (e.g. operate the doorbell pushbutton or test button).
- c) Wait for about 20 seconds, then test the function by triggering the transmitter again.

Setting the melody

You can freely set every provided tone to an event icon of your choice. The mode selector switch must be either set to 0 or 1.

- a) First trigger an alert (e.g. operate doorbell pushbutton).
- b) The icon assigned to the transmitter lights up and the previously chosen tone should be heard.
- c) Press and hold the melody setting button »🎵« (5) for two seconds. Let go of the button and press it repeatedly until you have found the desired melody (you can choose from 16 different tones).
- d) Once the icon ceases to flash, the selected tone has been set.

Deleting transmitters

Do the following if you wish to delete a transmitter from the device memory:

- a) Keep the function button »●« (3) pressed for some time. The event icons start to light up in sequence. Let go of the button as soon as the icon in question is lit.
- b) Now press the function button »●« and the melody button »🎵« (3)+(5) at the same time. All icons briefly light up and the transmitter is deleted from that icon.

Please note:

This procedure will delete **all** transmitters assigned to this event icon.

Status-LED

The receiver informs about the state of its own batteries as well as the batteries of the linked transmitters:

- A sufficient battery charge level of the receiver is indicated by the status LED flashing every 5 to 6 seconds in red.
- A low battery level of the receiver is indicated by the status LED flashing every two seconds in red. Please recharge the receiver as soon as possible (attach it to the power supply unit).
- If the battery level of a transmitter is low, the status LED lights up red every 2 seconds after the transmitter has transmitted its signal. Please replace the batteries of the according transmitter at the next possible opportunity.

Behaviour of the status LED when the power supply unit is connected:

Charging the battery: The status indicator lights up red. The red light is interrupted for half a second every 6 seconds.

Battery is charged and the unit is ready for operation: The status indicator lights up green. Every 6 seconds, the status indicator briefly lights up in red.

Use on rechargeable battery

Mains operation is recommended. The rechargeable batteries are primarily intended to power the unit in the event of a power failure.

The maximum operating time per charge is 3 weeks, depending on the frequency of incoming events.

The receiver should therefore only be disconnected from the power supply for a few days.

Recalling the last alert, cancelling the current alert

Press the »●« button to display the last registered alert again.

You can also cancel an active indicated alert by pressing this button.

AUX-socket and power connector

The »AUX«-socket (8) and the connector for the power supply (10) can be found on the rear of the receiver.

All Humantechnik modules can be connected to the AUX-output (vibrating pad, flash module, vibrating contraption etc, available as accessories). An indicated event will be forwarded to the connected modules which will then react accordingly.

The power connector is used to power the receiver via the power supply unit. This is used to recharge the batteries, but it can remain connected for continuous operation of the receiver.

Safety instructions for rechargeable batteries

Only use the specified battery type NiMH AAA (HR03)!

There is a risk of explosion if the wrong type of battery is used. Batteries must not be exposed to high or low extreme temperatures and low air pressure during use, storage or transport at high altitude.

Disposing of batteries in fires, ovens or mechanically crushing and cutting them may cause fire or explosion. Exposing batteries to extremely high temperatures may cause an explosion or leakage of flammable liquid or gas.

Batteries exposed to extremely low air pressure may explode or emit flammable liquid or gas.

Use only the original power supply with the receiver.

Maintenance and care

»signolux«-devices do not require any maintenance. If a unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

Warranty

»signolux« devices are very reliable products. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of these products.

Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your local disposal company or your local dealer.

Battery disposal

Please dispose of used rechargeable batteries and batteries properly, according to your local regulations.

Troubleshooting and solutions

Symptom	Cause	Solution
When trying to program a transmitter, a different icon lights up.	The transmitter you are trying to link has been already linked with the receiver.	Delete the transmitter from the unwanted icon before it can be assigned to a new icon.
The red status LED lights up with an alert.	This indicates a low battery in the according transmitter.	Insert a new battery in the transmitter.
The receiver does not work.	The batteries may be inserted incorrectly.	Ensure the correct polarity of the rechargeable batteries!
	The batteries are depleted.	Connect the receiver to the power supply.
	The triggered transmitter has not yet been paired with the »signalux« receiver.	Follow the linking procedure to assign it to an icon.
	The receiver is out of transmission range.	Try using the transmitter or receiver at a different location.
There is no alert tone.	Mode selector switch is on position 2	Check the position of the switch (9) on the rear of the receiver.
		Ensure that the volume is not set to the lowest setting. Rotate the control clockwise to turn up the volume
The flash light does not work...	Mode selector switch is on position 0	Check the position of the switch (9) on the rear of the receiver.

Symptoms	Cause	Solutions
The transmission range is reduced.	Metal structures and other building materials can reduce the transmission range.	If possible do not mount the transmitter or receiver near or on metal parts.
	Interference from other devices	Avoid setting up the »signolux« devices close to mobile phones and display screens.
	Walls and ceilings reduce the range.	Reduce the distance between transmitters and receivers or try a different position for the devices.
	Low battery charge reduces the range.	Charge the batteries by connecting the power supply unit.

Specifications

Work range	0 to 40°C
Radio frequency	868,425 MHz
Sound volume	90 dB at 1 m
Rechargeable battery type	4 x AAA NiMH 1,2 V HR03
Service time per charge (5 activations per day)	max. 3 weeks
Power supply unit	7,5 V DC / 1 A
Dimensions (H x Ø)	143 x 80 mm
Weight	250 g

This device complies with the following EC directives:

- 2011 / 65 / EC + 2015 / 863 / EC RoHS-Directive
- 2012 / 19 / EC WEEE-Directive
- 2014 / 53 / EC RED-Directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at www.humanteknik.com.

Humanteknik's UK authorised representative:
Sarabec Ltd.
15 High Force Road
MIDDLESBROUGH TS2 1RH
United Kingdom

Sarabec Ltd., hereby declares that this device complies with all UK statutory instruments.
UK declaration of conformity available from: Sarabec Ltd.

Technical specifications subject to change without prior notice.

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