EN

User manual

Page 2



Vibrating pager

FR

Mode d'emploi

Page 10

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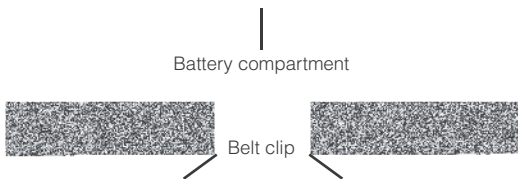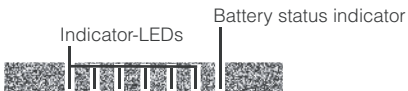Function button

Delete button

On/off switch

Socket for vibrating pillow and modules

Charging contacts

Folding stand

Congratulations on purchasing your »sign●lux pager«. We hope it will give you many years of reassuring operation. You have purchased a modern and reliable system. Please read these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features.

Package contents

Please check if all of the following components are included:

- »sign●lux-pager«
- carrying cord
- 2 pcs 1,5 V size AAA batteries
- manual
- warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

Function

The portable »sign●lux-pager« is a receiver for the signals of the various »sign●lux« transmitters and indicates these as vibration impulses.

6 indicator LEDs with symbols help to distinguish the signal source.

Signals and controls

Indicator-LEDs:

The 6 indicator-LEDs help to distinguish between the various signal sources. They light up for a total of 40 seconds when the according signal is received.

Battery status indicator:

The receiver informs about the state of the batteries of the receiver as well as the batteries of the transmitters:

- A sufficient battery level of the receiver is indicated by the red control lamp flashing every 5 to 6 seconds.
- A low battery level of the receiver is indicated by the red control lamp flashing in short sequence (once per second).

Please replace (or where applicable, recharge) the receiver batteries as soon as possible.

- A low battery level of a transmitter is indicated by the red control lamp shining together with the symbol of the according transmitter when it's signal is triggered.

Please replace (or where applicable, recharge) the batteries of the according transmitter as soon as possible.

Function button:

When you feel the »sign●lux-pager« vibrating, you can stop the vibration by pressing the function button on the receiver.

The indicator LEDs will stay illuminated, but will go out automatically after a total of 40 seconds.

Furthermore, you can press the function button to call up the most recent signal again (the function works for up to 3 minutes after receiving the signal). The function button is also used to program the receiver (see section 3 „Programming signals“).

Belt clip:

You can use the belt clip to attach the »sign●lux-pager« to your clothing.

On/off switch:

Use this switch to turn the pager on and off.

Delete button:

This button serves to remove transmitters assigned to an indicator symbol (see section 4 „Deleting transmitters from receiver memory“).

Setting up the system

1. Inserting the battery

There is a battery compartment on the underside of the »sign●lux-pager«. Open it by sliding the cover of the battery compartment away from the receiver using both your thumbs.

Insert two 1.5 V AAA batteries and close the receiver again.

2. Switching the unit on

Set the on/off switch to »on«. The »sign●lux-pager« is now ready to operate. After the unit has been switched on, the battery status LED lights up for about a second.

3. Programming signals and assignment to the symbols

A transmitter (e.g. doorbell push-button) can be assigned to each symbol. This assignment can be adapted by programming. Up to 6 transmitters can be assigned to each symbol. Please keep in mind that a transmitter can only be assigned if it is not yet known in the memory of the »sign●lux-pager«. If required,

you might first have to delete the transmitter from the memory (see section 4.)

- a) Keep the function button pressed for some time. The symbols begin to light up in sequence. Let go of the function button as soon as the desired symbol is lit. (e. g. doorbell symbol)
- b) Trigger the signal of the transmitter you want to assign to the lit up symbol. (e.g. operate the doorbell button).
- c) Wait for about 20 seconds and test the function.

4. Deleting transmitters from receiver memory

If you wish to remove the transmitters assigned to a symbol from the receiver memory:

- a) Keep the function button pressed for some time. The symbols begin to light up in sequence. Let go of the function button as soon as the desired symbol is lit.

- b) Now press the function button and the delete button at the same time. All symbols will briefly light up.

Please note that this will delete all transmitters assigned to the according symbol.

Meaning of the symbols

	doorbell	(green)
	telephone	(orange)
	person call	(yellow)
	babycry	(white)
	fire alarm	(red)
	alarm	(red)

Checking the last signal, deleting the current signal

Shortly pressing the function button, the last registered signal can be displayed again.

You can also “delete” a signal that was just activated by pressing this button.

Signal distinction

The various transmitter sources can be recognized by the different vibration patterns which are unique to each symbol:

Doorbell: Doorbell symbol (LED) is lit up in green

Telephone: Telephone symbol (LED) is lit up in orange

Person call: Person call symbol (LED) is lit up in yellow

Babycry: Babycry symbol (LED) is lit up in white

Fire alarm: Fire symbol (LED) is lit up in red

Alarm: Alarm symbol (LED) is lit up in red

Special accessories

The following modules (not included) can be plugged into to the AUX output connector:

- Vibrating pillow (A-3303-0)
- Flash module MF-1 (A-3320-0)
- Switch module MS-1* (A-3330-0)
- Acoustic module MA-1 (A-3340-0)

*Only for use with Type C (CEE 7/16 Europlug) and Type F (CEE 7/4 „Schuko“) outlet connectors

Transmission range

The signal range of the »sign●lux« transmitters can be up to 200 metres under ideal conditions. The following conditions may result in a reduced range:

- buildings or vegetation.
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the si-

gnal wave being weakened or even cancelled out by the reflected wave.

- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

Maintenance and care

All »sign●lux« devices do not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth.

Never use spirits, thinners or other organic solvents. Do not expose the devices to full sunlight for long periods.

In addition, they must be protected against excessive heat, moisture and severe mechanical shocks.

Note:

This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or

anything with an open flame, such as a lit candle, above or near the product.

Please make sure that the batteries are not exposed to excessive heat, such as sunlight, fire or anything similar.

Warranty

The »sign●lux-pager« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by unauthorised persons (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product serial number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products.

Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

This device complies to the following IC standards:


47 CFR Part 15; 2013
RSS-210 ISSUE 8

Technical data

Power supply:	2 pcs 1.5 V AAA battery or 1.2 V rechargeable battery
Battery endurance:	approx. 800 h with 1.5 V alkaline battery approx. 400 h with 1.5 V zinc carbon battery approx. 300 h with 1.2 V rechargeable battery
Frequency:	418 MHz or 915 MHz
Vibration:	Eccentric motor, different vibration sequences to differentiate between signals
Height:	85 mm
Width:	60 mm
Depth:	28 mm
Weight:	approx. 88 g
Operating range:	0° C – 40° C

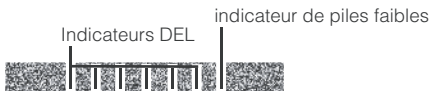Technical specifications subject to change without prior notice.

Logement de piles

This diagram shows a top-down view of the battery compartment. It consists of two dark, textured rectangular sections separated by a central gap. A vertical line above the gap is labeled 'Logement de piles'. A diagonal line from the bottom center points to the gap, which is labeled 'Clip ceinture'.

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This diagram shows a top-down view of a function key. It is a simple horizontal line with a short vertical line extending upwards from its right end.

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This diagram shows a top-down view of a power button. It is a simple horizontal line with a short vertical line extending upwards from its right end.

Contacts de chargement

This diagram shows a top-down view of charging contacts. It consists of two horizontal lines of equal length, with a short vertical line extending upwards from the center between them.

Bouton d'annulation

This diagram shows a top-down view of a cancel button. It is a simple horizontal line.

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This diagram shows a top-down view of a connector port. It is a simple horizontal line.

Pied escamotable

This diagram shows a top-down view of a retractable foot. It consists of two diagonal lines meeting at a central point, forming a V-shape.

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