DE

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Seite 2

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EN

**User manual**

Page 10

*signolux* receiver alarm clock

FR

**Mode d'emploi**

Page 18

Réveil *signolux*

NL

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Pagina 26

Digitale Flitswekker *signolux*

IT

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Pagina 34

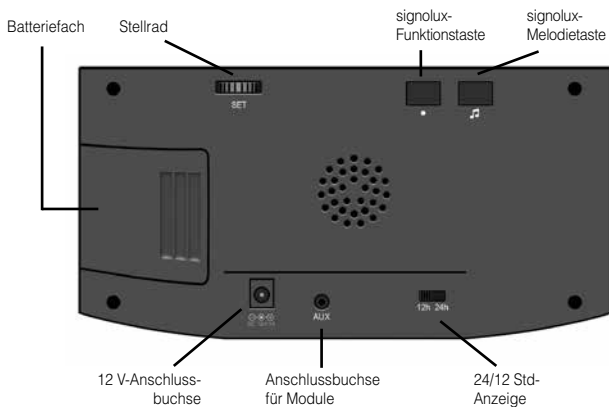
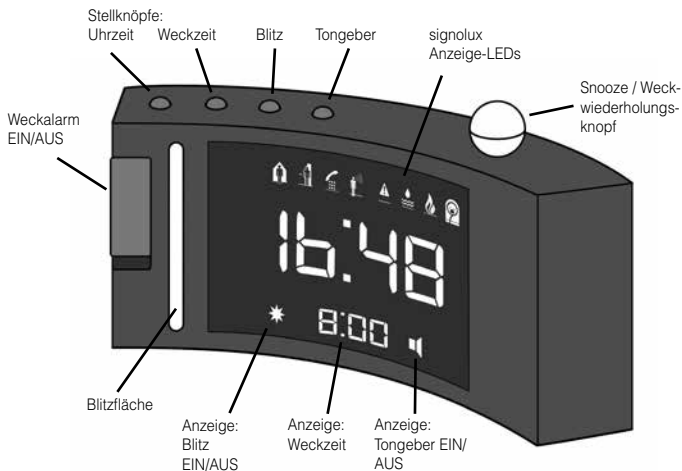
Sveglia digitale *signolux*

ES

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Página 42

Despertador receptor *signolux*

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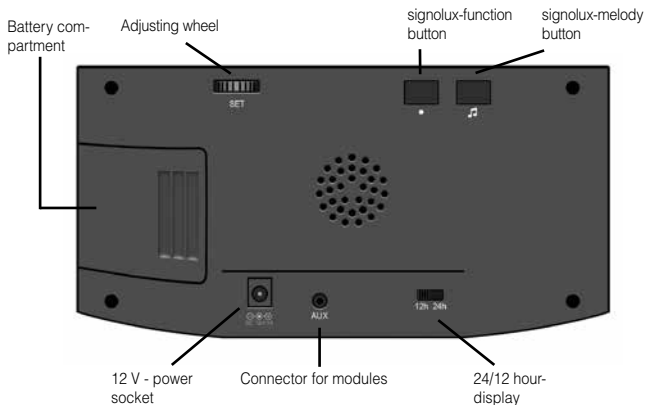
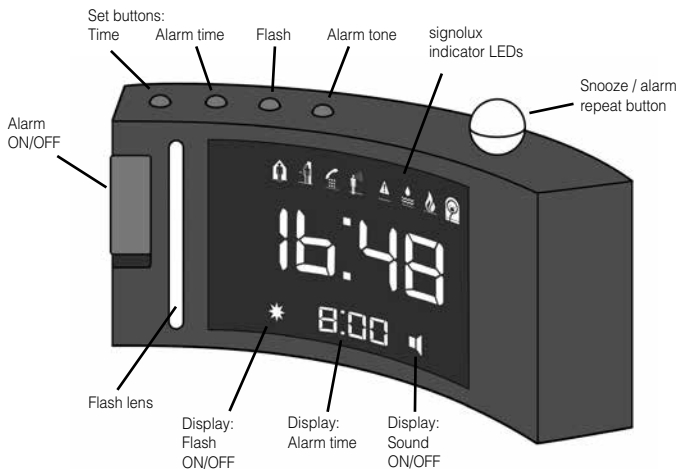
We would like to congratulate you on purchasing the »*signolux* alarm clock«. You have chosen a modern and reliable system. Kindly read the instruction manual carefully in order to use the device properly and to familiarise yourself with all the possibilities of the system.

### Package contents

Kindly check if all following parts are included:

- »*signolux* alarm clock«
- 12 volts power supply unit
- Instruction manual
- Warranty card

Should any parts be missing, kindly contact your specialist dealer or the manufacturer directly.

### Functioning principle

The »*signolux* alarm clock« is an electronic clock with a digital time display. It wakes you up with an acoustic signal which is adjustable in pitch and volume. It is also a complete *signolux* receiver, capable of displaying the alerts of all *signolux* transmitters. A received alert is displayed by flash and/or sound and can be identified by 8 illuminated event icons.

The following accessories can be connected to the »AUX« connector of the alarm clock:

- |                         |          |
|-------------------------|----------|
| - Vibrating pillow      | A-3303-0 |
| - Vibrating contraption | A-3310-0 |
| - Flash module          | A-3320-0 |
| - Acoustic module       | A-3340-0 |

## Setup

Connect the power supply unit to the 12 V power socket of the alarm clock and plug it into a 230 V power outlet. After a few seconds, the alarm clock is ready to receive event alerts.

### Setting the time

To set the time, press the time set button »« for one second. The hour digits will start to flash. Set the desired hour by turning the adjusting wheel (on the reverse side of the of the clock) to the left or right.

Press the time set button »« again. The minute digits will start to flash. Turn the adjusting wheel to the left or right to set the desired minutes.

Pressing the time set button »« again will end the time setting procedure and the set time is displayed.

### Setting the alarm time

To set the alarm time, press the alarm time button »« for one second. The hour digits of the alarm time will start to flash. Set the desired hour by turning the adjusting wheel to the left or right.

Press the alarm time button »« again. The minute digits of the alarm time will start to flash. Turn the adjusting wheel to the left or right to set the desired minutes.

Pressing the alarm time button »« again will end the alarm time setting procedure and the set alarm time is displayed.

## Alarm ON / OFF

The waking alarm can be switched on and off with the slider switch on the left side of the alarm clock.

Slider in upper position (green area is visible): Alarm is switched ON.

Slider in lower position: Alarm is switched OFF.

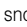
## Alarm tone ON / OFF

Press the alarm tone button »« for one second to switch the alarm tone ON or OFF. If the alarm tone is on, the speaker symbol will be visible in the display.

## Setting the alarm tone volume

Press and hold the alarm tone button »« while adjusting the volume of the alarm tone. After 2 seconds, the alarm tone can be heard. Keep the alarm tone button pressed and turn the adjusting wheel to the left or right to adjust the desired alarm tone volume. Release the alarm tone button to end the volume setting procedure.

## Setting the alarm tone pitch

While adjusting the alarm tone pitch, press and hold both the alarm tone button »« and the spherical snooze button at the same time. After 2 seconds, the alarm tone can be heard. Keep the alarm tone button and the snooze button pressed and turn the adjusting wheel to the left or right to adjust the desired alarm tone pitch.

Release the buttons to end the pitch setting procedure.

## Flash ON / OFF

Press and hold the flash button »« for one second to switch the flash ON or OFF. If the flash is on, the flash symbol will be visible in the display.

## Setting the nighttime display brightness

The brightness of the displayed digits is automatically adjusted according to the light conditions within the room. During the day, the digits are displayed with full brightness. In darkness, the set nighttime display brightness is activated.

To set the nighttime display brightness, press and hold the flash button »«. Keep the flash button pressed and turn the adjusting wheel to the left or right to adjust the desired display brightness for the night. Release the button to end the setting procedure.

If this adjustment is done during daylight conditions, the display will switch back to full brightness after a few seconds.

## Deactivating the illuminated digits

If the illuminated digits are too bright for you at night, even at their darkest setting, they can also be deactivated completely. This has no influence on the indication of events!

Digits off: Press and hold the buttons »« and »« simultaneously for 5 seconds. For confirmation, the digits flash 2 times and then switch off after 60 seconds.

Digits on: Press and hold the buttons »« and »« simultaneously for 5 seconds. For confirmation, the digits flash 4 times and then switch on again immediately.

The luminous digits can be activated for 60 seconds at any time by pressing any key.

### **Snooze function**

You can interrupt a current alarm by pressing on the spherical snooze button. The alarm will be repeated after 5 minutes.

### **12 / 24 hour mode**

The time can be displayed in a 12- or 24 hour mode.

The adjustment is made with the »**24h/12h**« switch on the reverse side of the alarm clock.

### **Power cut backup**

The battery compartment can hold 4 batteries of size AAA (not included). In the event of a power cut, these guarantee the continuation of the device settings, the time and the display of the *signalux* alerts.

**Please note:** During a power cut, the digits and the flash are not active as the power consumption is too high for the backup batteries!

## **Preset transmitter indication**

Signalux devices can be used together immediately and without prior pairing when delivered.

Which event icon is displayed depends on the type of transmitter. This information can be found in the user manual of the according transmitter.

## **Optional manual pairing**

### **Note 1:**

If a transmitter is to be displayed with an event icon of your own choice, this can be done as described in the chapter *Pairing transmitters and adapting them to the event icons*.

**However, this causes the alarm clock to lose the preset transmitter indication and all transmitters previously shown on the alarm clock must be individually paired.**

### **Note 2:**

Should there be repeated false alerts when using the preset transmitter indication, this may hint at a neighbouring *Signalux* installation causing interference. In this case, the problem can be solved by individually pairing all transmitter and receiver units within the setup (see *Pairing transmitters and adapting them to the event icons*).

## Reset to factory defaults

Delete the pairing of each event icon (see *Deleting transmitters from the receiver memory*).

When all previously assigned transmitters have been removed from the memory, the preset transmitter indication is restored and the transmitters are automatically displayed.

## Pairing transmitters and adapting them to the event icons:

Up to 8 transmitters (e.g. the doorbell push-button) can be assigned to each event icon.

This assignment can be adapted by pairing.

Kindly keep in mind that a transmitter can only be paired if it is not yet known in the memory of the *signolux* receiver. If required, you might first have to delete the transmitter from the memory.

a) Keep the function button »●« pressed for some time. The event icons light up in sequence. Release the button as soon as you see the desired icon.

b) Now trigger the signal using your transmitter (e.g. operate the doorbell pushbutton).

c) Wait for about 20 seconds and test the function.

## Deleting transmitters from the receiver memory

Do the following if you want to remove transmitter from the memory:

a) Keep the function button »●« pressed for some time. The event icons light up in sequence. Let go of the button as soon as you see the desired icon.

b) Now press the function button »●« and the melody button »🎵« until all icons briefly light up.

**Please note that this will remove all assigned transmitters from the selected event icon.**

## Setting the melody

You can choose the melody / sound effect for each event icon:

- First trigger a signal (e.g. operate the doorbell pushbutton).
- The icon assigned to the transmitter should light up and the previously chosen melody should be heard.
- Press and hold the melody setting button for 2 seconds. The icon starts to flash.
- Repeatedly press the melody setting button »« until you have found the desired melody / sound effect (you can choose from 15 different sounds).
- Once the icon ceases to flash, the selected melody has been assigned.

## Event icon description

The event icons stand for:

	Doorbell 1	green light
	Doorbell 2	green light
	Telephone	orange light
	Person call	yellow light
	Baby	white light
	Fire alarm	red light
	Water alarm	blue light
	Alarm	red light

## Signal distinction

\* only with attached vibrating pad

Doorbell 1		20 sec
LED		10 sec
vibration*		1 sec
Doorbell 2		20 sec
LED		10 sec
vibration*		2.0
Telephone		20 sec
LED		10 sec
vibration*		1.5
Person call		20 sec
LED		10 sec
vibration*		1.5
Alarm		20 sec
LED		10 sec
vibration*		3.5
Water alarm		20 sec
LED		10 sec
vibration*		1.0
Fire alarm		20 sec
LED		10 sec
vibration*		2.0
Baby		20 sec
LED		10 sec
vibration*		2.0

## Transmission range

The signal range of the *signolux* transmitters can be as much as 200 metres under ideal conditions. The following conditions may result in a reduced range:

- Buildings or vegetation.
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter or receiver at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

## Maintenance and care

The »*signolux* receiver alarm clock« does not require any maintenance. If it becomes dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not expose it to full sunlight.

In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

**Note:** This product is not protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, above or near the product.

## Warranty

The »*signolux* receiver alarm clock« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by unauthorised persons (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product serial number in any event.**

**ATTENTION:**

Use only commercially available batteries of type AAA (LR03, 1.5 V) for the power cut backup.

There is a risk of explosion if the wrong type of battery is used. The batteries must not be exposed to high or low extreme temperatures and low atmospheric pressure during use, storage or transport at high altitude.

Disposing of batteries in fires, ovens or mechanically crushing and cutting them can cause fires or explosions. Exposing batteries to extremely high temperatures may cause an explosion or leakage of flammable liquid or gas.

Batteries exposed to extremely low air pressure may explode or emit flammable liquid or gas.

Dispose of used batteries according to the battery manufacturer's instructions.

Use only the supplied mains adapter with the unit. If there is a need for a replacement power supply unit, please contact the shop where you purchased the product or a member of the Humantechnik Group directly.

**Disposal** of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units. You protect the environment and health of your fellow man by the correct disposal of this product.

Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, our communal disposal company or your local dealer.

### Technical data

Power supply:	DC12V /1000 mA
Power consumption:	approx. 2 W
Receiver frequency:	868,425 MHz
Dimensions:	212 mm x 103 mm x 70 mm
Weight:	337 g

Hereby, Sarabec Ltd declares that this device complies with all relevant UK Statutory Instruments.

The full declarations of conformity for this device are available from Sarabec Ltd., 15 High Force Road, MIDDLESBROUGH TS2 1RH. UK.

Sarabec's EU authorised representative:

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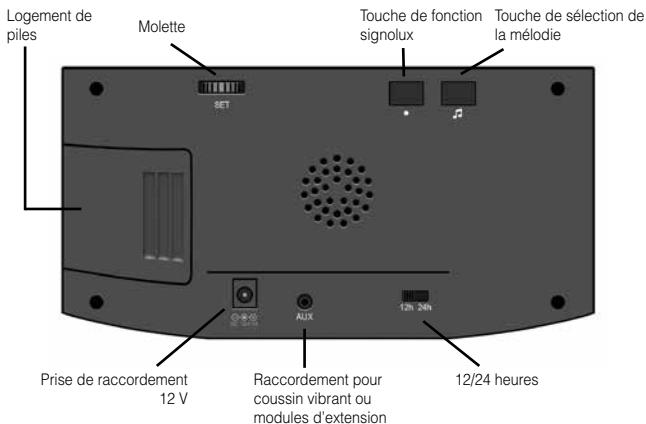
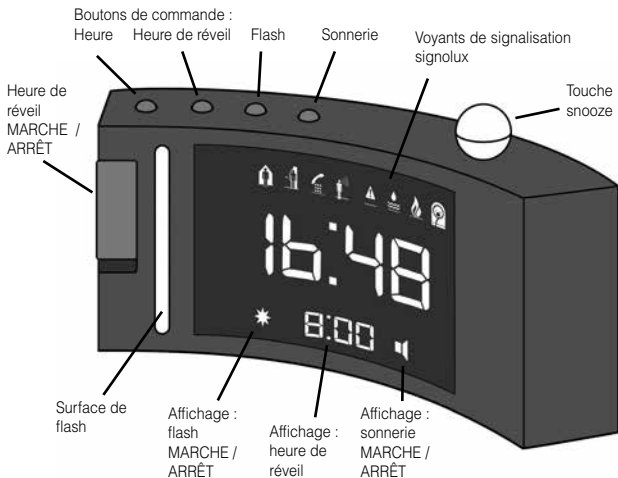
This device complies with the following EC directives:

- 2011 / 65 / EC + 2015 / 863 / EC RoHS directive
- 2012 / 19 / EC WEEE directive
- 2014 / 53 / EC RED directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at [www.humantechnik.com/service](http://www.humantechnik.com/service).

Technical specifications subject to change without prior notice.

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Campanello 1	 <small>The recorded sound is played back once.</small>
LED	 <small>The recorded sound is played back once.</small>
vibrazione*	 <small>ripetizione</small> acceso spento
Campanello 2	 <small>The recorded sound is played back once.</small>
LED	 <small>The recorded sound is played back once.</small>
vibrazione*	 <small>ripetizione</small> acceso spento
Telefono	 <small>The recorded sound is played back once.</small>
LED	 <small>The recorded sound is played back once.</small>
vibrazione*	 <small>ripetizione</small> acceso spento
Chiamapersona	 <small>The recorded sound is played back once.</small>
LED	 <small>The recorded sound is played back once.</small>
vibrazione*	 <small>ripetizione</small> acceso spento
Allarme	 <small>The recorded sound is played back once.</small>
LED	 <small>The recorded sound is played back once.</small>
vibrazione*	 <small>ripetizione</small> acceso spento
Allarme acqua	 <small>The recorded sound is played back once.</small>
LED	 <small>The recorded sound is played back once.</small>
vibrazione*	 <small>ripetizione</small> acceso spento
Allarme incendio	 <small>The recorded sound is played back once.</small>
LED	 <small>The recorded sound is played back once.</small>
vibrazione*	 <small>ripetizione</small> acceso spento
Baby	 <small>The recorded sound is played back once.</small>
LED	 <small>The recorded sound is played back once.</small>
vibrazione*	 <small>ripetizione</small> acceso spento

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**Humantechnik GmbH**Im Wörth 25
**Humantechnik GHL AG**Rastatterstrasse 9
**Sarabec Ltd**15 High Force Road