HUMANTECHNIK

signolux

**DE**

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Seite 2

Blitzlampe

**EN**

**User manual**

Page 10

Plug-In receiver

**FR**

**Mode d'emploi**

Page 18

Lampe-Flash

**NL**

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Pagina 26

Flitslamp

**IT**

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Pagina 34


Ricevitore radio portatile

**ES**

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Página 42

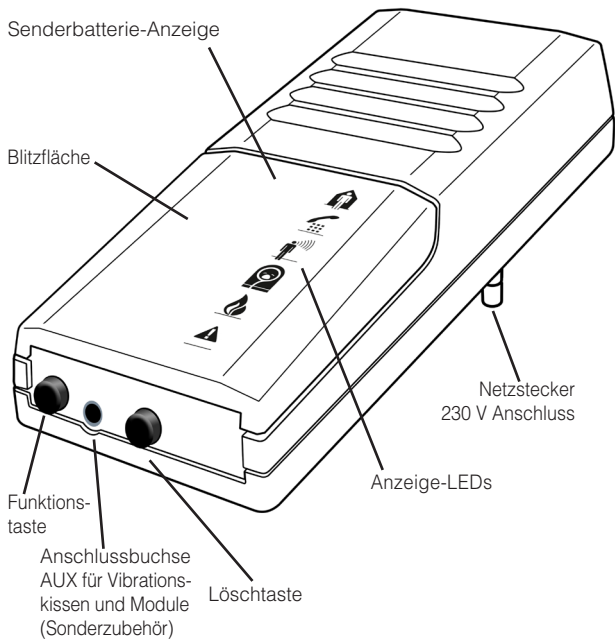Radorreceptor portátil

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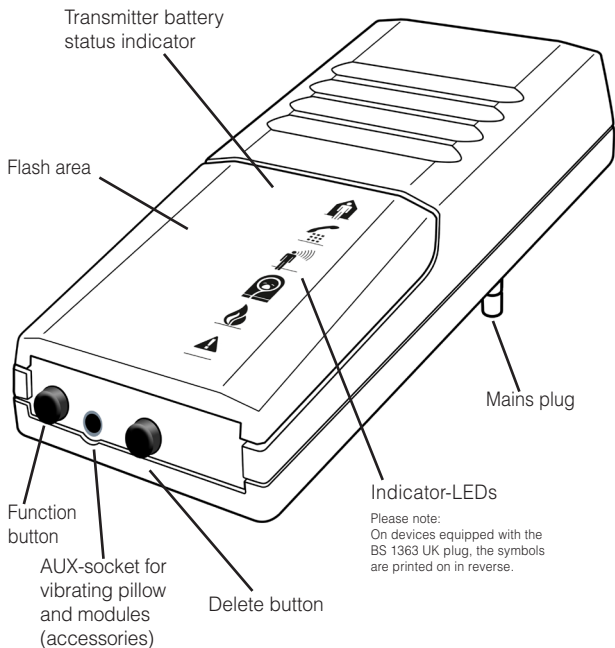
Congratulations on purchasing your »*signolux* plug-in receiver«. We hope it will give you many years of reassuring operation.

You have purchased a modern and reliable system.

Please read these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features.

### **Package contents**

Please check if all of the following components are included:

- »*signolux* plug-in receiver«
- manual
- warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

### **Function**

The »*signolux* plug-in receiver« is a receiver for the signals of the various »*signolux*« transmitters and indicates these as visible flashing light signals.

6 indicator LEDs with symbols help to distinguish the signal source.

## **Signals and controls**

### **Indicator-LEDs:**

The 6 indicator-LEDs help to distinguish between the various signal sources. They light up for 40 seconds after an incoming signal.

### **Transmitter battery status indicator:**

The receiver informs about the state of the batteries of the transmitters:

- A low battery level of a transmitter is indicated by the red control lamp shining together with the symbol of the according transmitter when its signal is triggered.

Please replace the batteries of the according transmitter as soon as possible.

### **Function button:**

When the »*signolux* plug-in receiver« is flashing, you can cancel the signal by pressing the function button.

Press it again to recall the last received signal.

The function button is also used to program the receiver (see section 2 „Programming signals“).

### **Delete button:**

This button serves to remove transmitters assigned to an indicator symbol (see section 3 „Deleting transmitters from receiver memory“).

## **Setting up the system**

### **1. Connect the device**

Plug the »*signolux* plug-in receiver« into a well visible mains socket.

The indicator LEDs light up once and the device is ready to use.

The »*signolux* plug-in receiver« is not waterproof. Therefore it is not advisable to use it in the open.

### **2. Programming signals and assignment to the symbols**

A transmitter (e.g. doorbell push-button) can be assigned to each symbol. This assignment can be adapted by programming. Up to 8 transmitters can be assigned to each symbol. Please keep in mind that a transmitter can only be assigned if it is not yet known in the memory of the »*signolux* plug-in receiver«. If required, you might first have to delete the transmitter from the memory (see section 3.)

a) Keep the function button pressed for some time. The symbols begin to light up in sequence. Let go of the function button as soon as the desired symbol is lit. (e. g. doorbell symbol)

b) Trigger the signal of the transmitter you want to assign to the lit up symbol. (e.g. operate the doorbell button).

c) Wait for about 20 seconds and test the function.

### **3. Deleting transmitters from receiver memory**

If you wish to remove the transmitters assigned to a symbol from the receiver memory:

a) Keep the function button pressed for a few seconds. The symbols begin to light up in sequence. Let go of the function button as soon as the desired symbol is lit.

b) Now press the function button and the delete button at the same time. All symbols will briefly light up.

**Please note that this will delete all transmitters assigned to the according symbol.**

## Meaning of the symbols

	doorbell	(green)
	telephone	(orange)
	person call	(yellow)
	babycry	(white)
	fire alarm	(red)
	alarm	(red)

## Checking the last signal, cancelling the current signal

By pressing the function button, the last registered signal can be displayed again.

You can also cancel a signal that was just activated by pressing this button.

## Special accessories

The following modules (not included) can be plugged into to the AUX output connector:

- Vibrating pillow (A-3303-0)
- Flash module MF-1 (A-3320-0)
- Switch module MS-1\* (A-3330-0)
- Acoustic module MA-1 (A-3340-0)

\*Only for use with Type C (CEE 7/16 Europlug) and Type F (CEE 7/4 „Schuko“) outlet connectors

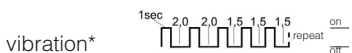
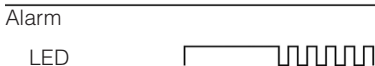
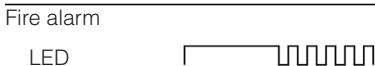
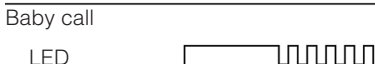
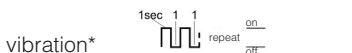
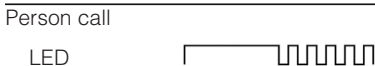
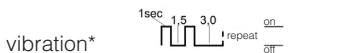
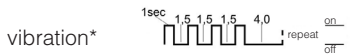
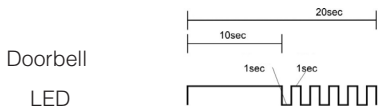
## Transmission range

The signal range of the »*signolux*« transmitters can be up to 200 metres under ideal conditions. The following conditions may result in a reduced range:

- Buildings or vegetation.
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

## Signal distinction

\* only with attached vibrating pad

## Maintenance and care

All »*signolux*« devices do not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth.

Never use spirits, thinners or other organic solvents. Do not expose the devices to full sunlight for long periods.

In addition, they must be protected against excessive heat, moisture and severe mechanical shocks.

### Note:

This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, above or near the product.

## Warranty

The »*signolux* Plug-In receiver« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by unauthorised persons (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product serial number in any event.**

**Disposal** of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products.

Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

This device satisfies the following EU directives:

- 2011/65/EG RoHS directive
- 2002/96/EG WEEE directive
- 1999/5/EG R&TTE directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at **[www.humantechnik.com](http://www.humantechnik.com)**.

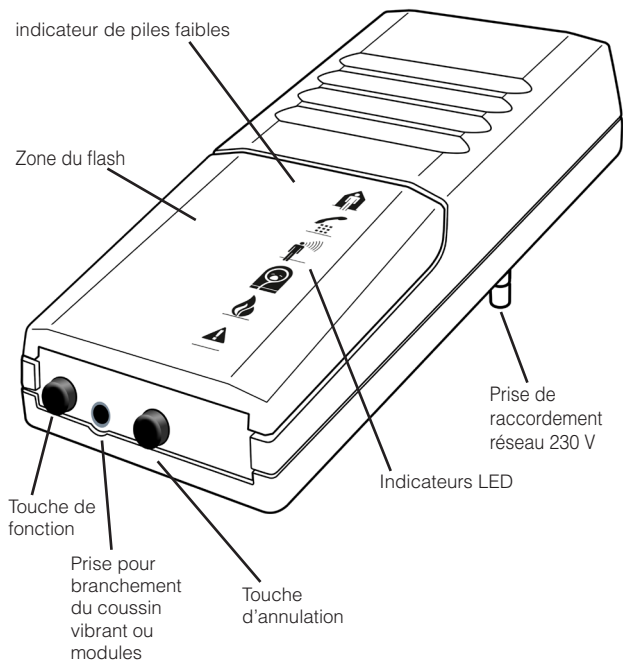
### Technical data

Power supply:	230 V, 50 Hz
Energy consumption:	approx. 1 W
Transmission frequency:	868,35 MHz
Height:	121 mm
Width:	65 mm
Length:	35 mm
Weight:	215 g

Technical specifications subject to change without prior notice.

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**Humantechnik GHL AG**Rastatterstrasse 9
**Sarabec Ltd**15 High Force Road