HUMANTECHNIK

signolux

DE

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Seite 2

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We would like to congratulate you on purchasing the modern and reliable »signolux« alerting system! Please read the user manual carefully in order to use the device properly and to familiarise yourself with all the possibilities this system offers.

Package contents

Please check if all following parts are included:

- Person call / Doorbell-pushbutton
- Battery (CR2032, already in place)
- Double-sided adhesive strip
- User manual
- Warranty card

Should any parts be missing, kindly contact your specialist dealer or the manufacturer directly.

Safety instructions

- Prior to putting the device into operation and using it, kindly read this manual thoroughly and in full.
- Store this operating manual in a safe place so that it is also accessible for other users at any time.
- Keep Signolux devices out of the reach of children! This packaging contains small parts which may present a choking hazard!
- Flashes of light can trigger epileptic seizures in people with photosensitive epilepsy (PSE).

Function

A »signolux« alerting system consists of at least one transmitter (e.g. person call pushbutton) and a »signolux« receiver. Up to 8 transmitters can be assigned to each event icon of the receivers.

If e.g. the doorbell pushbutton is pressed, a radio signal (868,425 MHz radio frequency) is transmitted to the »signolux« receiver. Depending on the receiver type, the received signals are indicated with light, sound or vibration.

Setup

The battery is already inserted in the device and the pushbutton is immediately ready for operation.

Preset transmitter indication

Signolux devices can be used together immediately and without prior pairing when delivered.

By default, the pushbutton will be indicated by receiver devices with the event icon »Doorbell 2«.

This transmitter can be displayed with an event icon of your own choice, this can be done as described in the following chapter.

However, this causes the according receiver to lose the *preset transmitter indication* and every transmitter previously indicated on the receiver must then be individually paired!

Individual icon assignment


To display an icon of your choice on a receiver for the personcall-pushbutton, proceed as follows:

- a) Keep the function button of the according receiver pressed. The event icons light up in sequence. Release the button when the desired icon is lit.
- b) Press the pushbutton.
- c) Wait for approx. 20 seconds and test the function.

Replacing the battery

In order to be able to insert the coin cell battery (type CR 2032) you first have to open the battery compartment.

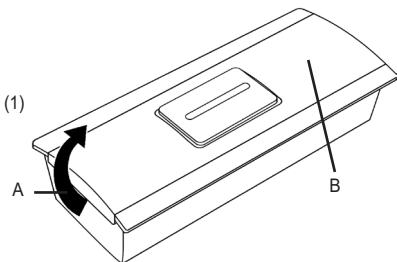
- a) Remove the transparent front by lifting it off its clip on one side of the housing (fig. 1 A). If necessary, use a small flatsided screwdriver to assist.

- b) Now it should be possible to unhook the front from the other side of the housing for removal.

c) You can now open the small screw (fig. 1 B) using a small Phillips screwdriver and remove the front of the call-push-button behind which the PCB with the battery is located.

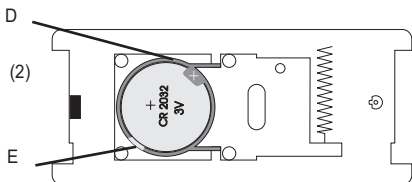e) Insert the battery (coin cell CR 2032) into the plastic support on the PCB (fig. 2 D).

e) If you would like to replace a battery, remove the old one with a small flat screwdriver from its plastic support by lifting the edge slightly (fig. 2 E). Afterwards you can insert a new battery. Please ensure the correct polarity of the battery.

f) Insert the front face into the housing of the pushbutton and press both firmly together until the housing parts lock in place. Please make sure that the rubber gasket sits correctly within the housing.

g) Carefully set the small screw (fig. 1 B) with a small Phillips screwdriver.

h) Hook the plastic front onto the clip of one side, then press the other side down until the front completely locks onto the housing. push button and press both firmly together until the housing parts lock in place.

Mounting the doorbell-pushbutton

You can mount the pushbutton using the supplied strips of double-sided tape. Take care to adhere it to a smooth, dry and clean surface.

Important note:

Kindly keep in mind that metal parts and other building materials may interfere with the transmission of the radio signal. As they can be either visible or hidden at the desired mounting location you should always carry out a functional test at the mounting location before starting the mounting process, and, if required, choose a different mounting or installation location if the reception is not guaranteed.

Specifications

| | |
|---|-----------------|
| Work range | -10 to 40°C |
| Radio frequency | 868,425 MHz |
| Range (open spaces) | up to 200 m |
| Transmission power | <10 mW |
| Battery | CR2032 |
| Battery service life (5 activations per day) | 18 months |
| Dimensions (HxWxD) | 40 x 80 x 16 mm |
| Weight | 41 g |

This device complies with the following EC standards:

- 2011 / 65 / + 2015 / 863 / EC RoHS - Directive
- 2012 / 19 / EC WEEE - Directive
- 2014 / 53 / EC RED - Directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at <http://www.humantechnik.com/service> .

Humantechnik's UK authorised representative:

Sarabec Ltd.
15 High Force Road
MIDDLESBROUGH TS2 1RH
United Kingdom

Sarabec Ltd., hereby declares that this device complies with all UK statutory instruments.

UK declaration of conformity available from:
Sarabec Ltd.

Use only commercially available batteries of type CR2032!

There is a risk of explosion if the wrong type of battery is used. The battery must not be exposed to high or low extreme temperatures and low atmospheric pressure during use, storage or transport at high altitude.

Disposing of batteries in fires, ovens or mechanically crushing and cutting them can cause fires or explosions. Exposing batteries to extremely high temperatures or extremely low air pressure may cause an explosion or leakage of flammable liquid or gas.

Disposal of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units.

You protect the environment and health of your fellow man by the correct disposal of this product.

Environment and health are endangered by incorrect disposal.

Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, our communal disposal company or your local dealer.

Dispose of used batteries according to your local disposal regulations.

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Humantechnik Finland Oy