HUMANTECHNIK

signolux

DE

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Seite 2

Gateway

EN

Operation Instructions

Page 14

Gateway

FR

Mode d'emploi

Page 26

Gateway

NL

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Pagina 38

Gateway

IT

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Pagina 50

Gateway

ES

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Página 62

Gateway

Funktionstaste

Internet-Status-LED

Löschtaste

Signalanzeige-LEDs

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Network socket

Power supply socket
(Micro-USB)

Thank you very much for purchasing the *Signolux-Gateway*. We expect this product to bring you many years of reliable service. Please read this manual thoroughly in order to operate the device properly and also to become accustomed with all features of the system.

Requirements

The following conditions must be given in order for the device to function:

- An installed Internet router with a functioning internet connection
 - The internet router must have a free LAN port
 - Within the settings of the router, DHCP must be enabled (Usually enabled by default)
 - An Android smartphone with OS version 4.01 or higher
- or
- An Apple iPhone with iOS 9 or higher

Scope of delivery

Please check if all following contents are included:

- *Signolux-Gateway*
- Network cable, 2 metres
- 5V power supply unit with Micro-USB-plug
- This manual and a warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

Function

An alerting system consists of at least one transmitter and one receiver.

The *Signolux-Gateway* acts as a receiver and recognises the alerts sent by the various Signolux transmitters within the house. The *Signolux-Gateway* displays these alerts and forwards them to a server on the internet, which in turn supplies the information for portable devices.

The alert can be displayed worldwide on a smartphone via an installable app.

Any amount of smartphones can be assigned to one Gateway device.

Please note: An interrupted internet connection at home or the smartphone being in a location without phone reception or WiFi will result in alerts not being displayed on the smartphone.

Setup

1. Connecting the network cable


Plug the supplied network cable into the network socket located on the rear panel of the device. Plug the other end of the network cable into a free LAN port of your internet router.

2. Connecting the power supply

Connect the Micro-USB-Plug of the power supply unit with the power-in-socket located on the rear panel of the *Signalux-Gateway*. Plug the power supply unit into a mains electricity socket.

The Gateway will configure itself automatically with the router. This procedure may take a few minutes to complete.

The *Signalux-Gateway* is now ready to receive the alerts of the Signalux system and to forward these in encrypted form to the internet.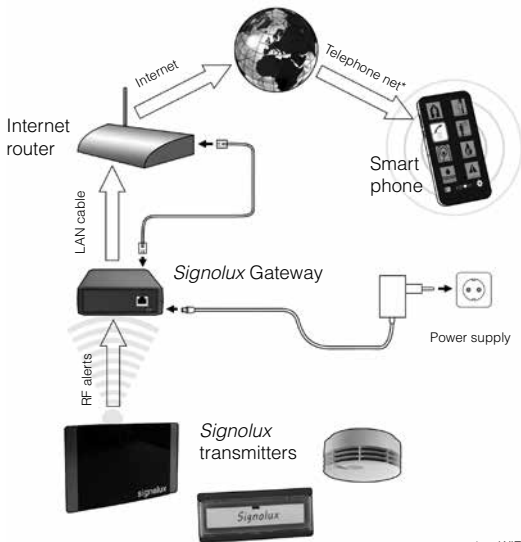

3a. Download and install the app

Search and download the app »**Humantechnik:Alert**« from the respective app store for Apple or Android devices and install it on your according device. For your convenience you will find the according QR-Codes on page 23.

3b. App configuration

After installing, launch the app and select the settings icon .

In the settings, you need to enter the ID number and code, which is unique to every *Signolux-Gateway* device.


You will find the ID and code on the label underneath the device as well as on the back cover of this manual.

After entering, select »Test settings«. The message »Testing connection...« is displayed.

After the message »Test successful!« has been indicated, select »Save settings« to save the ID-number and code entry.

The app will now receive alerts from the Signolux-Gateway.

Settings screen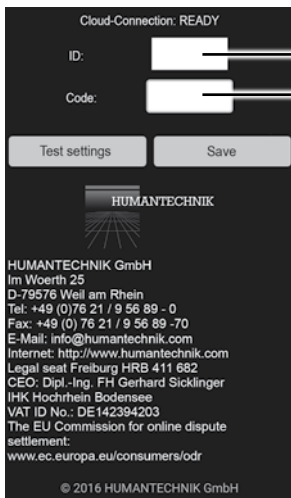

Cloud-Connection: READY

ID:

Code:

Test settings Save

HUMANTECHNIK

HUMANTECHNIK GmbH
Im Woerth 25
D-79576 Weil am Rhein
Tel: +49 (0)76 21 / 9 56 89 - 0
Fax: +49 (0) 76 21 / 9 56 89 -70
E-Mail: info@humantechnik.com
Internet: <http://www.humantechnik.com>
Legal seat Freiburg HRB 411 682
CEO: Dipl.-Ing. FH Gerhard Sicklinger
IHK Hochrhein Bodensee
VAT ID No.: DE142394203
The EU Commission for online dispute settlement:
www.ec.europa.eu/consumers/odr

© 2016 HUMANTECHNIK GmbH

Enter ID here


Enter code here

To return back to the main alert screen without alterations, press the „back“ key of your phone.

3c. Apple iOS: App configuration


After installing the app *Humantechnik:Alert* for iOS, please make sure that the following entries within the subfolder „Notifications“ of the iOS-settings are enabled (green):

- Allow Notifications
- Show in Notification Centre
- Sounds
- Badge App Icon
- Show on Lock Screen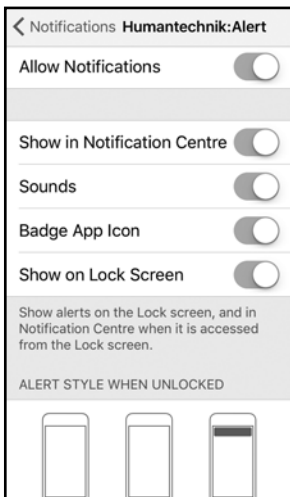

This ensures that notifications of the *Humantechnik:Alert* app can reach you at any time, even on the lock screen.

Upon first launch of the app, the operating system will ask for the permission to send notifications. Please select „Allow“.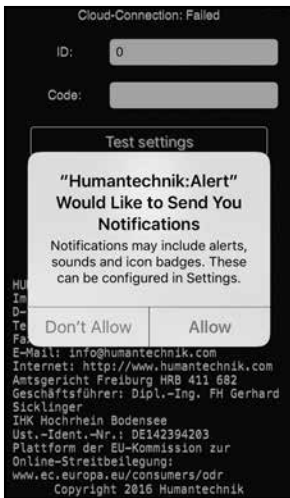

After completing the previous steps, select the settings icon . In the settings, you need to enter the ID number and code, which is unique to every *Signolux-Gateway* device (see the image on page 17). You will find the ID and code on the label underneath the device as well as on the back cover of this manual. After entering, select »Test settings«. The message »Testing connection...« is displayed. After the message »Test successful!« has been indicated, select »Save settings« to save the ID-number and code entry. The app is now ready to receive alerts from the *Signolux-Gateway*.

Alerting

1. Display on active app


The alert symbol is displayed immediately and can be confirmed by pressing on it.

2. Display when app active in background

A notification is displayed on the smartphone (similar to notifications of other messaging services like e.g. WhatsApp).

By tapping on the notification, the alert screen of the app is displayed and the according alert can be seen and confirmed by tapping on the illuminated event icon.

Alert screen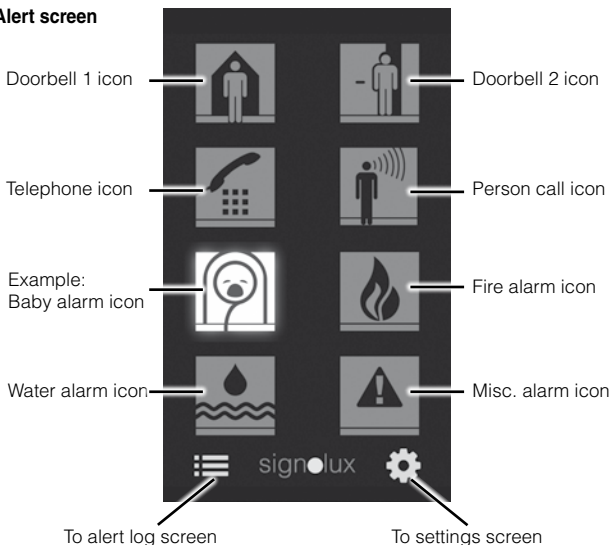

Please note: According to conditions within the internet, a delay of several seconds between the triggering of an alert on the *Signalux* system and the actual event indication on the app may occur.


3. Log of received alerts (30 entries)

Received alerts are stored in a list which shows the type of alert (e. g. doorbell, telephone etc.) and the time and date it came in.

The list can be opened by tapping the log icon ☰.

If required, the log can be cleared by touching the trashcan icon 🗑️.

Alert log screen

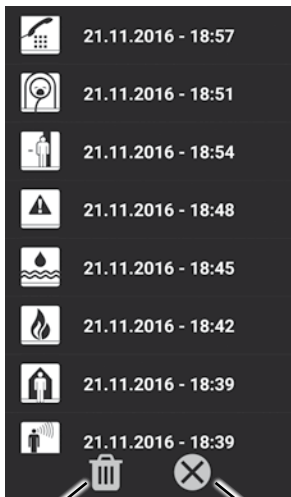Trashcan icon

Back to alert screen

Programming signals and assignment to the event icons

A transmitter (e.g. doorbell pushbutton) can be assigned to each event icon. This assignment can be adapted by programming. Up to 8 transmitters can be assigned to each icon.

Please keep in mind that a transmitter can only be assigned if it is not yet stored in the memory of the signalux-Gateway. If required, you might first have to delete the transmitter from the memory (see next paragraph).

- Keep the function button »●« pressed for some time. The event icons begin to light up in sequence. Let go of the function button as soon as the desired icon is lit. (e. g. doorbell icon).
- Trigger the signal of the transmitter you want to assign to the lit up icon. (e.g. operate the doorbell button).
- Wait for about 20 seconds and test the function.

Deleting transmitters from memory

To delete a transmitter from memory, do the following:

- Keep the function button »●« pressed for some time. The icons begin to light up in sequence. Let go of the function button as soon as the desired icon is lit.
- Now press the function button »●« and the delete button »○« at the same time. All event icons will briefly light up.

Please note that this will delete all transmitters assigned to the according icon.

Internet-Status-LED

LED is on: Internet connection is established and active.

LED is blinking: No internet connection. Please check the network cable connection or the settings of your internet router.

Transmission range

The radio signals of the *Signalux* transmitters within the house are designed to have a range of 200 meters under ideal conditions.

The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.

Placing the Gateway

The *signolux-Gateway* should be placed in the vicinity of the internet router. During the setup, please note the following:

- The Gateway should not be placed in proximity to strong sources of interference like TV-sets, computer monitors or mobile phones.
- The Gateway should not be placed on metal surfaces and objects, as the incoming signal strength can be weakened by the metal's shielding effect.

QR-Codes for app stores

For a convenient access to the according app providers you can also use the following QR-Codes:

Android

iOS

Notes on data usage

In order for the *Signolux-Gateway* to function it is necessary to transmit the ID, the code, the point of time and the type of alert to the server. No further data is transmitted.

The alerts are stored for 30 days, after which they are automatically deleted.

Users of Apple-devices receive their alerts also over the Apple push-notification servers to their units.

Maintenance and care

All *Signolux* devices do not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the Gateway where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The *Signolux-Gateway* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge. If possible, return the product in its original packaging, so consider not to dispose of the packaging.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units.

Help to protect the environment and health of your fellow men by the correct disposal of this product. Environment and health are endangered by careless disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Specifications

Power supply:	5V power supply unit with Micro-USB-connector
Receiver frequency:	868.3 MHz
Operating temperature:	0° C – 40° C
Height:	42 mm
Width:	119 mm
Depth:	138 mm
Weight:	260 g

This device complies to the following EC directives:

- 2011 / 65 / EC RoHS-directive
- 2012 / 19 / EC WEEE-directive
- 2014 / 53 / EC RED-directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at www.humantechnik.com.

Technical specifications subject to change without prior notice.

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Navigate after concluding the steps above indicated to the configurations (Symbol .

In the configurations you have to introduce for each Gateway the number ID available individually and the code (see illustration in page 65).

The number ID and the code you will find in the adhesive on the bottom side of the Gateway Signolux as well as in the back page of these instructions of handling.

Press next the button „Comprobar configuraciones“ It indicates a message „Comprobando conexión...“. After that has been indicated the message „¡Conexión exitosa!“ press please the button „Guardar“ to save the introduction of the number ID and the code.

The App now receives the messages from the Gateway Signolux.

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Sarabec Ltd