HUMANTECHNIK

signolux
soundmonitor

DE

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Seite 2

EN

User manual

Page 10

FR

Mode d'emploi

Page 18

NL

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Pagina 26

IT

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Pagina 34

ES

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Página 42

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Figure 1: View of top / side / front


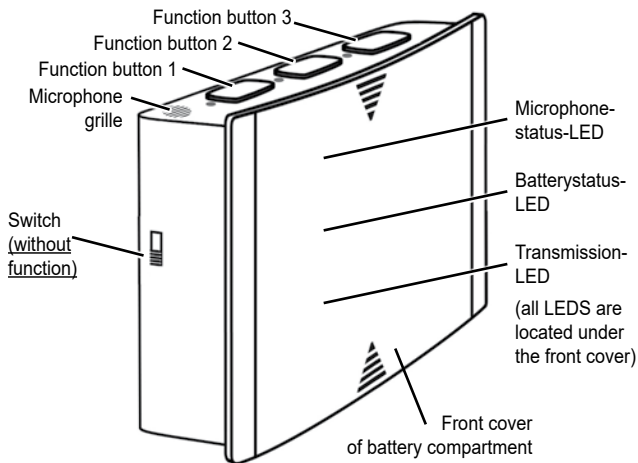
Figure 2: View of underside

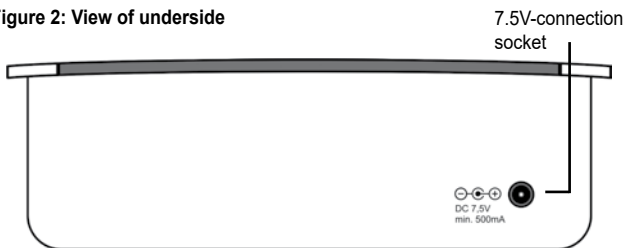Congratulations on purchasing your »*signolux soundmonitor*«.

You have chosen a highly modern and reliable system. Please read this manual thoroughly. It will tell you how to set up the system correctly and explain all the features this device has to offer.

Package contents

Please check if the following parts are included in the package:

- »soundmonitor«
- 3 x NiMH rechargeable battery, type AA, 1,2V (already inserted in the compartment)
- 7,5 V power supply unit
- Wall mount
- User manual
- Warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

Safety instructions

- Prior to putting the device into operation and using it, kindly read this manual thoroughly and in full.
- Store this user manual in a safe place so that it is also accessible for other users at any time.
- When using power tools always follow the instructions of the manufacturer and wear suitable protective equipment (e.g. protective goggles).

- Kindly check before starting to drill if there are any hidden electric cables or water pipes in the walls. In case of doubt we recommend using a cable detector.

Function principle

The »soundmonitor« has been designed to learn sounds and to recognise them if they occur again. Up to three different sounds can be memorized.

This makes it possible to bind special acoustic signals into the signolux alerting system. The soundmonitor scans the environmental noise around it for those sounds it has been taught to react to.

If a memorised sound has been recognised, the soundmonitor will send out a radio signal to all paired »signolux«-receivers within range.

The receivers indicate the radio signal with light-, vibration- or an acoustic alert (this depends upon the type of receiver).

Please note that the sound recognition only works with exactly repeatable sounds.

Inserting / replacing the rechargeable batteries (included and already inserted)

Remove the battery compartment lid: Push the front plastic cover upwards. A certain resistance, caused by the latching, has to be overcome. (Figure 3)

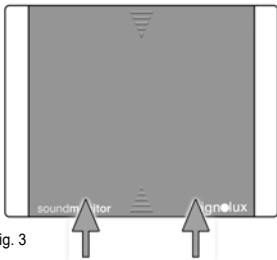
Fig. 3

The plastic cover can now be taken off, the battery compartment has become accessible. Observe the correct polarity when inserting / replacing the batteries (Figure 4).

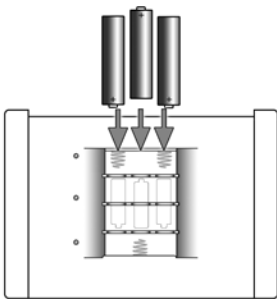
Fig. 4

Attach the plastic cover: Lay the cover onto the housing with approx 1 cm distance to the lower edge (Figure 5).

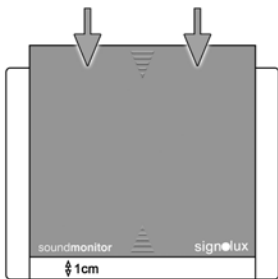
Fig. 5

Push the cover downwards until it clicks back into place. The front cover is now attached again.

Indicators and controls

3 indicator LEDs are located under the dark transparent front cover:

Microphone status indicator

This LED lights up whenever a random sound triggers the activation of the microphone analyser.

Battery status indicator

A red battery status LED indicates that the rechargeable batteries are almost depleted and must be recharged. During recharging, this LED will show an orange light. A permanent green light shows that the batteries are charged and the »soundmonitor« is active.

Transmission indicator

This LED indicates any outgoing transmission with a red light.

Function buttons

The correct functioning of the »soundmonitor« can be tested with any of the three function buttons. After pairing the »soundmonitor« with at least one receiver, the transmission and the triggered event indication on the receiver can be tested by pressing one of these buttons.

The function buttons also serve to teach the »soundmonitor« the sounds it will react to.

Setup

1. Choosing the best position

The »soundmonitor« scans and analyses the noises in its vicinity for the patterns of the sounds it has been taught.

In order to assist this function, the »soundmonitor« should be placed as close as possible to the sound source it is supposed to identify (for example placed on a wall underneath the door bell).

In order to ensure the largest possible transmission range, please note the following recommendations:

- The transmitter should not be placed close to interference-emitting devices such as TV sets, computer screens and mobile phones.

- The transmitter should not be placed on metal objects or surfaces due to the radio-shielding effect of the metal.

2. Power supply

Connect the power supply with the according socket on the underside of the »soundmonitor«.

Plug the power supply unit into a regular 230V mains electricity outlet.

3. Teaching the desired sounds

To teach a new sound, press and hold one of the function buttons, located on the upper face of the device, for 2 to 3 seconds until the adjacent LED of the button starts to blink.

Trigger the desired sound on the sound source. The sound should have a length of at least 1,5 seconds.

During the recording, the microphone status indicator must be lit constantly!

The function button LED will light up and go out after the sound has been recorded.

The new sound has been taught. The other two function buttons can also be programmed as described above.

Please note:

Every recognised sound is indicated by the same event icon on the receivers.

Important notice:

When teaching a sound, please avoid any other environmental noise.

The soundmonitor should remain in the position in which the according sound was recorded, in order to assist a secure recognition of the sound when it occurs again.

Please note:

This device can only recognise repeatable and unaltering electronic sounds. A mechanical sound source or voice cannot be identified.

Blackout-Backup

Please note that this device has been designed to be primarily powered over the power supply unit.

The rechargeable batteries serve only as an emergency backup during power cuts.

The standby provided by the batteries covers approx. 16 hours.

The batteries are charged when the »soundmonitor« is connected to the mains power supply.

The batteries have a charging time of approx. 12 hours.

4. Pairing the transmitter with the receivers

The »soundmonitor« has to be paired with every »signalux«-receiver, which is later supposed to react to it. By doing so, the indication type is also defined.

To register the transmitter, press and hold the according receiver's function key. The receiver's event indicator icons light up in sequence.

Release the function key as soon as the desired icon is lit up (for example the doorbell icon when catching the sound of the doorbell).

Now press one of the three function buttons of the »soundmonitor«.

Wait for approx. 20 seconds and then test the setup by pressing any of the function buttons. The receiver should now react, showing the according symbol.

This procedure needs to be repeated for every receiver unit.

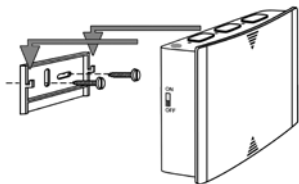
Please note:

Every recognised sound is indicated by the same event icon on the receivers.

Mounting the »soundmonitor«

The »soundmonitor« can be mounted using the supplied wall mount. To use it, fix the wall mount onto a wall (or rafter, cupboard etc.) When mounting onto a brick- or stone wall, you may need a power drill and stone bit to do so. Use the supplied wall plugs in this case.

Once fixed, hang the »soundmonitor« onto the wall mount bracket.

Maintenance and care

The »soundmonitor« does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Technical data

Power supply:	DC 7,5V / 1500mA, power backup by 3 rechargeable batteries 1,2V
Rech. battery standby:	approx. 16 hours
Frequency:	868,35 MHz
Max. transmitting power:	<10 mW
Temperature range:	0° C – 40° C
Microphone status indicator:	1 red LED
Battery status indicator:	1 red/green LED
Transmission indicator:	1 red LED
Recording indicators:	3 red LEDs
Height:	95 mm
Width:	112 mm
Depth:	38 mm
Weight:	320 g (including batteries)

This device complies with the requirements of the following EC directives:

- 2011 / 65 / EC RoHS-directive
- 2012 / 19 / EC WEEE-directive
- 2014 / 53 / EC RED-directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at www.humantechnik.com/service.

Technical specifications subject to change without prior notice.

Warranty

The »soundmonitor« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Rechargeable batteries

The included rechargeable batteries can be recycled. Please discard them through  the battery container or a specialist shop. Please discard only discharged batteries to assist environmental protection.

Disposal of used electrical and electronic units (applicable in the countries of the  European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units. You protect the environment and health of your fellow man by the correct disposal of this product.

Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, your communal disposal company or your local dealer.

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Sarabec Ltd