HUMANTECHNIK

signolux
alarmo

DE

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Seite 2

EN

User manual

Page 10

FR

Mode d'emploi

Page 18

NL

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Pagina 26

IT

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Pagina 34

ES

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Página 42

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Figure 1: View of top / side / front


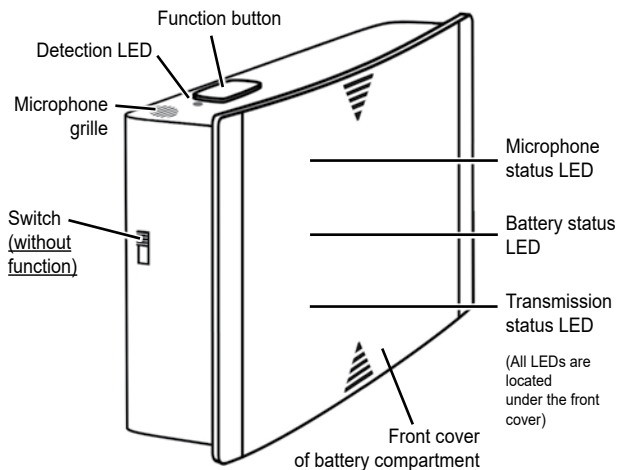
Figure 2: View of underside

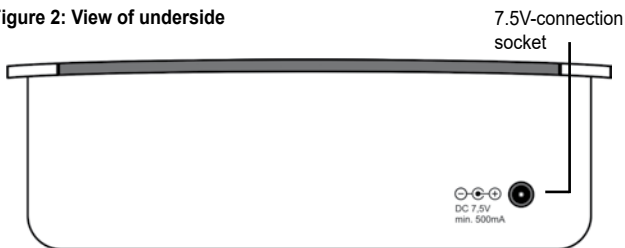Congratulations on purchasing your »alarmo« for the signolux alerting system.

You have chosen a highly modern and reliable device. Please read this manual thoroughly. It will tell you how to set up the system correctly and explain all the features it has to offer.

Package contents

Please check if the following parts are included in the package:

- »alarmo«
- 3 x NiMH rechargeable battery, type AA, 1,2V (already inserted in the compartment)
- 7,5 V power supply unit
- Wall mount
- User manual
- Warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

Safety instructions

- Prior to putting the device into operation and using it, kindly read this manual thoroughly and in full.
- Store this user manual in a safe place so that it is also accessible for other users at any time.
- When using power tools always follow the instructions of the manufacturer and wear suitable protective equipment (e.g. protective goggles).

- Kindly check before starting to drill if there are any hidden electric cables or water pipes in the walls. In case of doubt we recommend using a cable detector.

Function

»alarmo« has been conceived to enable the easy incorporation of already existing smoke detectors into the »signolux«-signalling system, without need for modification or wiring.

»alarmo« scans environmental noises in proximity for sound patterns typical of smoke detectors and fire alarm systems of any type.

False alarms, possibly caused by alarm sounds not actually within in the room (e. g. passing emergency vehicles, TV and movie sound effects) are prevented by means of intelligent analysis and filtering of the registered sound.

If an alarm sound is identified as such, »alarmo« transmits a radio signal to all accordingly registered »signolux« receivers within range, which will indicate the incident with light, vibration or sound - depending on the type of receiver.

Inserting / replacing the rechargeable batteries (included and already inserted)

Remove the battery compartment lid: Push the front plastic cover upwards. A certain resistance, caused by the latching, has to be overcome. (Figure 3)

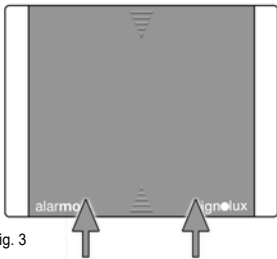
Fig. 3

The plastic cover can now be taken off, the battery compartment has become accessible. Observe the correct polarity when inserting / replacing the batteries (Figure 4).

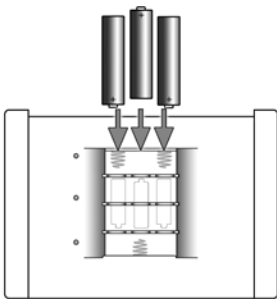
Fig. 4

Attach the plastic cover: Lay the cover onto the housing with approx 1 cm distance to the lower edge (Figure 5).

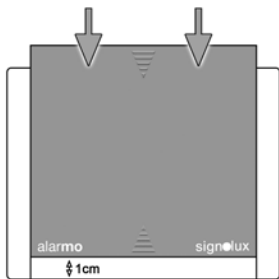
Fig. 5

Push the cover downwards until it clicks back into place. The front cover is now attached again.

Indicators and controls

Microphone status LED

This LED lights up when the environmental noise reaches a certain level.

Battery status LED

A red battery status LED indicates that the rechargeable batteries are almost depleted and must be recharged. During recharging, this LED will show an orange light. A permanent green light shows that the batteries are charged and the »alarmo« is active.

Transmission status LED

This LED indicates any outgoing transmission with a red light.

Function button

The correct functioning of the »alarm« can be tested with the function button. After pairing the »alarm« with at least one receiver (see page 14), the transmission and the triggered event indication on the receiver can be tested by pressing it.

Detection LED

This LED lights up when a sound has been identified as an alarm.

Setup

1. Choosing the best position

The »alarm« analyses the noises within its environment for sounds typical of smoke detectors and similar alarm systems.

In order to assist this function, »alarm« should be placed as close as possible to the sound source of the alarm, which is to be monitored (e. g. on a wall close to a smoke detector).

In order to gain the best possible transmission range, please note the following recommendations:

- The transmitter should not be placed close to interference-emitting devices such as TV sets, computer screens and mobile phones.

- The transmitter should not be placed on metal objects or surfaces due to the radio-shielding effect of the metal.

Note:

The device is not able of identifying, recording or even transmitting voices and spoken conversation.

Therefore there is no possible invasion or endangerment of the user's privacy.

2. Power supply

When using for the first time: Open the battery compartment and remove the battery stopper. Connect the power supply with the according socket on the underside of the »alarm«.

Plug the power supply unit into a regular 230V mains electricity outlet.

3. Blackout-Backup

Please note that this device has been designed to be primarily powered over the power supply unit.

The rechargeable batteries serve only as an emergency backup during power cuts.

The standby provided by the batteries covers approx. 16 hours.

The batteries have a charging time of approx. 12 hours.

4. Pairing the transmitter with the receivers

The »alarmo« has to be paired with every »signalux«-receiver, which is later supposed to react to it. By doing so, the indication type is also defined.

To register the transmitter, press and hold the according receiver's function key. The receiver's event indicator icons light up in sequence.

Release the function key as soon as the desired icon is lit up (for example the fire icon).

Now press the function button of the »alarmo«.

Wait for approx. 20 seconds and then test the setup by pressing any of the function buttons. The receiver should now react, showing the according symbol.

This procedure needs to be repeated for every receiver unit.

Important notice:

»alarmo«'s radio alarm signal is triggered by the computer-assisted positive analysis of environment sounds.

Despite of digital filtering and programmed threshold values, the possibility of a false alarm cannot be fully excluded, due to the almost limitless variety of possible sounds (e. g. siren-like custom ringtones on mobile phones).

Due to this fact, the Humantechnik GmbH will take no responsibility

for false alarms and their effects.

For this reason we advise to test the function of the »alarmo« by releasing a test alert on the alarm device (smoke detector, alarm system) which is to be monitored.

Sensitivity

»alarmo« has been developed and tested by considering and utilising many different types of alarm tones.


It is possible to alter the thresholds of your »alarmo«'s acoustic analysis, should it fail to respond to the monitored alarm device, even after repeated tests and alternating positions.

This involves opening the chassis and a certain process.

In this case, please contact the Humantechnik GmbH for advice.

Mounting the »alarmo«

The »alarmo« can be mounted using the supplied wall mount. To use it, fix the wall mount onto a wall (or rafter, cupboard etc.) When mounting onto a brick- or stone wall, you may need a power drill and stone bit to do so. Use the supplied wall plugs in this case. Once fixed, hang the »alarmo« onto the wall mount bracket.

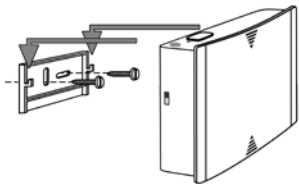Maintenance and care

The »alarmo« does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The »alarmo« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period. The warranty does not apply to damage caused by incorrect hand-ling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Technical data

| | |
|-----------------------------|--|
| Power supply: | Power supply unit DC 7,5V / 1500mA, power backup by 3 rechargeable batteries 1,2V |
| Rech. battery standby: | approx. 16 hours |
| Frequency: | 868,35 MHz |
| Maximum transmitting power: | <10 mW |
| Temperature range: | 0° C – 40° C |
| Detection LED: | 1 red LED |
| Microphone status LED: | 1 red LED |
| Transmission status LED: | 1 red LED |
| Battery status LED: | 1 red/green LED |
| Height: | 95 mm |
| Width: | 112 mm |
| Depth: | 38 mm |
| Weight: | 320 g (including batteries) |

This device complies with the requirements of the following EC directives:

- 2011 / 65 / EC RoHS-directive
- 2002 / 96 / EC WEEE-directive
- 2014 / 53 / EC RED-directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at www.humantechnik.com/service.

Technical specifications subject to change without prior notice.

Rechargeable batteries

The included rechargeable batteries can be recycled. Please discard them through the battery container or a specialist shop. Please discard only discharged batteries to assist environmental protection.

Disposal of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units. You protect the environment and health of your fellow man by the correct disposal of this product.

Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, your communal disposal company or your local dealer.

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Sarabec Ltd