HUMANTECHNIK

DE

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Seite 2

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7,5 V DC power input Sockets for vibrating pads

① ②

- ① This LED indicates the status of the internal battery of the charger.
Red: Battery is charging
Green: Battery is charged
- ② This LED indicates the status of the charger.
Green: The charger is on.
Red: The pager is inserted and provided with power.

Thank you for purchasing the charger for the portable Human-technik receivers. We hope it will serve you for a long time to come.

Please read this manual. It will tell you how to set up the system correctly and explain all the features.

Package contents

Please check if the following parts are included:

- Charger
- 2 x NiMH rechargeable batteries (Type AAA)
- Power supply unit
- User manual
- Warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

How to Start

1. Please first remove the batteries from the receiver and replace them with the rechargeable batteries included with the charger.
2. Connect the charger with the power supply by inserting the round plug into the DC jack on the back of the charger.
3. Plug the power supply into a power outlet. LED 2 lights up in green, indicating that the charger is functioning.

Rechargeable batteries

The charger comes with 2 NiMH rechargeable batteries. If the rechargeable batteries are properly charged and discharged, they will have a long service life.

Please make sure that the batteries are inserted correctly in the battery compartment of the portable receiver. The illustration on the bottom of the battery compartment indicates the correct direction.

Please note that the rechargeable batteries are not charged on delivery and must be completely charged (at least 10 hours) in the portable receiver in the charger first.

To recharge your portable receiver, please use only the rechargeable batteries included with the charger or the rechargeable replacement batteries (A-4966-0).

Charging the pager

Insert the pager in the charging bay. LED 2 will light up red when the pager is inserted. An LED on the pager indicates when it is fully charged.

LED 2 remains red as long as there is a pager present.

Sockets for vibrating pads and modules

There are two connector sockets marked VIB on the back of the recharging base. It is possible to connect up to two vibrating pads or other Humantechnik modules with the base.

When the portable receiver is placed in the charging dock, any incoming signal will be relayed to the connected accessories, which will then react accordingly.

Please note that the receiver must be switched on and it has to be in the charging dock to enable this function.

Backup power

The charger base is equipped with an inbuilt backup battery which will charge when the charger base is powered and which will provide backup power to the charger base in the case of a power grid black-out. This ensures the function of any connected vibrating pad in such cases. LED 1 indicates the status of the internal backup battery. (red: charging / green: charged)

Warranty

All Humantechnik products are very reliable. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge.

It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of these products.

Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material.

You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Rechargeable batteries

The provided rechargeable batteries can be recycled. Please dispose of the batteries through battery containers or a specialist shop. Please dispose only discharged batteries to guarantee environmental protection.

Specifications

Power supply:	DC 7.5 V
Inbuilt Lithium-ion battery:	2500 mAh
Visual signals:	1 LED red/green- Internal battery 1 LED red/green- Charger
Dimensions (HxWxD):	67 x 83 x 65 mm
Weight:	approx. 96 g
Temperature range:	0° C to 40° C

Technical specifications subject to change without prior notice.

CE mark

This device complies with the requirements of the following EU directives:

- 2011 / 65 / + 2015 / 863 / EU RoHS-directive
- 2012 / 19 / EC WEEE-directive
- 2014 / 30 / EC EMC-directive
- 2014 / 35 / EC LVD-directive

The conformity of the a.m. directives is confirmed by the CE mark on the device.

EC conformity declarations are available on the Internet under www.humantechnik.com/service.

UK CA

Hereby Sarabec Ltd declares that this device complies with all relevant UK Statutory Instruments.

The full declarations of conformity for this device are available from:

Sarabec Ltd., 15 High Force Road,
MIDDLESBROUGH TS2 1RH. UK.

Sarabec's EU authorised representative:
HUMANTECHNIK GmbH, Im Wörth 25,
D-79576 Weil am Rhein, Germany

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1. Please first remove the batteries from the Signolux receiver and replace them with the included rechargeable batteries with the base.
2. Connect the recharging base with the power supply by inserting the round plug into the DC jack on the back of the recharging base.
3. Plug the power supply into a power outlet. LED 2 lights up in green, indicating that the charger is functioning.

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