HUMANTECHNIK

signolux

DE

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Seite 2

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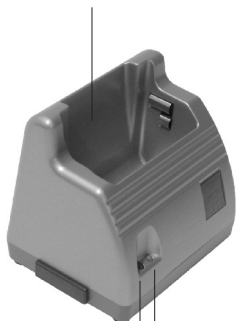
Recharging base for the portable Signolux receiver (A-2619xx)

Recharging Base for the
Portable RF Receiver

LED (red/ green)
Charging

LED (blue)
Ready

Slot for a
bridge element

12 V voltage input
(DC jack)

① ②

- ① The blue LED lights up when the *recharging* base is plugged in properly.
- ② Charging:
Red LED = Batteries are in the battery recharger and are in the process of being recharged
Green LED = Batteries are fully recharged

We would like to congratulate you on the purchase of the *recharging base* for the portable Signolux receiver and hope it will serve you for a long time to come.

Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

Standard components

Please check if all following components are included:

- *Recharging base* with bridge element and 4 safety screws
- 2 x NiMH rechargeable batteries (Type AAA)
- Power supply unit
- Operating instructions
- Guarantee card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

How to Start

1. Please first remove the batteries from the Signolux receiver and replace them with the included rechargeable batteries with the base.
2. Connect the *recharging base* with the power supply by inserting the round plug into the DC jack on the back of the *recharging base*.
3. Plug the power cord into a 230-volt power outlet.

Rechargeable Batteries

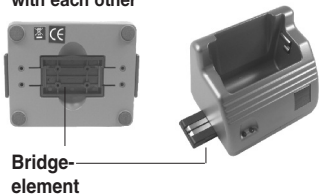
The *recharging base* comes with 2 NiMH rechargeable batteries. If the rechargeable batteries are properly charged and discharged, they will have a long service life.

Please make sure that the batteries are inserted correctly in the battery compartment of the portable signolux receiver. The illustration on the bottom of the battery compartment indicates the correct direction.

Please note that the rechargeable batteries are not charged on delivery and must be completely charged (at least 10 hours) in the portable signolux receiver in the *recharging base* first.

To recharge your portable signolux receiver, please use only the rechargeable batteries included with the *recharging base* or the rechargeable batteries that are available as special accessories (A-2994-0).

Connecting two *recharging bases* with each other

1. Unplug the two *recharging bases*.
2. Detach a bridge element from the bottom of one of the bases.
3. On the first *recharging base*, remove one of the plastic covers over the slot for the bridge element. Insert the bridge element into the slot.
4. Screw the safety screw into the bridge element through the bottom of the base.
5. Remove the opposite plastic cover on the second *recharging base*.
6. Set the second *recharging base* on the bridge element protruding out of the first *recharging base*.
7. Push the second *recharging base* over until it is flush with the first one.
8. Screw the safety screw into the bridge element through the bottom of the second *recharging base*.
9. Plug in one of the *recharging bases* into a power outlet by inserting the round plug into the DC jack on the back of the *recharging base*.

Use only one power supply unit to plug in the two *recharging bases*!

Maintenance and care

All *Signolux* devices are maintenance-free. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Please do not expose *Signolux* devices to direct sunlight for a long time and please protect them from excessive heat, moisture and severe mechanical shocks.

Note: This product is not protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

All *Signolux* devices are very reliable products. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Specifications

Power supply:

Via 12 V PSU to 110 V - 230 V, 50 Hz

Visual signals:

1 LED blue – Ready

1 LED red/ green – Recharging

Dimensions (HxWxD):

67 x 83 x 65 mm

Weight:

approx. 96 g

Operating temperature range:

0° C to 40° C

This product complies with European Union norms.

You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

Technical specifications subject to change without notice.

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Sarabec Ltd