HUMANTECHNIK

Triple-Bel

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GB **Operating Instructions** Page 6

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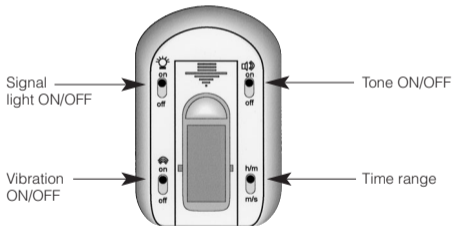
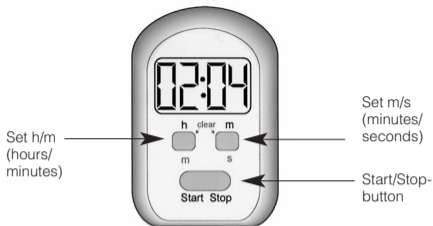
Time range

Setting h/m: maximum countdown time: 19 hours 59 minutes.

Setting m/s: maximum countdown time: 99 minutes 59 seconds.

Congratulations on purchasing the *Triple-Bel*. We hope it will give you many years of reassuring operation. Please read through these operating instructions carefully.

Setting the count down timer:

Set the display to 0:00 by pressing the „h/m“ and „m/s“ buttons at the same time.

Set the count down time by pressing the buttons „h/m“ and „m/s“.

Start the count down by pressing the „Start/Stop“ button once.

(Pressing the „Start/Stop“ button once again interrupts the count down.)

The signal is activated after the set time is reached. (Signalling light and/or vibration and/or acoustic signal).

Setting the count up timer:

Set the display to 0:00 by pressing the „h/m“ and „m/s“ buttons at the same time.

Start the count up by pressing the „Start/Stop“ button once.

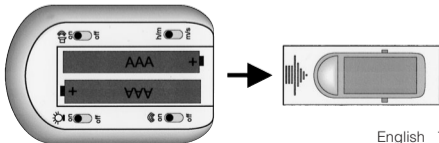
(Pressing the „Start/Stop“ button once again interrupts the count up.)

No signal can be activated in this operating mode!

Battery change

The *Triple-Bel* is operated with 2 alkali batteries (type AAA).

Open the battery compartment by pushing the cover into the direction of the arrow (see drawing). Insert 2 batteries according to the marking in the battery compartment.

Maintenance and care

The unit does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

Warranty

The *Triple-Bel* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt. **Always specify the product number in any event.**

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.



Technical data

Measurements:	72x48x30 mm (HxWxD)
Colour:	silver
Weight:	68 g
Power supply:	2 x 1,5 V micro alkali batteries (type AAA)

This product complies with European Union norms. You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

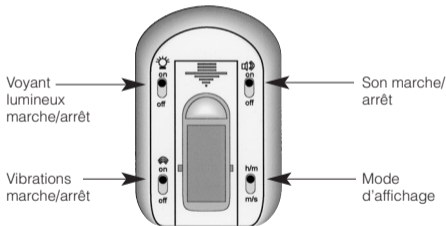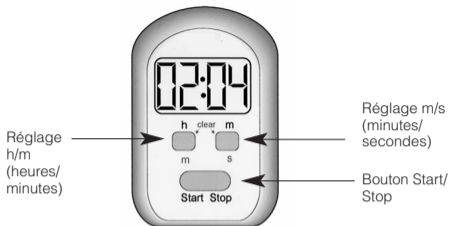Technical specifications subject to change without notice.

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Humantechnik GHL AG