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

Thank you for choosing the »travelTim«. You have purchased a modern and reliable product.

Operating principle

The »travelTim« is an electronic clock with a digital time display. It wakes up with an acoustic signal, with flashing light and/or with vibration.

Key functions

	Display Light / Stop Alarm
	Flashing light alarm, audible alarm and vibrating alarm
	Vibrating alarm (Only)
	Flashing light alarm and audible alarm
	Disable alarm
	Set alarm
	Set time
	Setting Locked
Zz	Snooze alarm
12/24 h	12/24 hour(s) switch

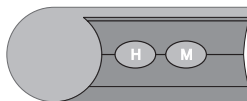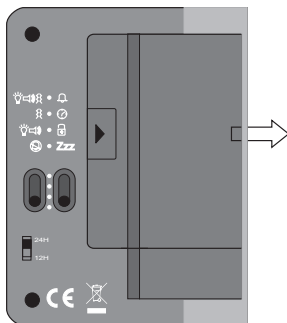Setting up the system

1. Inserting the battery

The »travelTim« is operated with 1 AAA Alkaline battery (Screen display) and 2 AA Alkaline batteries (Vibration). Open the battery compartment by sliding the cover of the battery compartment into the arrow direction of the »travelTim« using your thumb (see drawing). Insert 3 batteries according to the marking in the battery compartment.

2. Time setting

- Slide the right set switch to .
- Press the  buttons to set to actual time.
- After setting slide the switch to  (LOCK).
- Choose 12/24 hour(s) switch, located on the backside.

Note: If your »travelTim« is set to 12 hour(s) display „PM“ indicator will appear when post meridian hours are in effect.

3. Alarm setting

- Slide the right set switch to .
- Press  /  continuously until desired alarm time appears.
- After setting slide switch to  (LOCK).

4. Activate the alarm

- Choose flashing light, audible and vibrating alarm  or vibrating alarm (only)  or flashing light and audible alarm .
- When alarm activates slide the left switch to  to disable alarm. If the alarm is not switched off it will repeat every 24 hours.

5. Use the snooze alarm

- Choose alert preference.
- Slide the right set switch to **Zzz** position. **Zz** appears in the display.
- When alarm sounds: Press  to enjoy 8 minutes more rest. **Zz** will not repeat if alarm continues more than 4 minutes from start.
- Slide the left set switch to  to disable alarm.

Note:

The jack on the right side of the »travelTim« is an auxiliary output jack for an optional vibrating pillow or modules.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

Technical data

Dimensions:	95x70x30mm (H x W x D) with base folded in
Weight:	130g incl. batteries
Power supply:	1 x 1,5 V battery AAA (Typ LR03) and 2 x 1,5 V battery AA (Typ LR6) for the vibration motor
Color:	black/ green
Time of alarm:	4 min.

This product complies with European Union norms.

You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

Technical specifications subject to change without notice.

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