DE

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Seite 2

Digitalwecker DS-2

EN

**Operation Instructions**

Page 8

Digital alarm clock DS-2

FR

**Mode d'emploi**

Page 14

Réveil digital DS-2

NL

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Pagina 20

Digitale wekker DS-2

IT

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Pagina 26



Sveglia digitale DS-2

ES

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Página 32

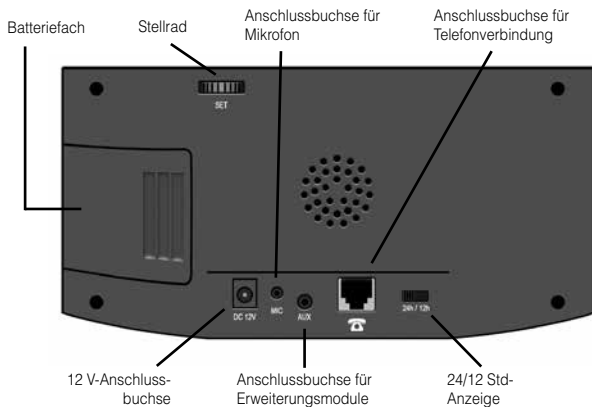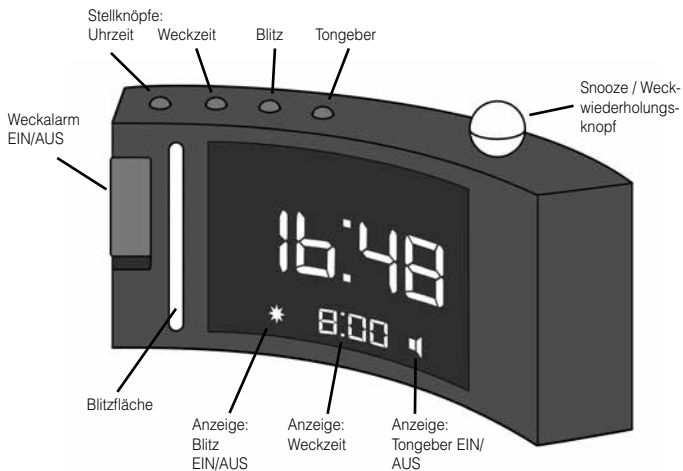Despertador digital DS-2

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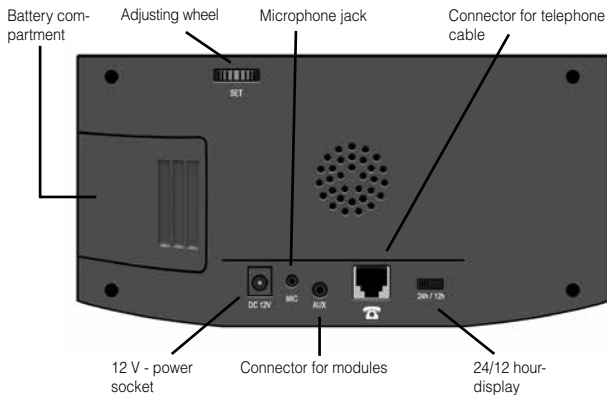
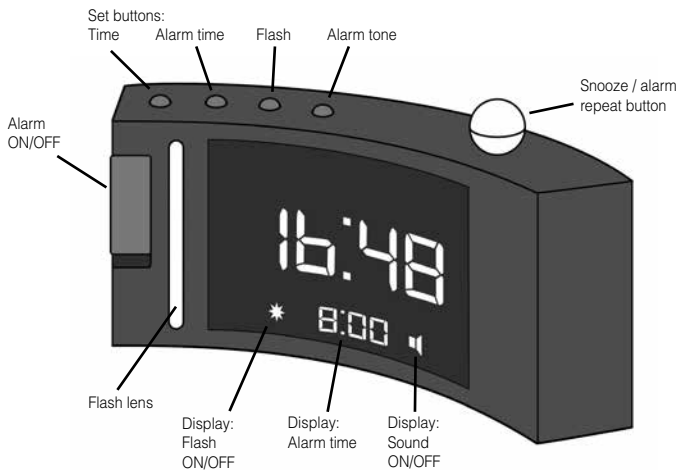
We would like to congratulate you on purchasing the »DS-2«. You have chosen a modern and reliable alarm clock. Kindly read the instruction manual carefully in order to use the device properly and to familiarise yourself with all its possibilities.

### Package contents

Kindly check if all following parts are included:

- »DS-2«
- 12 volts power supply unit
- Instruction manual
- Warranty card

Should any parts be missing, kindly contact your specialist dealer or the manufacturer directly.

### Functioning principle

The »DS-2« is an electronic clock with a digital time display.

It wakes you by flash and/or with an acoustic signal which is adjustable in pitch and volume.

Additionally, it can indicate incoming phone calls by flash and sound signals.

The following accessories can be connected to the »AUX« connector of the alarm clock:

- Vibrating pillow A-3303-0
- Vibration contraption A-3310-0
- Flash module A-3320-0
- Acoustic module A-3340-0
- Telephone cable (optional)
- Microphone cable (optional)

### Setup

Connect the power supply unit to the 12 V power socket of the alarm clock and plug it into a 230 V power outlet.

### Setting the time

To set the time, press the time set button »« for one second. The hour digits will start to flash. Set the desired hour by turning the adjusting wheel (on the reverse side of the of the clock) to the left or right.

Press the time set button »« again. The minute digits will start to flash. Turn the adjusting wheel to the left or right to set the desired minutes.

Pressing the time set button »« again will end the time setting procedure and the set time is displayed.

### Setting the alarm time

To set the alarm time, press the alarm time button »« for one second. The hour digits of the alarm time will start to flash. Set the desired hour by turning the adjusting wheel to the left or right.

Press the alarm time button »« again. The minute digits of the alarm time will start to flash. Turn the adjusting wheel to the left or right to set the desired minutes.

Pressing the alarm time button »« again will end the alarm time setting procedure and the set alarm time is displayed.

## Alarm ON / OFF

The waking alarm can be switched on and off with the slider switch on the left side of the alarm clock.

Slider in upper position (green area is visible): Alarm is switched ON.

Slider in lower position: Alarm is switched OFF.

## Alarm tone ON / OFF

Press the alarm tone button »« for one second to switch the alarm tone ON or OFF. If the alarm tone is on, the speaker symbol will be visible in the display.

## Setting the alarm tone volume

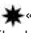
Press and hold the alarm tone button »« while adjusting the volume of the alarm tone. After 2 seconds, the alarm tone can be heard. Keep the alarm tone button pressed and turn the adjusting wheel to the left or right to adjust the desired alarm tone volume. Release the alarm tone button to end the volume setting procedure.

## Setting the alarm tone pitch

While adjusting the alarm tone pitch, press and hold both the alarm tone button »« and the spherical snooze button at the same time. After 2 seconds, the alarm tone can be heard. Keep the alarm tone button and the snooze button pressed and turn the adjusting wheel to the left or right to

adjust the desired alarm tone pitch. Release the buttons to end the pitch setting procedure.

## Flash ON / OFF

Press and hold the flash button »« for one second to switch the flash ON or OFF. If the flash is on, the flash symbol will be visible in the display.

## Setting the nighttime display brightness

The brightness of the displayed digits is automatically adjusted according to the light conditions within the room. During the day, the digits are displayed with full brightness. In darkness, the set nighttime display brightness is activated.

To set the nighttime display brightness, press and hold the flash button »«. Keep the flash button pressed and turn the adjusting wheel to the left or right to adjust the desired display brightness for the night. Release the button to end the setting procedure.

If this adjustment is done during daylight conditions, the display will switch back to full brightness after a few seconds.

## Snooze function

You can interrupt a current alarm by pressing on the spherical snooze button. The alarm will be repeated after 5 minutes.

## 12 / 24 hour mode

The time can be displayed in a 12- or 24 hour mode.

The adjustment is made with the »24h/12h« switch on the reverse side of the alarm clock.

## Power cut backup

The battery compartment can hold 4 batteries of size AAA (not included). In the event of a power cut, these guarantee the continuation of the set time.

**Please note:** During a power cut, the time display and the flash are not active as the power consumption is too high for the backup batteries!

## Telephone ring signalling

The »DS-2« can indicate incoming telephone calls by flash and sound. There are two connection possibilities to pick up an incoming phone call:

1. You can pick up the telephone ring signal by using a modem- or answering-machine type telephone cable (optionally available). The cable connects the alarm clock with a secondary connector of the telephone socket.
2. You can pick up the ring signal from the sound outlet of the telephone by using an external microphone (optionally available) which is fixed with a self-adhesive pad to the sound outlet.

The microphone cable is connected

to the "MIC" connection socket on the reverse side of the »DS-2«.

### Please note:

The ringing signal of the telephone is set factory-wise and cannot be altered with the tone adjustment buttons.

## Maintenance and care

The »DS-2« does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

## Warranty

The »DS-2« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

**Disposal** of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units. You help to protect the environment and health of your fellow men by the correct disposal of this product.

Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material.

You will receive further information on the recycling of products from your local community, our communal disposal company or your local dealer.

**Technical data**

Power supply:	DC 12 V / 1000 mA
Power consumption:	approx. 2 W
Height:	103 mm
Width:	212 mm
Depth:	70 mm
Weight:	337 g

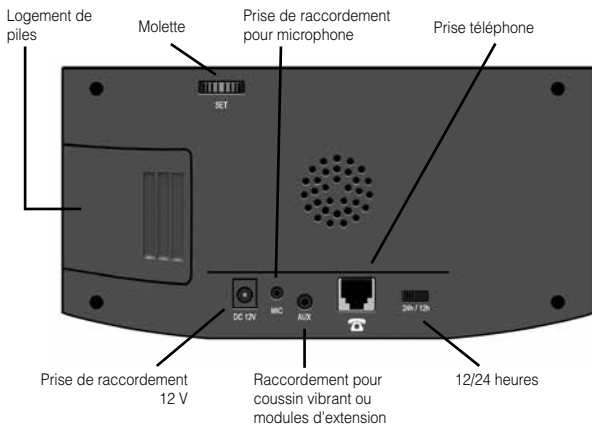
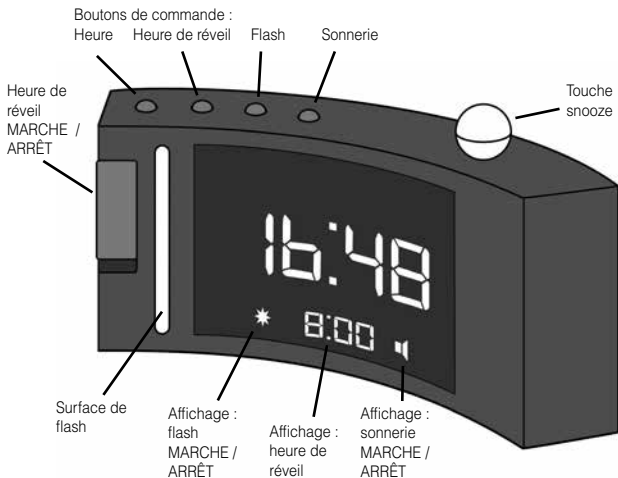
This device complies with the following EU directives:

- 2011 / 65 / EC RoHS directive
- 2012 / 19 / EC WEEE directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at [www.humantechnik.com](http://www.humantechnik.com).

Technical specifications subject to change without prior notice.

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**Humantechnik GmbH**Im Wörth 25
**Humantechnik GHL AG**Rastatterstrasse 9
**Sarabec Ltd**15 High Force Road