HUMANTECHNIK

lisa

D

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Seite 2

lisa time S

GB

Operating Instructions

Page 6

lisa time S

F

Mode d'emploi

Page 10

lisa time S

NL

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Pagina 14

lisa time S

I

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Pagina 18

lisa time S

E

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Página 22

lisa time S

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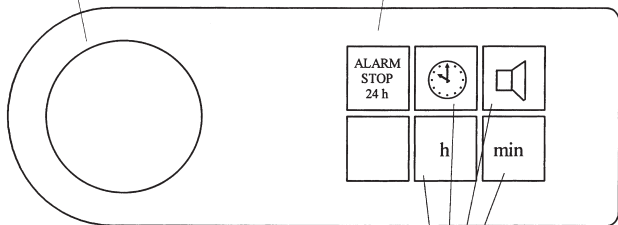
Snooze function
(alarm is repeated after 9 min)

Alarm stop for 24 hours


Connection socket for
special accessories:
vibrating pillow or modules

Setting buttons
Time/alarm setting

Switching output

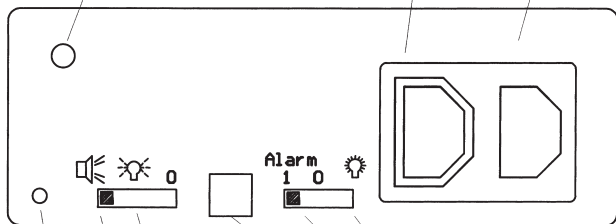Power connector

Display
brightness
setting

Flashing light
Flashing light and
audible alarm

Lamp ON
Alarm ON/OFF
Channel selection switch

We would like to congratulate you on the purchase of our *lisa time S* and hope that it will serve you well for a long time to come. You have chosen a modern and reliable system. Please read the operating instruction through carefully to be able to start the unit correctly and to familiarize yourself with all of the system features.

lisa time S is an electronic digital clock with a receiver to the *lisa* signalling system. It wakes you up using a flashing light and/or vibration and/or acoustic signal.

Additionally, the wake-up or *lisa* signal is issued through a switched socket.

The following special accessories can be connected to the alarm clock:

- Vibrating pillow/vibration system
- Flash module
- Acoustic module

Standard components

Please check if all following components are included:

- *lisa time S*
- 230 V mains lead
- 230 V switched socket
- Operating instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

Using your clock

Plug the mains plug of the *lisa time S* into a 230V mains socket. The display

flashes as a sign that the time is not correctly set. As soon as the time has been set the flashing stops.

Setting the time

Keep the button  and the **h** or **min** button pressed simultaneously to set the desired time.

Setting the alarm time

Keep the button  and the **h** or **min** button pressed simultaneously to set the desired alarm time.

Checking the alarm time set

Press the button  the alarm time is shown in the display of the alarm clock. A flashing dot between the hour and the minutes on the display shows that the alarm has been set.

Alarm signal

After reaching the alarm time set the *lisa time S* starts to issue flashing lights and/or an acoustic signal depending on the setting. If a vibration cushion is connected the vibration pulse is also simultaneously triggered. With the switched socket the alarm signal can be passed to a lamp or a vibration system. If an alarm signal is not interrupted by pressing the »Alarm Stop 9 min« or the »ALARM STOP 24 h« button, the alarm signal is repeated after 4 minutes (auto snooze).

Interrupting/switching off the alarm signal:

For 9 minutes (snooze):

By pressing the snooze button
»Snooze function«

For 24 hours:

By pressing the »ALARM STOP 24 h«

For a longer period (weekend, holiday):

Reset the switch on the back to **0**

Special features of *lisa time S*

- When using the alarm clock for the first time it is possible that the internal battery needs to be recharged. This means that the alarm clock will only be fully functional after about six hours.
- *lisa time S* is also a receiver for the *lisa* signalling system. From the 4 signal lights you can recognise which alarm signals (door, telephone, baby, alarm or beeper) are being issued. As with all *lisa* devices *lisa time S* has a switch to change channels (also see *lisa* operating instructions).
- The brightness of the display can be set by a potentiometer.
- The built-in battery can bridge a power failure of at least one hour. During this time the time and alarm setting remain saved.

- If a light is connected it can be switched on when the switch is in the "light on" position. Please note that when the switch is in this position the switch output does not have an alarm function.

Maintenance and care

The *lisa time S* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The *lisa time S* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units. You protect the environment and health of

your fellow man by the correct disposal of this product.

Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, your communal disposal company or your local dealer.

Technical data

| | |
|--------------------------------------|---------------|
| Power supply: | 230 V, 50 Hz |
| Power consumption: | approx. 1 W |
| Maximum load of the switched socket: | 500 W |
| Height: | 57 mm |
| Width: | 170 mm |
| Depth: | 190 mm |
| Weight: | approx. 930 g |

This product complies with European Union norms.

You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

Technical specifications subject to change without notice.

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Humantechnik GHL AG