HUMANTECHNIK

*lisa time flash*

**D**

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Seite 2

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**GB**

## **Operating Instructions**

Page 6

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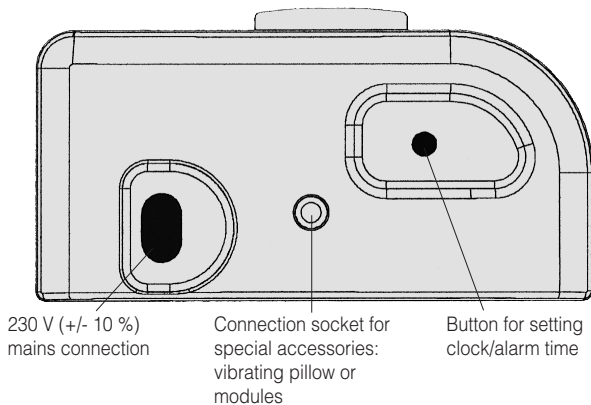
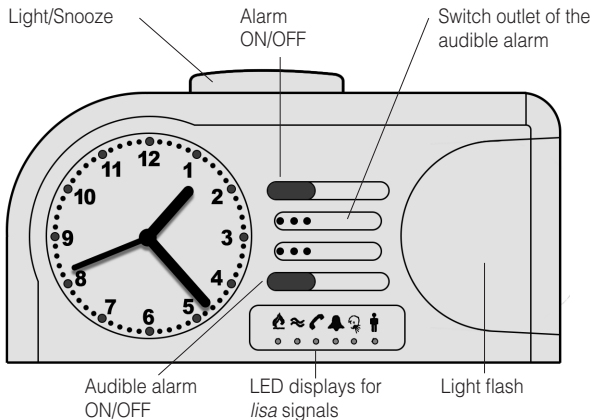
We would like to congratulate you on the purchase of our *lisa time flash* and hope that it will serve you well for a long time to come. You have chosen a modern and reliable system. Please read through the operating instruction carefully to be able to start the unit correctly and to familiarize yourself with all of the system features.

The *lisa time flash* is an electronic clock with an analogue time display. It wakes you up with a flashing light and/or an acoustic alarm. The following special accessories can be connected to the alarm clock:

- Vibrating pillow
- Flash module
- Switch module
- Acoustic module.

In addition, the *lisa time flash* has a receiver for the *lisa* signaling system.

### **Standard components**

Please check if all following components are included:

- *lisa time flash*
- 230 V power supply unit
- Operating instructions
- Warranty card

### **Startup**

Connect the supplied mains lead to the socket on the alarm clock and plug it into a 230 V power outlet.

### **Time setting**

Carefully pull out the button for setting the clock/alarm time and, with

the button pulled out, turn it in the direction of the arrow until the desired time is set.

Note: Please do not use excessive force when pulling out the button for setting the clock/alarm time!

### **Alarm setting**

Turn the button for setting the clock/alarm time in the direction of the arrow until the required alarm time is set.

### **Switching on the alarm**

Switch on the alarm using the ON/OFF switch.

Switch not pressed: Alarm switched on (green outline visible around switch).

Switch pressed: Alarm switched off.

### **Switching on the audible alarm**

Switch on the audible alarm using the audible alarm ON/OFF switch.

Switch not pressed: Audible alarm switched on (green outline visible around switch).

Switch pressed: Audible alarm switched off.

### **Snooze Alarm**

Press the Light/Snooze button to suspend a alarm which is in progress. The alarm is triggered again after about 4 minutes.

### Switching on the clock face illumination

Press the Light/Snooze button to switch on the clock face illumination for about 4 seconds.

### Special aspects of the alarm clock

- The alarm signal is repeated after 4 minutes (Autosnooze) unless you press the Light/Snooze button or the alarm ON/OFF button.
- The *lisa time flash* is also used as a receiver for the *lisa* light signaling system. The 6 signal lamps on the front of the clock let you know which transmitter signals are active (fire alarm, water alarm, telephone, doorbell, babymonitor, person call or alarm).
- Like all *lisa* units, the *lisa time flash* has a channel selection switch (see also the *lisa* operating instructions). This switch is located on the underside of the unit.
- The quartz mechanism of the alarm clock has a backup power supply. The clock will continue to operate for about 2 hours if there is a power failure. The alarm function and the illumination are deactivated during this period.
- The clock face illumination is also activated when the alarm is triggered.
- As well as the green marking of the Alarm ON/OFF switch, the Snooze button is also illuminated when the alarm is triggered.

- You can select between two different alarm types.  
The alarm type is altered as described below:  
-- Switch off the alarm --  
Press and hold the Snooze button for 3 seconds  
-- Switch the alarm back on --  
The Snooze button flashes to indicate that the second alarm type has been selected.

### Maintenance and care

The *lisa time flash* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

## Warranty

The *lisa time flash* is very safe to operate and provides highly reliable service. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will not only be carried out under warranty if the complete warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

**Disposal** of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling

of electrical and electronic units. You protect the environment and health of your fellow man by the correct disposal of this product. Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, your communal disposal company or your local dealer.

## Technical data

Power supply: 230 V, 50 Hz

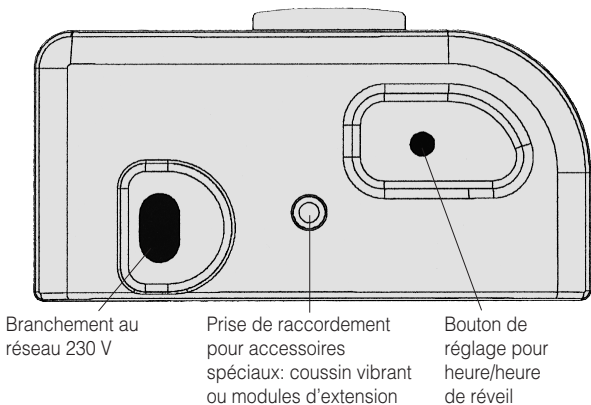
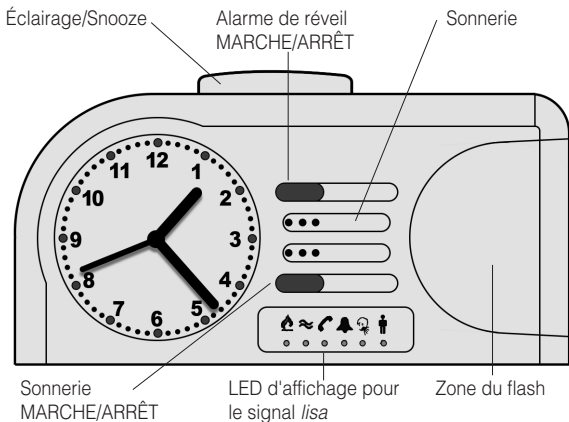
Power consumption: 3 W

This device satisfies the following EU directives:

- 2011/65/EG RoHS directive
  - 2002/96/EG WEEE directive
  - 1999/5/EG R&TTE directive
  - 2004/108/EG EMC directive
  - 2006/95/EG Low voltage directive
- Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at [www.humanteknik.com](http://www.humanteknik.com).

Technical specifications subject to change without notice.

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**Sarabec Ltd**