HUMANTECHNIK

lisa vibro time

D

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Seite 2

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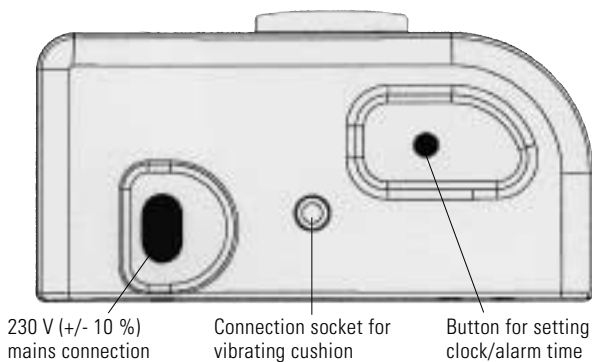
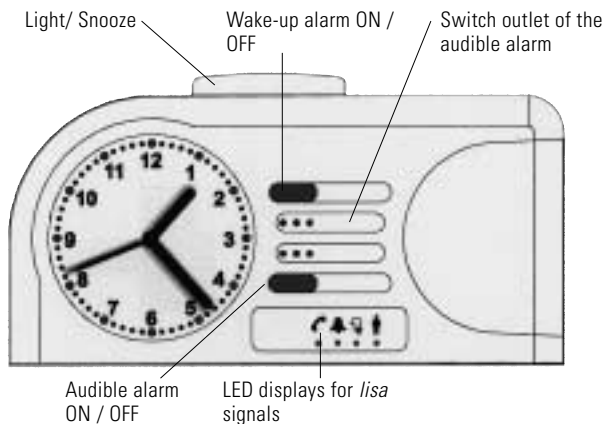
Stromversorgung: 230 V, 50 Hz

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Congratulations on purchasing your *lisa vibro time*. You have selected a modern and reliable system. The *lisa vibro time* is an electronic clock with an analogue time display. It wakes you up using vibration and/or an audible alarm (a vibrating cushion is supplied with the product). In addition, the *lisa vibro time* has a receiver for the *lisa* signaling system.

Standard components

- *lisa vibro time*
- vibrating cushion
- 230 V power supply unit
- Warranty card

Startup

Connect the supplied mains lead to the socket on the alarm clock and plug it into a 230 V power outlet.

Time setting

Carefully pull out the button for setting the clock/alarm time and, with the button pulled out, turn it in the direction of the arrow until the desired time is set.

Note: Please do not use excessive force when pulling out the button for setting the clock/alarm time!

Setting the wake-up time

Turn the button for setting the clock/alarm time in the direction of the arrow until the required alarm time is set.

Switching on the wake-up alarm

Switch on the wake-up alarm using the ON / OFF switch.

Switch not pressed: Wake-up alarm switched on (green outline visible around switch).

Switch pressed: Wake-up alarm switched off.

Switching on the audible alarm

Switch on the audible alarm using the Audible alarm ON / OFF switch.

Switch not pressed - Audible alarm switched on (green outline visible around switch).

Switch pressed: Audible alarm switched off.

Snooze Alarm

Press the Light/Snooze button to suspend a wake-up alarm which is in progress. The wake-up alarm is triggered again after about 5 minutes.

Face illumination (back light)

Touching the Light/Snooze Alarm/*lisa* signal cancel switch will light the face.

Switching on the clock face illumination

Press the Light/Snooze button to switch on the clock face illumination for about 4 seconds.

Special aspects of the alarm clock

- The alarm signal is repeated after 4 minutes (Autosnooze) unless you press the Light/Snooze button or the Wake-up alarm ON / OFF button.
- The *lisa vibro time* is also used as a receiver for the *lisa* light signaling system.

The 4 signal lamps on the front of the clock let you know which transmitter signals are active (door, phone, baby alarm, alarm or pager).

- Like all *lisa* units, the *lisa vibro time* has a channel selection switch (see also the *lisa* operating instructions). This switch is located on the underside of the unit.
- The quartz mechanism of the alarm clock has a backup power supply.

The clock will continue to operate for about 2 hours if there is a power failure.

The wake-up alarm function and the illumination are deactivated during this period.

- The clock face illumination is also activated when the alarm is triggered.
- As well as the green marking of the Alarm ON / OFF switch, the Snooze button is also illuminated when the alarm is triggered.
- You can select between two different alarm types.

The alarm type is altered as described below:

- Switch off the alarm -- Press and hold the Snooze button for 3 seconds --

Switch the alarm back on --

The Snooze button flashes to indicate that the second alarm type has been selected.

Maintenance and care

The *lisa vibro time* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The *lisa vibro time* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer.

The warranty covers the repair of the product and returning it to you free of charge during the warranty period. It is essential that you send the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorized. Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer as well as a copy of the sales slip. Always specify the product number in any event.

Disposal

Disposal of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

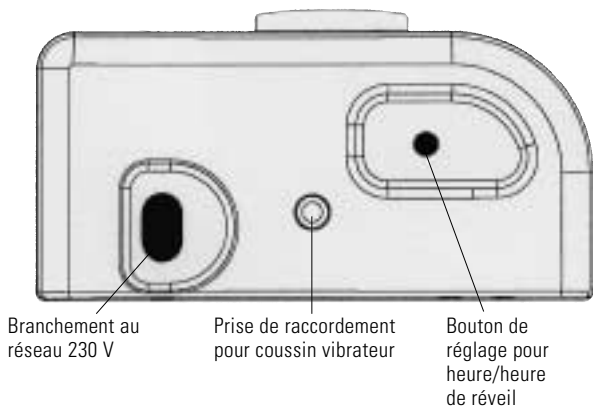
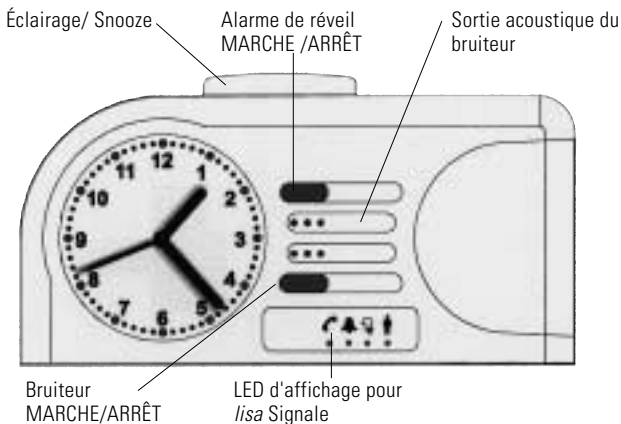
The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units. You protect the environment and health of your fellow man by the correct disposal of this product. Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, your communal disposal company or your local dealer.

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Humantechnik GHL AG