HUMANTECHNIK

lisa
RF time

DE

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Seite 2

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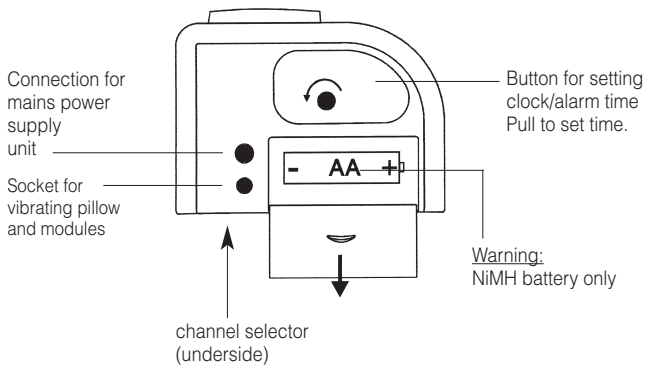
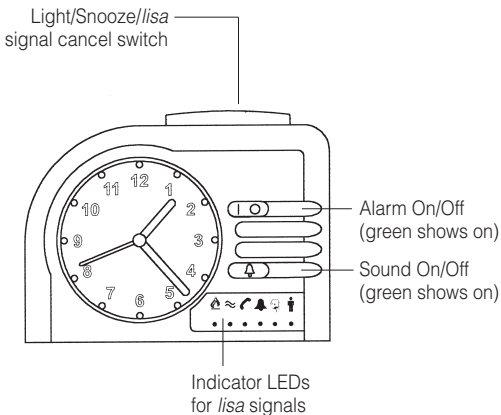
We would like to congratulate you on the purchase of our *lisa RF time* and hope that it will serve you well for a long time to come. You have chosen a modern and reliable system.

Please read through the operating instruction carefully to be able to start the unit correctly and to familiarize yourself with all of the system features.

Standard components

Please check if all following components are included:

- *lisa RF time*
- vibrating pad
- 230 V power supply unit
- Operating Instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

Operating principle

The *lisa RF time* alarm clock is an electronic clock with an analog dial. It is also a complete receiver for the *lisa* radio transmitters. It possesses a powerful vibrating pad, which alerts you to a signal, as well as 6 LEDs which indicate the type of signals that are being received.

Installing the *lisa RF time*

Open the battery compartment of the *lisa RF time* and remove the paper at the + contact of the battery.

Connect the mains power supply unit to the *lisa RF time* and plug into a mains socket.

Time setting

Lightly pull the time and alarm setting knob and turn it in the direction of the arrow.

Warning: Pull the knob out carefully and only turn it in the direction of the arrow!

Alarm setting

Turn the time and alarm setting knob in the direction of the arrow.

Switching on the alarm

Push the alarm On/Off button. When you see the green marking the alarm is on.

Switching on the sound

Push the sound On/Off button. When you see the green marking the sound is on.

Snooze Alarm

Touching the Light/Snooze Alarm/*lisa* signal cancel switch will stop the alarm and it will start again in approximately 5 minutes.

Face illumination (back light)

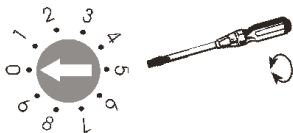
Touching the Light/Snooze Alarm/*lisa* signal cancel switch will light the face for approx 4 seconds.

Setting the correct radio channel

A special digital transmission process ensures that the radio signals are picked up without errors. This process also makes it possible to set 10 different radio channels.

When supplied from the factory, all *lisa RF time* are set to channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building.

Set the required channel by using a small screwdriver to turn the channel selector switch, which is located on the lower side of the clock, to the required

channel number. Use a small screwdriver to adjust this switch. All transmitters and receivers must be set to the same channel.

Transmission Signal Recognition

The *lisa RF time* converts the various transmission signals into varying sequences of vibrating and signal displays.

Telephone: Telephone LED lights

Vibration V P14 V P30 _____

Door I: Door LED lights

Vibration V P14 V P14 B P14 V P40 _____

Door II: Door LED flashes

Vibration V P20 V P20 V P20 V P40 _____

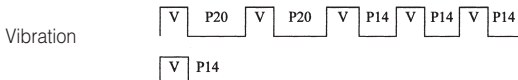
Baby monitor: Baby LED lights

Vibration V P35 V P40 _____


Call button: Call button LED lights

Vibration V P12 V P12 _____


Alarm: All LEDs flashes

Water alarm: Water alarm LED lights

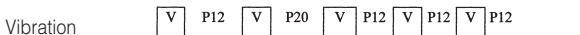Fire alarm: Fire alarm LED lights

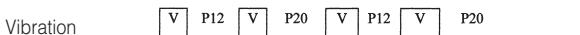P12 Pause 1,2 s

P14 Pause 1,4 s

P20 Pause 2 s

P35 Pause 3,5 s

P40 Pause 4 s

V Vibration

The indicator LEDs remain on for 40 seconds.

Touching the Light/Snooze switch cancel any vibration caused by *lisa* signals. The indicator LEDs light the normal time (40 seconds).

Recalling the most recent signal

The *lisa RF time* stores each signal it receives for a period of 3 minutes. During this time, you can call up the signal again by pressing the Light/Snooze/*lisa* signal cancel switch button. The signal is then displayed on the indicator LEDs. A vibration signal is not triggered.

Radio range

Note the following points when using the *lisa RF time*:

The radio signals are designed to have a range of 80 meters under optimum conditions. The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation.
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.

- Positioning of the transmitter at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

Power failure backup

The battery can provide backup power for a mains power outage of up to 72 hours. All functions are available during this period.

Battery empty

If the pager LED lights up without a *lisa* signal the battery voltage is too low.

Symptom: Flat battery indicator (Pager LED) lights up permanently.

Cause: Power supply unit isn't connected to the *lisa RF time*.

Battery is faulty - replace battery with a new one.

Attention: Use only a NiMH battery. After the unit has been disconnected

from the mains supply for several months it must be plugged into the mains supply for several hours before the unit will function reliably on batteries again.

Maintenance and care

The *lisa RF time* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

Warranty

The *lisa RF time* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units. You protect the environment and health of your fellow man by the correct disposal of this product. Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, your commu-

nal disposal company or your local dealer.

Technical data

Power supply:

12 V power supply unit at 230 V, 50 Hz

Power consumpt.: approx. 1 W

Frequency: 868.35 MHz

Height: 85 mm

Width: 120 mm

Depth: 50 mm

Weight: approx. 185 g

This product complies with European Union norms.

CE compliance declarations are available on the Internet at www.humanteknik.com in the "service" section.

Technical specifications subject to change without notice.

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Sarabec Ltd