HUMANTECHNIK

*lisa* ((•))  
RF time flash

**D**

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Seite 2

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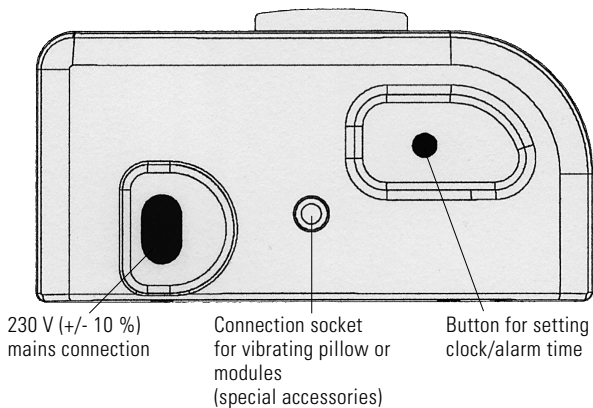
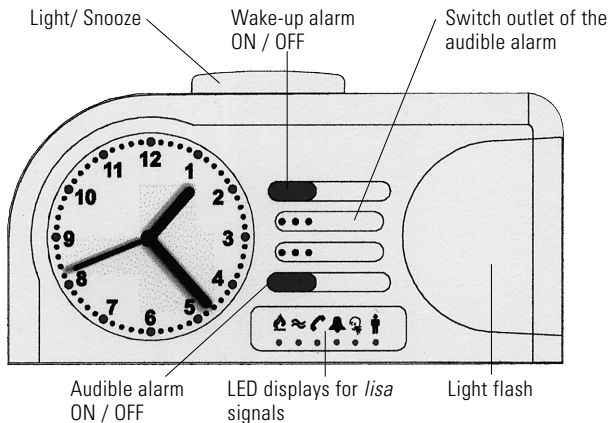
We would like to congratulate you on the purchase of our *lisa RF time flash* and hope that it will serve you well for a long time to come. You have chosen a modern and reliable system. Please read the operating instruction through carefully to be able to start the unit correctly and to familiarize yourself with all of the system features.

The *lisa RF time flash* is an electronic clock with an analogue time display and a receiver for the radio signals from various *lisa* radio transmitters. When a radio signal is detected, you are alerted in the form of a flashing light / acoustic alarm and 6 light-emitting diodes. It is also possible to connect a vibrating pillow or modules (special accessories).

### **Standard components**

Please check if all following components are included:

- *lisa RF time flash*
- 230 V power supply unit
- Operating instruction
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

### **Startup**

Connect the supplied mains lead to the socket on the alarm clock and plug it into a 230 V (+/- 10 %) power outlet.

### **Time setting**

Lightly pull the time and alarm setting knob and turn it in the direction of the arrow.

Warning: Pull the knob out carefully and only turn it in the direction of the arrow!

### **Alarm setting**

Turn the time and alarm setting knob in the direction of the arrow.

### **Switching on the alarm**

Push the Alarm On/Off button. When you see the green marking the alarm is on.

### **Switching on the sound**

Push the Sound On/Off button. When you see the green marking the sound is on.

### **Snooze Alarm**

Touching the Light/Snooze Alarm/*lisa* signal cancel switch will stop the alarm and it will start again in approximately 5 minutes.

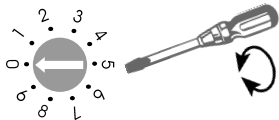
### **Face illumination (back light)**

Touching the Light/Snooze Alarm/*lisa* signal cancel switch will light the face.

### **Setting the correct radio channel**

A special digital transmission process ensures that the radio signals are received error-free. This process also makes it possible to set 10 different radio channels. When sup-

plied from the factory, the *lisa RF time flash* is set to channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building. Set the channel you want by adjusting the channel selector switch on the underside of the unit to the required channel number. Use a small screwdriver to adjust this switch. All transmitters and receivers within your system must be set to the same channel.

## Transmission Signal Recognition

The *lisa RF time flash* converts the various transmission signals into varying sequences of flash and signal displays.

**Telephone:** Telephone LED lights

Flashing - sequence B P14 B P30 W

**Doorbell I:** Doorbell LED lights

Flashing - sequence B P14 B P14 B P14 B P40 W

**Doorbell II:** Doorbell LED flashes

Flashing - sequence B P20 B P20 B P20 B P40 W

**Babymonitor:** Baby Monitor LED lights

Flashing - sequence B P35 B P40 W

**Pager:** Pager LED lights

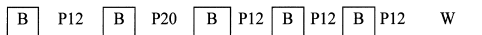
Flashing - sequence B P12 B P12 W

**Alarm:** All LEDs flash

Flashing - sequence B P20 B P20 B P14 B P14 B P14 W


## Water alarm: Water Alarm LED lights

Flashing -  
sequence

## Fire alarm: Fire Alarm LED lights

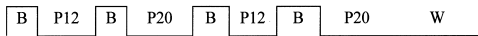Flashing -  
sequence

**P12** Pause 1,2 seconds

**P14** Pause 1,4 seconds

**P20** Pause 2,0 seconds

**B** flash

**P30** Pause 3,0 seconds

**P35** Pause 3,5 seconds

**P40** Pause 4,0 seconds

**W** repeat

The indicator LEDs remain on for 40 seconds.

Touching the Light/Snooze switch cancel any flash alarm caused by *lisa* signals. The indicator LEDs light the normal time (40 seconds).

## Recalling the most recent signal

The *lisa RF time flash* stores each signal it receives for a period of 3 minutes. During this time, you can call up the signal again by pressing the Light/Snooze *lisa* signal cancel switch button. The signal is then displayed on the indicator LEDs. A flash signal is not triggered.

## Radio range

The *lisa* transmitters are designed to have a radio signal range of 80 meters under optimum conditions.

The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones.  
Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavorable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.

- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

### Special aspects

- The alarm signal is repeated after 4 minutes (Autosnooze) unless you press the Light/Snooze button or the Wake-up alarm ON / OFF button.
- The clock face illumination is also activated when the alarm is triggered.
- As well as the green marking of the Alarm ON / OFF switch, the Snooze button is also illuminated when the alarm is triggered.
- You can select between two different alarm types. The alarm type is altered as described below:
  - Switch off the alarm --  
Press and hold the Snooze button for 3 seconds.
  - Switch the alarm back on.  
The Snooze button flashes to indicate that the second alarm type has been selected.

### Maintenance and care

The *lisa RF time flash* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the *lisa RF time flash* where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

### Warranty

The *lisa RF time flash* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge during the warranty period. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorized to do so (do not break the seal on the unit).

Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer or the original warranty certificate if you purchased the unit directly from the manufacturer.

**Always specify the product number in all cases.**

### Disposal

Disposal of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units.

You protect the environment and health of your fellow man by the correct disposal of this product. Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, your communal disposal company or your local dealer.

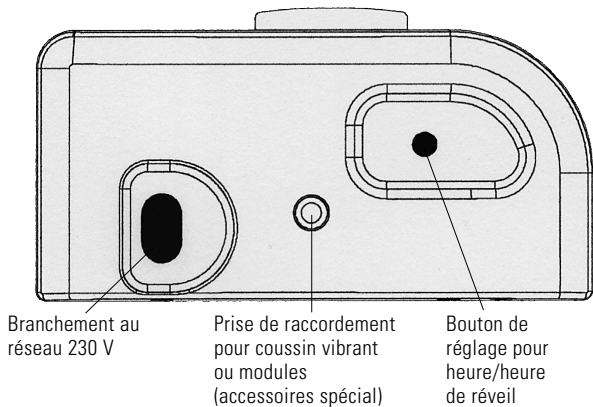
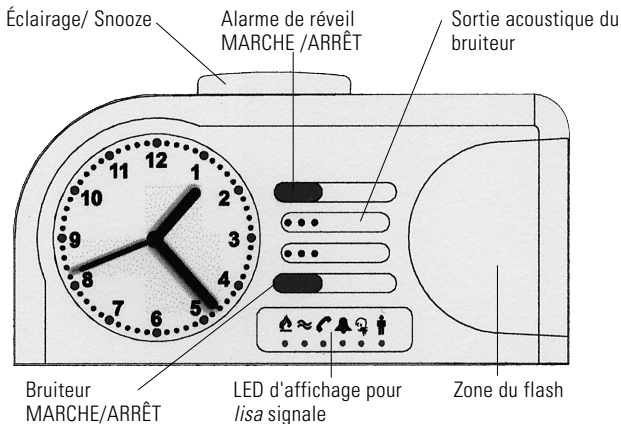
### Technical data:

Power supply:	230 V, 50 Hz
Power consumpt.	3 W
Frequency:	868.35 MHz

This product complies with European Union norms. You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

Technical specifications subject to change without notice.

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## **Humantechnik GHL AG**