lisa
RF time flash

DE

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Seite 2

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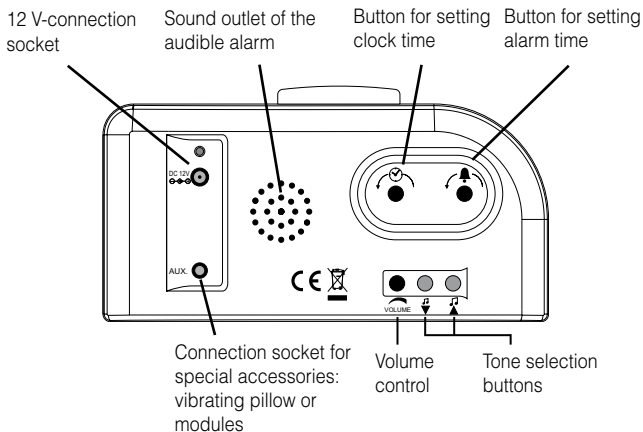
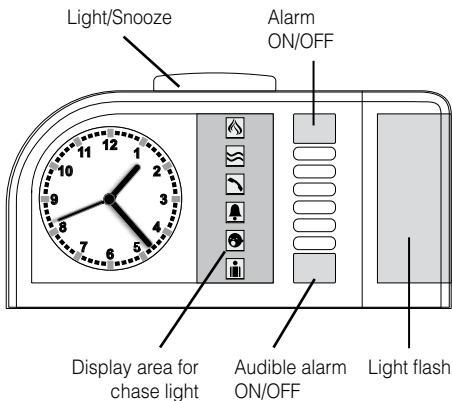
We would like to congratulate you on the purchase of our »*lisa* RF time flash« alarm clock. You have chosen a modern and reliable system. Please read the operating instruction carefully to be able to start the unit correctly and to familiarize yourself with all of the system's features.

Standard components

Please check if all following components are included:

- »*lisa* RF time flash«
- 12V power supply unit
- Operating instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

Functioning principle

The »*lisa* RF time flash« is an electronic clock with an analogue time display and a receiver for the radio signals from various *lisa* radio transmitters. When a radio signal is detected, you are alerted in the form of a flashing light/acoustic alarm and 6 light-emitting diodes.

The following special accessories can be connected to the alarm clock:

- Vibrating pillow
- Flash module
- Switch module
- Acoustic module

Startup

Plug in the power supply unit of the »*lisa* RF time flash« into a 230 V power outlet and connect the cable with the 12 V connection socket.

Time setting

Turn the button for setting the time  in the direction of the arrow until the desired time is set.

Alarm setting

Turn the button for setting the alarm  in the direction of the arrow until the required alarm time is set.

Switching on the alarm

Switch on the alarm using the ON/OFF switch.

Switch not pressed: Alarm switched on (green outline visible around switch).

Switch pressed: alarm switched off.

Switching on the audible alarm

Switch on the audible alarm using the audible alarm ON/OFF switch.

Switch not pressed: Audible alarm switched on (green outline visible around switch).

Switch pressed: Audible alarm switched off.

Select alarm tone/melody

You can select the alarm tone/melody with the tone selection buttons. You can choose from 10 different types of tone sequences.

Snooze Alarm

Press the Light/Snooze button to suspend an alarm which is in progress. The alarm is triggered again after about 5 minutes.

Switching on the clock face illumination

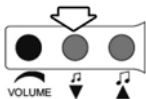
Press the Light/Snooze button to switch on the clock face illumination for about 6 seconds.

Switching the flash on and off

The flash feature of the *lisa* RF time-flash alarm clock can be switched on and off.

To switch the flash off, please proceed as follows:

- Switch off the alarm and the tone (Switches are pressed in, the green edge is not visible).
- Now press the alarm repeat button on the top side of the alarm clock and keep it held down.
- Press six times on the left tone adjuster button (situated at the right side of the volume control).

When pressing the button you will hear a beep tone.

- Let go of the alarm repeat button. The flash is now switched off.

To switch the flash on, please proceed as described above, but press six times on the right tone adjuster button.

The flash is now switched on again.

Setting the correct radio channel

A special digital transmission process ensures that the radio signals are received error-free. This process also makes it possible to set 10 different radio channels. When supplied from the factory, the »lisa RF time flash« is set to channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building. Set the channel you want by adjusting the channel selector switch on the underside of the unit to the required channel number. Use a small screwdriver to adjust this switch. All transmitters and receivers within your system must be set to the same channel.

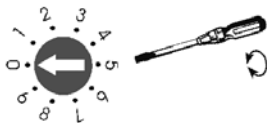
Fig. setting the correct radio channel.

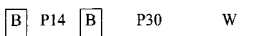
Transmission Signal Recognition

The »lisa RF time flash« converts the various transmission signals into varying sequences of flash and signal displays.

Telephone:

Telephone LED is lit

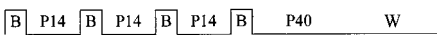
Flashing-
sequence

Doorbell I:

Doorbell LED is lit

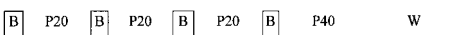
Flashing-
sequence

Doorbell II:

Doorbell LED flashes

Flashing-
sequence

Babymonitor:

Baby LED is lit

Flashing-
sequence

Pager:

Pager LED is lit

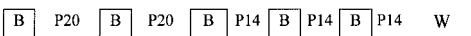
Flashing-
sequence

Alarm:

All LEDs flash

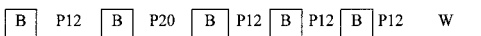
Flashing-
sequence

Water alarm:

Water Alarm LED is lit

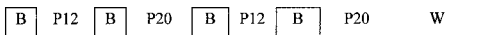
Flashing-
sequence

Fire alarm:

Fire Alarm LED is lit

Flashing-
sequence

P12 Pause 1,2 seconds

P30 Pause 3,0 seconds

P14 Pause 1,4 seconds

P35 Pause 3,5 seconds

P20 Pause 2,0 seconds

P40 Pause 4,0 seconds

B Flash

W Repeat

The indicator LEDs remain on for 40 seconds.

Touching the Light/Snooze switch cancel any flash alarm caused by *lisa* signals.
The indicator LEDs light the normal time (40 seconds).

Recalling the most recent signal

The »lisa RF time flash« stores each signal it receives for a period of 3 minutes.

During this time, you can call up the signal again by pressing the Light/Snooze lisa signal cancel switch button. The signal is then displayed on the indicator LEDs. A flash signal is not triggered.

Radio range

The lisa transmitters are designed to have a radio signal range of 80 meters under optimum conditions.

The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavorable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there maybe many other sources of radio waves which can disrupt the original signal.

- Having units operating at similar frequencies located close to one another may also result in mutual interference.

Special aspects

- The alarm signal is repeated after 5 minutes (autosnooze) unless you press the Light/Snooze button or the alarm ON/OFF button.
- The quartz mechanism of the alarm clock has a backup power supply. The clock will continue to operate for about 2 hours if there is a power failure. The alarm function and the illumination are deactivated during this period.
- The clock face illumination is also activated when the alarm is triggered.
- In addition to the green marking of the Alarm ON/OFF switch, the snooze button is also illuminated when the alarm is triggered.
- The »lisa RF time flash« converts the various transmission signals (Telephone, Doorbell I+II, Baby-monitor, Pager, Alarm, Water alarm, Fire alarm) into varying sequences of flash and signal displays.

Maintenance and care

The »lisa RF time flash« does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The »lisa RF time flash« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units. You protect the environment

and health of your fellow man by the correct disposal of this product. Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, your communal disposal company or your local dealer.

Technical data

Power supply:	230 V, 50 Hz; 12V DC 1000mA
Power consumption:	approx. 2 W
Height:	86 mm
Width:	175 mm
Depth:	53 mm
Weight:	337 g

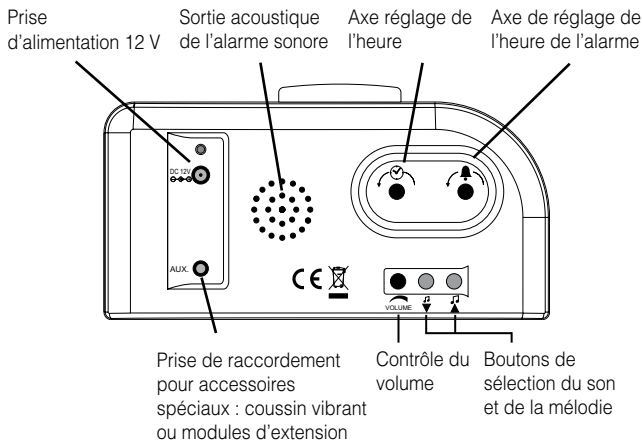
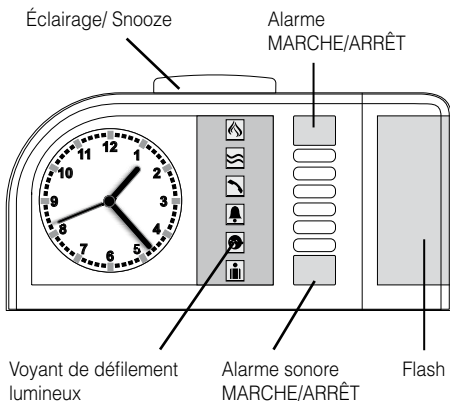
This device satisfies the following EU directives:

- 2011/65/EC RoHS directive
- 2002/96/EC WEEE directive
- 1999/5/EC R&TTE directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at www.humantechnik.com.

Technical specifications subject to change without notice.

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Sarabec Ltd