DE

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Seite 2

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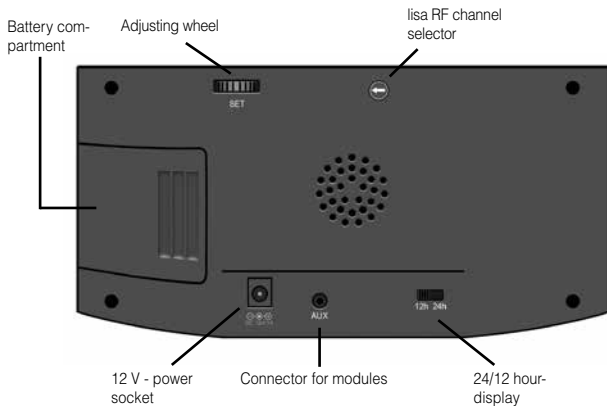
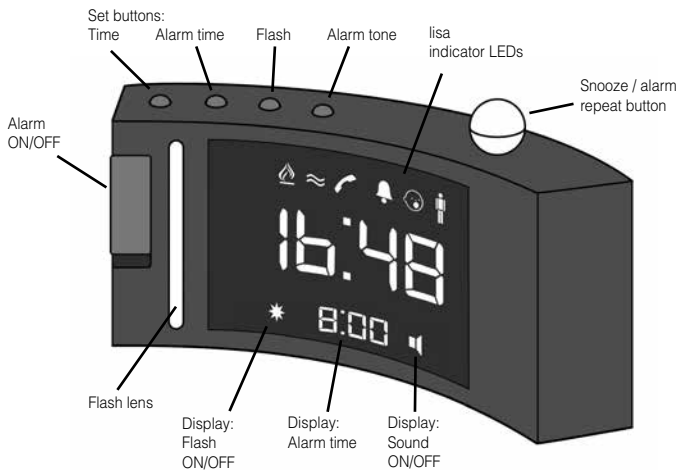
We would like to congratulate you on purchasing the »*lisa* DS-2/RF«. You have chosen a modern and reliable system. Kindly read the instruction manual carefully in order to use the device properly and to familiarise yourself with all the possibilities of the system.

Package contents

Kindly check if all following parts are included:

- »*lisa* DS-2/RF«
- 12 volts power supply unit
- Instruction manual
- Warranty card

Should any parts be missing, kindly contact your specialist dealer or the manufacturer directly.

Function

The »*lisa* DS-2/RF« is an electronic clock with a digital time display. It wakes you up with an acoustic signal which is adjustable in pitch and volume. It is also a complete *lisa* receiver, capable of displaying the signals of all *lisa* transmitters. A received signal is displayed with flash and/or sound and identified by 6 indicator LEDs.

The following accessories can be connected to the »AUX« connector of the alarm clock:

- | | |
|--------------------|----------|
| - Vibrating pillow | A-3303-0 |
| - Flash module | A-3320-0 |

Setup

Connect the power supply unit to the 12 V power socket of the alarm clock and plug it into a 230 V power outlet.

Setting the time

To set the time, press the time set button »« for one second. The hour digits will start to flash. Set the desired hour by turning the adjusting wheel (on the reverse side of the of the clock) to the left or right.

Press the time set button »« again. The minute digits will start to flash. Turn the adjusting wheel to the left or right to set the desired minutes.

Pressing the time set button »« again will end the time setting procedure and the set time is displayed.

Setting the alarm time

To set the alarm time, press the alarm time button »« for one second. The hour digits of the alarm time will start to flash. Set the desired hour by turning the adjusting wheel to the left or right.

Press the alarm time button »« again. The minute digits of the alarm time will start to flash. Turn the adjusting wheel to the left or right to set the desired minutes.

Pressing the alarm time button »« again will end the alarm time setting procedure and the set alarm time is displayed.

Alarm ON / OFF

The waking alarm can be switched on and off with the slider switch on the left side of the alarm clock.

Slider in upper position (green area is visible): Alarm is switched ON.

Slider in lower position: Alarm is switched OFF.

Alarm tone ON / OFF

Press the alarm tone button »« for one second to switch the alarm tone ON or OFF. If the alarm tone is on, the speaker symbol will be visible in the display.

Setting the alarm tone volume

Press and hold the alarm tone button »« while adjusting the volume of the alarm tone. After 2 seconds, the alarm tone can be heard. Keep the alarm tone button pressed and turn the adjusting wheel to the left or right to adjust the desired alarm tone volume. Release the alarm tone button to end the volume setting procedure.

Setting the alarm tone pitch

While adjusting the alarm tone pitch, press and hold both the alarm tone button »« and the spherical snooze button at the same time. After 2 seconds, the alarm tone can be heard.

Keep the alarm tone button and the snooze button pressed and turn the adjusting wheel to the left or right to adjust the desired alarm tone pitch. Release the buttons to end the pitch setting procedure.

Flash ON / OFF

Press and hold the flash button »« for one second to switch the flash ON or OFF. If the flash is on, the flash symbol will be visible in the display.

Setting the nighttime display brightness

The brightness of the displayed digits is automatically adjusted according to the light conditions within the room. During the day, the digits are displayed with full brightness. In darkness, the set nighttime display brightness is activated.

To set the nighttime display brightness, press and hold the flash button »«. Keep the flash button pressed and turn the adjusting wheel to the left or right to adjust the desired display brightness for the night. Release the button to end the setting procedure. If this adjustment is done during daylight conditions, the display will switch back to full brightness after a few seconds.

Deactivating the illuminated digits

If the illuminated digits are too bright for you at night, even at their darkest setting, they can also be deactivated completely. This has no influence on all other functions!

Digits off: Press and hold the buttons »« and »« simultaneously for 5 seconds. For confirmation, the digits flash 2 times and then switch off after 60 seconds.

Digits on: Press and hold the buttons »« and »« simultaneously for 5 seconds. For confirmation, the digits flash 4 times and then switch on again immediately.

The luminous digits can be activated for 60 seconds at any time by pressing any key.

Snooze function

You can interrupt a current alarm by pressing on the spherical snooze button. The alarm will be repeated after 5 minutes.

12 / 24 hour mode

The time can be displayed in a 12- or 24 hour mode.

The adjustment is made with the »**24h/12h**« switch on the reverse side of the alarm clock.

Power cut backup

The battery compartment can hold 4 alkaline batteries of size AAA (not included).

In the event of a power cut, these guarantee the continuation of the set time and the display of the *lisa* signals.

Please note: During a power cut, the time display and the flash are not active as the power consumption is too high for the backup batteries!

Only use the specified battery type AAA (LR03) !

There is a risk of explosion if the wrong type of battery is used. The batteries must not be exposed to high or low extreme temperatures and low air pressure during use, storage or transport at high altitude.

Disposing of batteries in fires, ovens or mechanically crushing and cutting them can cause fires or explosions. Exposing batteries to extremely high temperatures may cause an explosion or leakage of flammable liquid or gas.

Batteries exposed to extremely low air pressure may explode or emit flammable liquid or gas.

Use only the original type power supply with the alarm clock.

***lisa* receiver functions:**

The alarm clock is equipped with all functions of a *lisa* RF receiver.

Recalling the most recent signal

The »*lisa* DS-2/RF« stores each signal it receives for a period of 3 minutes.

During this time, you can call up the signal again by pressing the snooze button. The signal is then displayed on the indicator LEDs. A flash signal is not triggered.

Setting the correct radio channel

A special digital transmission process ensures that the radio signals are received error-free. This process also makes it possible to set 10 different radio channels.

When supplied from the factory, the »*lisa* DS-2/RF« is set to channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building.

Set the desired channel by adjusting the channel selector switch on the rear side of the unit to the required channel number. Use a small screwdriver to adjust this switch. All transmitters and receivers within your system must be set to the same channel.

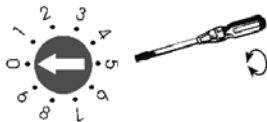
Fig. setting the correct radio channel.

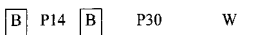
Transmission Signal Recognition

The »/isa DS-2/RF« converts the various transmission signals into varying sequences of flash and signal displays.

Telephone:

Telephone LED lights

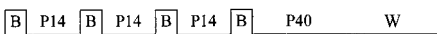
Flashing-
sequence

Doorbell I:

Türklingel-LED leuchtet

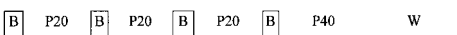
Flashing-
sequence

Doorbell II:

Doorbell LED flashes

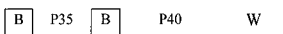
Flashing-
sequence

Babymonitor:

Baby LED lights

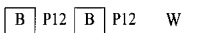
Flashing-
sequence

Pager:

Pager LED lights

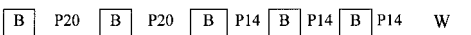
Flashing-
sequence

Alarm:

All LEDs flash

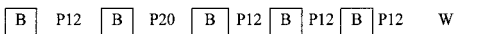
Flashing-
sequence

Water alarm:

Water Alarm LED lights

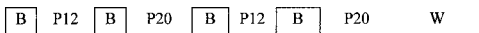
Flashing-
sequence

Fire alarm:

Fire Alarm LED lights

Flashing-
sequence

P12 Pause 1,2 seconds

P30 Pause 3,0 seconds

P14 Pause 1,4 seconds

P35 Pause 3,5 seconds

P20 Pause 2,0 seconds

P40 Pause 4,0 seconds

B Flash

W Repeat

The indicator LEDs remain on for 40 seconds.

Touching the Light/Snooze switch cancel any flash alarm caused by /isa signals.
The indicator LEDs light the normal time (40 seconds).

Radio range

The lisa transmitters are designed to have a radio signal range of 80 meters under optimum conditions.

The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavorable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there maybe many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

Maintenance and care

The »lisa DS-2/RF« does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic

solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The »lisa DS-2/RF« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units. You protect the environment and health of your fellow man by the correct disposal of this product.

Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, our communal disposal company or your local dealer.

Technical data

Power supply:	DC 12V / 1000mA
Powercut backup:	4 pcs AAA (LR03) alkaline batteries
Receiver frequency:	868.35 MHz
Power consumption:	approx. 2 W
Height:	103 mm
Width:	212 mm
Depth:	70 mm
Weight:	337 g

Hereby, Sarabec Ltd declares that this device complies with all relevant UK Statutory Instruments. The full declarations of conformity for this device are available from Sarabec Ltd., 15 High Force Road, MIDDLESBROUGH TS2 1RH. UK.

Sarabec's EU authorised representative:

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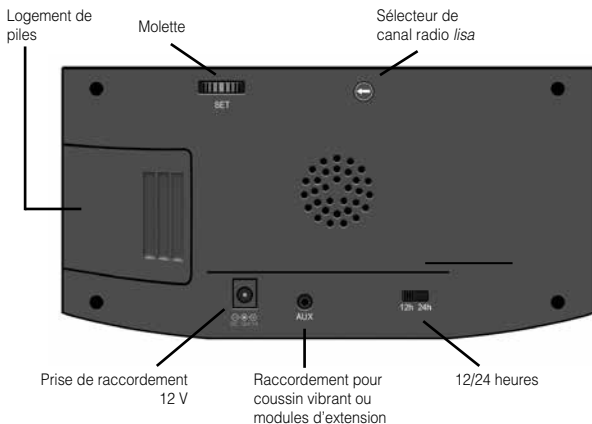
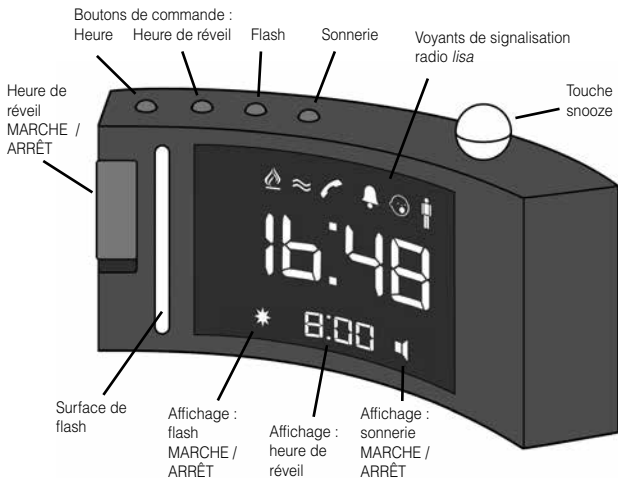
This device complies with the following EC directives:

- 2011 / 65 / EC + 2015 / 863 / EC RoHS-directive
- 2012 / 19 / EC WEEE-directive
- 2014 / 53 / EC RED-directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at www.humanteknik.com.

Technical specifications subject to change without prior notice.

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