signolux
Visual Signal System

DE

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Vibrationskissen

EN

User manual

Vibrating pad

FR

Mode d'emploi

Coussin vibrant

NL

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Trilkussen

IT

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Cuscino vibrante

ES

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Cojín vibratorio

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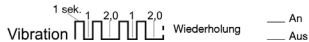
Alarm: Alarmsymbol (LED) rot

Alle Werte in Sekunden: z.B. 1,5 = 1,5 Sekunden

With the »signolux« alerting system, you have chosen a modern and reliable system. Please read this manual thoroughly in order to use the device correctly.

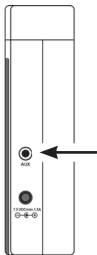
Safety instructions

Prior to putting the device into operation and using it, kindly read this manual thoroughly and in full. Keep this user manual in a safe place so that it is also accessible for other users at any time.

Functioning principle

This »signolux« vibrating pad has no batteries and can **only** be used together with the »signolux«-receivers, with exception of the standard receiver A-2614-0 and the plug-in receiver A-2644-0/W! The vibrating pad informs you about incoming event alerts through vibration.

Setup

Connect the vibrating pad with the »AUX« socket of the respective receiver. Trigger an alert to test the function (e.g. press the doorbell button).

Please note:

This vibrating pad works **only** with signolux-receivers! It is not compatible with with the standard receiver A-2614-0, the flash lamp A-2644-0/W or devices of the »lisa« system!

Event identification:

Doorbell 1: Doorbell icon 1 (green LED) is active

Doorbell 2: Doorbell icon 2 (green LED) is active

Telephone: Telephone icon (orange LED) is active

Person call: Person icon (yellow LED) is active

Babycry: Baby icon (white LED) is active

Fire alarm: Fire icon (red LED) is active

Water alarm: Water icon (blue LED) is active

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Humantechnik GmbH
Humantechnik GHL AG
Sarabec Ltd.