HUMANTECHNIK

***MS-1***

**D**

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Seite 2

Schaltmodul MS-1

**GB**

**Operating Instructions**

Page 6

Switch Module MS-1

**F**

**Mode d'emploi**

Page 10

Module de commutation MS-1

**NL**

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Pagina 14

Schakelmodule MS-1

**I**

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Pagina 18

Modulo switch MS-1

**E**

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Página 22

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

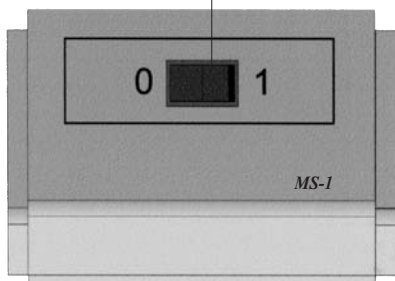
On/off switch for the switched socket

Front

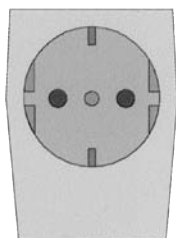Switched socket

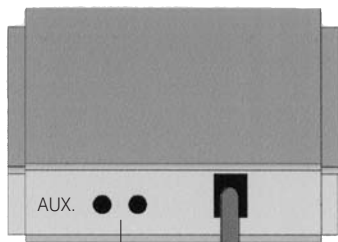Connector socket for  
signal source/ modules/  
vibrating pillow

Back

**The *switch module MS-1* can be connected to each of our units that has a connector socket (3.5 mm jack) for the vibrating pillow.**

We would like to congratulate you on the purchase of our *switch module MS-1* and hope that it will serve you well for a long time to come.

You have chosen a modern and reliable system. Please read the operating instruction through carefully to be able to start the unit correctly and to familiarize yourself with all of the system features.

With the *switch module MS-1* you can extend every unit that has a connector socket (3.5 mm jack) for the vibrating pillow with a 230 V switched socket.

### **Standard components**

Please check if all following components are included:

- *Switch module MS-1*
- Connecting cable
- Operating instruction
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

### **Installation**

Attach the connecting cable to one of the two connector sockets for the signal source at the back of the *switch module MS-1*. Plug the other end of the cable into a unit with a connector socket for a vibrating pillow e.g. alarm clock DS-1, VC-1/VC-10, time *flash/lisa* time *flash*, *lisa* receivers (table flash lamp, RF flash lamp, RF blink lamp, mobile RF receiver).

Plug the plug of the *switch module MS-1* into a 230 V mains socket and a lamp into the switched socket of the module. Please make sure that the lamp is switched on.

The lamp can be switched on and off with the switch at the front of the *switch module MS-1*.

### Note

A vibrating pillow or another module (e.g. acoustic module) can be plugged into the empty connector socket at the back of the *switch module MS-1*.

### **Signalling**

Upcoming signals (e.g. wake-up alarm, *lisa* alarm) are signalled by the lamp being switched on and off.

### **Maintenance and care**

The *switch module MS-1* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shocks.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

### **Warranty**

The *switch module MS-1* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

**Disposal** of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

### Technical data

Power supply: 230 V / 50 Hz  
Power consumption: approx. 1 watt

This device satisfies the following EU directives:

- 2002/95/EG RoHS directive
- 2002/96/EG WEEE directive
- 2004/108/EG EMC directive
- 2006/95/EG Low voltage directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at [www.humanteknik.com](http://www.humanteknik.com).

Technical specifications subject to change without notice.

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**Sarabec Ltd**