HUMANTECHNIK

MA-1

D

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Seite 2

Akustikmodul MA-1

GB

Operating Instructions

Page 6

Acoustic module MA-1

F

Mode d'emploi

Page 10

Module acoustique MA-1

NL

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Pagina 14

Akoestische module MA-1

I

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Pagina 18



Modulo acustico MA-1

E

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Página 22

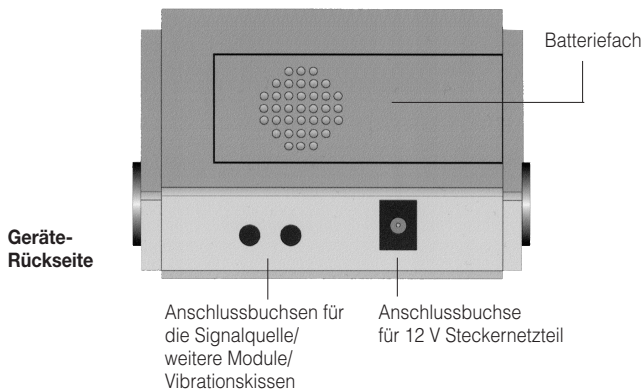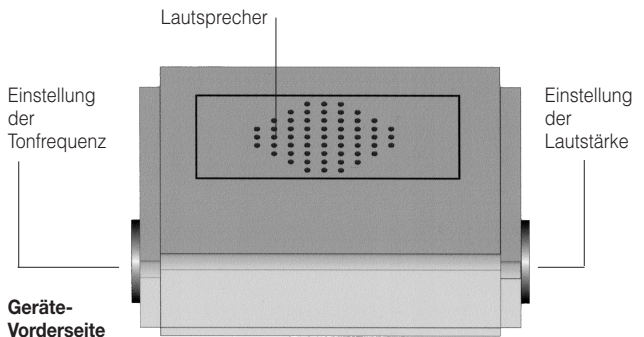Módulo acústico MA-1

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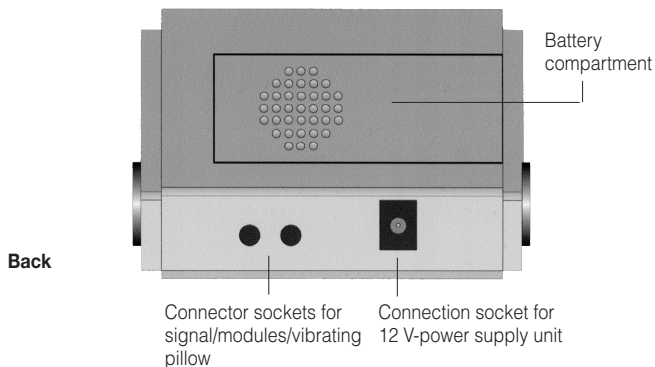
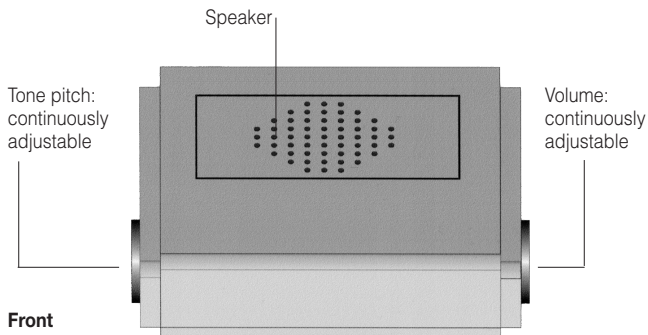
The *acoustic module MA-1* can be connected to each of our units that has a connector socket (3.5 mm jack) for the vibrating pillow.

We would like to congratulate you on the purchase of our *acoustic module MA-1* and hope that it will serve you well for a long time to come.

You have chosen a modern and reliable system. Please read the operating instruction through carefully to be able to start the unit correctly and to familiarize yourself with all of the system features.

With the *Acoustic Module MA-1* you can extend every unit that has a connector socket (3.5 mm jack) for the vibrating pillow with a very loud acoustic tone.

Standard components

Please check if all listed items are included:

- *Acoustic module MA-1*
- Connecting cable
- 12 V-power supply unit
- Operating instructions
- Warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

Installation

Attach the connecting cable to one of the two connector sockets for the signal source at the back of the *acoustic module MA-1*. Plug the other end of the cable into a unit with a connector

socket for a vibrating pillow, e.g. alarm clock DS-1, VC-1/VC-10, time flash/*lisa* time flash, *lisa* receiver (table flash lamp, RF flash lamp, RF blink lamp, portable radio receiver).

Plug the 12 V-power supply unit in the respective socket at the back of the *acoustic module MA-1*.

Note

A vibrating pillow or another module (e.g. flash module) can be plugged into the empty connector socket at the back of the *acoustic module MA-1*.

Signalling

Upcoming signals (e.g. wake-up alarm, *lisa* alarm) are signalled by a very loud acoustic tone.

Special features

Volume: continuously adjustable (max. 100 dB)

Tone pitch: continuously adjustable (100-2000 Hz)

Power failure backup: with a 9 V battery. (Battery not included.)

The *acoustic module MA-1* can also be operated on just a 9 V battery.

Maintenance and care

The *acoustic module MA-1* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

Warranty

The *acoustic module MA-1* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people who are not authorised to do so (destruction of the seal on the unit). Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You

protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical Data

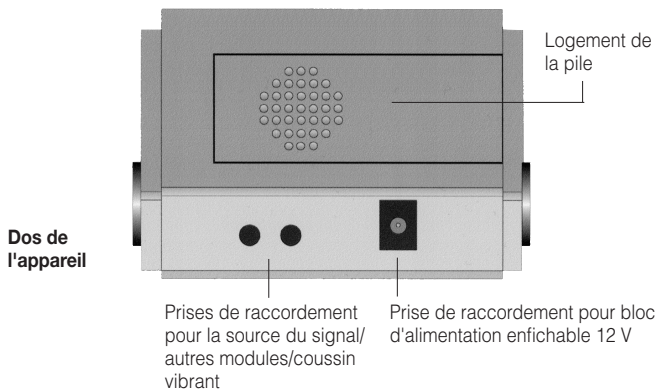
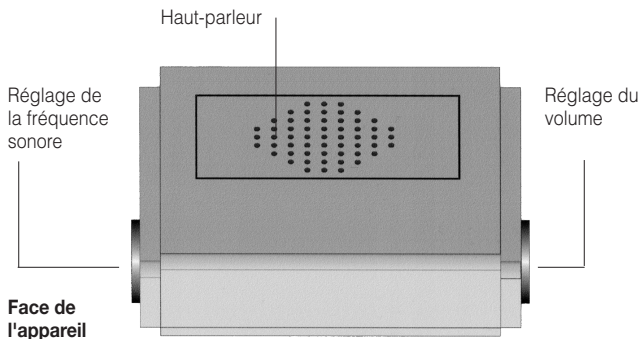
Power supply: 12 V-power supply unit at 230 V, 50 Hz
or with a 9 V battery

Tone pitch: 100-2000 Hz continuously adjustable

This product complies with European Union norms.
You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

Technical specifications subject to change without notice.

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Sarabec Ltd