**introson**<sup>IR</sup>

**EN**

**User manual**

Page 4

**DE**

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Seite 9

**FR**

**Mode d'emploi**

Page 13

**NL**

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Pagina 17

**ES**

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Página 21

**IT**

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Pagina 25

**SE**

**Bruksanvisning**

Sida 29

**NO**

**Betjeningsveiledning**

Side 33

**FI**

**Käyttöohje**

Sivu 37

**HU**

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


Oldal 41

**PL**

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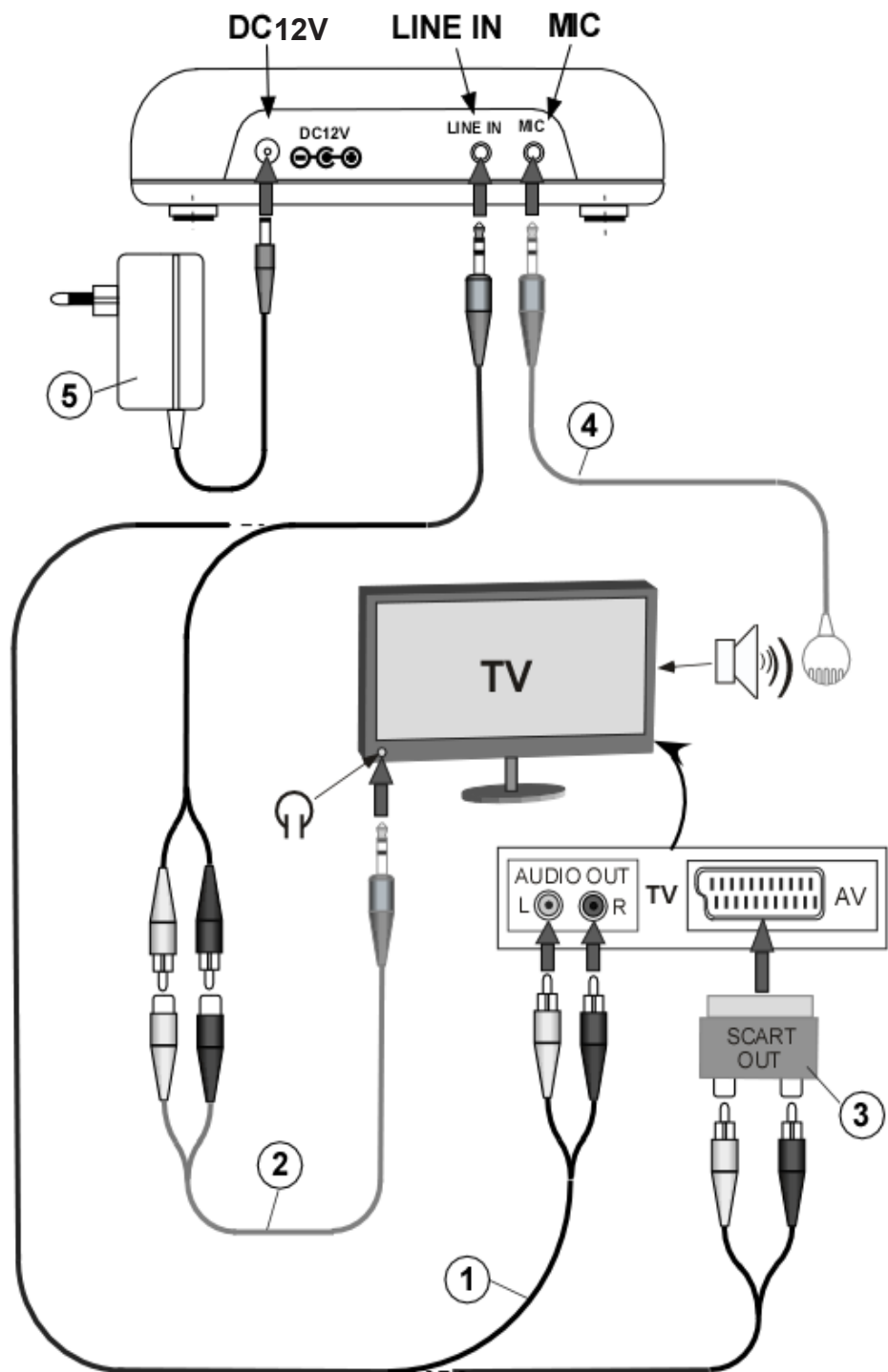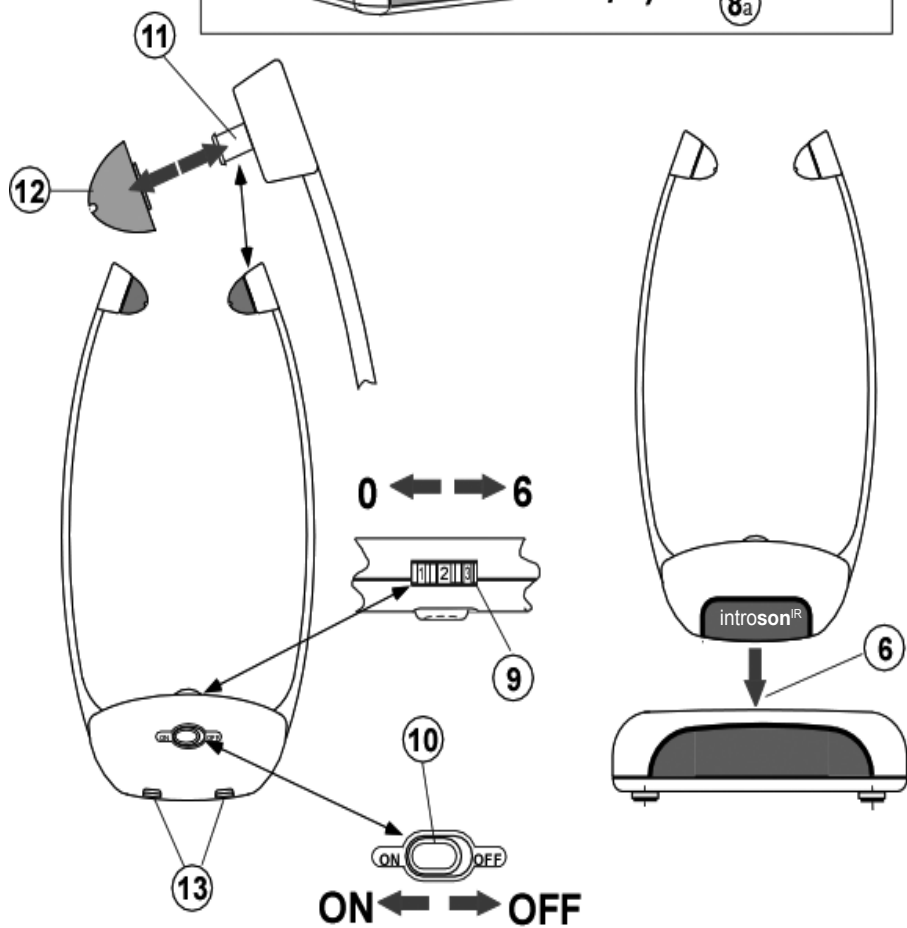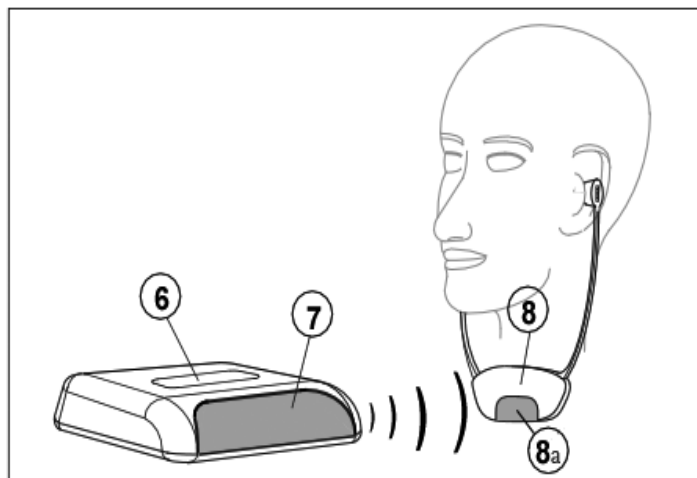Strona 45

## TV listening system with infrared transmission

### Safety Precautions

- Before you start up and use this device, please be sure to read through all these instructions carefully.
- Keep these operating instructions in a safe place so that other users have access to them any time they might need them.
- The device and the power supply unit in particular are designed for use in dry indoor rooms only. Proper operation is guaranteed at a room temperature of 10 °C to 35 °C.
- Do not place the infrared transmitter near heating vents or radiators and avoid any exposure to direct sunlight. Direct sunlight may have a negative effect on infrared sound transmission.
- Always handle the device and any accessories with care and set them up in as dust-free and clean an environment as possible.
- Clean the transmitter and the receiver regularly. The soft silicone earpieces should be cleaned regularly to always ensure proper hygiene. To replace them, please use the soft silicone earpieces that are sold as special accessories.
- Varnishes and furniture polishes can react aggressively with the rubber feet on the transmitter and leave unsightly marks on the surfaces of your furniture.
- Make sure that you only ever use the power supply unit provided with the device to supply electricity to the transmitter.
- Should you not plan to use the system for the period of several days, we recommend that you pull the plug out of the wall outlet.

#### Important note:

- Your receiver can be set to a high volume. To protect your hearing or the hearing of people with normal hearing, particularly of children, from any damage caused by excessively loud volume, please observe the following information:
  1. Set the volume to low before you put on the receiver.
  2. While using the device, set the volume to a level that is comfortable for you.
- If you would like to use this device while using a cardiac pacemaker or any other medical device, please observe the following information:
  1. Before you use this product together with a pacemaker, consult your doctor or the manufacturer of the pacemaker to make sure that none of the functions of the pacemaker will be affected.
  2. If you use a cardiac pacemaker or any other medical device, be sure to follow the safety precautions/safety guidelines provided by your doctor or the manufacturer of the pacemaker before you use the *introsön*<sup>®</sup>.

## Standard components

Please check if all listed items are included:

- *introsom*<sup>IR</sup>- Transmitter and Receiver
- 12 V plug-in PSU
- Connection cable
- Operating Instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

## Connecting the infrared transmitter

- Plug in the audio connection cable ① that comes with the device into the "LINE IN" connection jack of the transmitter and the other end into the connection jack of your sound source (e.g. the TV).

Other connection adapters are available optionally: a 3.5-mm stereo cinch jack adapter ② which can be connected to a headphone jack, or a SCART connector ③, which can be plugged into the SCART jack of the audio source (marked with AV).

If your sound source is not equipped with a headphone jack or any other connection options, then you can clip a microphone ④ onto the sound source. This microphone is then plugged into the corresponding MIC jack on the transmitter. The microphone is sold separately.

- Connect the power cord that comes with the device ⑤ to the transmitter using the "DC12V" connection plug. Then plug the power cord itself into a 230V electrical wall outlet.
- Set the under-the-chin receiver into the charging tray of the transmitter without using any excessive force ⑥. A red LED will light up to indicate that the device is being charged up.
- Before the device is used for the first time, the receiver with the built-in rechargeable battery should be charged for 24 hours.

## Setting up or adjusting the transmitter

- Place the transmitter close to the sound source in a suitable position. There must be a clear line of sight between the front of the transmitter (polished front surface ⑦) and the receiver ⑧. The polished surface of the transmitter must be aimed at the front of the transmitter.

## Using the receiver

- Use the volume control ⑨ to initially set the receiver to a low volume to protect your hearing. Turn the volume control in the direction of the "6" to increase the volume and in the direction of the "0" to decrease the volume.
- Use the sliding switch ⑩, on the rear side of the under-the-chin receiver to switch the device "ON" or "OFF". Make sure that the sound source connected is switched on and adjust the volume on the receiver to a level that is comfortable for you.

### Replacing the silicone earpieces

- The skin-neutral silicone earpieces ⑫ should be replaced if they become dirty. The earpieces can be removed from the holder by gently twisting them ⑪. The new silicone earpieces can be put on by gently twisting them and pressing them onto the holders.

### Battery recharging contacts

- We recommend that you keep the battery contacts ⑬ clean at all times so that the battery can always recharge properly and fully (using the recharging contacts in the recharging tray of the transmitter and the recharging contacts on the receiver).

### Replacing the battery

- After a long service life, the rechargeable battery of the receiver might need to be replaced. To open the lid of the battery compartment 8a , slide it downwards and exchange the rechargeable battery (item no. A-4966-0).

### Warranty

This product is very safe to operate and provides highly reliable service. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw away the packaging. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the complete warranty card is returned accompanied by a copy of the dealer's invoice/till receipt. **Always specify the product number in any event.**

**Disposal** of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units.

You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

**Technical data**

Modulation frequency:	Stereo
Carrier frequency:	2,3 / 2,8 MHz
Frequency response:	20 - 20000 Hz
Distortion:	< 1 %
Noise level:	typ. 60 dB
Operating range:	0° C to 40° C

**Headset receiver:**

Operating time per charge:	Connection cable
Maximum volume SPL:	approx. 120 dBA
Weight approx.	approx. 56 g

**Transmitter:**

Power consumption:	approx. 3.6 VA
Operating voltage:	12 Volt DC
Supply voltage:	230 Volt AC 50 Hz
Weight:	approx. 172 g

This product complies with European Union norms.

You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

Technical specifications subject to change without notice.

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**Sarabec Ltd**