HUMANTECHNIK

sonumaxx<sup>BT</sup>

DE

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Seite 2

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
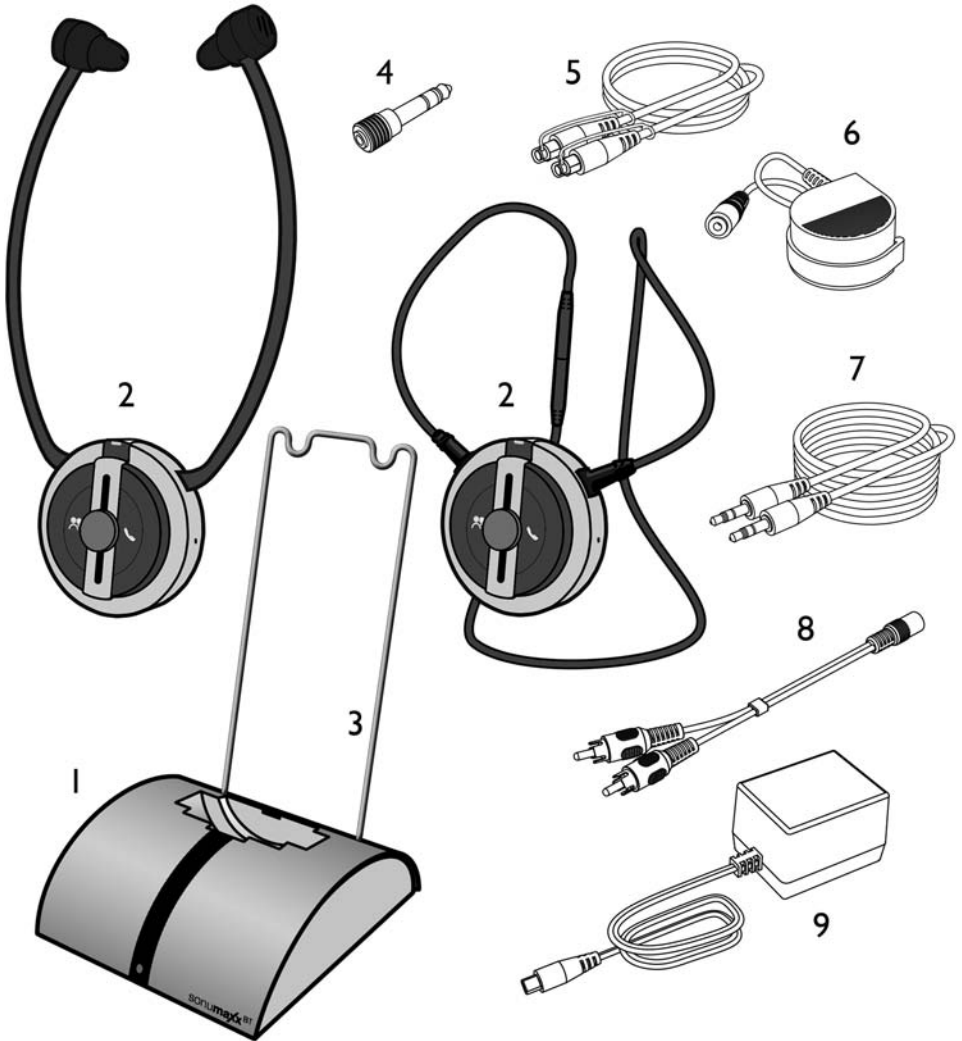
## Scope of delivery

- 1 Transmitter
  - 2 Stethoset **or** loop receiver
  - 3 Bracket for loop receiver (only included in LR set)
  - 4 6.3 mm to 3.5 mm adapter plug
  - 5 Optical Toslink cable
  - 6 External microphone
  - 7 Audio cable (both ends with 3.5 mm stereo plug)
  - 8 RCA adapter cable
  - 9 AC/DC adapter
- 1 pair of additional big silicone ear adapters (Headphone version, not shown)
  - Warranty card (not shown)

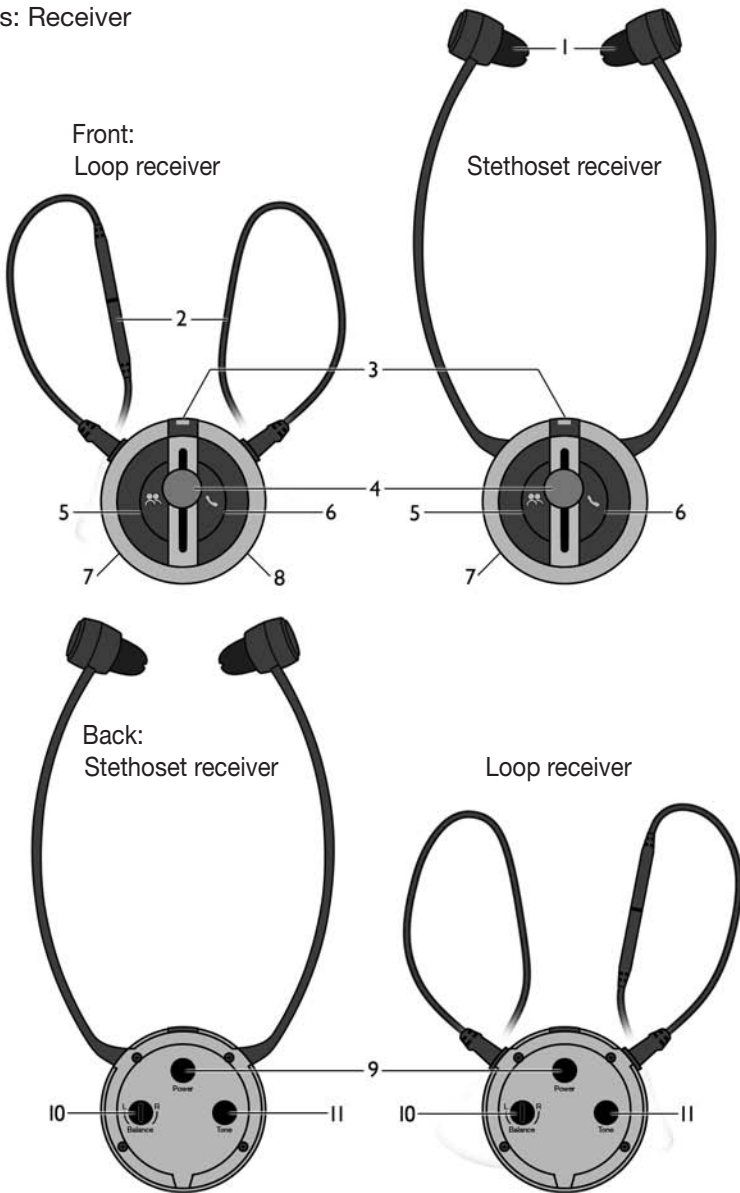
Controls: Receiver

- |   |                                    |    |                                     |
|---|------------------------------------|----|-------------------------------------|
| 1 | Adjustable earbuds                 | 6  | Telephone button                    |
| 2 | Loop receiver with safety coupling | 7  | USB charging jack                   |
| 3 | Indicator LED                      | 8  | 3.5 mm earphone jack (Type LR only) |
| 4 | Volume control                     | 9  | Power button                        |
| 5 | MIC button                         | 10 | Sound balance control               |
|   |                                    | 11 | Tone button                         |

## Controls: Transmitter

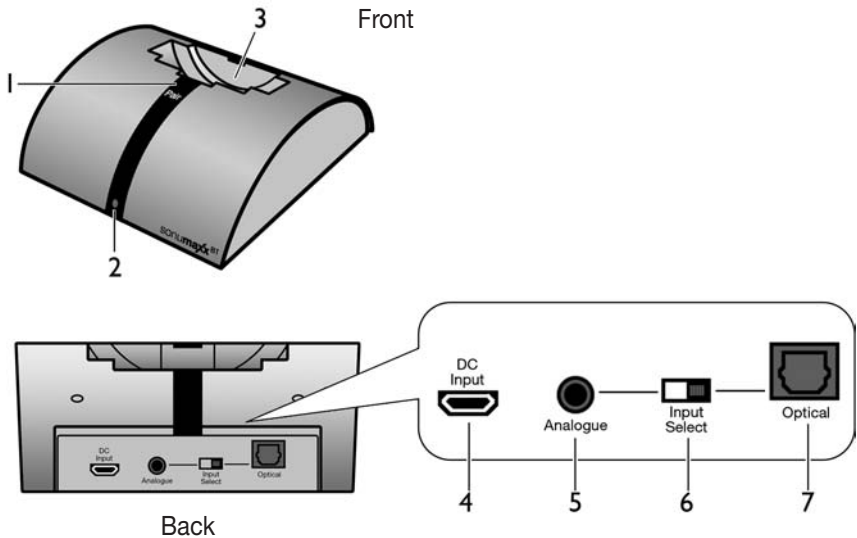- |   |                                |   |                                 |
|---|--------------------------------|---|---------------------------------|
| 1 | Pair button                    | 5 | Analogue audio input            |
| 2 | Indicator LED                  | 6 | Input selector                  |
| 3 | Charging bay                   | 7 | Optical digital input (Toslink) |
| 4 | Power input (Micro USB socket) |   |                                 |

## Introduction

This digital headset uses Bluetooth transmission technology and offers sound amplification so users can adjust volume to their listening needs.

Simply connect the transmitter to a desired audio source: DVD player, CD player, TV set, Hi-Fi system or MP3 player.

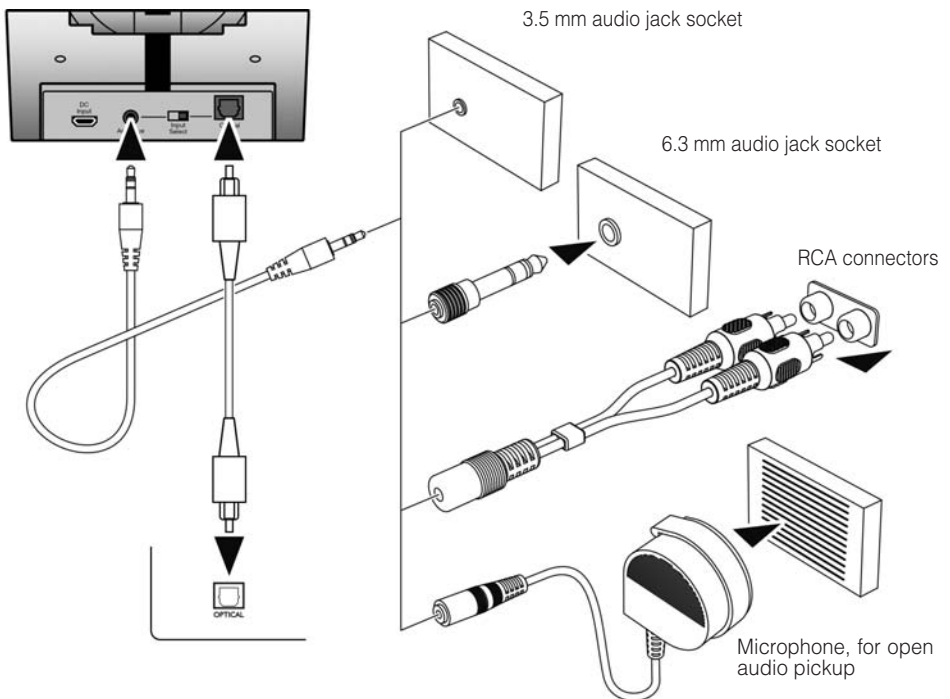
The receiver also allows you to take incoming phone calls from a mobile phone - or listen to music transmitted by your phone. The transmitter signal can also be received on other Bluetooth receiving devices.

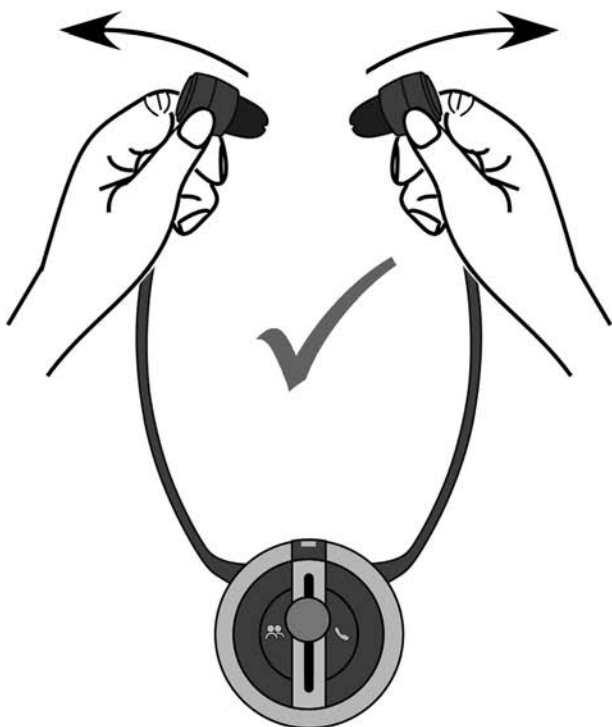
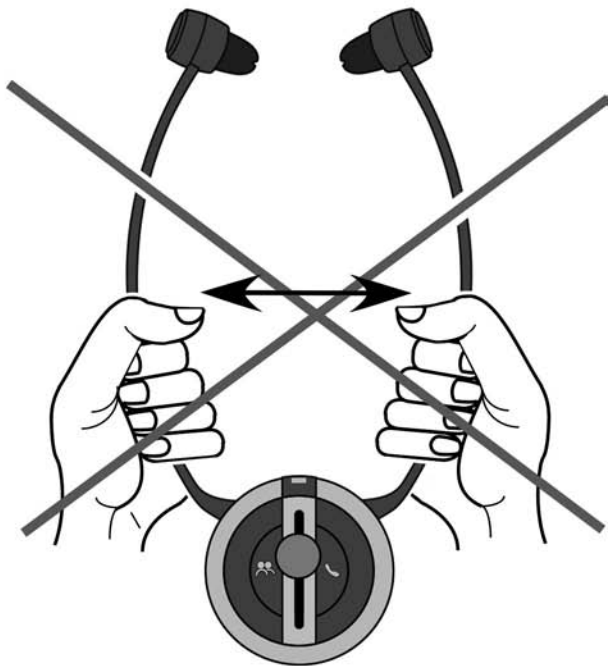
## Features

1. Bluetooth technology
2. Charging station
3. Optical digital input
4. RF transmission auto off (if no incoming audio signal)
5. Mute button, for holding discussions
6. Telephone receiver function
7. External microphone for sound pick up
8. Sound balance adjustment
9. Tone adjustment

## Installation: Transmitter

1. Connect the power adapter to the DC input USB socket located on the back of the transmitter panel.
2. Connect the power adapter to an electrical outlet.
3. The RCA adapter cable, audio cable, 6.3 mm to 3.5 mm adapter plug, Toslink cable and external microphone serve to connect various types of audio sources (such as a DVD or CD player, TV, HiFi stereo or MP3 player).
4. Push the input selector switch, located on the rear panel of the transmitter, toward the side on which the audio input cable has been connected.

The blue indicator LED starts flashing for 5 seconds after plugging the power supply into the outlet. The transmitter and receiver are already linked by default. If an active linked receiver is present, the indicator LED is lit constantly and the system is connected and ready to use.

If there is no active linked receiver, the transmitter automatically goes into pairing mode: The indicator LED blinks rapidly for 3½ minutes. During this time, the transmitter can be found and paired to any type of Bluetooth receiver (mobile phone, streamer, hearing aids). If no receiver is paired within this time, the indicator LED shuts off and the transmitter enters standby mode.

The transmitter can be called out of standby mode at any time by pressing the pair button. The process described above is then repeated.

## Setup

1. Activate the audio source connected to the transmitter, on (e. g. TV or HiFi stereo set).
2. Press the receiver's power button for two seconds to switch it on. The indicator LED on the receiver will flash blue.
3. Plug in the power supply connected to the transmitter or, if the transmitter is already connected, the transmitter will find the receiver automatically.
4. The indicator LEDs on the transmitter and receiver will now be constantly lit in blue, this indicates a functioning connection. Adjust the volume as desired (green slider up or down) and put on the headset.
5. The receiver's indicator LED will start to flash slowly if the battery runs low. In this case, recharge the receiver.
6. Switch off the receiver after use by pressing the power button for 2 seconds. The receiver also switches off automatically when placed in the charging bay of the transmitter.

The loop receiver (Type LR) features a 3.5 mm audio jack on the side, which enables you to connect a regular set of earphones (not included). The loop can be opened: To do so, pull the safety coupling apart.

Insert the included bracket into the holes provided on the transmitter. The bracket is used to easily store the loop cable.

## Notes

Please make sure that the integrated battery is fully charged before its first use. Charge the receiver for about 5 hours.

## Charging the battery

1. Place the receiver in the transmitter charging bay. Ensure that the receiver contacts are touching the contact blades of the transmitter.
2. The charging process will start automatically. The indicator LED on the receiver will flash green during the charging process.
3. After the charging process is completed, the indicator LED on the receiver will be constantly lit in green.

Alternatively, the receiver can be charged directly using the transmitter power supply or a mobile phone charger with micro USB connection. Use the charging jack on the side of the receiver to do so. It is also possible to charge it during operation using a power bank. This can extend the run time of the receiver.

## Bluetooth functions

### Transmitter Bluetooth functions:

The transmitter is used for audio transmission from any audio source (e.g. hearing aids). It can be paired with any Bluetooth receiver device. To pair the transmitter, set it to pair mode (plug transmitter into a power supply or briefly press the Pair button and seek/pair the transmitter with the desired receiving device (follow the operating manual for the receiving device if necessary).

Please note: Only one receiving device can be paired at a time. It is not possible to operate the transmitter with multiple receivers at the same time.

The transmitter can save a maximum of 4 different receivers.

The transmitter always pairs first to the last paired receiver within its range.

### Receiver Bluetooth functions:

The receiver is used to play audio signals emitted by Bluetooth transmitters. It is possible to have the receiver paired with an SonumaxxBT transmitter while being connected to a cell phone at the same time. This allows you to receive incoming calls with the receiver. This only functions with a Bluetooth-supporting cell phone. The receiver can save a maximum of 4 devices.

### Pair SonumaxxBT receiver with other Bluetooth transmitters

The SonumaxxBT receiver can be paired with various Bluetooth transmitters, such as with a smartphone in order to listen to music and make phone calls.

1. Press the receiver for 2 seconds to activate it.
2. Press the telephone button on the receiver until the blue LED on the receiver starts to flash. The receiver is now ready to pair with a cell phone.
3. Pair the receiver and the cell phone. See the handbook for your cell phone for instructions.

4. The telephone and receiver are now paired. The receiver will display this with a constantly lit blue LED. Now you can use the receiver to listen to music through the cell phone or make calls.

Please note: The receiver will always connect first to the last connected device.

Once you turn on the SonumaxxBT receiver and a Bluetooth enabled cell phone is in range, the receiver will automatically connect to the cell phone.

To connect the receiver with the SonumaxxBT transmitter once again, turn off the Bluetooth function on your cell phone or move the cell phone out of range. The SonumaxxBT receiver will then automatically connect to the SonumaxxBT transmitter when it is turned on.

### **Connect SonumaxxBT set for telephone calls with simultaneous transmission from the SonumaxxBT.**

This procedure only has to be completed once. A functional pairing between the SonumaxxBT transmitter and SonumaxxBT receiver is required. This has already been completed as a factory default.

1. Press the power button on the back of the receiver for 2 seconds to turn it on. The receiver will connect to the transmitter automatically. The blue LED on the transmitter will indicate they are connected.
2. Press the telephone button on the receiver until the blue LED on the receiver starts to flash. The receiver is now ready to pair with a cell phone.
3. Pair the receiver and the cell phone. See the handbook for your cell phone for instructions.
4. The telephone and receiver are now paired. The receiver will display this with a constantly lit blue LED. The devices are now working together and prepared.
5. Press the power button for 2 seconds to activate the SonumaxxBT receiver, or activate it by placing it in the SonumaxxBT transmitter.
6. Turn off the Bluetooth function on your cell phone.
7. Press the power button on the back of the receiver for 2 seconds to turn it on. The receiver will connect to the transmitter automatically. The blue LED on the transmitter will indicate they are connected.
8. Turn the Bluetooth function on your cell phone back on. The SonumaxxBT should be listed in the list of recognised devices on your cell phone. Now, connect the cell phone with the SonumaxxBT receiver using the menu on your cell phone. Now the cell phone and SonumaxxBT transmitter are connected at the same time.

Please note: Only the telephone function is Bluetooth enabled; you cannot listen to music through your cell phone in this mode.

The receiver will always connect first to the last connected device.

Once you turn on the SonumaxxBT receiver and a Bluetooth enabled cell phone is in range, the receiver will automatically connect to the cell phone.

To connect only the receiver with the SonumaxxBT transmitter, turn off the Bluetooth function on your cell phone or move the cell phone out of range. The SonumaxxBT receiver will then automatically connect to the SonumaxxBT transmitter when it is turned on.

### **Use SonumaxxBT set for telephone calls with simultaneous transmission from the SonumaxxBT.**

A pairing between the sonumaxxBT transmitter, SonumaxxBT receiver, and cell phone, as described in the section “Connect SonumaxxBT” set for telephone calls with simultaneous transmission from the SonumaxxBT.”

1. Turn off the Bluetooth function on your cell phone.
2. Press the power button on the back of the receiver for 2 seconds to turn it on. The receiver will connect to the transmitter automatically. The blue LED on the transmitter will indicate they are connected.
3. Turn the Bluetooth function on your cell phone back on. The SonumaxxBT should be listed in the list of recognised devices on your cell phone. Now, connect the cell phone with the SonumaxxBT receiver. Now your cell phone and the SonumaxxBT transmitter are connected at the same time.

### **Taking phone calls:**

If the receiver is linked to an audio source and a Bluetooth phone, an incoming phone call will be indicated by a ring tone and the audio transmission will be interrupted. The incoming call can be taken by pressing the telephone button, and the integrated microphone allows you to talk on the phone.

To end the phone call, press the telephone button again. Should your conversation partner have ended the call first, then it is not necessary to press the telephone button again. The previous audio transmission will then be picked up once again.

### **Communication function of the receiver:**

The audio transmission can be interrupted at any time, for example in order to talk to someone else in the room: To do so, press the MIC button on the receiver 1x. The integrated microphone will then pick up the sound within the room and amplify it over the receiver. The volume can be adjusted as usual. To end the communication mode, press the MIC button once more. The green indicator light will go off, and the previous audio transmission will continue.

## Tone adjustment:

If necessary, the tone can be adjusted on the receiver in 4 levels by repeatedly pressing the tone button:

1. Press once: Treble boost - The setting is indicated by one short beep.
2. Press twice: Treble and bass boost - The setting is indicated by two short beeps.
3. Press three times: Bass boost - The setting is indicated by three short beeps.
4. Press four times: Normal - The setting is indicated by one long beep.

The receiver saves the selected setting for the next use.

## Balance adjustment:

The sound balance can be adjusted on the receiver if required. Turn the sound balance control to the left or right in order to balance out the incoming audio signal.

## Trouble shooting:

### No sound:

- Make sure that the power supply is securely plugged into the outlet and that the power supply cable is properly inserted into the DC input on the transmitter.
- Ensure the receiver is properly linked to the transmitter.
- The receiver battery may be low. Recharge the receiver.
- Make sure that the audio source is feeding a signal to the transmitter, check if the TV, stereo set or other sound source is turned on.
- Check the volume setting.
- The connected audio source may not be in playback mode.
- Check which transmitter the receiver is connected to. Deactivate Bluetooth function on the unnecessary Bluetooth device.

Only loop receiver: Make sure that the safety coupling is correctly pushed together.

## Interference:

- The receiver battery may be low. Recharge the receiver.
- The distance between transmitter and receiver may be too large. Decrease the distance.
- The audio input signal may be too low. Raise the volume of the audio source.

## Follow the following instructions if the receiver cannot be linked to the transmitter:

1. Unplug the transmitter from the power supply.
2. Turn the receiver on.
3. Plug the transmitter back into the power supply.
4. Press the telephone button on the receiver until the blue LED on the receiver flashes quickly.
5. Press the Pair button on the transmitter.
6. The transmitter and receiver will pair; once the two blue LEDs are constantly lit, the connection has been made.

## Specifications:

Dimensions of transmitter:	H 42 mm x W 108 mm x D 108 mm
Dimensions of receiver housing:	Ø 65 mm x D 28 mm
Dimensions of stethoset receiver:	H 233 mm x W 122 mm x D 28 mm
Transmitter:	
Power feed:	approx. 4 VA
Power supply:	5 Volt DC
Power adapter supply:	100-240 Volt 50-60 Hz
Weight:	110 g
Modulation type:	Stereo
Carrier frequency:	2.4 GHz
Transmission output:	10 mW
Range:	approx. 15 meters
Frequency response:	30 - 20000 Hz
Distortion factor:	< 1 %
Signal-to-noise ratio:	> 70 dB
Temperature working range:	0° C – 55° C
Receiver:	
Run time per full charge:	approx. 7 h
Charging time:	approx. 3 h (2h for 90% charge)
Rechargeable battery (integrated):	3.7 V lithium polymer
Maximum volume:	125 dB
Weight of stethoset receiver:	66 g
Weight of loop receiver:	80 g

## Safety warnings:

The equipment is designed for listening to domestic audio sources. Other uses are outside the intended use for this product. Using the equipment in a manner not in accordance with these instructions may cause damage or contamination. Such damages are excluded from the warranty. The equipment is not waterproof and should be kept away from all fluids. Do not place near any heat sources. Do not place any items on top of the equipment that may cause it to malfunction.

This device has a high volume output. Setting the volume too high may damage your hearing!

## Warranty

Should a malfunction occur despite proper operation, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. If possible, send in the product in its original packaging. Do not discard the original packaging.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned, accompanied by a copy of the dealer's invoice/receipt.

**Always specify the product serial number in all communications.**

**Disposal** of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system for such devices).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary domestic waste but has to be returned to a collection point for recycling electric and electronic devices. By disposing of this product correctly, you help protect the environment and human health. Incorrect disposal may be hazardous to the environment and to health. Material recycling helps to reduce the consumption of raw materials. You can receive further information on how to recycle this product from your community, your communal disposal company or your local dealer.

This device complies with the following EU directives:

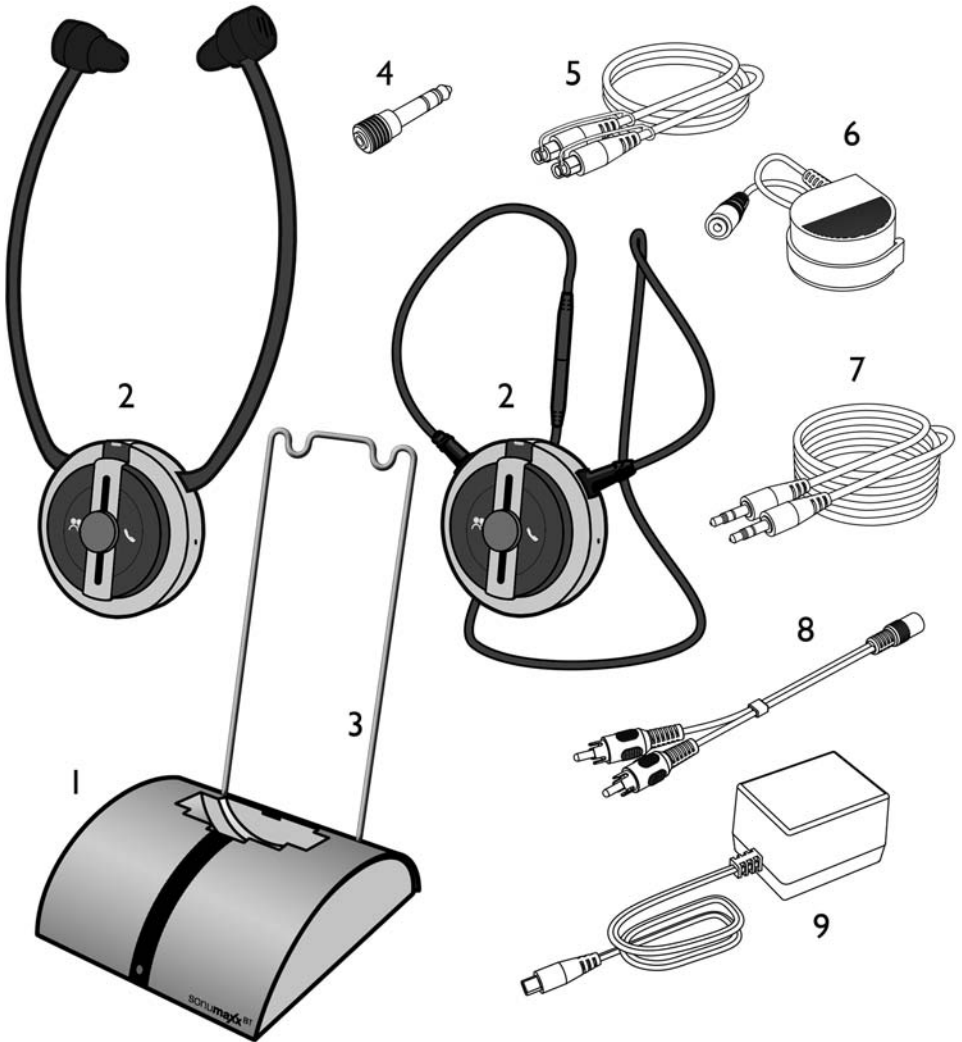
- 2011/ 65/EU      RoHS Directive
- 2012/19/EU      WEEE Directive
- 2014/53/EU      R&TTE Directive
- EN 301 489-17 V3.1.1 : 2017
- EN 301 489-1 V2.1.1 : 2017
- EN 300 328 V2.1.1 : 2016
- EN 62479 : 2010
- 2014/35/EU      Low Voltage Directive
- EN 60065 : 2014

Compliance with the directives listed above is confirmed by the CE seal on the device.  
CE compliance declarations are available on the internet at [www.humanteknik.com](http://www.humanteknik.com).

Technical specifications subject to change without notice.

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**Sarabec Ltd**