HUMANTECHNIK

earis®
MAX

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Introduction

Welcome to the user manual for the EarisMAX transmitter – your versatile solution for high-quality audio streaming. The EarisMAX transmitter combines advanced Bluetooth® technology with a plethora of connectivity options to provide you with a seamless and customizable audio experience.

With a wide range of connectivity options, you can easily connect the EarisMAX transmitter to various devices, including televisions, PCs, tablets, and more. The innovative Auracast™ technology even allows you to wirelessly transmit high-quality audio with minimal latency, ensuring an immersive listening experience.

The EarisMAX transmitter is designed to connect audio sources that do not have a Bluetooth® transmission function to Bluetooth®-enabled receivers (pairing with compatible headphones or hearing aids).


Please read this user manual carefully to properly set up the device and become familiar with all of its features.

Package Contents

- EarisMAX transmitter
- USB-A to USB-C cable
- Toslink cable
- Audio adapter RCA to 3.5mm jack
- 3.5mm jack connection cable
- User manual
- Safety Instructions
- Warranty Card

Product overview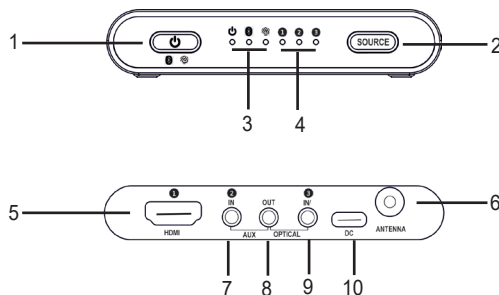

1. Power On/Off / Bluetooth® Selection
2. Input Selection Button
3. Bluetooth®-Status LED
4. Input Status LEDs 1-3
5. Input 1 / HDMI Input
6. Antenna Connection
7. Input 2 / Analog
8. Output / miniToslink / Analog
9. Input 3 / miniToslink / Microphone
10. USB-C 5V DC Input

Connecting the EarisMAX Transmitter to Power

The EarisMAX transmitter is powered via the USB-C 5VDC (10) input. This can be done using a USB port on the TV or with a standard USB power adapter that delivers at least 300mA at 5VDC.

We recommend using a direct USB connection on the TV. When the TV is turned off, the EarisMAX transmitter will also be turned off simultaneously. CAUTION: There are TV models where the USB port turns off only after a few minutes.

When the EarisMAX transmitter is disconnected from power, the previously set Bluetooth modes remain stored in the device.

Turning On and Off the EarisMAX Transmitter

Press and hold the Power On/Off button (1) for 3 seconds to turn on the EarisMAX transmitter. The status LED (3) will illuminate green.

Press the Power On/Off button (1) again for 3 seconds to turn off the EarisMAX transmitter.

Note: When the EarisMAX transmitter is disconnected from power and then reconnected, all previous settings remain intact.


13 English

Connecting the EarisMAX Transmitter to an Audio Source

Turn off the audio source before connecting the transmitter. Place the transmitter near the audio source. Be careful not to place the transmitter in close proximity to thick reinforced concrete walls and other emitting devices, as this may affect the range. You have several options to connect the EarisMAX transmitter to an audio source (e.g., TV, stereo system, etc.). Check what connection options your audio source provides. The fold-out page of this user manual provides an overview of the available options. We recommend using digital sources such as Toslink/SPDIF or HDMI preferably. If these connections are not available, using the analog (jack) audio output is recommended. If none of these connections are available, sound can be transmitted to the transmitter via a microphone. However, note that the sound quality will not be very high-quality corresponding to the microphone used. If you are already using the Toslink/SPDIF or analog output of your TV device for a soundbar or speakers, it is possible to loop the signal through the EarisMAX to continue using these devices.

Connecting the EarisMAX Transmitter to Digital Output (S/PDIF / Toslink)


You can achieve very good audio quality by connecting the transmitter to your audio source using the optical Toslink cable (digital). Please note that the digital input can only transmit audio sources in PCM/Stereo standard. Please refer to the user manual of the connected audio source on how to switch the audio.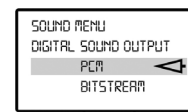

If a combined digital signal (bitstream) is used, audio playback is not possible. Remove the protective caps from the two Toslink connectors on the cable. Connect the mini-Toslink cable to the input port 3 of the transmitter and connect it to your audio source. To select input 3, press the input selection button (2) after powering on the device by pressing the power on/off button (1) for 3 seconds. The input status LED 3 should illuminate green.

Connecting the EarisMAX Transmitter via HDMI

You can achieve very good audio quality by connecting the transmitter to your audio source using HDMI ARC (HDMI Audio Return Channel). Please note that the digital input can only transmit audio sources in PCM/Stereo standard. Please refer to the user manual of the connected audio source on how to switch the audio.

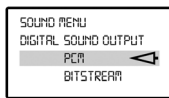When using a combined digital signal (bitstream), audio playback is not possible. Connect the HDMI cable to the input port 5 of the transmitter and connect it to your audio source. Make sure to use the HDMI ARC port of your audio source; otherwise, no audio will be transmitted.

To select input 1, press the input selection button (2) after powering on the device by pressing the power on/off button (1) for 3 seconds. The input status LED 1 should illuminate green.

Connecting the EarisMAX Transmitter to RCA Jack

You can achieve good audio quality by connecting the transmitter to your audio source using an analog audio cable.

Connect the analog cable to the input port (2) of the transmitter and connect it to your audio source. Ensure that the white plug is connected to the white jack of the audio source and that the red plug is connected to the red jack of the audio source.

To select input 2, press the input selection button (2) after powering on the device by pressing the power on/off button (1) for 5 seconds. The input status LED 2 should illuminate green.

Connecting the EarisMAX Transmitter to Headphone Jack

Connect the analog cable to the input port 2 of the transmitter and plug the audio cable into the headphone jack of your audio source.

To select input 2, press the input selection button (2) after powering on the device by pressing the power on/off button (1) for 5 seconds. The input status LED 2 should illuminate green.

Set the volume of your audio source to a moderate level.

Some TVs mute the speakers when something is connected to the headphone jack. On some TVs, this muting can be turned off via the TV menu. Alternatively, use a different connection method.

Connecting the EarisMAX Transmitter to Device Speaker

The transmitter can capture audio from a speaker using a stereo/mono electret microphone. The microphone is not included and can be purchased from your retailer. Connect the analog cable to the input port 3 of the transmitter. Attach the microphone to the speaker of your audio source.

To select input 3, press the input selection button (2) after powering on the device by pressing the power on/off button (1) for 5 seconds. The input status LED 3 should illuminate orange.

15 English

Please note that the volume of your audio source directly affects the volume at the receiver. Also, note that this method provides the lowest audio quality among the given connection options.

Toslink Input and Output (Passthrough)

If you already have a soundbar or another device connected to the Toslink port of your TV, the EarisMAX transmitter allows you to loop the Toslink signal through the EarisMAX transmitter, enabling simultaneous operation of the EarisMAX transmitter and the soundbar.

To do this, connect the EarisMAX as described in „Connecting the EarisMAX Transmitter to Digital Output (S/PDIF / Toslink)“.

You will also need a miniToslink to Toslink cable. Connect the miniToslink end to the output/miniToslink/analog (8) and connect the Toslink end to the soundbar.

Analog Audio Input and Output (Passthrough)

If you already have a soundbar or another device connected to the audio output of your TV, the EarisMAX transmitter allows you to loop the audio signal through the EarisMAX transmitter, enabling simultaneous operation of the EarisMAX transmitter and the soundbar.

To do this, connect the EarisMAX as described in „Connecting the EarisMAX Transmitter to RCA Jack“. You will also need a stereo audio cable with a 3.5mm jack for the EarisMAX transmitter. Connect the cable to the output miniToslink/analog (8) and connect the audio jack to the soundbar.

Bluetooth® Device Pairing with the Transmitter

Here, you will find everything you need to know about the versatile Bluetooth® connection options offered by this device. From classic Bluetooth pairing to advanced technologies like Auracast™ and Bluetooth® LE Audio - we will guide you through the world of wireless audio connectivity.

Auracast™.

Auracast™ is an innovative technology specifically designed for high-quality, low-latency audio transmissions. With Auracast™, you can transmit audio content of the highest quality from your transmitter to **multiple** compatible receivers **simultaneously**, creating an immersive listening experience without disruptive delays. In this chapter, you will learn how to optimize the use of Auracast™ to get the most out of your audio experience.

Bluetooth® LE Audio.

Bluetooth® LE Audio is an innovative evolution of Bluetooth® that offers improved audio quality and efficiency, specifically designed for hearing aids. This technology enables reliable wireless audio playback with minimal power consumption and supports

English 16

new features such as multi-channel audio and enhanced synchronization. In this chapter, you will learn how to use Bluetooth® LE Audio with your EarisMAX transmitter to enjoy a premium listening experience.

****Bluetooth Classic:****

Bluetooth Classic is the traditional Bluetooth® technology used for wireless data transmission, including audio signals, between various devices. With Bluetooth® Classic, you can easily connect your EarisMAX transmitter to a variety of Bluetooth®-enabled devices such as earbuds and headphones.

Connecting with Auracast™

We recommend choosing this connection when you have playback devices that support Auracast™.

Press the Bluetooth® selection button (1) to select the Auracast™ symbol. The Bluetooth® status LED (3) will illuminate green when the Auracast™ symbol is selected.

Note that the device will restart each time you switch, and all existing connections will be interrupted. The Auracast™ channel is now activated. By default, you will find the channel under the name "EarisMAX Transmitter". Now you can connect an unlimited number of playback devices to the EarisMAX transmitter via Auracast™.

The exact procedure for connecting may vary depending on your specific playback device. We recommend consulting the user manual of your playback device for the correct pairing steps.

Important Note:

The Auracast™ specifications allow for different sampling rates. Most playback devices support various sampling rates. Our EarisMAX transmitter is factory-set to a sampling rate of 48 kHz to ensure the highest possible sound quality. However, it may occur that playback devices, such as hearing aids, only support a sampling rate of 24 kHz. If you wish to use our EarisMAX transmitter with such devices, you can switch the sampling rate.

Switching the Auracast™ Sampling Rate

- The EarisMAX transmitter must be turned on and in Auracast™ mode.
- Press both buttons simultaneously for 3 seconds.
- The status LEDs 1-3 will blink 3 times slowly when the switch to the 24 kHz sampling rate is complete.
- Press both buttons simultaneously for 3 seconds again to switch the sampling rate back.
- The status LEDs 1-3 will blink 3 times quickly when the switch to the 48 kHz sampling rate is complete.

Connecting with Bluetooth® LE Audio

We recommend choosing this connection when you have hearing aids that support Bluetooth® LE Audio.

Press the Bluetooth® selection button (1) to select the Bluetooth® symbol until the Bluetooth® status LED (3) blinks bright blue at the Bluetooth® symbol. Note that the device will restart each time you switch, and all existing connections will be interrupted. The Bluetooth® LE Audio channel is now ready for pairing. Now you can connect your hearing aid to the EarisMAX transmitter via Bluetooth® LE Audio.

The exact procedure for connecting may vary depending on your specific playback device. We recommend consulting the user manual of your playback device for the correct pairing steps.

Connecting with Bluetooth®

We recommend choosing this connection when you have headphones that do not support Auracast™.

Press the Bluetooth® selection button (1) to select the Bluetooth® symbol until the Bluetooth® status LED (3) blinks blue at the Bluetooth® symbol. Note that the device will restart each time you switch, and all existing connections will be interrupted. The Bluetooth® channel is now ready for pairing. Now you can connect your playback device to the EarisMAX transmitter via Bluetooth®.

The exact procedure for connecting may vary depending on your specific playback device. We recommend consulting the user manual of your playback device for the correct pairing steps.

Setting an Auracast™ Password

Using the earisMAX Transmitter software for Windows, you can assign an Auracast™ password to the Earis MAX transmitter. This protects your Auracast™ channel from misuse. The earisMAX Transmitter software can be downloaded and installed from the Humantechnik website. Follow the instructions of the software to set a password.

EarisMAX Transmitter Update

Using the earisMAX Transmitter software for Windows, you can update the Earis MAX transmitter to the latest firmware. The earisMAX Transmitter software can be downloaded and installed from the Humantechnik website. Follow the instructions of the software to install the latest firmware.

Technical data

Bluetooth:	5.3
Codec:	LC3, SBC
Audio Latency:	<50ms
Transmission Range:	up to 50m
Power Input:	USB-C 5VDC 300mA
RF Transmission Power:	max. 10dBm
Operating Temperature:	0°C to 40°C
Weight:	90g

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Sarabec Ltd