HUMANTECHNIK

Crescendo 50

D

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Seite 2

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GB

Operating Instructions

Page 8

Personal Listener

F

Mode d'emploi

Page 14

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NL

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Pagina 20

Hoorversterker

I

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Pagina 26

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E

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Página 32

Amplificador audiométrico

Bedienelemente

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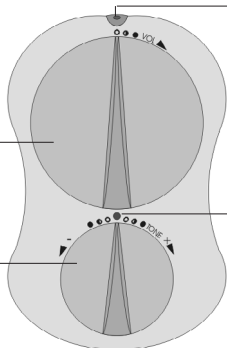
Lautstärke-
regler

Hochton-
Tieftonregler

Internes
Mikrofon

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

Batteriefach

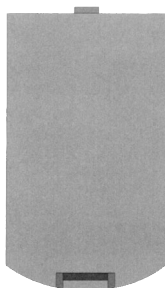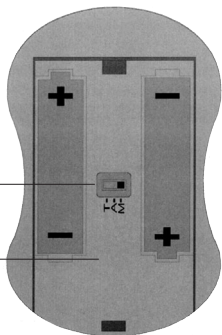Batteriefachdeckel

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Kopfhörer-/
Teleschlingen-
Buchse

Internes Mikrofon

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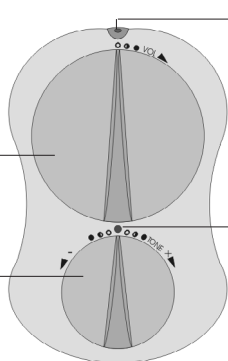
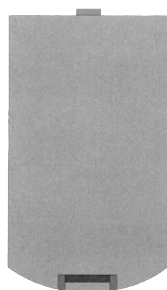
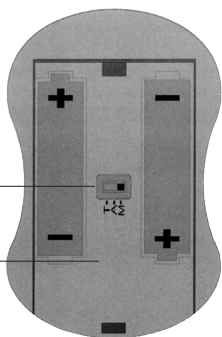
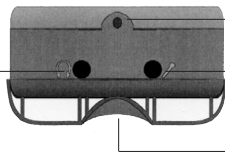
Headphone/
teleloop jack

Built-in microphone

Jack for external microphone / audio cable

Clip

Congratulations on purchasing your *Crescendo 50* amplifier. You have selected a modern and reliable system. Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

Standard components

Please check if all following components are included:

- *Crescendo 50* amplifier with internal microphone
- 2 x 1,5 V AA batteries
- Plug-in microphone (external microphone) with extension cable
- DIR-Headset with stereo plug
- Belt clip with neck cord
- Operating instructions
- Warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

Operating principle

The *Crescendo 50* uses either the built-in microphone or the external microphone to pick up acoustic signals, amplifies them and emits them via the headphone jack. The amplifier is equipped with a pick up coil which enables you to receive signals from an inductive loop system.

Inserting batteries

The amplifier is operated with 2 AA batteries (Type LR6 or AA).

The battery compartment is located at the back of the amplifier.

To insert the batteries:

1. To open the battery compartment, press the tab against the cover and then lift the cover.
2. Insert the two batteries according to the polarity marks.
3. Close the battery compartment.

WARNING: The sound output levels from this system can be very high. It is recommended that the volume control should be turned to the »OFF« position before changing the battery or plugging in or unplugging the headphones or microphone.

Signal reception

Acoustic signals can be picked up by the amplifier in four different ways:

1. The acoustic signals are picked up by the built-in microphone, whereby the operating mode selector in the battery compartment must be set to »M« (factory default setting for the *Crescendo 50*).
2. Acoustic signals from television sets or other audio devices can be picked up by the plug-in microphone included with the system. Simply plug this microphone into the microphone jack. (If an external microphone is used, the built-in microphone will be switched off automatically.) In addition, the operating mode selector in the battery compartment must be set to »M«.
3. Audio signals can also be picked up by a stereo/audio cable. In this case, the operating mode selector in the battery compartment must be set to »A«. Connect the audio cable with the *Crescendo 50* and the audio source (e.g. television set).
4. To pick up signals from an induction loop system, the operating mode selector in the battery compartment must be set to »T«. Please note that any microphone connected must be disconnected first.

Switching the device on

To switch the amplifier on, turn the volume knob clockwise until you click past a noticeable resistance. The ON/OFF indicator will light up to show you that the device has been switched on.

Setting the volume/tone

Adjust the volume control to set the device to the volume you prefer. Use the treble/bass control knob to adjust the tone; the treble and the bass will be increased accordingly.

Switching the device off

To switch the amplifier off, turn the volume knob counter-clockwise until you click past a noticeable resistance. The green ON/OFF indicator will switch off.

Adjusting the operating mode selector

In order to use the amplifier's different options for picking up signals, the selector in the battery compartment must be set to the corresponding position. Set the operator mode selector as follows:

1. Open the battery compartment cover as described in »Putting in the batteries«.
2. Use a small screwdriver to set the switch to the desired position.

The following three positions are available:

- a. Position »M«: Operation using the built-in/external microphone. (If an external microphone is used, the built-in microphone will be switched off automatically.)
- b. Position »A«: Switch position for the direct connection of any audio device using a 3.5 mm stereo/audio cable.
- c. Position »T«: Operation as an induction loop receiver.

What to do if the amplifier doesn't work

Make sure that the microphone cable or the headphone cable/teleloop cable is not damaged in any way.

No sound

- Headphone/teleloop and microphone plugged into the wrong jack
 - Check connections.
- Batteries dead
 - Replace old batteries with new ones.
- Operating mode selector set to wrong position
 - Set the operating mode selector in the battery compartment to »M« to use the built-in microphone or the external plug-in microphone;
 - If set to »T«, you are using your amplifier as an induction loop receiver.

Quiet or distorted sound

- Batteries almost dead
 - Replace old batteries with new ones.

Tinny or muted sound

- Tone control knob set incorrectly
 - Set the control knob to a middle setting.

Note: The *Crescendo 50* is not designed to operate with headphones or neck-loops with mono jack plugs. Using these may cause damage to the amplifier.

Maintenance and care

The *Crescendo 50* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the *Crescendo 50* where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The *Crescendo 50* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer. The warranty covers the repair of the product and returning it to you free of charge during the warranty period. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical data

Dimensions:	26 x 62 x 95 mm (H x W x D)
Colour:	silver/grey
Weight:	110 g
Amplification:	max. 50 dB
Power supply:	2 x 1,5 V mignon batteries (type LR6 or AA)

This product complies with European Union norms.
You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

Technical specifications subject to change without notice.

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Humantechnik GHL AG