»Crescendo 60«

DE

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Seite 2

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EN

Operation Instructions

Page 18

Personal Listening System

FR

Mode d'emploi

Page 34

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NL

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Pagina 50

Hoorversterker

IT

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Pagina 66

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
ES

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Página 82

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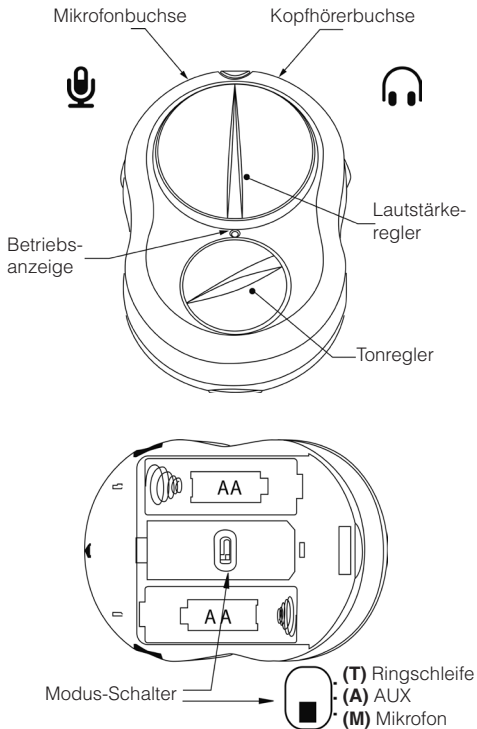Bedienelemente

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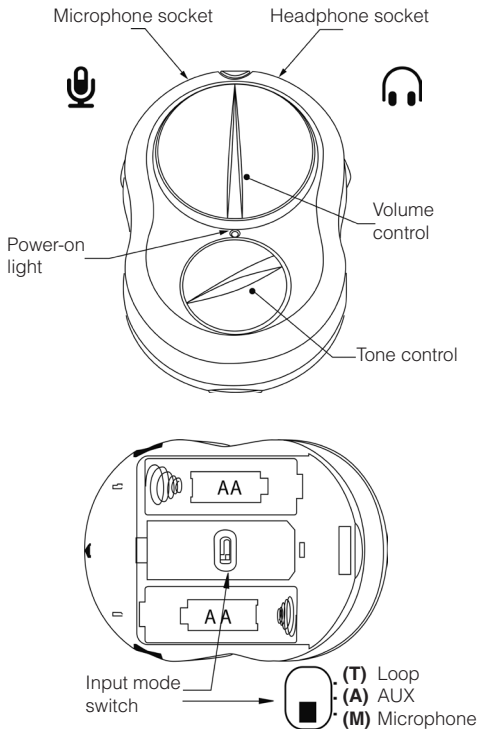
Operating controls

Thank you for choosing the *Crescendo 60* personal listening system. Please read these operating instructions to benefit from all the capabilities of the system and it will give you many hours of reliable listening performance.

Package contents

Please check if the following parts are included:

- »*Crescendo 60*«
- 2x batteries type AA / LR6
- Stethoscope headphones
- Plug-in microphone and lead
- Belt clip
- Neck cord
- Manual and warranty card

Should any parts be missing, kindly contact your specialist dealer or the manufacturer directly.

Function

The *Crescendo 60* will help you to communicate with confidence. It is a battery powered amplifier that can receive sound from various sources, and then amplify that sound to the way you like to listen. The *Crescendo 60* has a wide-ranging tone control that can boost the high and low frequencies according to your hearing needs.

WARNING!

The sound output from this system can be very high!

Switch off before changing the batteries or plugging or unplugging any accessories.

PLEASE NOTE:

The manufacturer cannot be held responsible for damage caused by not using the *Crescendo 60* in accordance with these safety precautions.

Do not open the casing of the *Crescendo 60* except to change batteries or operate the mode switch. No user-serviceable parts can be found inside.

Ensure mains powered equipment is switched off before connecting to the *Crescendo 60*.

Do not expose the *Crescendo 60* to rain or any other forms of moisture.

Do not expose the *Crescendo 60* to direct sunlight or extreme heat such as radiators.

Only AA batteries of the type specified can be used. Do not mix old and new batteries.

Use rechargeable batteries in accordance with the instructions supplied with them.

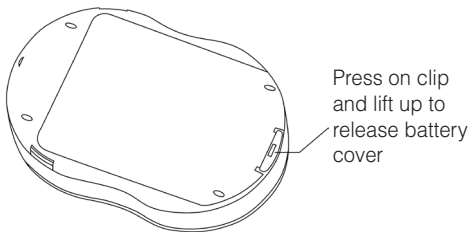
If the *Crescendo 60* is not to be used for a long time, remove the batteries.

Inserting Batteries


The unit uses 2 AA alkaline batteries (LR6 or AA).

Remove belt clip if fitted (see below).

Remove the battery cover located on the rear of the amplifier by pressing in the clip on the end of the unit and lifting the cover upward.

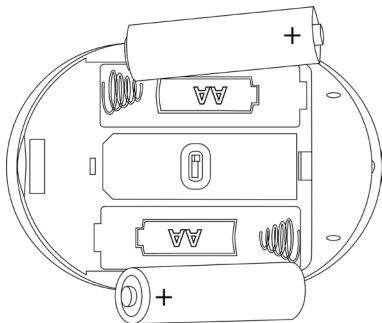Insert the two batteries according to the symbol shown in the battery compartments.

Replace the battery cover.

Basic Operation

The *Crescendo 60* is delivered set for microphone use.

Insert batteries as shown above.

With the amplifier switched off, check volume control is turned to left until click felt. Control will point upwards and power light will be off.

Plug a microphone into the microphone socket with the microphone symbol.

Plug either headphones or a neck loop (for hearing aid users with loop facility) into the headphone socket with the headphone symbol.

Wear the headphones or a neck loop. Switch the amplifier on. Turn the volume control clockwise past the “click”.

A green LED will light up to indicate the amplifier is on. When colour changes to red the batteries are coming to the end of their useful life.

Adjust the volume to suit your listening level and the tone control to boost the high or low tones to adjust clarity.

When finished turn the amplifier off by turning the volume control anti-clockwise until it “clicks”. The green LED will go out.

Input Options

- Microphone

Select »**M**« on the mode switch.

For general listening using either a plug in or a long lead microphone to get right to the sound source.

Plug the microphone into the socket with the Microphone symbol.

- Direct connection (AUX in)

Select »**A**« on the mode switch.

To listen with maximum clarity to TV or Audio from the SCART, RCA Phono or Jack outputs. Plug the appropriate lead into the socket with the Microphone symbol.

- Loop receiver

Select »**T**« on the mode switch.

Check or listen to an existing loop system using headset or headphones.

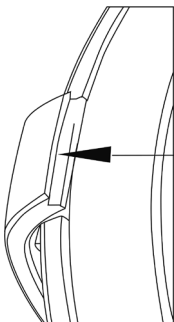
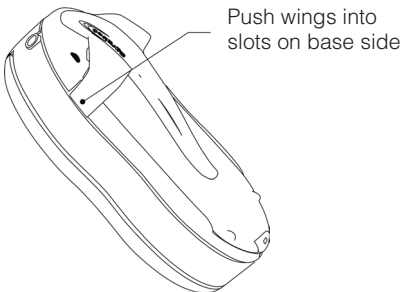
Listening Options

The following accessories with 3.5 mm jack plug can be connected to the headphone socket:

- Stethoscope Headset (included)
- Headphones:
Wear any conventional over-the-head headphones with 32 ohm impedance with a 3.5mm stereo jack plug.
- Earphones
- Neck Loop:
Can be worn around the neck to transmit the sound directly to the hearing aid on the »**T**« or loop setting.
- Ear hook:
Worn adjacent to the behind the ear hearing aid to transmit the sound directly to the hearing aid on the »**T**« or loop setting.
- Direct connection to hearing aid:
Direct connection to a hearing aid with the appropriate facility and lead.

Beltclip / neckcord

To fit the belt clip, simply press the wings of the clip into the position shown:


To remove the clip press one of the wings away from the body.

Do not try to remove the clip by lifting the clip away from the battery cover as you may damage the parts.

To fit the neck cord remove belt clip and thread the cord as shown, and replace the belt clip.

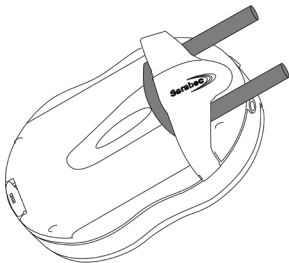The neck cord has a safety release feature to avoid you getting trapped by the cord.

Troubleshooting

- No Sound:

Check if headset / microphone are in the correct sockets.

Check operating mode.

Batteries may be exhausted - fit new ones

Headset or microphone may be faulty.

- Low or distorted sound:

Batteries nearly exhausted - fit new ones.

Check operating mode.

- Feedback or squealing sound:

Microphone is too close to headset and volume is set too loud.

- Crackling or sound goes on and off:

Dirty plugs - clean plugs

Faulty leads, wiggle leads to find faulty lead - change lead!

- Tinny or muffled sound:

Tone control in wrong position.

Maintenance and care

Headset ends can be removed for cleaning and can be washed in warm soapy water but must be thoroughly dried before replacing!

If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition, it must be protected

against excessive heat, moisture and severe mechanical shocks.

Warranty

Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is desirable that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units.

You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal.

Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Specifications

Power requirements:

2 X 1.5V AA/LR06 Alkaline batteries.

Battery endurance:

Battery life giving over 80 hours use with a 32 ohm headset and average listening levels.

Normal or rechargeable batteries may be used with reduced life.

Dimensions:

L 95mm / W 62mm / H 25mm (without belt clip)

Weight:

70g without batteries, approx. 120g with batteries.

Material:

ABS plastic casing.

Performance:

Frequency Response: 200Hz-10 kHz +/- 3dB

Maximum Gain: 60dB

Tone Control: +/- 8dB at 200Hz

+/- 8db at 5kHz

Maximum audio signal output:

>125dB with the included headphones

This device complies to the following EC standards:

- 2011 / 65 / EC RoHS-Directive
- 2012 / 19 / EC WEEE-Directive
- 2014 / 30 / EC EMC-Directive
- 2014 / 35 / EC LVD-Directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at www.humantechnik.com.

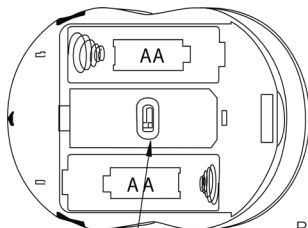
Technical specifications subject to change without prior notice.

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Sarabec Ltd