HUMANTECHNIK

soundshuttle

**D**

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Seite 2

Tragbares Ringschleifensystem

**GB**

**Operating Instructions**

Page 6

Portable induction loop system

**F**

**Mode d'emploi**

Page 10

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
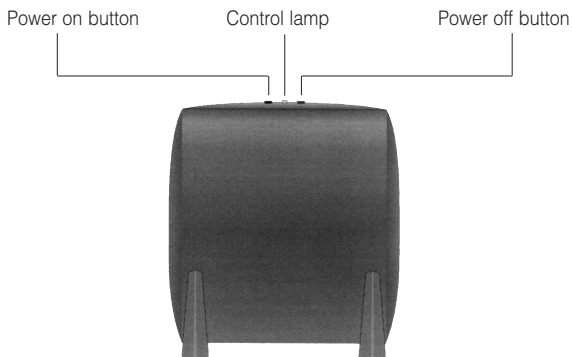
### Operating elements - front

### Operating elements - back

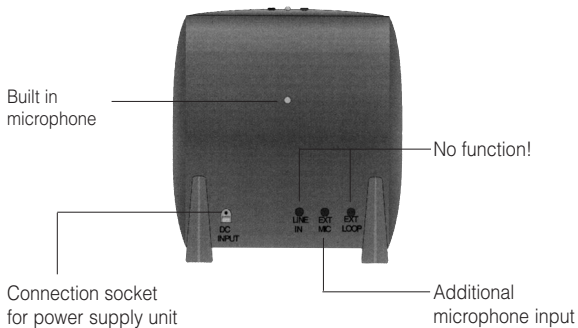We would like to congratulate you on purchasing your *soundshuttle*. You have chosen a modern, reliable device. Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

### **How it works**

The *soundshuttle* uses its built-in microphone to record sounds and emits them via the built-in induction loop. The signals emitted in this way can be received by a hearing aid that is set to the "T" or "MT" position.

### **Package contents**

The following parts are included in the package:

- *soundshuttle*
- Power supply unit
- Operating instructions
- Warranty card

### **Setting up the *soundshuttle***

The induction loop system is already completely installed and set up in the housing. All you have to do is to place the *soundshuttle* at the desired position between you and the person you are talking to and connect the power supply unit. Please note that the microphone on the back of the device should be facing the speaker.

### **Switching the device on**

To switch the induction loop system on, press the green button on the top of the device. Once the device is switched on, the control lamp will light up green.

### **Recording/ emitting signals**

If sounds are received by the microphone on the back of the device and emitted by the device as magnetic signals, the control lamp will begin to flash orange.

### **Additional microphone input (3.5 mm plug)**

An external microphone can be connected to the additional microphone input. (e.g. microphone cable A-4901-0) (If an external microphone is used, the built-in microphone will be switched off automatically.)

### **Switching the device off**

To switch the induction loop system off, press and hold the red button on the top of the device until the control lamp goes off.

### **Power supply for the *soundshuttle***

The *soundshuttle* can be operated using the built-in rechargeable battery or the power supply unit. If the battery is fully charged, the device can be used for up to 4 hours. A dead battery has to be recharged for approximately 16 hours until it is fully charged once again. You can also use the *soundshuttle* while it is being charged, but please note that this will lengthen the time required to fully recharge the battery. The device can remain connected to the power supply voltage for as long as desired.

### **Maintenance and care**

The *soundshuttle* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

### **Warranty**

The *soundshuttle* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer. The warranty covers the repair of the product and returning it to you free of charge during the warranty period. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer as well as a copy of the sales slip.

**Always specify the product number in any event.**

**Disposal** of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

### Specifications

Dimensions:	180 x 170 x 65 mm (HxWxD)
Weight:	420 g
Microphone sensitivity:	Up to 60 dB $\pm$ 3 dB
Power supply:	Primary 100-240 V 50-60 Hz AC/ secondary 15 V DC or built-in 12 V NiMH rechargeable battery
Output power:	max. 7 W

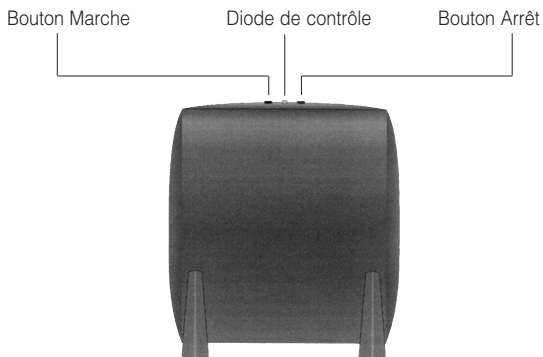
This product complies with European Union norms.  
You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

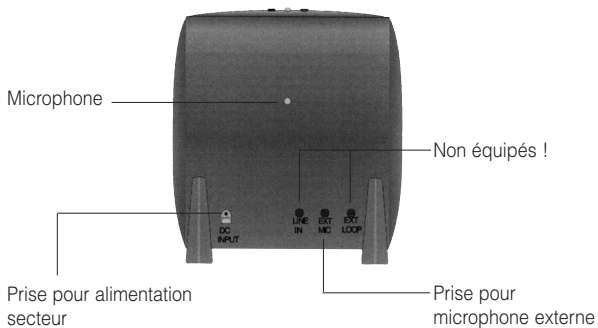
Technical specifications subject to change without notice.

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## **Humantechnik GHL AG**