# AUDIOropa

## LA-70

DE

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Seite 2

Class D Ringschleifen-  
verstärker

EN

### **User manual**

Page 12

Class D loop amplifier

FR

### **Mode d'emploi**

Page 22

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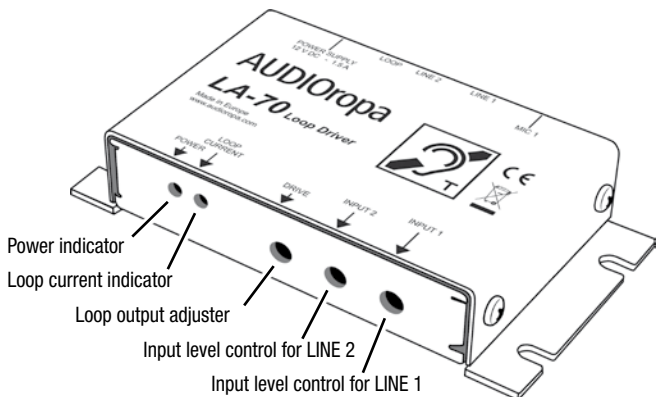
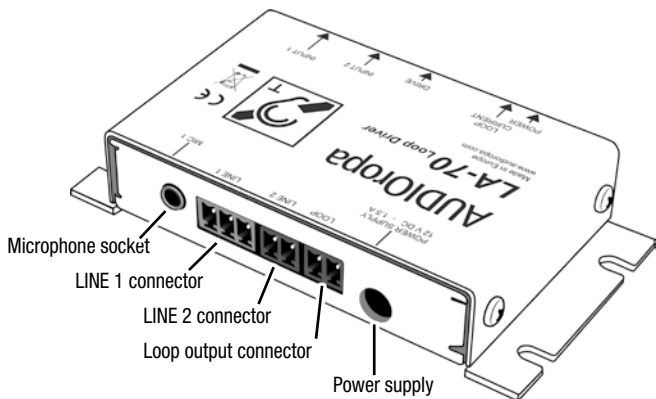
## Control elements

We would like to congratulate you on purchasing the »LA-70« Class D loop amplifier! You have chosen a modern and reliable device. First, please read this manual carefully to become familiar with the device and all its options before using it.

## **Contents of package**

Please check if all the listed parts are included:

- »LA-70«
- 12 V-power adapter
- Power lead
- Terminal connectors: 2x 2-pole, 1x 3-pole
- Adhesive loop symbols
- User manual
- Warranty card

Should any parts be missing, please contact your dealer or the manufacturer directly.

## **Concept of function**

The »LA-70« takes in sound via a microphone or direct connection lead from an electronic audio source and sends a signal into a loop cable which is placed around the listening area. The sound is transmitted by induction, as »magnetic sound waves«, to an accordingly equipped hearing aid.

## **Indicators, controls and connectors**

*POWER:* This indicator LED is lit permanently if the amplifier is provided with electric power.

*LOOP CURRENT:* This LED indicates that a signal is transmitted to the loop cable and that it is functioning correctly.

*DRIVE:* This control adjusts the loop current.

*INPUT 1:* Adjusts the input level for LINE 1.

*INPUT 2:* Adjusts the input level for LINE 2.

*POWER SUPPLY:* Connect the power adapter to this socket.

*LOOP:* A 2-pole terminal connector with the attached loop cable is plugged into this port.

*LINE 2:* Input port for audio-in 2, 2-pole terminal connector. Possible connection modes: Line, low-impedance, 100V.

*LINE 1:* Input port for audio-in 1, 3-pole terminal connector. Possible connection modes: Line, microphone.

*MIC 1:* Microphone port, 3.5 mm jack with phantom power.

- When using only one audio-in port, adjust the input level in such a way that the maximum output is reached.

### **Please note:**

- Disconnect the power from the device before attaching or removing components.
- Turn the DRIVE control to the lowest setting before powering the device.

## **Connecting the loop cable**

In order to connect a loop cable or the loop pad (available as accessory), take the free ends of the cable, insert them into one of the included 2-pole terminal connectors and fix them by tightening the screws. Please make sure that only the metal of the cable ends is wedged in the terminal clamps. It is not necessary to regard a certain polarity. Plug the terminal connector into the port marked with "LOOP".

## **Audio signal input**

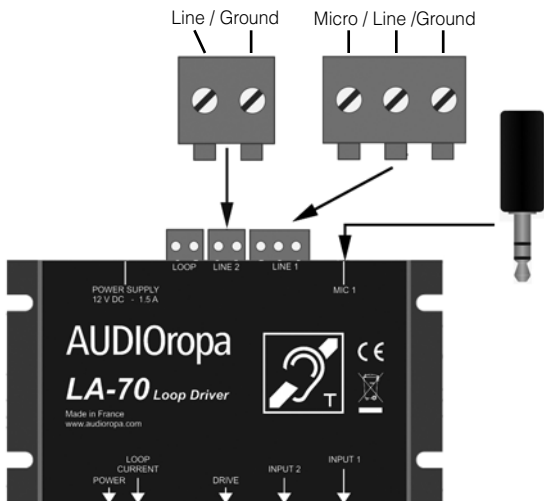
The ports MIC1, LINE 1 and LINE 2 serve as inputs for audio signals.

Microphones and outputs from stereo sets and PA-systems can be connected:

MIC1 is intended for a 3-pole 3.5 mm jack plug with phantom powering, for connecting condenser microphones.

LINE 1 offers a 3-pole connection for microphones as well as regular audio input signals.

LINE 2 has a 2-pole connection for regular audio input types.

## Use of the loop system

- Users of hearing aids should now switch to T-coil reception (switch to „T“ or „MT“).
- Set the volume to the preferred level.

Attention:

Before attaching or removing lines to the loop amplifier, disconnect it from the power supply first.

## **Troubleshooting**

### No sound:

- Check all connections.
- Check if the power indicator is lit.
- Check if the audio source is connected.

### Low sound level:

- Incorrect placement of loop or loop pad (accessory).
- Volume is set too low. Check the volume setting of the audio source and the induction receiver (hearing aid).

### Distorted sound:

- Loop output is set too high. Lower the adjustment.
- Connection cable is damaged.

### Background noise (buzzing or humming):

- If the noise remains after switching the »LA-70« off with the induction receiver being on, check the surrounding for sources of interference like TV-sets, fluorescent lights or dimmer switches. Keep the »LA-70« switched off and deactivate on device after the other until the interference disappears.
- If the noise disappears together with switching the »LA-70« off, a system error or a damaged connection lead may be present.

## Maintenance and care

The »LA-70« does not require any maintenance. If the unit becomes dirty, simply wipe it clean with a soft, damp cloth.

Note: Disconnect the »LA-70« from the power supply first. Never use spirits, thinners or other organic solvents. Do not place the »LA-70« loop amplifier where it will be exposed to full sunlight

for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

## Warranty

The »LA-70« is a very reliable product. Should a malfunction occur despite the unit being set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. If possible send the product back in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the product seal). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

**Disposal** of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

## Specification

Power supply unit:	Primary: 100-240 V / 50-60 Hz AC Secondary: 12 V DC 1.5 A
Output power:	max. 20 W
Peak current:	3.5 A
RMS current:	2.1 A at 1 kHz
Compressor:	Variable 1 : 1 to 20 : 1
Noise reduction:	Bandwith limits for 8 KHz total gain, 16KHz to -6dB
THD:	THD+N <0,5% à 1kHz
Protection:	Thermal, short-circuit and start
Dimensions:	100 x 50 x 21 mm (LxHxD)
Weight:	78 g

This device satisfies the following EU directives:

- 2011/65/EC RoHS-directive
- 2012/19/EC WEEE-directive
- 2014/30/EC EMV-directive
- 2014/35/EC Low voltage directive

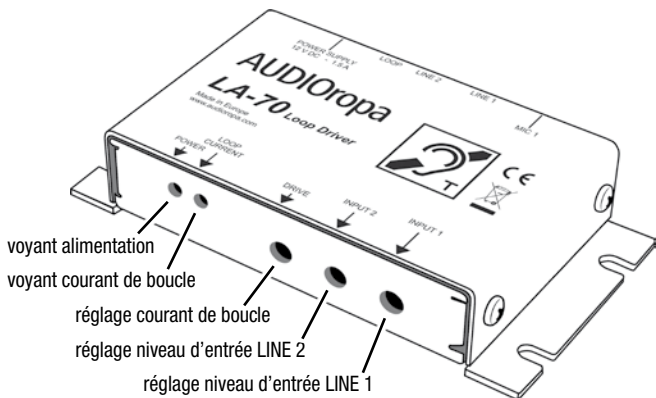
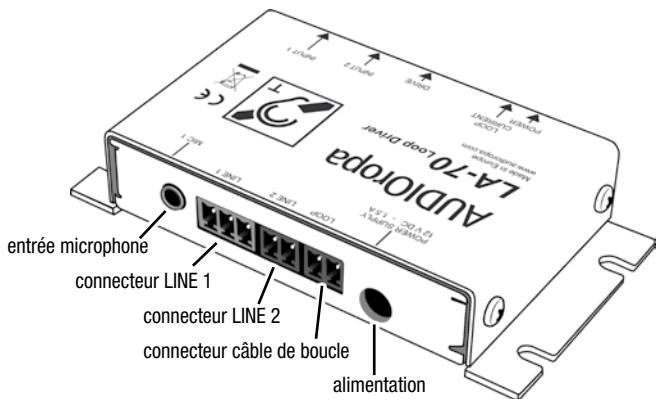
Compliance with the directives listed above is confirmed by the CE seal on the device.

The complete CE compliance declaration is available on the internet at [www.audioropa.com](http://www.audioropa.com).

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**Sarabec Ltd**