**DE**

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Seite 2

**Class D Ringschleifenverstärker**

**EN**

**User manual**

Page 20

**Class D loop driver**

**FR**

**Mode d'emploi**

Page 38

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# INHALT

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# INDEX

1. Introduction.....	21
1.1 Warning notes.....	21
2. Presentation of the loop driver.....	21
2.1 Description.....	21
2.2 The range.....	21
2.3 Package contents.....	22
2.4 Advice and safety.....	22
3. Function principle.....	23
4. Indicators, connectors and controls.....	23
4.1 Indicators.....	23
4.2 Multi-loop output.....	23
4.3 Front panel and settings.....	24
4.4 Rear panel and settings.....	25
4.5 Rack mounting.....	26
4.6 Adjustments and connections.....	28
4.7 Combining multiple drivers.....	31
5. Setting up.....	33
5.1 Setting a single loop.....	33
5.2 Setting up a master-and-slave configuration.....	33
5.3 Securing settings.....	34
5.4 Metal loss compensation adjustment.....	34
5.5 Operation of the fault contact.....	34
5.6 Audio Input.....	35
Maintenance and care.....	36
Warranty.....	36
Disposal.....	36
Specifications.....	37

# 1. Introduction

Thank you for having purchased the *PRO LOOP D5* class D loop driver!  
Please take a few moments to read this manual. It will ensure you the best use of the product and many years of service.

## 2. Presentation of the loop driver

### 2.1 Description

The PRO LOOP Class D series consists of loop drivers made to equip rooms with audio support for people with hearing loss.

This series has been developed under strict and rigorous specifications. These devices have been designed to facilitate their use and their installation.

### 2.2 The range

The PRO LOOP D5 belongs to a new generation of induction loop drivers.

This driver is amongst the most compact on the market while offering the necessary features (AGL, MLC, compressor, etc.) to ensure installation for rooms up to 300 m<sup>2</sup> with a single device.

This coverage makes it possible to comply with the international IEC-60118-4 standard.

### 2.3 Package contents

The following items are included:

- PROLOOP D5 induction loop driver
- Power cable 1.5 m, connectors CEE 7/7 - C13
- 2x 3-point connectors
- 1x 2-point connector, output
- 1x 2-point connector, input
- Rack mounting brackets and screws
- Adhesive loop-indication signs
- Warranty card

Should any of these items be missing, please contact your local retailer or the manufacturer.

## 2.4 Advice and safety

The majority of problems with induction loops happen when the installation has not been properly planned so take time before starting the installation.

Ideally, the loop driver should be placed near the area to be covered. This may involve placing the loop driver on a panel, under a desk or under a counter.

To ideally position the loop in the area to be equipped, it is important to take the users of the system into account.

For example, if you only need to have the speaker and the client, a reduced coverage with a centered loop may be a better solution than a loop around the perimeter of the room. It will limit radiation and increase confidentiality.

- To prevent accidents or personal injury caused by electrical shocks, never place any type of container filled with a liquid, such as a vase, on the device.
- Never pull on the power cord to remove the plug from the wall outlet; always pull the plug.
- Do not operate the device near heat sources or in rooms with high humidity. (Operating temperature range 0-45°C).
- Do not cover the air vents so that any heat generated by the device can be dissipated by air circulation.
- Depending on the stress it is exposed to, the device can develop high temperatures in the heat sink and at the air vents. Be careful not to touch these elements - risk of burns!
- Disconnect the mains plug from the socket in order to switch off the appliance completely, as standby operation and thus low energy consumption takes place when the appliance is switched off via the power switch.
- An installation must be carried out by qualified personnel.
- The device must be out of reach of unauthorized persons.
- The device may only be operated by persons with appropriate knowledge.
- The device is only to be used for operating inductive loop systems!
- To prevent potential injury, do not position this device higher than 2 metres from a horizontal surface.
- Connect the loop driver only to wiring which complies to IEC 60364.

## 3. Function principle

An induction loop system is installed mainly in a room. The installation consists of an electrical wire which is laid around or within the room, forming one or more loops. Both wire ends are connected to the loop driver.

Hearing aids are often equipped with a coil, often called »T Position« or »T-coil«. This coil, placed inside the prosthesis, is capable of reacting to the magnetic field generated by the loop wire of the room. By induction, the electrical signal present in the loop will be

reflected in the coil of the hearing aid. The coil transmits the signal of the loop to the internal amplifier of the prosthesis, which will then bring it to the ear of the hearing-impaired person - free of environmental disturbances.

The sound source can be anything. In a cinema, for example, the sound of the film will be fed into the loop. In a conference room, it's the sound of the orator's microphone. In a train station, the voice of the station master, etc.

The loop can be installed at the floor or ceiling level, more precisely between 1.20m and 1.70m from the listening height (ears). Therefore it is to be considered if the potential audience is standing or seated. The loop is integrated in the building, as is the electrical installation.

The presence of a hearing loop is often indicated by a blue or yellow symbol representing a barred ear and a letter »T«.

Adhesive signs bearing this symbol are included in the package. Please use them to indicate that hearing impaired people can benefit from the otherwise hidden installation of the inductive loop.

## **4. Indicators, Connectors and Controls**

### **4.1 Indicators**

The loop driver incorporates a monitoring system which checks the main functions of the device, such as the power amplifier, the integrity of the loop cable connected to the driver and the inputs.

If one of the functions is faulty or does not work, an according LED on the front panel of the amplifier lights up and the fault relay opens (NO).

### **4.2 Multi-loop output**


The PROLOOP D5 driver has two 0° and 90° phase-shifted outputs and one IN-Input on the rear panel. (see image on page 25). These features offer the possibility to link multiple drivers with each other.

Depending on the design of the layout, it is therefore possible to control the external »overspill« of the field and/or the homogeneity of coverage in the coverage area.

This multiple output feature is important if an optimal loop system is to be achieved.


For more application-specific information on using the device and planning the best loop layout according to your project, please contact us at [info@audioropa.com](mailto:info@audioropa.com).

### 4.3 Front panel and controls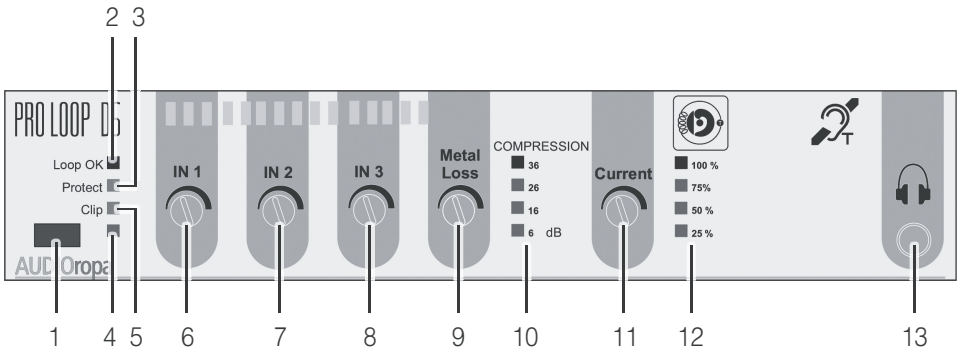

- 1 ON / OFF power button.
- 2 LED »Loop OK«. This green LED is displayed when the loop is in working condition.
- 3 LED »Protect« error display. This red LED lights up when the driver unit for the loop overloads or the loop is defective.
- 4 Power LED, ON / OFF display. This light indicates the device is on power.
- 5 LED »Clip« error indicator. This red LED lights up when the input level for the loop is too high.
- 6 Input 1 setting (Line or microphone). This adjusts the signal level of INPUT 1.
- 7 Input 2 setting (Line or microphone). This adjuster is for the signal level of INPUT 2.
- 8 Input 3 setting (100V). This controls the level of INPUT3.
- 9 Metal Loss Compensation: Adjust to reduce interference with metal structures.
- 10 Input level indicator.
- 11 Loop current output adjuster. Controls the electric current in the loop.
- 12 Loop OK output indicator.
- 13 6.3 mm headphone jack for monitoring.

## 4.4 Rear panel and connectors

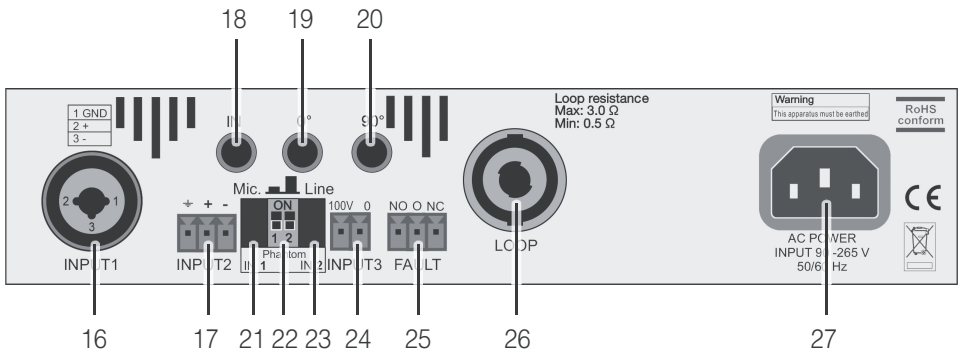
- 16 INPUT 1 Combo: Mic or Line input for external audio. Accepts XLR and 6.35mm jack.
- 17 INPUT 2: Mic or Line input for external audio. 3 port block type Phoenix input.
- 18 Input. This input is used to connect the output of another loop driver in Master/Slave configurations.
- 19 Output to slave amplifier »0°«. This output can be connected to the input of another loop driver in Master/Slave configurations.
- 20 Output to slave amplifier »90°«. This 90° phase-shifted output can be connected to the input of another loop driver in LOS-Master/Slave configurations.
- 21 Button for line or microphone selection for INPUT 1. Switch the input according to the source (microphone or line).
- 22 Phantom power feed switch. Activates the supply voltage for microphones on INPUT 1 and/or INPUT 2.
- 23 Button for line or microphone selection for INPUT 2. Switch the input according to the source (microphone or line).
- 24 2 port block type Phoenix type input: 100V priority. This terminal block input allows to connect an external audio input from a 100V sound system.
- 25 NO / NC fault relay. Allows system status information to be forwarded via a relay.
- 26 Loop output »speakON« connector. Connects the inductive loop to the driver.
- 27 AC power cord connection.

## 4.5 Rack mounting

### 4.5.1 Ventilation and rack mounting

Leave a space of 1U (44mm) above the loop driver.

Leave a space of at least 168 mm behind the loop driver.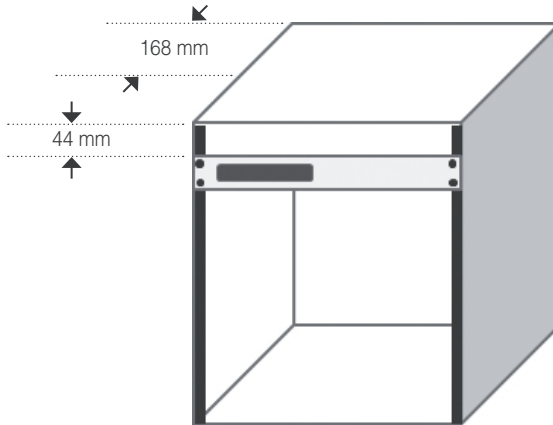

#### **Attention, Warning, Danger :**


The loop driver contains an advanced protection circuit, which allows it to reduce the power output to maintain safe operating temperatures. Insufficient ventilation may cause the amplifier output power to be reduced during normal operation (indicated by the red PROTECT LEDs lighting up).

To reduce the risk of thermal limitation and to allow proper heat dissipation, it is recommended to keep the space directly above and behind the device clear.

### 4.5.2 Rack integration

Fit the included rack mounting brackets as shown in the following image, using the provided mounting screws.

Then install the loop driver into the rack.
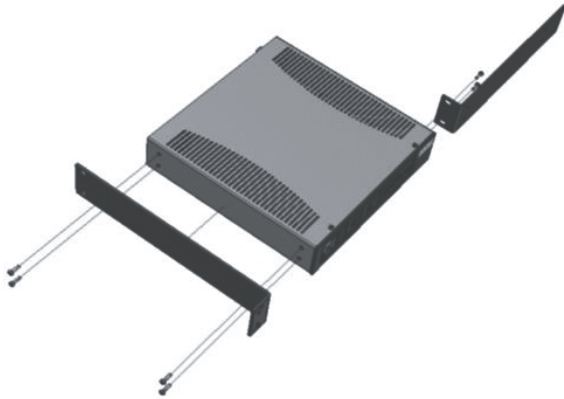


### 4.5.3 Rack-mounting two loop drivers

Fit the included rack mounting brackets as shown in the image below, using the provided mounting screws.


Attach the amplifiers to each other using the brackets.

Finally, install the loop drivers in the rack.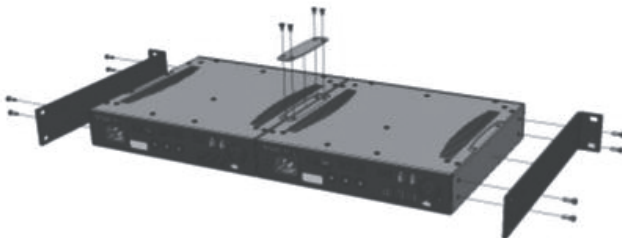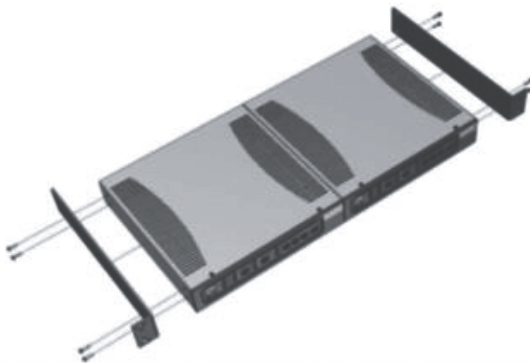

### 4.5.3 Mounting the loop driver on a wall

Using the provided screws, fasten the brackets to the amplifier as shown in the picture below. Then attach the amplifier to the according wall.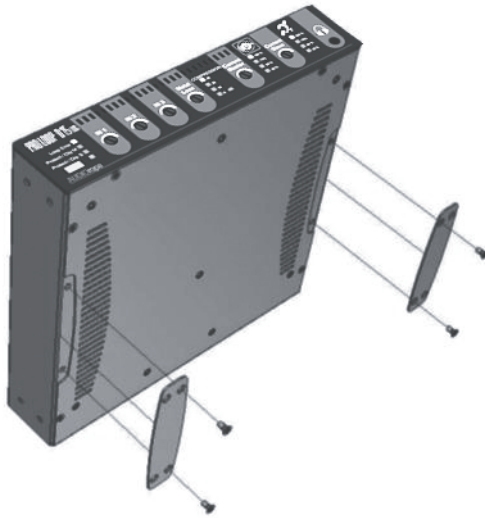

## 4.6 Adjustment and connection

### 4.6.1 Loop connection

The induction loop is connected via the »speakON« connector located on the rear panel of the driver.

The cables connecting the actual loop and the driver must be placed close and parallel to each other. If possible, twist them. This avoids inductive interference which may be caused by other transformers in the vicinity.

### 4.6.2 Audio inputs

Audio sources connect via the 3 inputs of the amplifier provided for this purpose.

The driver has 3 types of input :

INPUT 1: Line or Micro

INPUT 2: Line or Micro

INPUT 3: 100 V

### 4.6.3 Priority 100V input

INPUT 3 (100V) on the rear panel of the driver is designed to prioritize PA systems in order to facilitate emergency announcements and other important notifications.

If several active audio sources are fed to the inputs of the driver, it will always be INPUT 3 that has priority over the others. Incoming signals on INPUT 3 will mute INPUT 1 and INPUT 2.

In some cases, and if the configuration allows it, we recommend to connect the sound system on INPUT 1 and INPUT 2 of the driver and the PA source on the INPUT 3.

### 4.6.4 Input and outputs jack 6.35

On the back of the amplifier there are 3 jacks 6.35mm, these in- and outputs can connect several PRO LOOP D drivers together to create large single-phased loop systems, or low overspill systems by using the 90° phase-shifted output.

Various configuration possibilities are described in section 4.7.

### 4.6.5 Line output

It is possible to connect a recorder and other gear to the line output of the loop amplifier. The line output of the amplifier is the 6.35mm jack marked »0 °«, which is also used to link several amplifiers when installing complex systems (double loop). A further 6.35mm jack socket can be found on the front of the device, for easy monitoring with headphones.

### 4.6.6 Power supply

PRO LOOP D drivers have a direct power supply of 230V AC (or 115V), of a power of 300VA.


### 4.6.7 Status relay

The status output is used to send information about the status of the loop driver to external devices via a NO / NC relay.

### 4.6.8 Connectors :


Symmetric:

- Pin 1 : Mass/Ground/Shield
- Pin 2 : Hot spot (+)
- Pin 3 : Cold spot (-)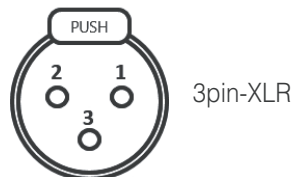

Asymmetric:

- Pin 1 : Mass/Ground
- Pin 2 : Signal (+)
- Pin 3 : Connected to mass (pin 1)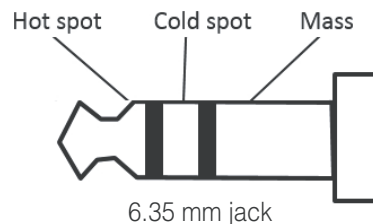

#### 4.6.9 Power on/off


The unit is powered up using the grey switch on the front panel of the loop driver. If the device is powered, the Power LED lights up.


To switch the unit off, press the Power switch on the front of the driver again. Please note that the device enters a stand-by mode when switched off. Disconnect the mains plug from the socket in order to switch off the appliance completely.

#### 4.6.10 Loop OK

This green LED on the front of the amplifier monitors the function of the loop. If the loop is disrupted, short-circuited or if the loop resistance is not between 0.5 and 3 Ohm, the »Loop OK« LED will not be lit and the »Protect« and/or »Clip« LEDs will light up instead.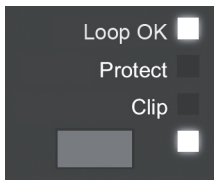

Loop in working order

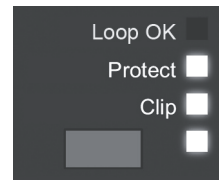Loop(s) faulty, input too high  
or driver overloaded


#### 4.6.11 Protect / Clip

The »Protect« and »Clip« LEDs indicate the status of the device. These red LEDs will light up if:

- The ohmic resistance of the inductive loop is not between 0.5 and 3 Ohm
- The respective driver section is overloaded or overheated
- The input level is too high, resulting in clipping of the audio signal.

## 4.7 Combining multiple drivers


### 4.7.1 Master over Slave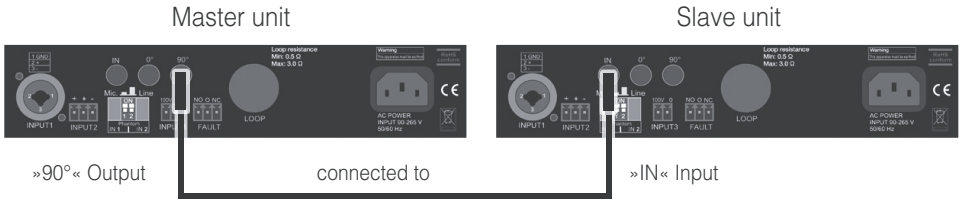

To establish a low overspill or high coverage system:

- Connect the respective inductive loops to the »speakON« connectors on the drivers.
- Connect an audio source to INPUT 1, 2 or 3 of the Master driver unit.
- Connect the »90°« output jack socket of the Master unit to the »IN« jack socket of the Slave unit using a 6.35mm to 6.35mm jack cord.
- Power all components up and adjust accordingly.

### 4.7.1 Slave to Slave

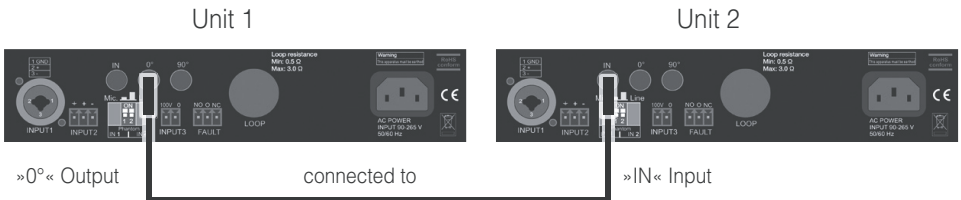
To establish a high coverage system:

- Connect the »0°« jack socket of the loop driver number 1 to the IN jack of the next loop driver.
- Power all components up and adjust accordingly.

This allows you to utilise several drivers on the same input. This can be of use if several rooms are on the same audio transmission.

## 4.7 Combining multiple drivers

### 4.7.1 Master to Slave over Slave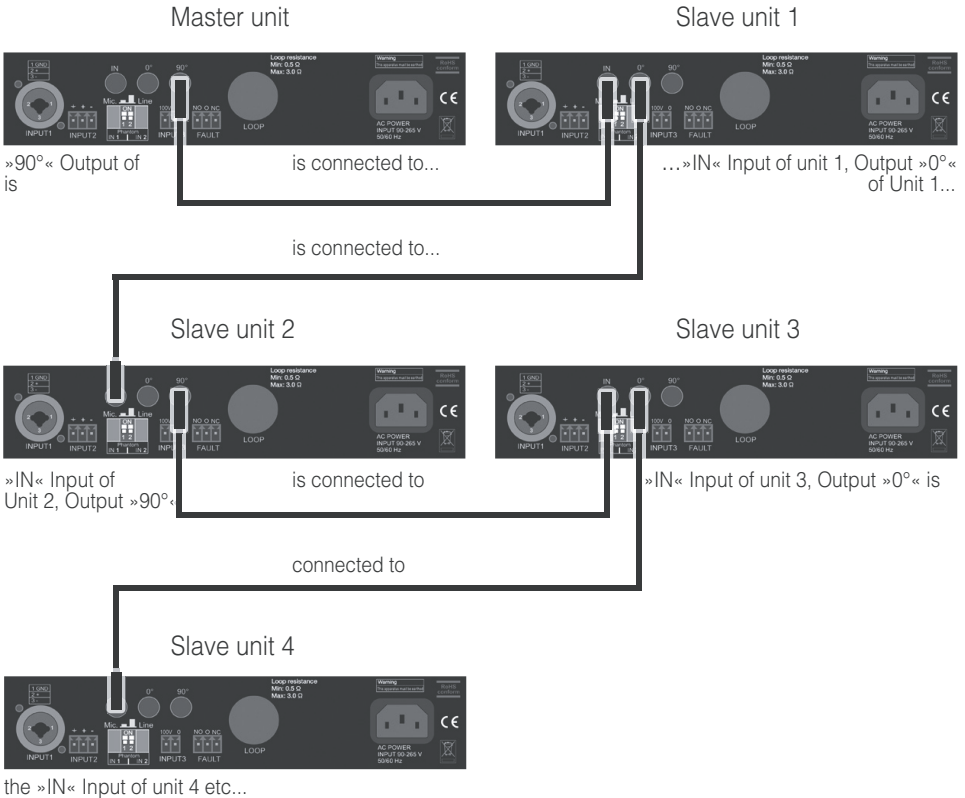

To establish a low overspill or high coverage system for very large areas:

- Connect the »90°« jack socket of the Master loop driver to the »IN« jack of the next loop driver (Slave unit 2).
- Connect the »0°« jack socket of unit 2 to the »IN« jack of the next loop driver (unit 3).
- Connect the »90°« jack socket of unit 3 to the »IN« jack of the next loop driver (unit 4).
- Connect the »0°« jack socket of unit 4 to the »IN« jack of the next loop driver (unit 5).
- Repeat this sequence as needed
- Connect the respective phase-shifted loops to the »speakON« connectors on the according drivers.
- Power all components up and adjust accordingly.

This allows you to install an inductive loop system in vast areas like exhibition halls and stadiums.

## 5. Setup

### 5.1 Setting up a simple perimeter loop

- Power up your amplifier and check that all potentiometers are at 0.
- Connect your loop to the »speakON« loop connector.
- Connect a 1 kHz sinusoidal source tone to one of the inputs.
- Increase the input signal on the front panel of the PRO LOOP driver to between 26 and 36 dB of the input level indicator.
- Increase the output current via the loop output adjuster on the front panel of the driver to reach between 75% and 100% on the output level indicator.
- Take a field strength meter and take a first measurement in the center of your room.
- Adjust the settings to -3dB in the center of the zone.
- Perform further adjustments to achieve a IEC60118-4 compliant installation.

### 5.2 Setting up a Master and Slave configuration

#### 5.2.1 Master driver

Drivers of the PRO LOOP D series can use a secondary (or more) unit for low-overspill loop installations, daisy-chained with an alternating 90° phase shift.

In cases where a very large coverage area is needed, several units can be combined in Master- and Slave setups to achieve this. The Master driver unit is the unit with the audio source connected at INPUT 1, 2 or 3. This is the basic signal, the phase shift modules within in the units will shift this signal by 90 ° or 0 ° to pass it to the next slave driver.

There is no particular setting needed to determine which driver will be a Slave unit: The Master driver is the only driver that will have INPUT 1, 2 or 3 in use, see section 4.7.1.

#### 5.2.1 Master loop

- Power up your Master driver and check that all controls are set to 0.
- Connect your primary loop to the »speakON« Master loop connector
- Connect a 1 kHz sinusoidal source tone to one of the inputs
- Increase the input signal on the front panel of the PRO LOOP driver to between 26 and 36 dB of the input level indicator.
- Increase the output current via the Master output adjuster on the front panel of the driver to reach between 75% and 100% on the output level indicator.
- Take a field strength meter and take a first measurement in the center of your room.

See the following section 5.2.2.

### 5.2.2 Slave loop

- Connect your secondary loop to the »speakON« Slave loop connector.
- Disconnect the primary loop from the »speakON« Master loop connector
- Increase the output current via the Slave output adjuster on the front panel of the driver to reach between 75% and 100% on the output level indicator.
- Take a field strength meter and take a second measurement in the center of your room.

### 5.2.3 Final adjustments

Connect the two loops and then adjust the settings until you reach a minimum signal of -3dB in the area with the lowest reception within the intended coverage area. Carry out the test procedures to perform an IEC60118-4 compliant installation.

For more details on the IEC60118-4 standard settings contact us at [info@audioropa.com](mailto:info@audioropa.com) or contact your local dealer.

### 5.3 Securing the settings

The amplifiers are supplied with a set of 8 small adhesive circles, these are intended to close the setting controls to avoid unwanted adjustments.

### 5.4 Metal loss compensation adjustment

If you recognise a distortion of the signal due to magnetic pollution, turn the Metal Loss control on the front panel of the driver clockwise.

Check your measurements in high frequencies with a sine wave tone of 4kHz and adjust the compensation settings to achieve acceptable measurements to IEC60118-4.

### 5.5 Operation of the fault contact


When the amplifier is working properly the »NO / NC« relay is activated, giving a connection between the »NC« (Normally Closed) pin and the »O« pin.

If the error detection registers an operating problem (disrupted loop, inadequate ohmic loop resistance, overloaded driver, etc.) the »NO / NC« relay is disabled, giving a connection between the »NO« (Normally Open) pin and the »O« pin.

## 5.6 Audio Input

### 5.6.1 Sensitivity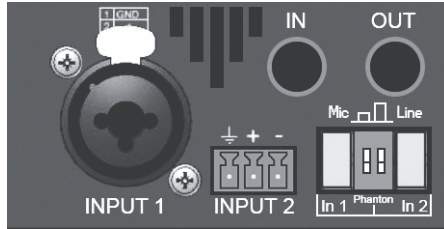

The levels of INPUT 1 and 2 can be set according to the audio source used.

If the connected audio source is a microphone, switch the according grey button to the »Mic« position, as indicated on the rear panel of the device.

If the connected audio source connected to one of the inputs is of line-type, switch the according grey button to the »Line« position.

### 5.6.2 Phantom power

The DIP switch on the rear panel of the amplifier (see depiction above) can activate phantom power for microphones which require it.

If the audio source connected to one of the inputs is a phantom-powered microphone, set the the according DIP switch to the upper position.

If the audio source connected to the inputs is a microphone which does not require phantom power, or if it is not a microphone, set the DIP switch to the down position.

## Maintenance and care

The »PRO LOOP D5« does not require any maintenance. If the unit becomes dirty, simply wipe it clean with a soft, damp cloth. Note: Disconnect the »PRO LOOP D5« from the power supply first. Never use spirits, thinners or other organic solvents. Do not place the »PRO LOOP D5« where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

When not used, store the device in a dry place, protected from dust.

## Warranty

The »PRO LOOP D5« is a very reliable product. Should a malfunction occur despite the unit being set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge.

It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the product seal). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

**Disposal** of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units.

You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal.

Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

## Specifications

Height / Width / Depth:	45 mm / 215 mm / 220 mm
Weight:	1.5 kg
Power supply:	110 - 250 Vac, ~ 50 / 60 Hz
Passive protection:	Fuse 6.3 A T, not accessible
Maximum area:	300 m <sup>2</sup> according to IEC 60118-4
Maximum input power:	< = 157 VA
Loop output	
Max. current:	5.8 A RMS / 16.4 A peak-to-peak
Max. tension:	34 V RMS / 96 V peak-to-peak
Connector:	Neutrik NL4 speakON-socket, layout 1+/-1-
Frequency range:	100-5000 Hz ( ± 3 dB)
Distortion:	< 1% at rated power output, 1 kHz
INPUT 1 and 2:	0,5 mV to 100 mV / 10 kOhm (MIC) or 25 mV to 4 V / 10 kOhm (LINE) Combo-XLR-3 socket (INPUT 1) Phoenix contact (INPUT 2) 12V toggable phantom power
INPUT 3:	100V-Input for connecting PA-systems
Automatic Gain Control (AGC)	Dynamics: up to 36dB
Environmental working range:	0°C - 45°C, up to 2000 metres MASL
Power consumption during idle:	10,5 W,at 230V AC, connected 1 Ω- loop, at room temperature after 30 min. stabilizing time
Max. power consumption:	25 W

This device complies with the following EC directives:

- 2017 / 2102 / EC RoHS-directive
- 2012 / 19 / EC WEEE-directive
- 2014 / 35 / EC Low voltage directive
- 2014 / 30 / EC Electromagnetic Compatibility

Compliance with the directives listed above is confirmed by the CE seal on the device.  
CE compliance declarations are available on the Internet at [www.humantechnik.com](http://www.humantechnik.com).

Technical specifications subject to change without prior notice.

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**Sarabec Ltd**