HUMANTECHNIK

cm-1

D

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Seite 2

Kommunikationssystem

GB

Operating Instructions

Page 22

Communication system

F

Mode d'emploi

Page 42

Système de communication

NL

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Pagina 62

Communicatiesysteem

I

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Pagina 82

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E

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Página 102

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S

Bruksanvisning

Sida 122

Kommunikationssystem

Allgemeine Informationen

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General information

The *CM-1* communication system makes it noticeably easier for hearing-impaired people to take part in conversations in difficult situations.


Each time you switch it on, take care to ensure that the volume on the *CM-1* receiver is set correctly to suit your personal needs in order to avoid possible damage resulting from excessive volume. To do this, turn the volume control down low.

The *CM-1* communication system contains small parts which can be swallowed and is not suitable for children under the age of 3.

If the device is not used properly, the manufacturer will not accept any liability for damage and injury.

General information	page 22
Standard scope of delivery	24
Commissioning and description of the controls	
The <i>CM-1</i> communication system	25
<i>CM-1</i> receiver	25
<i>CM-1</i> transmitter	29
CM charger	32
Interconnecting two CM chargers	34
Transport pouch	34
The <i>CM-1</i> communication system in everyday use	
At a presentation	35
Meeting around a table	36
In direct dialogue	37
On a trip – in a car, in a train compartment or on an airplane	37
Enjoying music or TV programmes	38
Changing the transmission frequency	38
Appendix	
Safety instructions	39
Maintenance and care	40
Warranty	40
Environmental advice/Disposal	40
Technical data	41
CE symbol	41

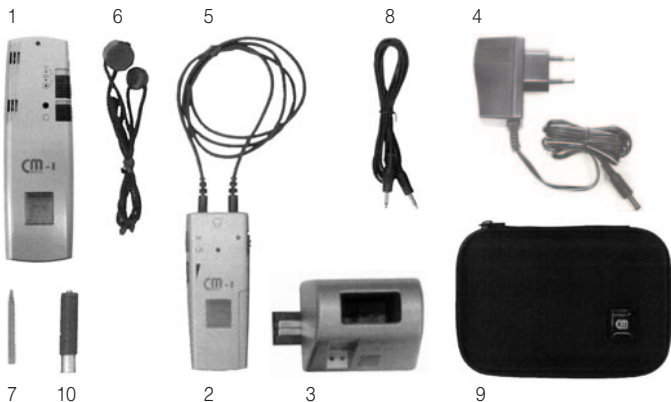Standard scope of delivery

The complete pack contains the following components:


No.	Device component	Quantity delivered
1	CM-1 transmitter	1
2	CM-1 receiver and ring loop	1
3	CM charger with a bridge element and safety screws	2
4	Power supply unit	1
5	Teleloop (induction loop)	1
6	Carrying loop	1
7	Pin to set the transmitter frequency	1
8	Audio cable	1
9	Transport poche	1
10	Battery »AAA«	3
	Operating instructions	1
	Warranty card	1

Commissioning and description of the controls

The *CM-1* communication system

The *CM-1* communication system is used for the wireless transmission of speech, music and sounds between a *CM-1* transmitter and a *CM-1* receiver which may be up to 30 m apart.

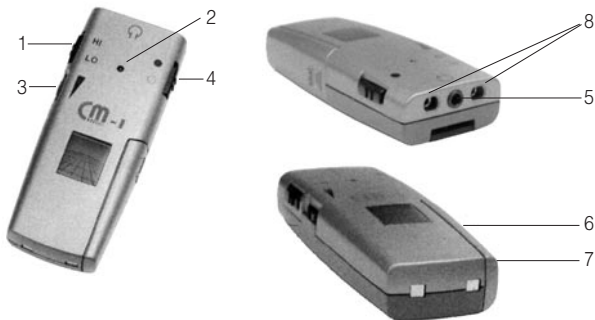CM-1 receiver

No.	Control feature	Function and application	<i>CM-1</i> receiver
1	HI/LO selector switch	Switch setting	HI Accentuates the transmission of the treble tones LO Accentuates the transmission of the bass tones
2	LED (red)	Battery charge status when device is switched on	<ul style="list-style-type: none"> ● Continuous Battery is adequately charged light ○ Flashing light Will need to be recharged. ○ No light Battery discharged. Place the <i>CM-1</i> receiver on the CM charger to re-charge the battery.

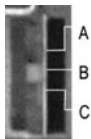
No. Control feature Function and Application *CM-1* receiver

3 Volume control Switch setting 0 No effective tone transmission low volume. (Recommended setting during commissioning)
1,2 3,4 Medium volume
5,6 High volume

4 On/off switch Switch setting On ● Push the switch up to activate your *CM-1* receiver
Off ○ Push the switch down to deactivate your *CM-1* receiver

5 Headset jack Option for connecting a headset directly

6 Frequency selector switch There is a slide switch for setting the following 3 frequency ranges in the battery compartment of the *CM-1* receiver:


Switch setting A 863,500 MHz
B 864,000 MHz
C 864,500 MHz

7 Battery compartment cover The battery compartment is located on the bottom right of the device. The battery is inserted into it.

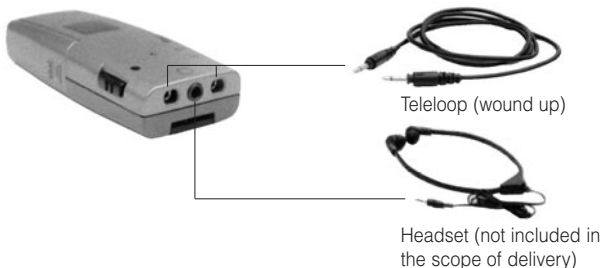8 Teleloop Teleloop socket

Inserting the teleloop into the *CM-1* receiver:

The *CM-1* receiver can be used in two ways:

- via the teleloop which is supplied
- via a headset (not included in the scope of delivery)

In order to ensure successful transmission via a teleloop, your hearing aid must have an integrated telecoil (T-coil) with which the sound information is transmitted inductively to your hearing aid. Hearing aids generally tend to have a small T/MT switch which can be used to set your hearing aid to inductive reception.

1. Insert the two plugs fully into the jacks on the top of the device. The marking on the side of the plugs points towards the headset jack.
2. Turn the plugs through 90° so that the markings on the side point towards the top of the device.

Inserting the battery into the *CM-1* receiver:

The device runs on a rechargeable "AAA"-type battery. Where necessary, only use original replacement batteries.

It can also run on a non-rechargeable battery. If you run your *CM-1* communication system on disposable batteries, don't place the receiver onto the CM charger.

Ensure that the battery is placed the right way around into the battery compartment. The stipulated installation direction is indicated on the bottom of the battery compartment.

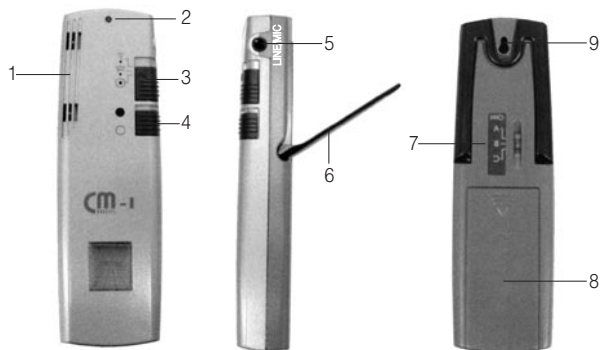
N.B. This battery is not adequately charged upon delivery and has to be charged fully on the CM charger first of all.

1. Switch off your *CM-1* receiver.
2. Remove the battery compartment cover from your *CM-1* receiver.
3. Place the supplied battery into the battery compartment (note the polarity!).
4. Place the cover onto the battery compartment.
5. Place the *CM-1* receiver into one of the CM chargers.

The right-hand LED on the CM recharger lights up in red during the entire charging process. After 10 hours, the battery is fully recharged and the LED now lights up in green.

CM-1 transmitter

No. Control feature Function and application CM-1 transmitter

1	Microphone unit	Two microphones a defined distance from the area where the sound is picked up.
2	LED (red)	Battery charge status when device is switched on
	●	Continuous light Battery is adequately charged
	◐	Flashing light Will need to be recharged.
	○	No light Battery discharged. Place the CM-1 transmitter on the CM charger to recharge the battery.

No. Control feature Function and application *CM-1* transmitter

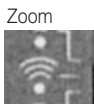
- 3 Mode selector switch

Switch setting

Dialogue: audibly reduces disruptive sounds from the sides and from behind. When the device is switched on, the change to this switch position is indicated acoustically by three signal pulses.

Meeting: audibly reduces disruptive sounds from behind and, at the same time, amplifies signals from in front. When the device is switched on, the change to this switch position is indicated acoustically by two signal pulses.

Presentation: amplifies the signals from all directions. When the device is switched on, the change to this switch position is indicated acoustically by a signal pulse.

- 4 On/off switch

Switch setting

Push the switch up to activate your *CM-1* transmitter
Push the switch down to deactivate your *CM-1* transmitter

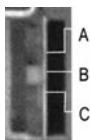
- 5 Audio/microphone in jack

Facility for connecting the *CM-1* transmitter directly to your television or music system. Use the audio cable supplied. Instead of an audio cable, you can also connect an external microphone (special accessory). (If an external microphone is used, the built-in microphone will be switched off automatically.)

No. Control feature Function and application *CM-1* transmitter

6 Stand Fould-out stand

7 Frequency selector switch There is a slide switch on the back of the device for setting the following 3 frequency ranges:

Switch setting	A	863,500 MHz
	B	864,000 MHz
	C	864,500 MHz

Slide the selector switch to the required frequency position using the supplied pin.

Bear in mind at all times that the *CM-1* transmitter and the *CM-1* receiver have to be set to the same frequency ranges. Different frequency ranges prevent each transmission.

8 Battery compartment cover The battery compartment is located on the back of the device and holds both batteries.

9 Recording device pendant cord Insert the knob of the pendant cord into the circular part of the device and engage the knob perceptibly by pushing it towards the tip of the device.

Inserting the batteries into the *CM-1* transmitter:

The device runs on two rechargeable "AAA"-type batteries. Where necessary, only use original replacement batteries.

It can also run on non-rechargeable batteries. If you run your *CM-1* communication system on disposable batteries, don't place the transmitter onto the *CM* charger.

Ensure that the batteries are placed the right way around into the battery compartment. The stipulated installation direction is indicated on the bottom of the battery compartment.

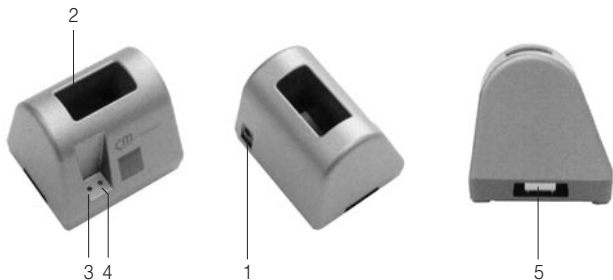
N.B. This battery is not adequately charged upon delivery and has to be charged fully on the CM charger first of all.

1. Switch off the *CM-1* transmitter.
2. Remove the battery compartment cover
3. Place the supplied batteries into the battery compartment (note the polarity!).
4. Place the cover onto the battery compartment.
5. Place the *CM-1* transmitter onto one of the CM chargers.

The right-hand LED on the CM recharger lights up in red during the entire charging process. After 10 hours, the battery is fully recharged and the LED now lights up in green.

CM charger

No.	Control feature	Function and application	CM-1 charger	
1	12 V voltage input	Jack for the power supply unit on the back of the CM-charger.		
2	Charging cradle for device components	3-pin charging cradle to hold the <i>CM-1</i> transmitter unit or <i>CM-1</i> receiver unit upright.		
3	LED (blue)	Standby indicator	lights	Power supply is connected
4	LED (red/green)	Charge indicator	red green	Battery is recharging Battery is fully recharged
5	Bridge element connector slot	There is a connector slot in the base under a plastic cover. A bridge element can be pushed into it to enable two CM chargers to be connected.		
				
6	Bridge element	Bridge element for connecting two CM chargers electrically.		

Installation of the CM charger

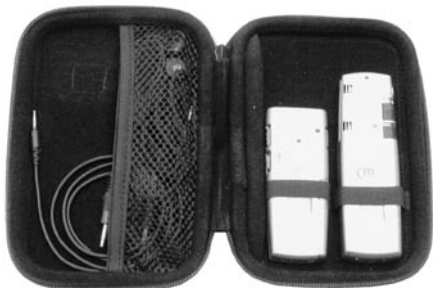
1. Plug the power supply unit into the 230 V mains.
2. Connect the CM charger to the power supply unit by inserting the round connector of the power supply unit into the DC jack on the back of the CM charger.

Interconnecting two CM chargers

1. Disconnect the two chargers, which you wish to join, from the mains supply.
2. Detach one bridge element from the base of one of the chargers.
3. On the first charger, remove the plastic covers from the bridge element connector slot. Slide the bridge element into the connector slot.
4. Fix the bridge element in place through the underside of the device base using the supplied screws.
5. Remove the corresponding plastic cover from the second charger.
6. Place the second charger onto the bridge element that protrudes out from the first charger.
7. Slide the second charger flush against the first charger.
8. Fix the second charger in place through the underside of the device base onto the bridge element using the supplied screws.
9. Connect one of the two chargers to the power supply unit by inserting the round connector of the power supply unit into the DC jack on the back of the CM charger.

! Where there are two chargers connected together, only use one power supply unit!

Transport pouch


The *CM-1* communication system in everyday use

You can influence the tone of the sound transmitted by varying the tone pattern on the HI/LO selector switch: the HI position accentuates the transmission of the treble tones.

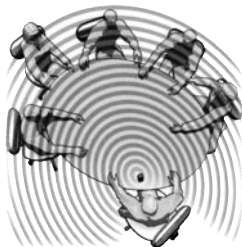
At a presentation

- You are seated in groups around a number of tables in a presentation room, for example.
- Sound is distributed evenly throughout the room via a loudspeaker system.
- Occasional questions are put to the speaker from people seated at the groups of tables.
- Use "**Omni**" transmission mode.

1. Set your hearing aid to "T" or "MT" in order to transmit sound information directly to the T-coil in your hearing aid via the teleloop, or connect a headset to a *CM-1* receiver.
2. Slide the mode selector switch to the Omni position.
3. Position your *CM-1* transmitter as centrally as possible, e.g. on the conference table.
4. Use the fold-out stand to raise the microphone unit somewhat and to mechanically dampen jolting noises from the surface of the table.
5. Switch the *CM-1* transmitter and *CM-1* receiver on at the on/off switch.
6. Adjust the transmission volume to a comfortable level by setting the volume controller on your *CM-1* receiver to an appropriate level.

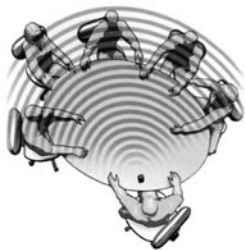Meeting "around a table"

- You are taking part in the meeting shown.
- The other people are speaking at different levels.
- Use the **"Zoom"** transmission mode to reduce disruptive sounds from behind perceptibly and, at the same time, to amplify speech in front of you.

1. Set your hearing aid to "T" or "MT" in order to transmit sound information directly to the T-coil in your hearing aid via the teleloop, or connect a headset to a *CM-1* receiver.
2. Slide the mode selector switch to the Zoom position.
3. Position your *CM-1* transmitter as close as possible to the speaker, whereby the device's microphone unit should be pointing as directly as possible towards the speaker.
4. Switch the *CM-1* transmitter and *CM-1* receiver on at the on/off switch.
5. Adjust the speech volume to a comfortable level by setting the volume controller on your *CM-1* receiver to an appropriate level.

In "direct" dialogue

- You are having a more private conversation with another person.
- You would like to concentrate fully on this person.
- You are distracted by all the other types of noise.
- Use **"Super Zoom"** transmission mode to reduce all noises from the sides and from behind perceptibly and to filter out the other person.

1. Change the position of your *CM-1* transmitter so that it is pointing as directly as possible towards the other person.
2. Slide the mode selector switch to the Super Zoom position.
3. Adjust the transmission volume to a level which is reasonable for the conversation.

On a trip – in a car, in a train compartment or on an airplane

You are on a journey, for instance as part of a travel group on a train or bus, accompanied by someone in the car or traveling alone by air. You want to be able to chat to the person sitting beside you during the trip, however the person isn't sitting on the same side as your hearing aid.

The situation is compounded by all the other disruptive sounds on board, especially the permanent noise from the air-conditioning, in an airplane for example.

Just like having a private conversation at a meeting, you need to amplify all the sounds made by the other person with the greatest possible concentration.

The *CM-1* produces the desired effect in Super Zoom transmission mode by reducing all noises from the sides and from behind perceptibly and by filtering out the conversation with the other person.

Enjoying music or TV programmes

There is an Audio/microphone in jack on the *CM-1* transmitter which you can connect directly to your television or music system using the supplied audio cable. How to connect a device:

1. Switch off the *CM-1* transmitter and the *CM-1* receiver.
2. Set the volume control on the *CM-1* receiver to a low value.
3. Insert one of the two cinch plugs (\varnothing 3.5mm) into the Audio/microphone in jack of the *CM-1* transmitter.
4. Insert the other cinch plug into the audio output jack of your television or music system.
5. Switch the *CM-1* transmitter and the *CM-1* receiver back on.
6. Adjust the transmission volume to a comfortable level.

Changing the transmission frequency

If you experience transmission interference, this may be due to the fact that there are two *CM-1* systems being operated in close proximity to one another. The *CM-1* transmitter and *CM-1* receiver are capable of communicating in three different frequency ranges. The frequency ranges are characterized by the letters "A", "B" and "C" and apply to transmitters and receivers alike.

Bear in mind at all times that the *CM-1* transmitter and the *CM-1* receiver have to be set to the same frequency ranges. Discrepancy in frequency ranges will prevent any transmission.

1. Switch off the *CM-1* transmitter and the *CM-1* receiver.
2. Open the battery compartment on your *CM-1* receiver and take out the battery.
3. Use the supplied pin to adjust the frequency selector switch inside the battery compartment.
4. Re-insert the battery and place the cover on the battery compartment.
5. Turn the *CM-1* transmitter around so that it is facing away from you.
6. Use the supplied pin to adjust the frequency selector switch on the back of the *CM-1* transmitter to the same value that you had previously selected for your *CM-1* receiver.
7. Switch the *CM-1* transmitter and the *CM-1* receiver back on.

Appendix

Safety instructions

The *CM-1* communication system is designed in accordance with the current state of the art and recognized safety rules. Pay attention to the following safety instructions:

- Position the charger on a non-slip base.
- Lay the connecting cable so that it cannot be tripped over.
- Never expose the device to the following sources of influence:
 - heaters
 - direct sunlight
 - moisture
- Protect the device from damp, dust, aggressive fluids and vapors.
- Only connect the connecting cables to the designated sockets / jacks.
- Only connect approved accessories.
- Under no circumstances may modifications of a technical or non-technical nature be made to the communication system.
- The *CM-1* communication system contains small parts which can be swallowed and is not suitable for children under the age of 3.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

Maintenance and care

The *CM-1* communication system does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

Warranty

The *CM-1* communication system is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorized to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Environmental advice/Disposal

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical data

CM-1 transmitter

Dimensions (H x W x D):	19 x 37x120 mm
Weight:	64 g
Power supply:	2 x 1,2 V NiMH-batteries (LR03 type)
Charge time (following complete discharge):	max. 10 h
Operating time (load-dependent):	max. 10-12 h

CM-1 receiver

Dimensions (H x W x D):	20 x 39 x 96 mm
Weight:	45 g
Power supply:	1 x 1,2 V NiMH-batteries (LR03 type)
Charge time (following complete discharge):	max. 10 h
Operating time (load-dependent):	max. 10-12 h

CM charger

Dimensions (H x W x D):	67 x 83 x 65 mm
Weight:	94g
Power supply via a PSU:	Primary 220/230 V 50-60 Hz AC / Secondary 12 V DC

CE symbol

This product conforms to the requirements of EU Directive 1999/5/EC on radio equipment and telecommunications terminal equipment and the mutual recognition of their conformity. Conformity with the afore-mentioned directive is confirmed by the CE symbol on the device.

Technical specifications subject to change without notice.

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